


# Liberty Hemi Wheel Lock

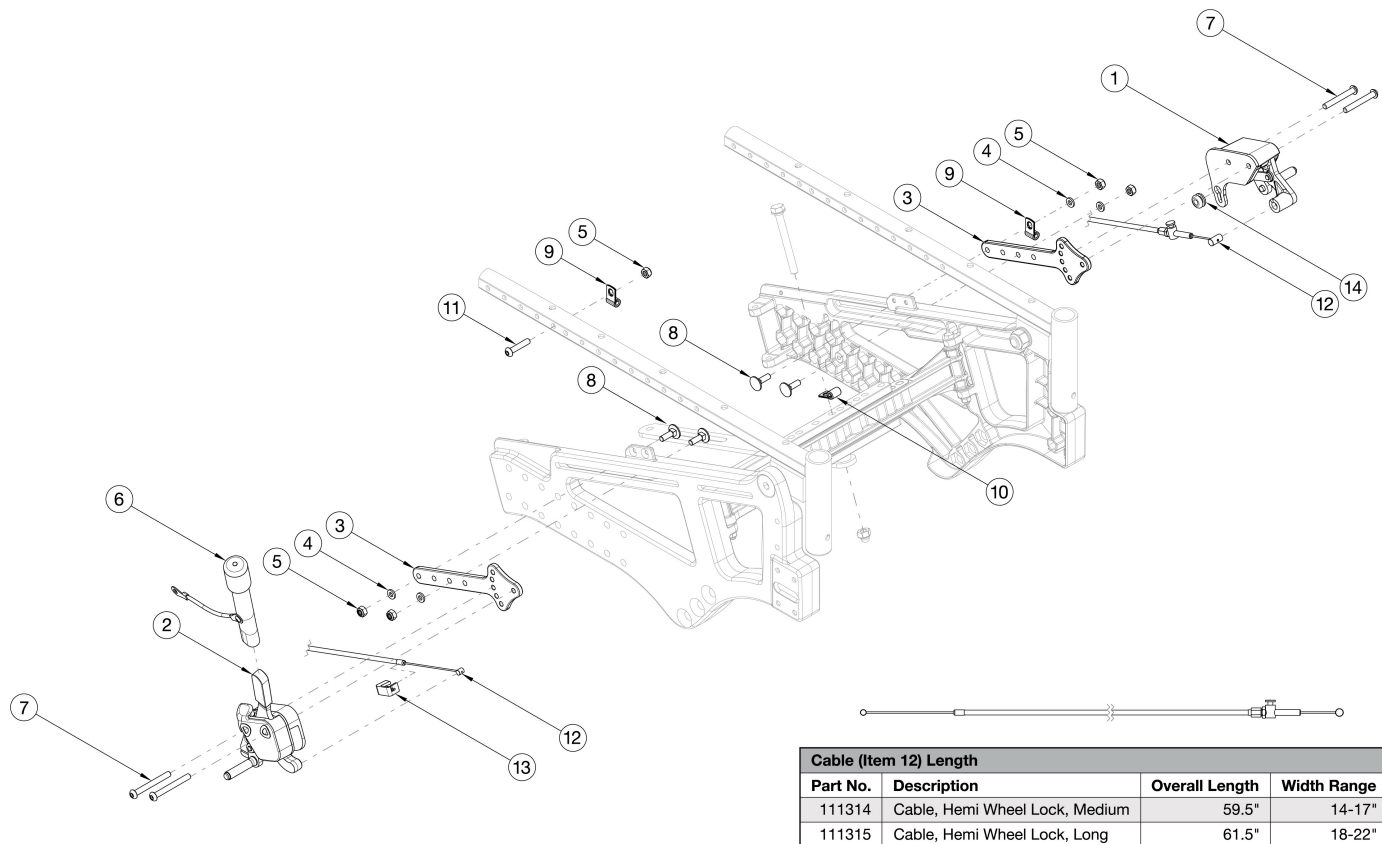

Cable (Item 12) Length			
Part No.	Description	Overall Length	Width Range
111314	Cable, Hemi Wheel Lock, Medium	59.5"	14-17"
111315	Cable, Hemi Wheel Lock, Long	61.5"	18-22"

Available for retrofit within the field.

Item Number	Part Number	Description	Quantity
1a, 2a, 3-12a, 13, 14	112391	Hemi Wheel Lock Assembly with Medium Cable, Right Side Mounted	1
1b, 2b, 3-12a, 13, 14	112392	Hemi Wheel Lock Assembly with Medium Cable, Left Side Mounted	1
1a, 2a, 3-12b, 13, 14	112393	Hemi Wheel Lock Assembly with Long Cable, Right Side Mounted	1
1b, 2b, 3-12b, 13, 14	112394	Hemi Wheel Lock Assembly with Long Cable, Left Side Mounted	1
1a	111591	Wheel Lock Follower Assembly without Mount, Left	1
1b	111592	Wheel Lock Follower Assembly without Mount, Right	1
2a	111593	Hemi Wheel Lock, Master Assembly, Right without Mount	1
2b	111594	Hemi Wheel Lock, Master Assembly, Left without Mount	1
3	111220	Hemi Wheel Lock Mount, Liberty	1
4	100746	M6 Flat Washer Blk Zn	4

Item Number	Part Number	Description	Quantity
5	100658	M6 Nylock Nut Blk Zn	5
6	200369	Wheel Lock Extension	1
7	100670	M6 x 45 BTN SHCS BLK ZC	4
8	103032	M6 x 20 Carriage Bolt, Blk Zn	4
9	001239	P-Clip 3/16"	2
10	111597	P-Clip, 1/4", 3/8" Hole	1
11	102625	M6 X 30 BHCS BLK ZC	1
12a	111314	Hemi Wheel Lock Cable, Medium <i>Used with Seat Widths 14-17"</i>	1
12b	111315	Hemi Wheel Lock Cable, Long <i>Used with Seat Widths 18-22"</i>	1
13	115879	Hemi Wheel Lock Cable Retaining Clip with Instruction	1
14	115743	Wheel Lock Retaining Grommet	1