

Wheel Lock Setup




Quickie IRIS

15-Jul-13

- No-Go
- Setup Not Available
- *Forward
- Rearward
- Short Rearward

Attendant Wheel Lock POS

Rear Wheel Size	Roller Pos	Axle Pos							Order Kit #	*Cable & **Slide Block
		12	13	14	15	16	17	18		
12	1		2	2	1	1	1	1	108138 Rearward is Reverse (will need to flip the block)	*Short Cable and **Standard Slide Block
	2		2	2	1	1	1	1		
	3		3	3	2	2	1	1		
	4			3	3	2	2	1		
	5				3	3	2	2		
	6					3	3	2		
16	1		3	2	2	1	1	1	108139 for 20-24 Mag Wheels OR 108140 for 20-24 Spoke Wheels Rearward is Reverse (will need to flip the block)	*Long Cable and **Standard Slide Block
	2	1*	3	3	2	2	1	1		
	3			3	3	2	2	1		
	4				3	3	2	2		
	5					3	3	2		
	6						3	3		
20	1	2*	2*	1*	3	3	2	2	108139 for 20-24 Mag Wheels OR 108140 for 20-24 Spoke Wheels Rearward is Reverse (will need to flip the block)	*Long Cable and **Standard Slide Block
	2	2*	2*	1*	1*	3	3	2		
	3	3*	2*	2*	1*	1*	3	3		
	4		3*	2*	2*	1*	1*	3		
	5			3*	2*	2*	1*	1*		
	6				3*	2*	2*	1*		
22	1	2*	2*	1*	1*	3	3	2	108139 for 20-24 Mag Wheels OR 108140 for 20-24 Spoke Wheels Rearward is Reverse (will need to flip the block)	*Long Cable and **Standard Slide Block
	2	3*	2*	2*	1*	1*	3	3		
	3	3*	3*	2*	2*	1*	1*	3		
	4		3*	3*	2*	2*	1*	1*		
	5			3*	3*	2*	2*	1*		
	6				3*	3*	2*	2*		
24	1	3*	2*	2*	1*	1*	3	3	108139 for 20-24 Mag Wheels OR 108140 for 20-24 Spoke Wheels Rearward is Reverse (will need to flip the block)	*Long Cable and **Standard Slide Block
	2	3*	2*	2*	2*	1*	1*	3		
	3		3*	3*	2*	2*	1*	1*		
	4			3*	3*	2*	2*	1*		
	5				3*	3*	2*	2*		
	6					3*	3*	2*		

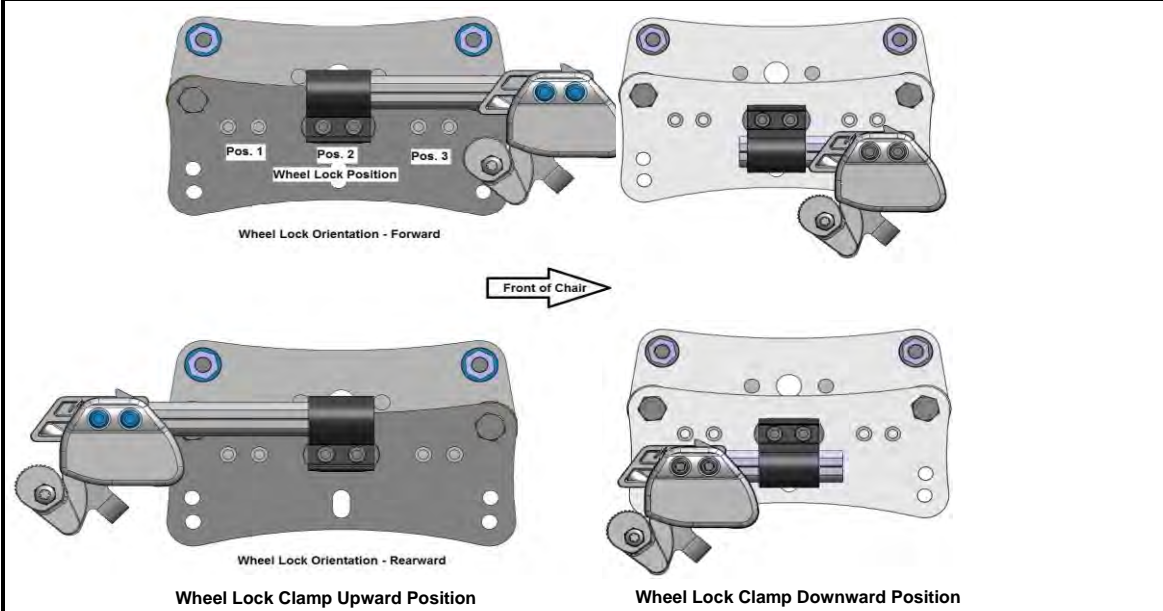

Notes:


- * When Axle Plate Orientation is in 1-4 (ENG910), install wheel lock clamp in upward position
- * When Axle Plate Orientation is in 5-8 (ENG910), install wheel lock clamp in downward position

Wheel Lock Setup

- No-Go
- Setup Not Available
- *Forward
- Rearward
- Short Rearward

Short Rearward Attendant Wheel Lock POS

Rear Wheel Size	Roller Pos	Axle Pos							Order Kit #	*Cable & **Slide Block
		12	13	14	15	16	17	18		
12	1	3							126252 Rearward is Reverse	*Short Cable and **Short Slide Block
	2	3								
	3									
	4									
	5									
	6									
	6									
16	1	3							126252 Rearward is Reverse	*Short Cable and **Short Slide Block
	2									
	3									
	4									
	5									
	6									
	6									
20	1									
	2									
	3									
	4									
	5									
	6									
	6									
22	1									
	2									
	3									
	4									
	5									
	6									
	6									
24	1									
	2									
	3									
	4									
	5									
	6									
	6									

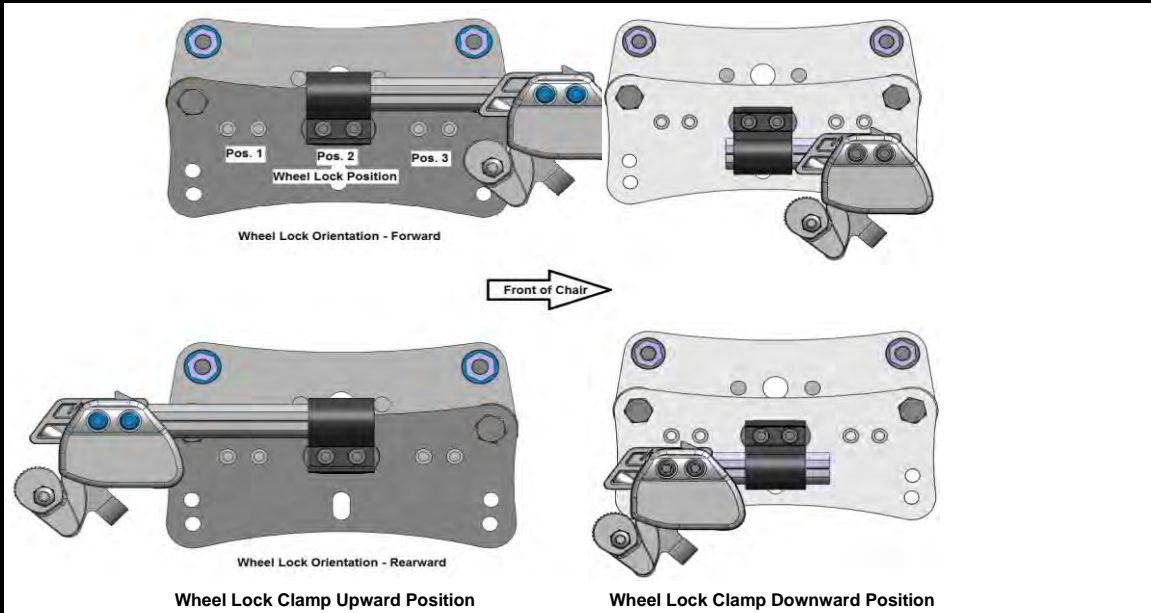

Notes:

- * When Axle Plate Orientation is in 1-4 (ENG910), install wheel lock clamp in upward position
- * When Axle Plate Orientation is in 5-8 (ENG910), install wheel lock clamp in downward position