


# (Discontinued 1) Focus CR Dual Hand Tilt Mechanism Stroller Handle Back

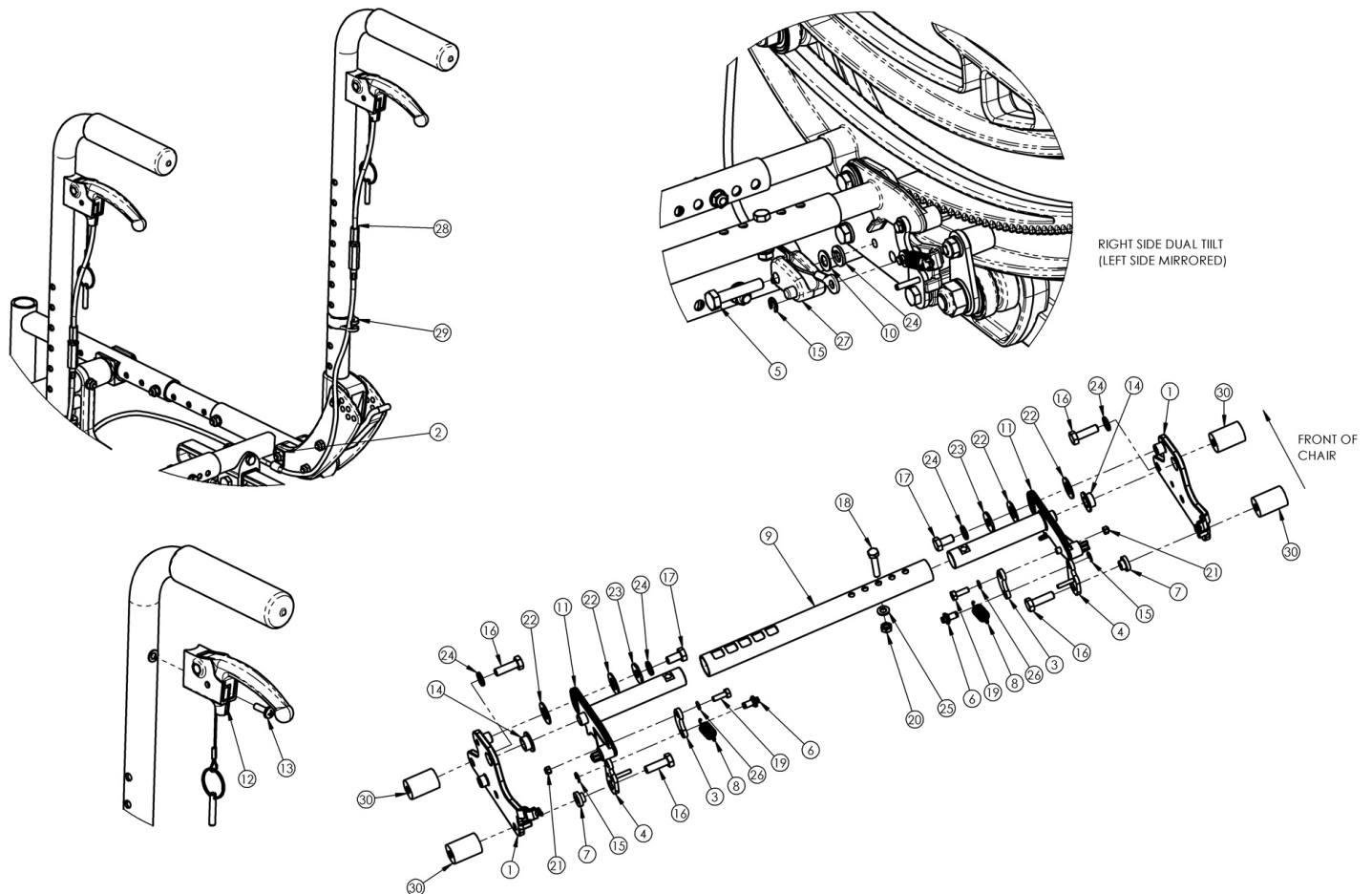

*This Focus CR Hand Tilt is discontinued and was used on chairs with serial numbers prior to FCR0025162 and FCHD002316.*

Item Number	Part Number	Description	Quantity	Retail
2, 12, 13, 28a, 29	503777	Dual Hand Tilt / Trigger Kit for Short Stroller Handle/Fixed Height/Recline Back, Focus	1	\$288.62
2, 12, 13, 28b, 29	503778	Dual Hand Tilt / Trigger Kit for Tall Stroller Handle/Fixed Height/Recline Back, Focus	1	\$290.76
1a, 14	503779	Tilt Lock Mount Bracket w/ Bearing Kit, Right, Focus, Dual Fixed Height	1	\$19.27
1b, 14	503780	Tilt Lock Mount Bracket w/ Bearing Kit, Left, Focus, Dual Fixed Height	1	\$19.27
2	001239	P-Clip 3/16"	2	\$6.30
3	003247	Link, Drive, Tilt	2	\$12.87
4a	002084	Toggle, Drive, Tilt, R	1	\$11.28
4b	002085	Toggle, Drive, Tilt, L	1	\$20.45

Item Number	Part Number	Description	Quantity	Retail
5	101824	M6 x 30 HHCS Blk Zn	2	\$2.10
6	003364	Pin, Toggle, Tilt	2	\$7.25
9a	002094	Growth Strut, Tilt Lock, Small	1	\$33.62
9b	002095	Growth Strut, Tilt Lock, Med	1	\$35.72
9c	002096	Growth Strut, Tilt Lock, Large	1	\$37.82
10	102803	Washer Flat .250 x .500 x .027 HX BLZ	2	\$1.58
11a	002411	Cog Lever, Tilt Lock, R	1	\$38.59
11b	002412	Cog Lever, Tilt Lock, L	1	\$38.59
12, 13	110139	Lever Assembly with Lanyard Vertical Tilt	2	\$34.00
13	100734	M5 x 16 PHMS ZC	2	\$1.58
16a	101823	M6 x 20 HHCS Blk Zn	4	\$1.05
16b	100674	M6 x 45 CLS10.9 DIN931 HHCS ZC <i>Used with Heavy Duty option/spacer</i>	2	\$1.58
16c	101826	M6 x 50 HHCS Blk Zn <i>Used with Heavy Duty option/spacer</i>	2	\$2.10
17	101822	M6 x 12 HHCS BLKZC w/Patch	2	\$1.05
18	101821	M5 x 25 HHCS Blk Zn	1	\$1.05
19	102783	M3 x 12 HHCS, CL8.8, Blk Zn	2	\$1.05
20	100657	M5 Nylock Nut Blk Zn	1	\$0.53
21	102784	Nut, Nylock, M3, Blk	2	\$1.05
22	102082	Washer Nylon .406IDX.75ODX.040	4	\$0.56
23	101853	M6 Washer Blk Zn	2	\$0.53
24	101835	M6 Lock Washer Blk Zn	5	\$0.53
25	101856	Flat Washer, M5, Blk Zn	1	\$0.53
26	102785	Flat Washer, M3, Blk Zn	2	\$0.53
27	003387	Cable Mount, Tilt	2	\$3.68
28a	002100	Cable, Tilt, Short <i>Used with short back height range</i>	2	\$107.16
28b	002101	Cable, Tilt, Med	2	\$107.16

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>	<b>Retail</b>
		<i>Used with tall back height range</i>		
29	001117	Cable Clip 1"	2	\$3.83
30	001405	Spacer, .69OD x .24ID x .1.01Lg <i>Only used with Heavy Duty Option</i>	4	\$26.05