


Ethos Tipping Lever

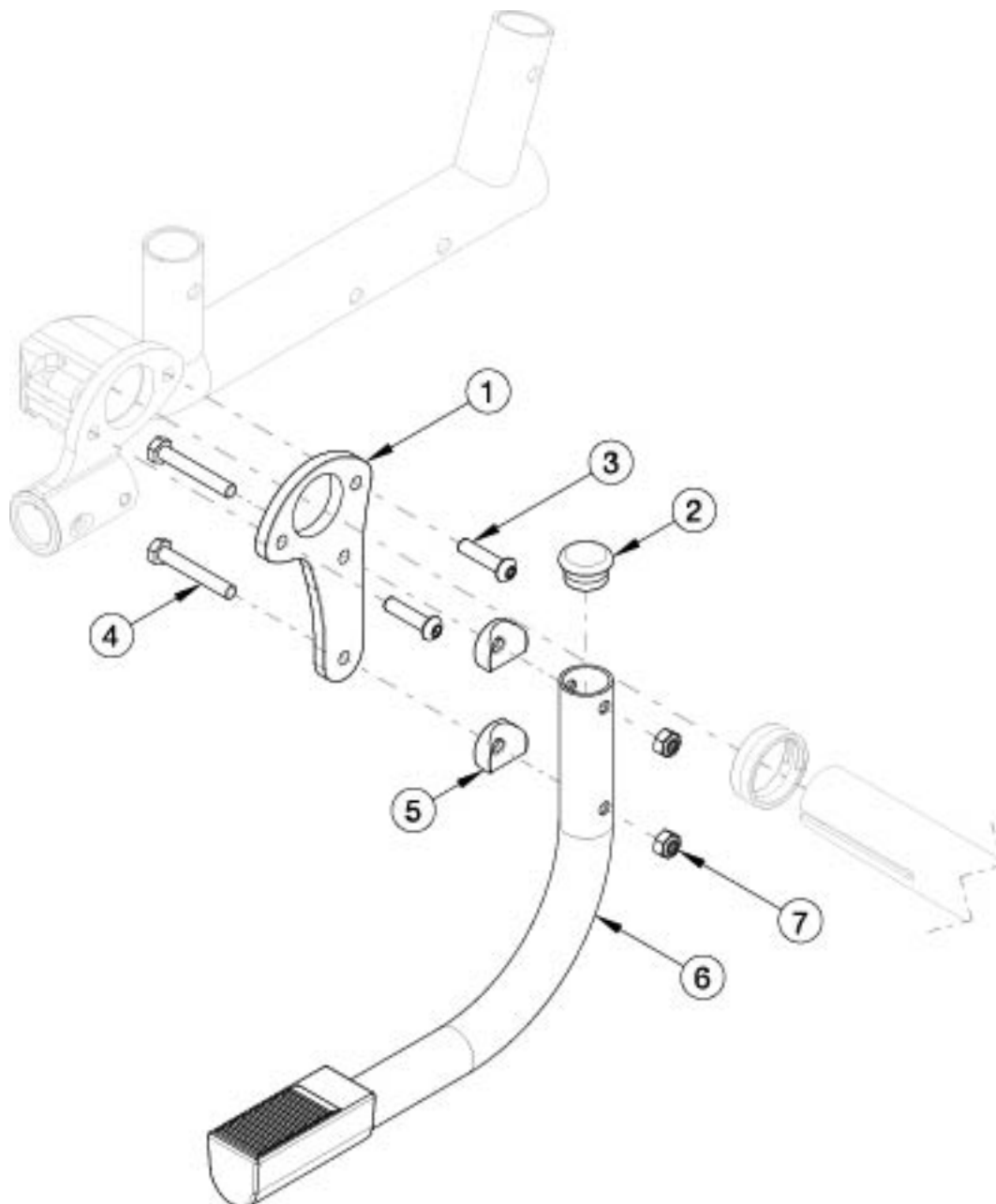

Not available if right and left User Activated Anti Tips are present

Item Number	Part Number	Description	Quantity
1 - 4a, 5 - 9	119634	Tipping Lever Kit Ethos <i>Used with 24" - 26" Wheels</i>	1
1 - 4b, 5 - 9	120810	Tipping Lever Short Kit Ethos <i>Used with 22" Wheels</i>	1
1	119606	Ethos Tipping Lever Adapter Plate	1

Item Number	Part Number	Description	Quantity
3	115573	M6 x 25, BHCS, CLS 10.9, NYL, BLZ	2
4	100681	M6 x 40 CLS8.8 DIN931 HHCS ZC	2
5	000054	Saddle 7/8"	2
6a, 2	121380	Tipping Lever Tube Ethos <i>Used with 24" - 26" Wheels</i>	1
6b, 2	121381	Tipping Lever Tube Short Ethos <i>Used with 22" Wheels</i>	1
7	100658	M6 Nylock Nut Blk Zn	2