


Arc Rear Side Frame

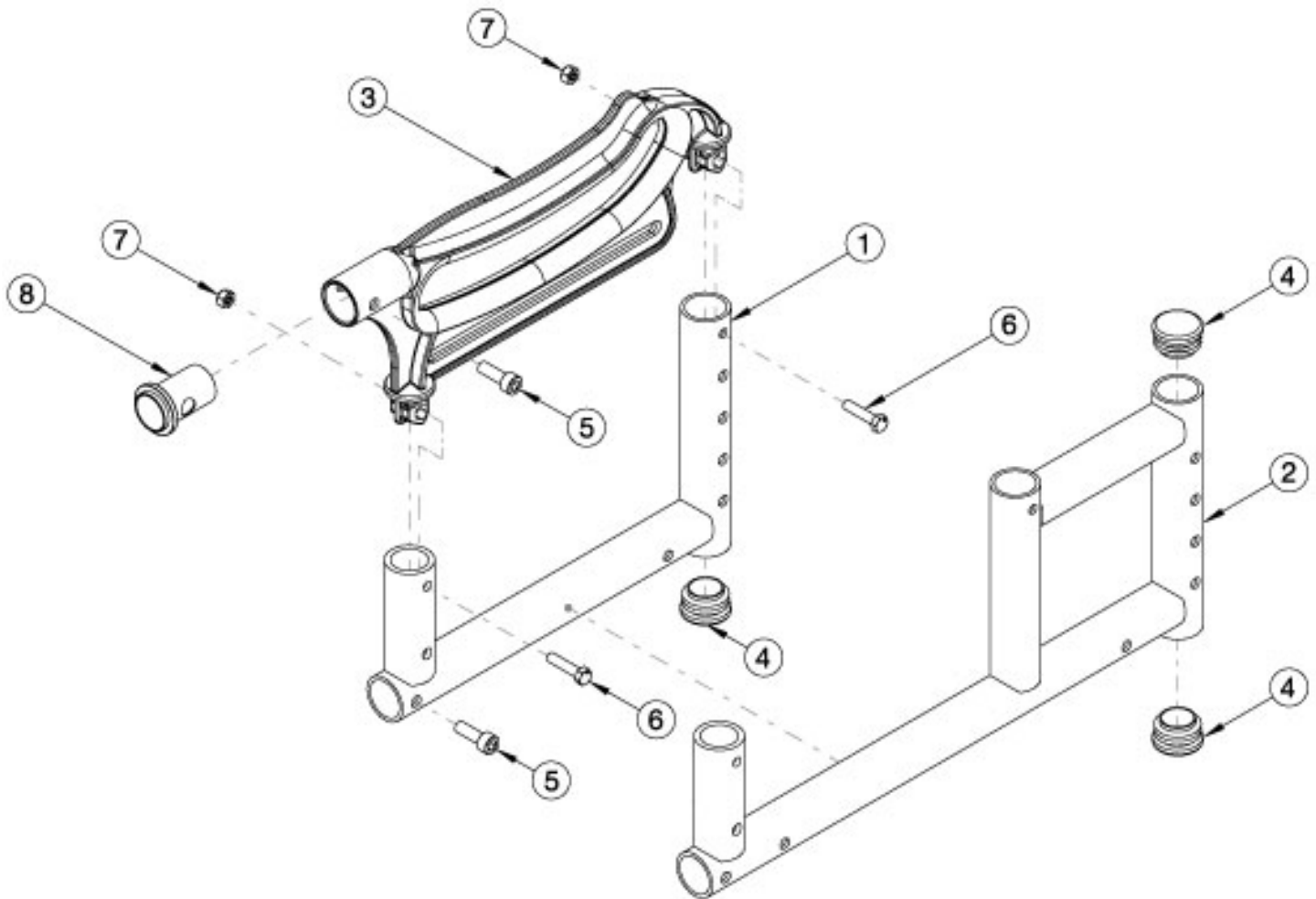

When switching to extended frames/width new tilt cables may be needed.

Item Number	Part Number	Description	Quantity
1a, 4	113117_ _ _	Rear Side Frame, Right, Arc	1
1b, 4	113118_ _ _	Rear Side Frame, Left, Arc	1
2a, 4	113119_ _ _	Rear Side Frame, Extended, Right, Arc <i>Used when the Reclining Backrest or Vent/Battery Tray is present.</i>	1
2b, 4	113120_ _ _	Rear Side Frame, Extended, Left, Arc <i>Used when the Reclining Backrest or Vent/Battery Tray is present.</i>	1
3	110164	Roller Guide	1
4	100804	Tube End Plug 1"	1
5	101844	M6 x 18 SHCS Blk Zn	2

Item Number	Part Number	Description	Quantity
6	101820	M5 x 22 HHCS Blk Zn	2
7	100657	M5 Nylock Nut Blk Zn	2
8	110149	Bushing, Pivot, X-Hinge	1