


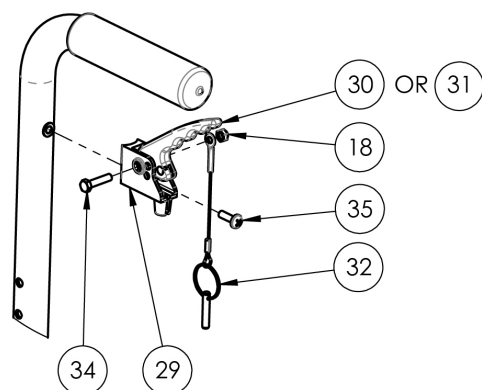

*This Focus CR Hand Tilt is discontinued and was used on serial numbers prior to FCR0013026.*

| Item Number | Part Number | Description                    | Quantity |
|-------------|-------------|--------------------------------|----------|
| 6a          | 001254      | Growth Strut Short TILT        | 1        |
| 6c          | 001256      | Growth Strut Long TILT         | 1        |
| 8           | 001109      | Spring Tilt Lock Right TILT    | 1        |
| 9           | 001110      | Spring Tilt Lock Left TILT     | 1        |
| 10          | 001104      | Retainer Lock Lever TILT       | 2        |
| 11          | 001112      | Mount Cable 90 Deg TILT        | 1        |
| 12          | 001111      | Bearing GFI-0607-04 TILT       | 2        |
| 13          | 101823      | M6 X 20 Hhcs Blk Zn            | 4        |
| 14          | 101821      | M5 x 25 HHCS Blk Zn            | 1        |
| 15          | 101818      | M4 x 10 FHCS BLKZC w/Patch     | 2        |
| 16          | 101844      | M6 x 18 SHCS Blk Zn            | 2        |
| 17          | 100658      | M6 Nylock Nut Blk Zn           | 2        |
| 18          | 100657      | M5 Nylock Nut Blk Zn           | 1        |
| 19          | 100746      | M6 Flat Washer Blk Zn          | 2        |
| 20          | 101853      | M6 Washer Blk Zn               | 2        |
| 21          | 101835      | M6 Lock Washer Blk Zn          | 4        |
| 22          | 101856      | Flat Washer, M5, Blk Zn        | 1        |
| 23          | 102082      | Washer Nylon .406IDX.75ODX.040 | 4        |
| 24          | 101842      | M3 x 8 SHCS Blk Zn             | 1        |
| 25a         | 001113      | Cable Tilt Short TILT          | 1        |
| 25b         | 001114      | Cable Tilt Medium TILT         | 1        |
| 26a         | 001115      | Cable Tilt Long TILT           | 1        |
| 26b         | 001116      | Cable Tilt Extra Long TILT     | 1        |
| 27          | 001117      | Cable Clip 1"                  | 1        |
| 28          | 001118      | Cable Clip 7/8"                | 1        |
| 33          | 001239      | P-Clip 3/16"                   | 1        |

| Item Number | Part Number | Description         | Quantity |
|-------------|-------------|---------------------|----------|
| 34          | 101820      | M5 x 22 HHCS Blk Zn | 1        |
| 35          | 100734      | M5 x 16 PHMS ZC     | 1        |