


# Rogue / Ethos Single Rigid Stroller Handle

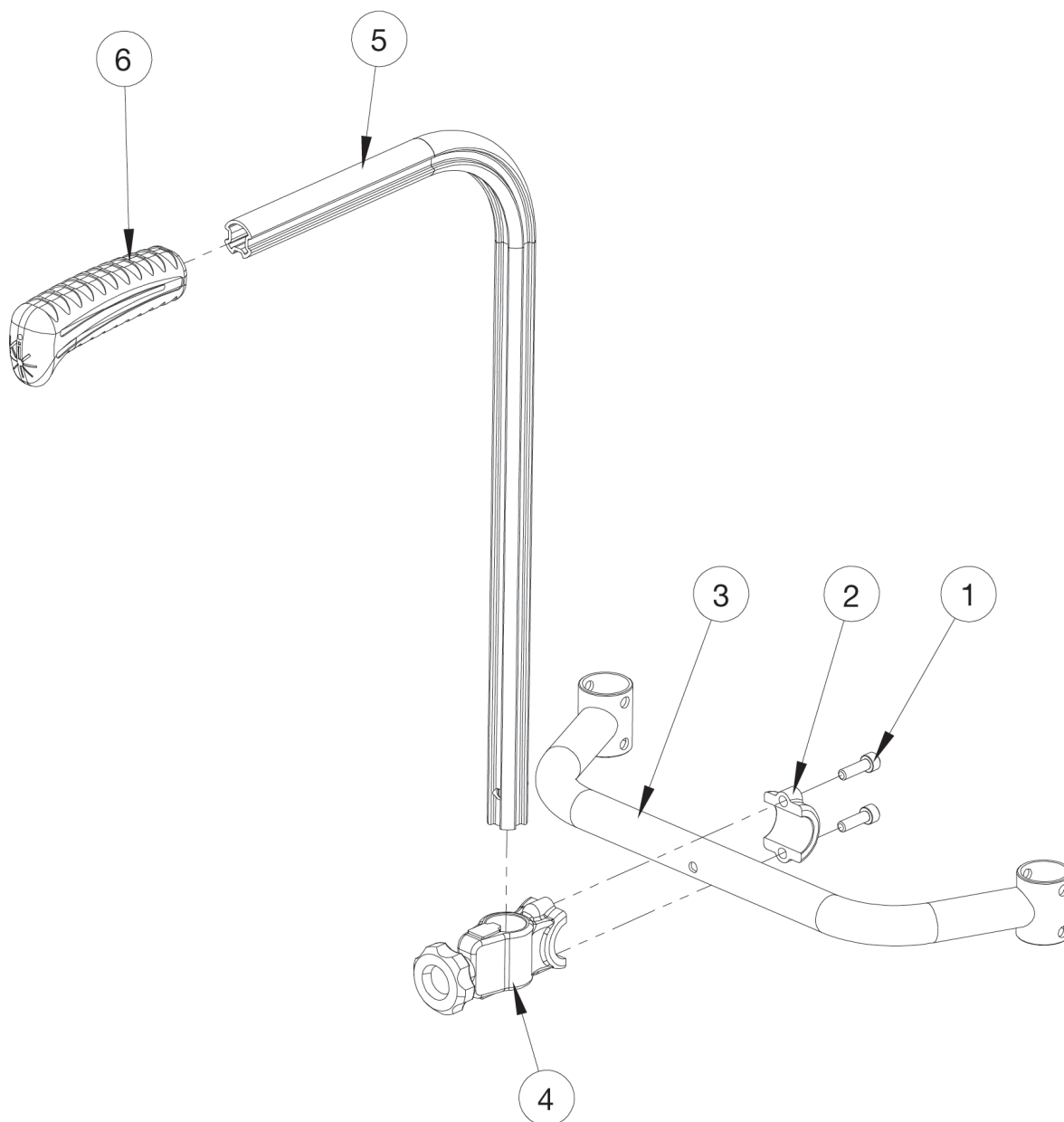

| Item Number | Part Number | Description                                | Quantity |
|-------------|-------------|--------------------------------------------|----------|
| 1,2,4       | 122138      | Stroller Handle Clamp Assembly             | 1        |
| 1,2,4,5a,6  | 125269      | Stroller Handle Short Assembly Each, Rigid | 1        |

| Item Number | Part Number | Description                                                                                                                                                 | Quantity |
|-------------|-------------|-------------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
|             |             | <i>Used on chairs with 15" or greater rear seat height and drum brake is not present.</i>                                                                   |          |
| 1,2,4,5b,6  | 125270      | Stroller Handle Tall Assembly Each, Rigid<br><i>Used on chairs with less than 15" rear seat height and drum brake is not present.</i>                       | 1        |
| 5a,6        | 125271      | Stroller Handle Tube Short, Rigid<br><i>Used on chairs with 15" or greater rear seat height and drum brake is not present. Replaced part number 506137.</i> | 1        |
| 5a          | 122559      | Stroller Handle with Snap Button Short<br><i>Used on chairs with 15" or greater rear seat height and drum brake is not present.</i>                         | 1        |
| 5b          | 122560      | Stroller Handle with Snap Button Tall<br><i>Used on chairs with less than 15" rear seat height and drum brake is not present.</i>                           | 1        |
| 6.0         | 109433      | Ergo Push Handle Grip 7/8" Standard with Instruction                                                                                                        | 1        |
| 3c          | 123621      | Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 12" Wide Single Stroller Handle                                                                                | 1        |
| 3d          | 123622      | Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 13" Wide Single Stroller Handle                                                                                | 1        |
| 3e          | 123623      | Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 14" Wide Single Stroller Handle                                                                                | 1        |
| 3f          | 123624      | Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 15" Wide Single Stroller Handle                                                                                | 1        |
| 3g          | 123625      | Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 16" Wide Single Stroller Handle                                                                                | 1        |
| 3h          | 123626      | Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 17" Wide Single Stroller Handle                                                                                | 1        |
| 3i          | 123627      | Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 18" Wide Single Stroller Handle                                                                                | 1        |
| 3j          | 123628      | Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 19" Wide Single Stroller Handle                                                                                | 1        |
| 3k          | 123629      | Rogue/Ethos HA Back Non HA R-Bar 8.813" Tall 20" Wide Single Stroller Handle                                                                                | 1        |
| 3n          | 123632      | Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 12" Wide Single Stroller Handle                                                                                | 1        |
| 3o          | 123633      | Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 13" Wide Single Stroller Handle                                                                                | 1        |
| 3p          | 123634      | Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 14" Wide Single Stroller Handle                                                                                | 1        |
| 3q          | 123635      | Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 15" Wide Single Stroller Handle                                                                                | 1        |
| 3r          | 123636      | Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 16" Wide Single Stroller Handle                                                                                | 1        |
| 3s          | 123637      | Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 17" Wide Single Stroller Handle                                                                                | 1        |
| 3t          | 123638      | Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 18" Wide Single Stroller Handle                                                                                | 1        |
| 3u          | 123639      | Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 19" Wide Single Stroller Handle                                                                                | 1        |

| Item Number | Part Number | Description                                                                  | Quantity |
|-------------|-------------|------------------------------------------------------------------------------|----------|
| 3v          | 123640      | Rogue/Ethos HA Back Non HA R-Bar 7.813" Tall 20" Wide Single Stroller Handle | 1        |
| 3y          | 123643      | Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 12" Wide Single Stroller Handle | 1        |
| 3z          | 123644      | Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 13" Wide Single Stroller Handle | 1        |
| 3aa         | 123645      | Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 14" Wide Single Stroller Handle | 1        |
| 3ab         | 123646      | Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 15" Wide Single Stroller Handle | 1        |
| 3ac         | 123647      | Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 16" Wide Single Stroller Handle | 1        |
| 3ad         | 123648      | Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 17" Wide Single Stroller Handle | 1        |
| 3ae         | 123649      | Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 18" Wide Single Stroller Handle | 1        |
| 3af         | 123650      | Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 19" Wide Single Stroller Handle | 1        |
| 3ag         | 123651      | Rogue/Ethos HA Back Non HA R-Bar 6.813" Tall 20" Wide Single Stroller Handle | 1        |
| 3ah         | 123606_ _ _ | Adjustable Height Rigidizer Bar 8" Single Stroller Handle                    | 1        |
| 3ai         | 123607_ _ _ | Adjustable Height Rigidizer Bar 9" Single Stroller Handle                    | 1        |
| 3aj         | 123608_ _ _ | Adjustable Height Rigidizer Bar 10" Single Stroller Handle                   | 1        |
| 3ak         | 123609_ _ _ | Adjustable Height Rigidizer Bar 11" Single Stroller Handle                   | 1        |
| 3al         | 123610_ _ _ | Adjustable Height Rigidizer Bar 12" Single Stroller Handle                   | 1        |
| 3am         | 123611_ _ _ | Adjustable Height Rigidizer Bar 13" Single Stroller Handle                   | 1        |
| 3an         | 123612_ _ _ | Adjustable Height Rigidizer Bar 14" Single Stroller Handle                   | 1        |
| 3ao         | 123613_ _ _ | Adjustable Height Rigidizer Bar 15" Single Stroller Handle                   | 1        |
| 3ap         | 123614_ _ _ | Adjustable Height Rigidizer Bar 16" Single Stroller Handle                   | 1        |
| 3aq         | 123615_ _ _ | Adjustable Height Rigidizer Bar 17" Single Stroller Handle                   | 1        |
| 3ar         | 123616_ _ _ | Adjustable Height Rigidizer Bar 18" Single Stroller Handle                   | 1        |
| 3as         | 123617_ _ _ | Adjustable Height Rigidizer Bar 19" Single Stroller Handle                   | 1        |
| 3at         | 123618_ _ _ | Adjustable Height Rigidizer Bar 20" Single Stroller Handle                   | 1        |