


Catalyst E Aluminum Cross Braces

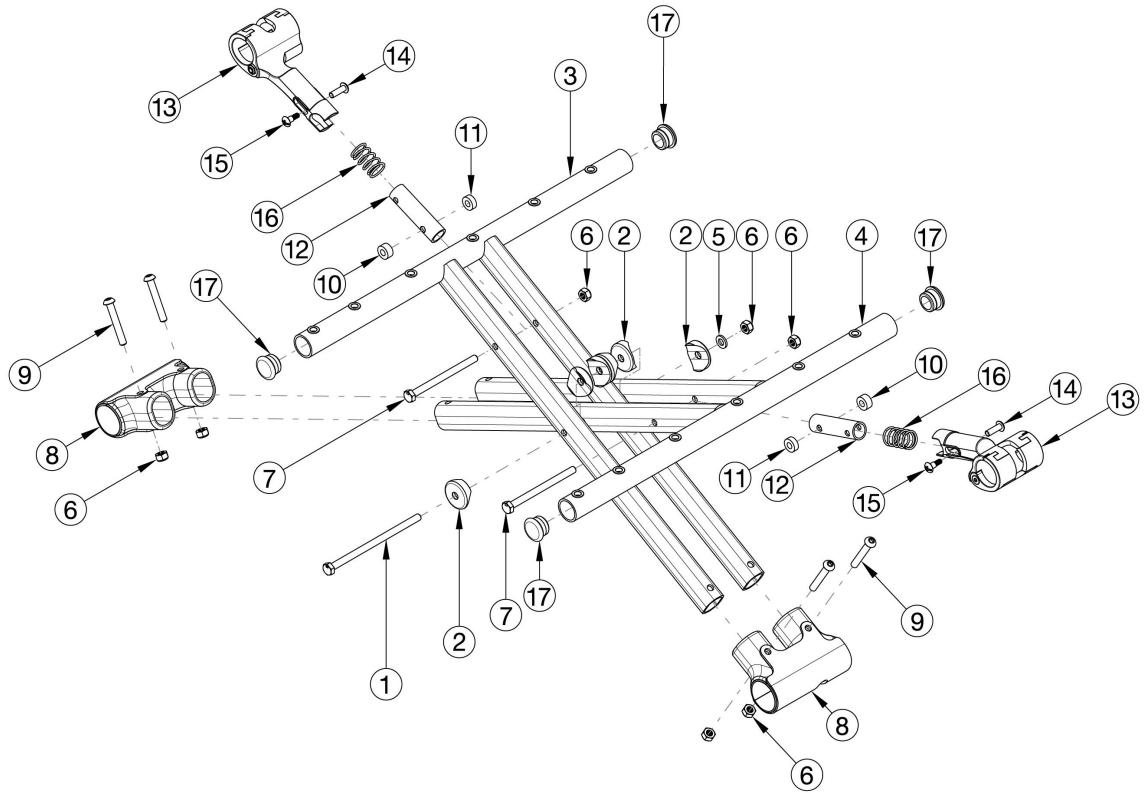

Cross Brace Overall Width	
Seat Width	Cross Brace Measurement
14"-16"	Approx. 1" less than seat width
17"	Approx. 1.25" less than seat width
18"-21"	Approx. 1.5" less than seat width
22"	Approx. 1.75" less than seat width

Cross Brace Overall Depth	
Seat Depth	Cross Brace Measurement
14"-20"	Approx. 0.375" less than seat depth

Item Number	Part Number	Description	Quantity
12a, 13 - 16	115770	Upper Pivot Assembly, 14" Wide	1
12b, 13-16	117402	Upper Pivot Assembly, 15" Wide	1
12c, 13-16	115771	Upper Pivot Assembly, 16" Wide	1
12d, 13-16	117403	Upper Pivot Assembly, 17" Wide	1

Item Number	Part Number	Description	Quantity
12e, 13-16	115772	Upper Pivot Assembly, 18" Wide	1
12f, 13-16	117404	Upper Pivot Assembly, 19" Wide	1
12g, 13-16	115773	Upper Pivot Assembly, 20" Wide	1
12h, 13-16	115774	Upper Pivot Assembly, 21" / 22" Wide	1
6, 8, 9	115775	Lower Pivot Assembly	1
1 - 2, 5 - 7, 10 - 11	115776	Cross Brace Hardware Kit	1
3a, 17	115791	Cross Brace, Right, 14x16	1
4a, 17	115777	Cross Brace, Left, 14x16	1
3b, 17	115792	Cross Brace, Right, 14x18	1
4b, 17	115778	Cross Brace, Left, 14x18	1
3c, 17	117405	Cross Brace, Right, 15x16	1
4c, 17	117406	Cross Brace, Left, 15x16	1
3d, 17	117407	Cross Brace, Right, 15x18	1
4d, 17	117408	Cross Brace, Left, 15x18	1
3e, 17	117409	Cross Brace, Right, 15x20	1
4e, 17	117410	Cross Brace, Left, 15x20	1
3f, 17	115793	Cross Brace, Right, 16x16	1
4f, 17	115779	Cross Brace, Left, 16x16	1
3g, 17	115794	Cross Brace, Right, 16x18	1
4g, 17	115780	Cross Brace, Left, 16x18	1
3h, 17	115795	Cross Brace, Right, 16x20	1
4h, 17	115781	Cross Brace, Left, 16x20	1
3i, 17	117413	Cross Brace, Right, 17x16	1
4i, 17	117414	Cross Brace, Left, 17x16	1
3j, 17	117415	Cross Brace, Right, 17x18	1
4j, 17	117416	Cross Brace, Left, 17x18	1
3k, 17	117417	Cross Brace, Right, 17x20	1
4k, 17	117418	Cross Brace, Left, 17x20	1

Item Number	Part Number	Description	Quantity
3l, 17	115796	Cross Brace, Right, 18x16	1
4l, 17	115782	Cross Brace, Left, 18x16	1
3m, 17	115797	Cross Brace, Right, 18x18	1
4m, 17	115783	Cross Brace, Left, 18x18	1
3n, 17	115798	Cross Brace, Right, 18x20	1
4n, 17	115784	Cross Brace, Left, 18x20	1
3o, 17	117419	Cross Brace, Right, 19x16	1
4o, 17	117420	Cross Brace, Left, 19x16	1
3p, 17	117421	Cross Brace, Right, 19x18	1
4p, 17	117422	Cross Brace, Left, 19x18	1
3q, 17	117423	Cross Brace, Right, 19x20	1
4q, 17	117424	Cross Brace, Left, 19x20	1
3r, 17	115799	Cross Brace, Right, 20x16	1
4r, 17	115785	Cross Brace, Left, 20x16	1
3s, 17	115800	Cross Brace, Right, 20x18	1
4s, 17	115786	Cross Brace, Left, 20x18	1
3t, 17	115801	Cross Brace, Right, 20x20	1
4t, 17	115787	Cross Brace, Left, 20x20	1
3u, 17	117425	Cross Brace, Right, 21x16	1
4u, 17	117426	Cross Brace, Left, 21x16	1
3v, 17	117427	Cross Brace, Right, 21x18	1
4v, 17	117428	Cross Brace, Left, 21x18	1
3w, 17	117429	Cross Brace, Right, 21x20	1
4w, 17	117430	Cross Brace, Left, 21x20	1
3x, 17	115802	Cross Brace, Right, 22x16	1
4x, 17	115788	Cross Brace, Left, 22x16	1
3y, 17	115803	Cross Brace, Right, 22x18	1
4y, 17	115789	Cross Brace, Left, 22x18	1

Item Number	Part Number	Description	Quantity
3z, 17	115804	Cross Brace, Right, 22x20	1
4z, 17	115790	Cross Brace, Left, 22x20	1
1	100678	M6 x 110 CLS8.8 DIN931 HHCS ZC	1
2	002807	Cross Tube Saddle CAT	6
5	100746	M6 Flat Washer Blk Zn	1
7	100660	M6 x 80mm CLS8.8 DIN931 HHCS ZC	2
10	100880	1/4" Plastic Spacer	2
11	105019	3/16" Plastic Spacer	2