

parts manual

parts manual

QUICKIE®

 Shark

JUNE 2016

Sunrise Medical appreciates your business. For your ordering convenience, our Customer Service Representatives can be contacted at:

**To place orders:
(800)333-4000**

Quickie Technical Assistance:

You can reach our Technical Support Team by choosing prompt 1 followed by the # sign, then prompt 2 for Technical Support. Choose option 1 for Quickie Technical Support.

**For Assistance with a Web Order,
or to sign up for sunrisemedical.com
(800) 333-4000 Option 5**

**FOR YOUR CONVENIENCE YOU MAY PLACE ORDERS THROUGH
OUR WEBSITE OR BY FAX 24 HOURS A DAY:**

www.sunrisemedical.com


Fax: (800)300-7502

For easy reference please fill in your Customer Number and Sunrise Account Manager's name.


Customer Account Number _____
Ship to Account Number _____
Sunrise Account Manager _____

*Part prices and configurations or specifications of products included in this Parts Manual are subject to change without prior notice.


QUICKIE SHARK / SHARK-S	4
BRAKE	4
SHARK - BRAKE.....	4
FOOTREST	4
SHARK - FOOTREST	5
STEERING FORK	5
SHARK - STEERING FORK HIGH.....	6
SHARK - STEERING FORK LOW	7
WHEEL.....	9
SHARK - DRIVE WHEEL 26" WITH EXTERNAL CASSETTE HUB	9
SHARK - DRIVE WHEEL 26" WITH INTERNAL GEARING HUB WITH REVERSING DRUM BRAKE	10
SHARK - REAR WHEEL	11
FRAME.....	12
SHARK / SHARK S - FRAME SHORT/LONG.....	13
SHARK / SHARK S - FRAME WITH REMOVABLE FORK	14
SHARK / SHARK S - STAND WITH X-LINK HINGE.....	16
SHARK / SHARK S - STAND FOR REMOVABLE FORK	17
BACKREST	17
SHARK - BACKREST HIGH.....	18
SHARK - BACKREST LOW	20
SHARK - BACKTUBE.....	21
GEARING SYSTEM / SPINDAL.....	21
SHARK - SPINDAL HOUSING	22
SHARK - STANDARD CRANK.....	25
SHARK - V-CRANK.....	26
SHARK - SPECIAL V - CRANK.....	27
SHARK - CRANK SHAFT / CHAIN RING	28
SHARK - HANDLES	29
SHARK - HUB SHIFTER.....	31
SHARK - GEARING SYSTEM.....	32
SHARK - GEARING SYSTEM AND BRAKE.....	34
SEAT	35
SHARK - SEAT UPHOLSTERY	36
ACCESSORIES	36
SHARK - FRONT FENDER & LIGHT	37
SHARK - REAR FENDER & LIGHT	38
TIRE PUMP	39
Duplicate of Title.....	39

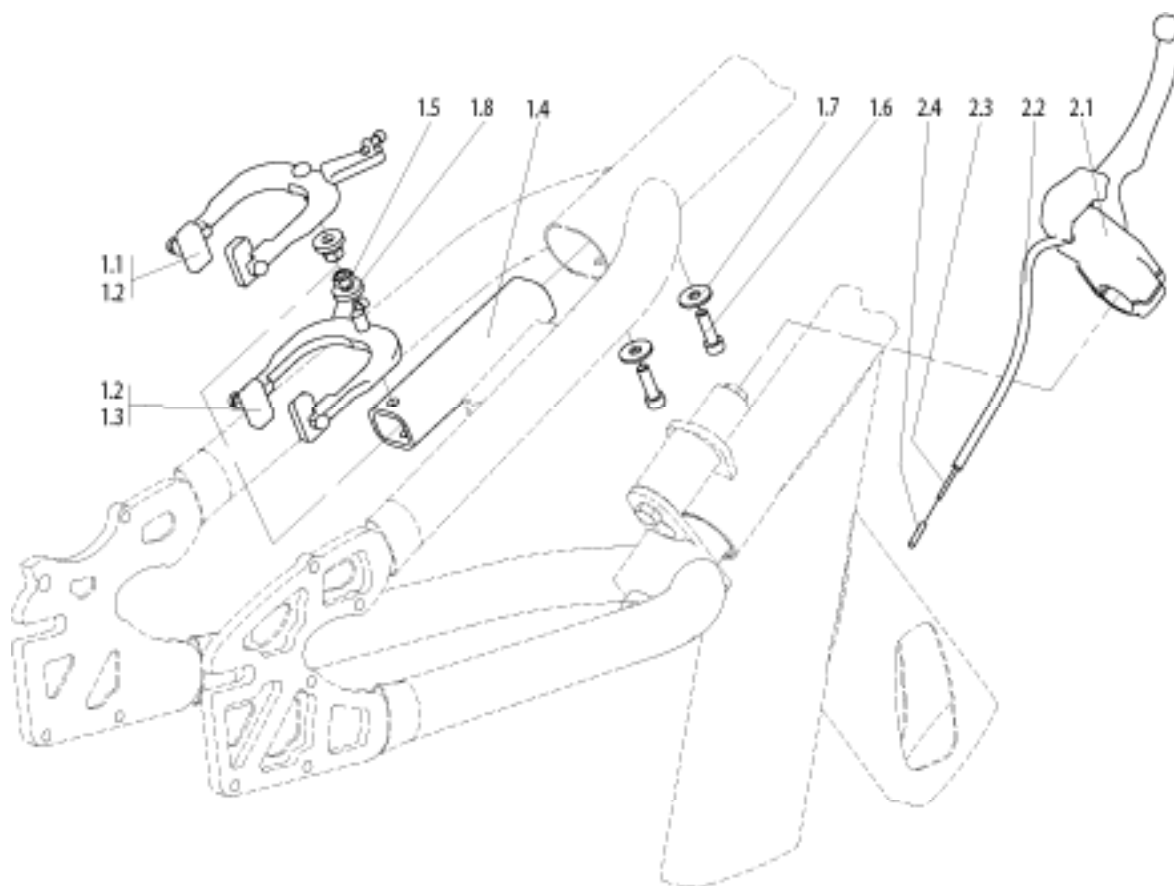


Pos.	Item Number	Description	Remarks
1.1	000 228 014	Caliper brake Racing	
1.2	000 228 019	Caliper brake Touring	
1.3	000 228 054	Caliper brake f. 7-Speed	
1.4	471 812 130	Teleskoprohr für Bremse	
1.5	000 128 007-ET	Nut M 6 DIN 985	25 Pcs.
1.6	000 102 029-ET	Nut M 6 x 20 DIN 912	20 Pcs.
1.7	000 130 116-ET	Washer A 6,4 DIN 9021	25 Pcs.
1.8	000 130 121-ET	Washer A 6,4 DIN 7349	25 Pcs.
2.1	000 228 022	Brake lever	
2.2	000 228 077	Housing brake 340 mm, Fork high / Fork low	
2.3	000 228 076-ET	Brake cable	
2.4	000 228 079-ET	Brake cable end cap	10 Pcs.

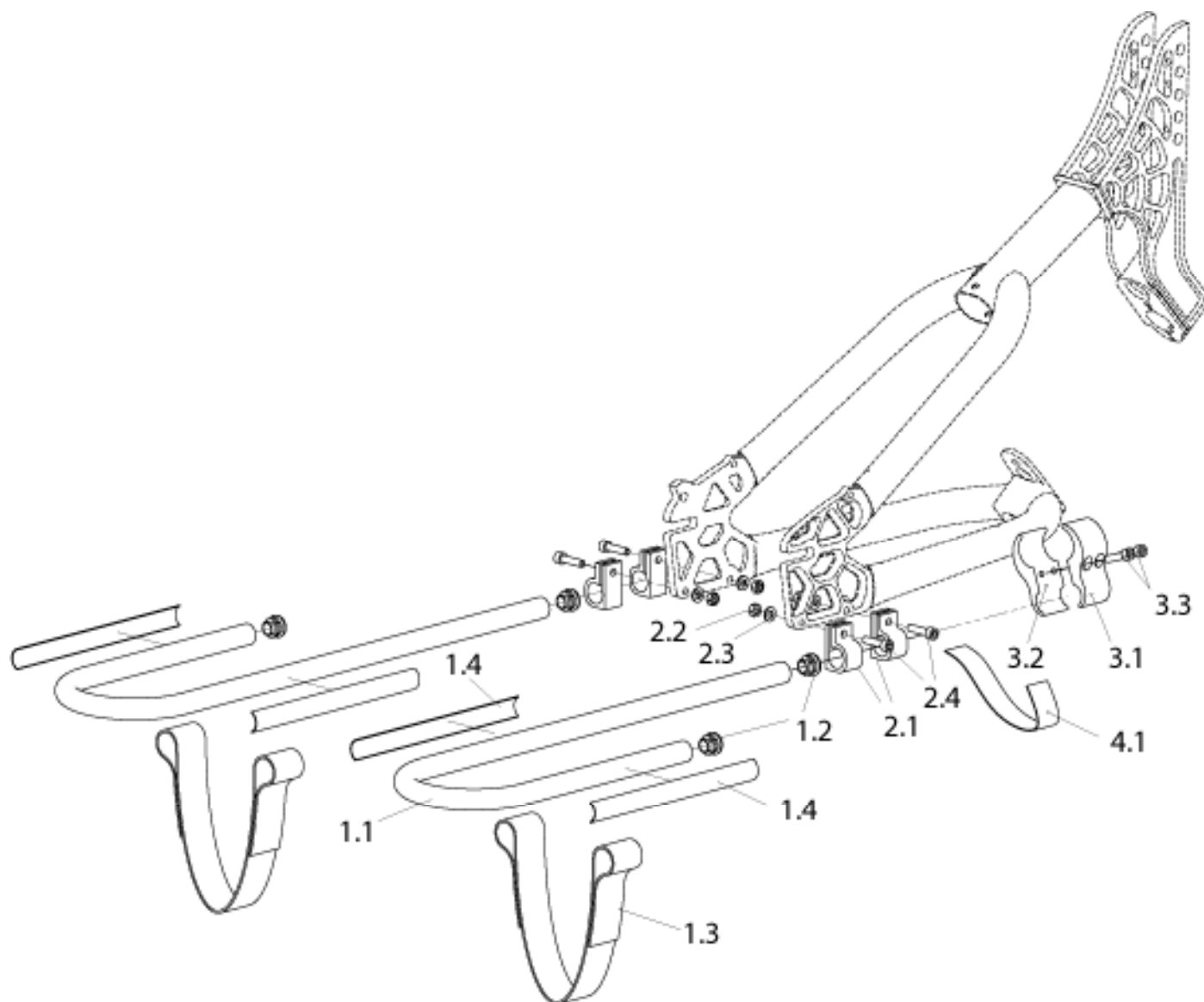


Pos.	Item Number	Description	Remarks
1.1-1.4,2.1-2.4	470 821 201	Foot rest for long legs cpl.	
1.1-1.4,3.1-3.3	470 821 202	Foot rest for short legs cpl.	
1.1	470 821 200	Foot rest	
1.2	000 282 042-ET	Louvered cap 18,7 mm	10 Pcs.
1.3	470 821 003	Velcro Foot strap	
1.4	615 840 012	Soft strip 200 mm	
2.1	471 821 001	Foot rest clamp	
2.2	000 128 007-ET	Hex nut M 6 DIN 985	25 Pcs.
2.3	000 130 109-ET	Washer A 6,4 DIN 125	25 Pcs.
2.4	000 102 006-ET	Soc. Screw M 6 x 25 DIN 912	25 Pcs.
3.1	615 821 002	Clamp inside	
3.2	615 821 003	Clamp outside	
3.3	000 102 006-ET	Soc. Screw M 6 x 25 DIN 912	25 Pcs.
4.1	615 840 008.01	Strap to fix legs to fork	

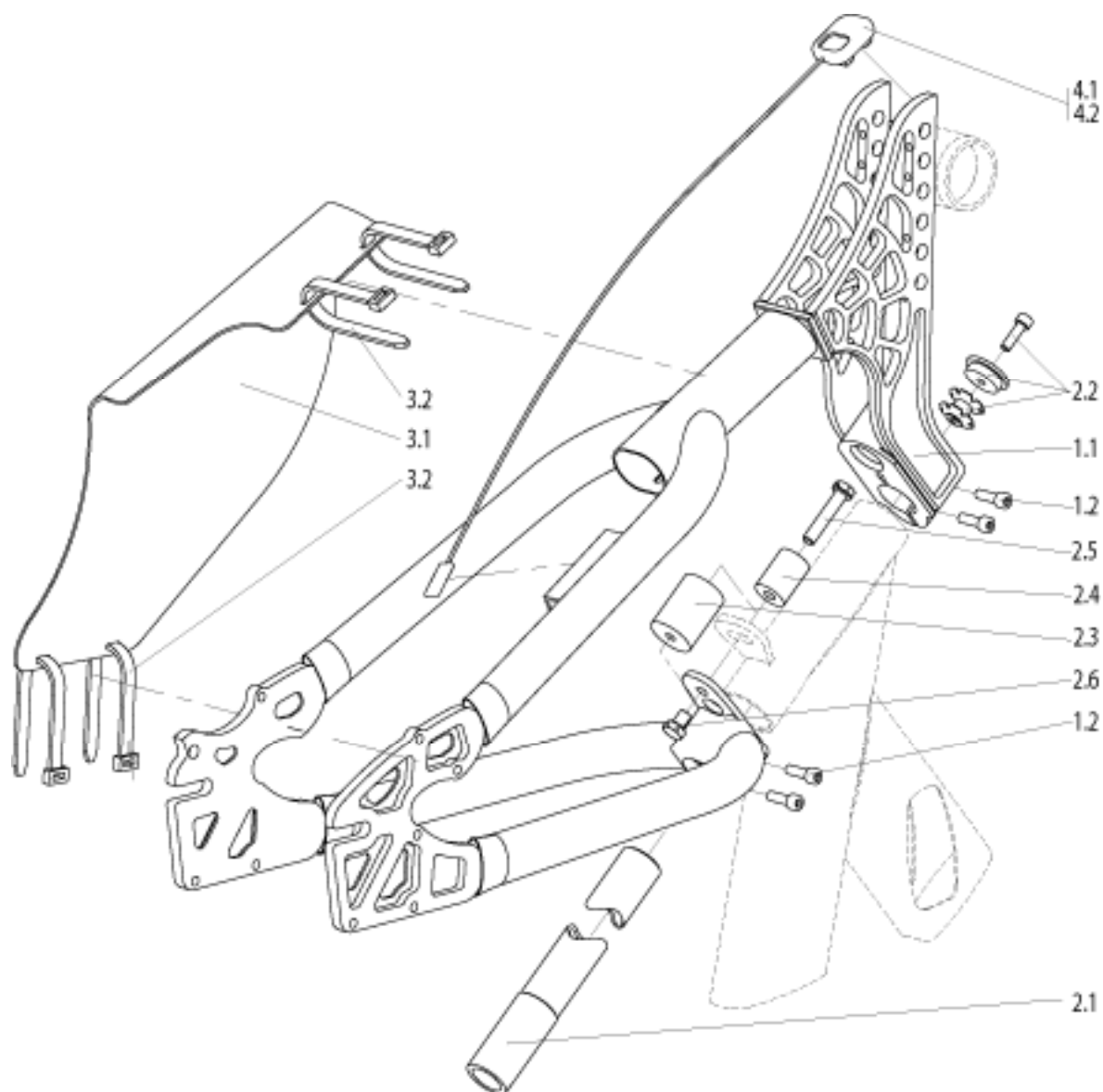


Pos.	Item Number	Description	Remarks
1.1-1.2	471 812 095S	Steering fork high	
1.2	000 102 029-ET	Soc. screw M 6 x 20 DIN 912	25 Pcs.
2.1-2.6	471 812 096	Steering tube	
2.2	000 228 300S	Headset	
2.3	000 282 111	Damper GMT-Puffer, Typ C	
2.4	471 861 002	Clamp tube	
2.5	000 100 036-ET	Hex. screw M 8 x 40 DIN 933	25 Pcs.
2.6	000 100 005-ET	Hex. screw M 8 x 16 DIN 933	25 Pcs.
3.1-3.2	471 812 097	Chain guard high cpl.	
3.2	000 280 010-ET	Zip tie 178 x 4,8 mm	10 Pcs.
4.1	471 812 014	Sport computer cpl.	
4.1	000 228 004	Sport computer	
4.2	471 812 015	Speedo meter & O-Ring cpl.	
4.2	000 228 006	Speedo meter	
o.B.	000 282 034	O-Ring 70 x 2	

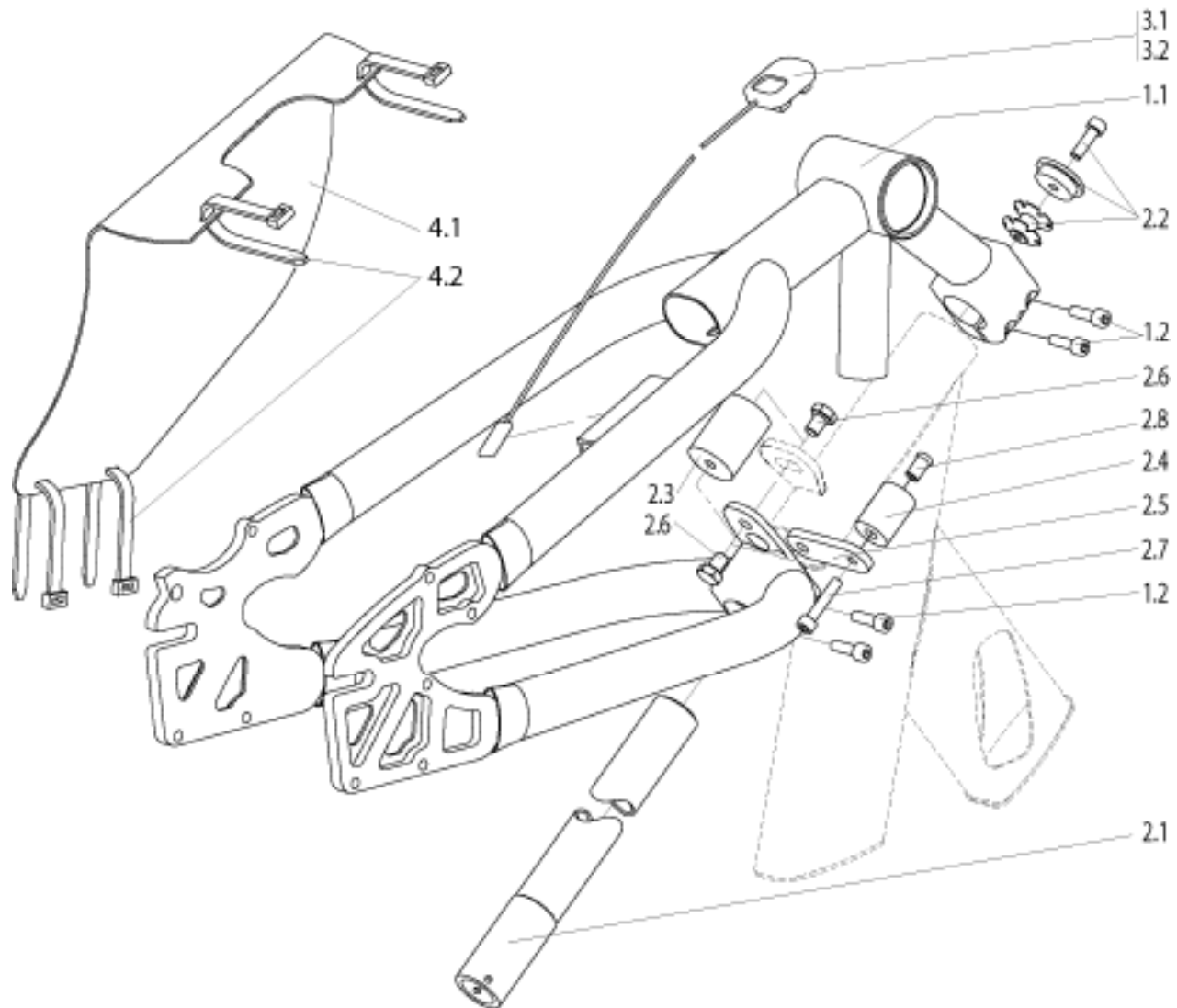


Pos.	Item Number	Description	Remarks
1.1-1.2	471 812 094S	Steering fork low	
1.2	000 102 029-ET	Soc. screw M 6 x 20 DIN 912	25 Pcs.
2.1-2.8	471 812 093	Steering tube	
2.2	000 228 300S	Head set	
2.3	000 282 111	Damper GMT-Puffer, Typ C	
2.4	471 861 002	Clamp tube	
2.5	471 861 003	Brake bracket	
2.6	000 100 005-ET	Hex. screw M 8 x 16 DIN 933	25 Pcs.
2.7	000 102 012-ET	Hex. screw M 8 x 30 DIN 933	25 Pcs.
2.8	840 826 007-ET	Socket nut	10 Pcs.
3.1	471 812 014	Sport computer cpl.	
3.1	000 228 004	Sport computer	
3.2	471 812 015	Speedo meter cpl.	
3.2	000 228 006	Speedo meter	
4.1-4.2	471 812 098	Kettenschutz tief kpl.	
4.2	000 280 010-ET	Kabelflachbinder 178 x 4,8 mm	10 Pcs.
o.B.	000 282 034	O-Ring 70 x 2	

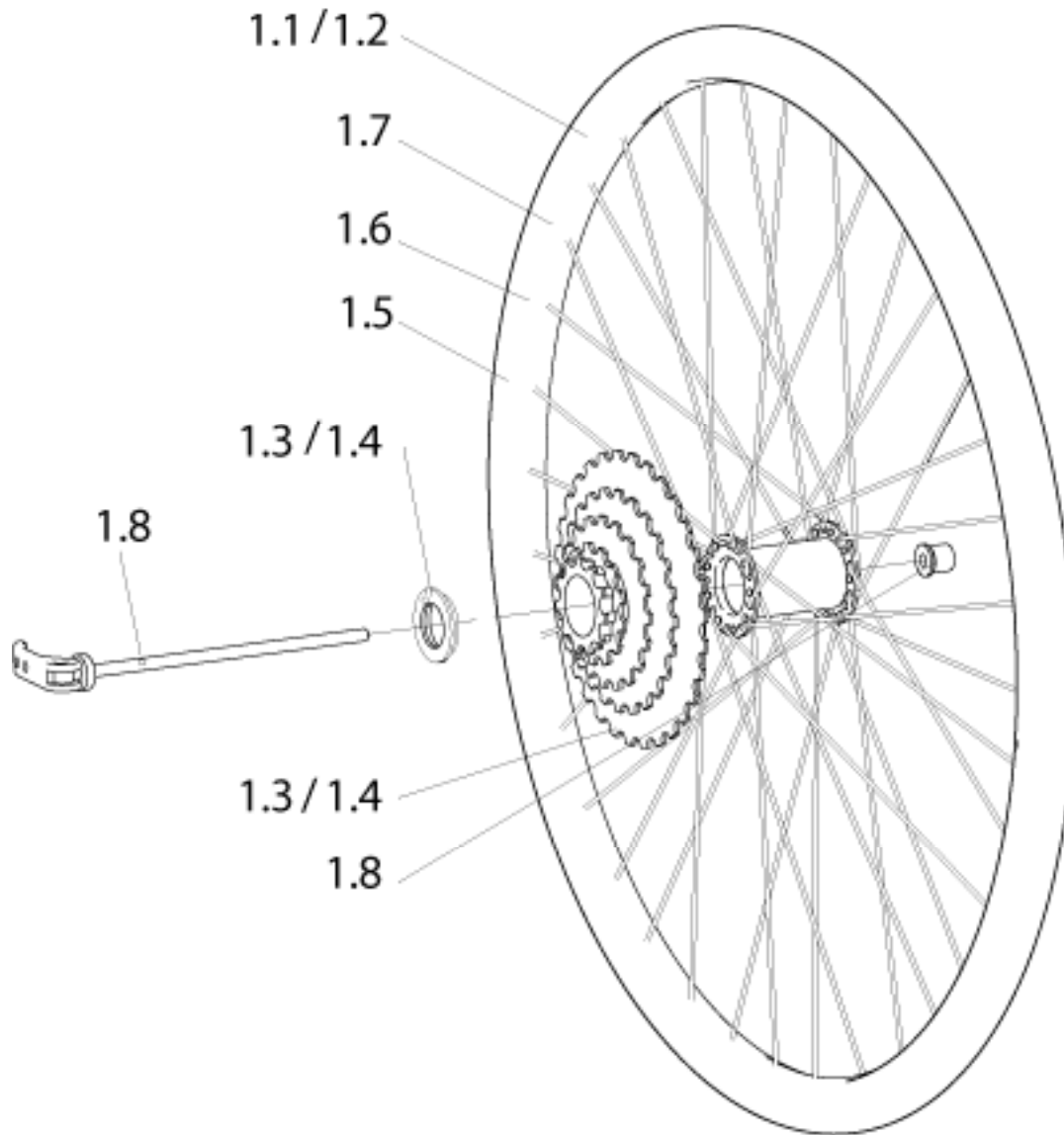

Pos.	Item Number	Description	Remarks
1.1-1.2,1.5	000 850 333	DRIVE WHL 26" HUB DT240S	
1.1-1.2,1.5	000 850 331	DRV WHL 26" HUB SHIMANO DEORE	
1.1-1.2,1.5-1.7	471 851 001	Drive wheel 26"; Hub DT240S DRNO	complete assembly with tire and tube
1.3	000 228 100	Shark-S: Cassette 9-speed, SRAM PG990, 11-34 teeth	
1.4	000 228 061	Shark: Cassette 9-speed, SRAM PG950, 11-34 teeth	
1.5	000 283 154	Rim band 26"	
1.6	000 283 108	Air tube 26" , 20-571	
1.7	000 283 043	Tyre 26", 25-559	
1.8	000 233 006	Wheel spanner	

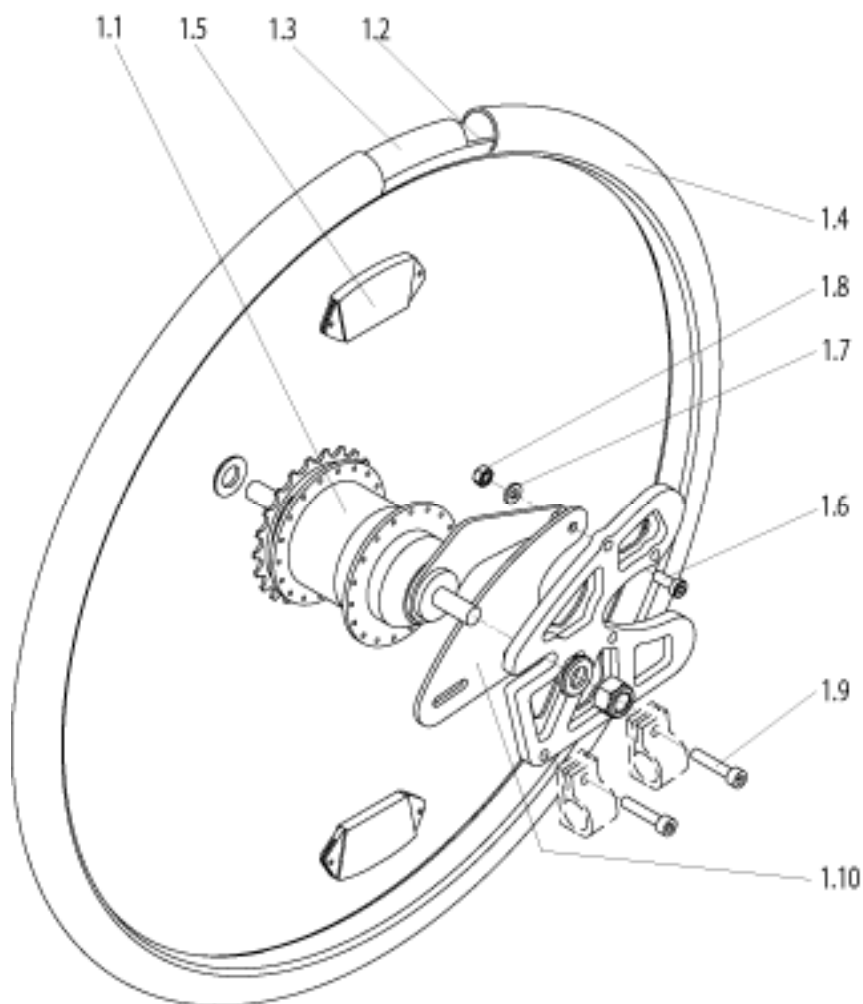


Pos.	Item Number	Description	Remarks
1.1	000 850 338	Drive wheel 26" (559) 7-Speed Hub cpl.	
1.2	000 283 154	Rim band 26"	
1.3	000 283 108	Air tube 26", 20-571	
1.4	000 283 045	Tire 26" x 1.35	
1.4	000 283 043	Tire 26" x 1.10	
1.5	000 280 001-ET	Spoke reflector	10 Pcs.
1.6	000 102 011-ET	Button head M 6 x 20 Torx	25 Pcs.
1.7	000 130 107-ET	Plastic washer 15 x 6,4 x 4 PA	10 Pcs.
1.8	000 128 007-ET	Nut M 6 DIN 985	25 Pcs.
1.9	000 102 012-ET	Hose clamp 30 x 15 mm	25 Pcs.
1.10	471 853 014	Drehmomentstütze	
1.6-1.10	471 853 015	Drehmomentstütze kpl. für 7-Gangschaltung	

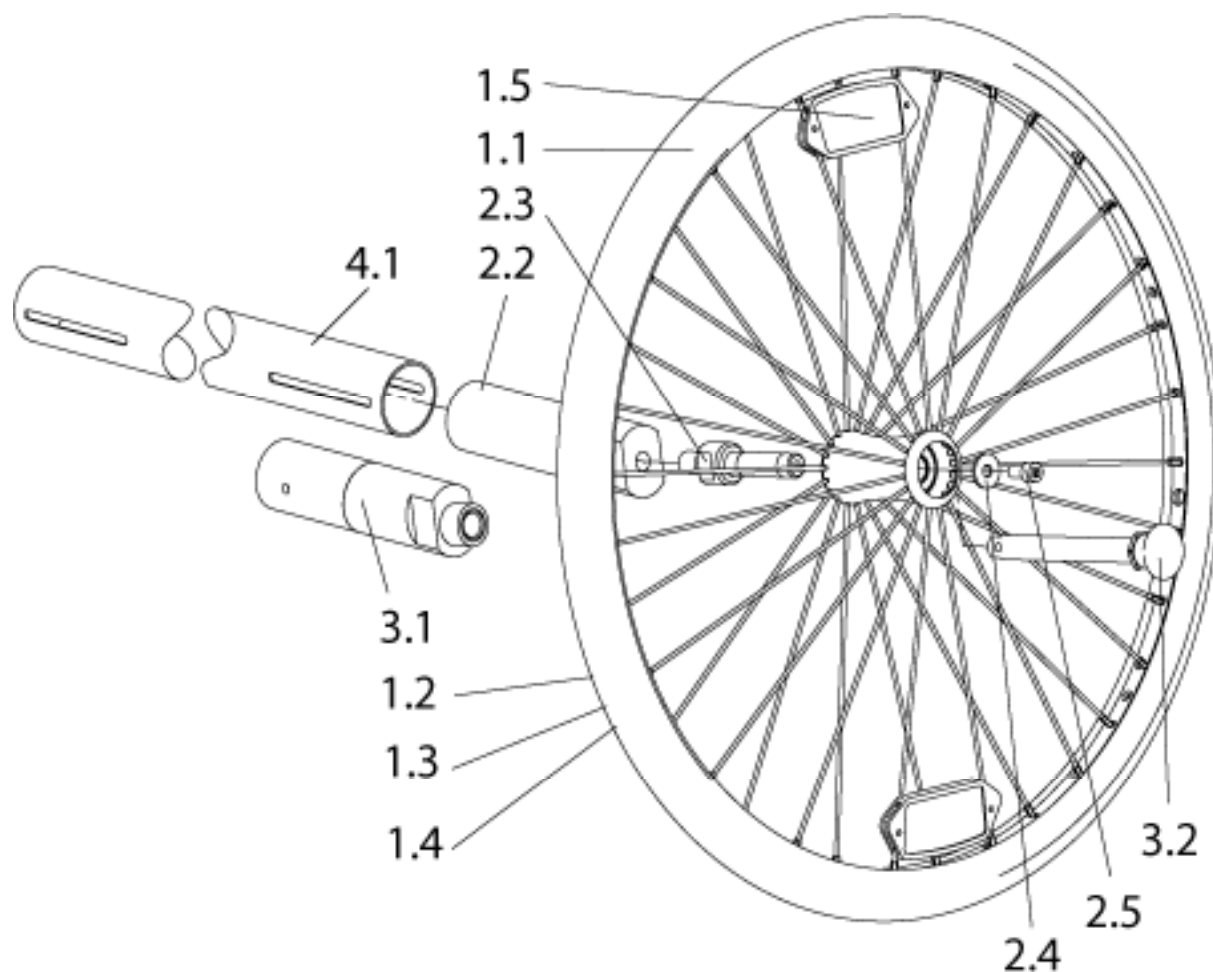


Pos.	Item Number	Description	Remarks
1.1-1.5	000 850 173	Rear wheel 20" w. design hub, tire 20" x 1 1/8"	
1.1-1.5	000 850 174	Rear wheel 20" w. design hub, tire 20" x 1 1.35"	
1.1-1.5	000 850 175	Rear wheel 26" w. design hub, tire 26" x 1 1/8"	
1.1-1.5	000 850 176	Rear wheel 26" w. design hub, tire 26" x 1 1.35"	
1.1	000 850 100	Rear wheel 20" w. design hub (w./o. tire)	
1.1	000 850 340	Rear wheel 26" w. design hub (w./o. tire)	
1.2	000 283 159	Rim band 20"	
1.2	000 283 154	Rim band 26"	
1.3	000 283 106	Air tube 20" 28/32-406/451	
1.3	000 283 108	Air tube 26", 20-571	
1.3	000 283 103	Air tube 26" x 1 3/8"	
1.4	000 283 042	Tire 20" x 1 1/8", 28-406	
1.4	000 283 046	Tire 20" x 1.35", 35-406	
1.4	000 283 043	Tire 26" x 1.10", 28-559	
1.4	000 283 045	Tire 26" x 1.35", 35-559	
1.5	000 280 001-ET	Spoke reflector	10 Pcs.
2.2	471 853 001	Axle sleeve 10° f. Stem	
2.3	471 853 002	Stem	
2.4	000 130 116-ET	Washer A 6,4 DIN 125	25 Pcs.
2.5	000 102 002-ET	Soc. screw M 6 x 12 DIN 912	25 Pcs.
3.1	615 853 001.03	Axle sleeve 10° f. QR axle	

Pos.	Item Number	Description	Remarks
3.2	000 220 299	QR axle f. Design wheel	
4.1	471 853 003	Axle tube	

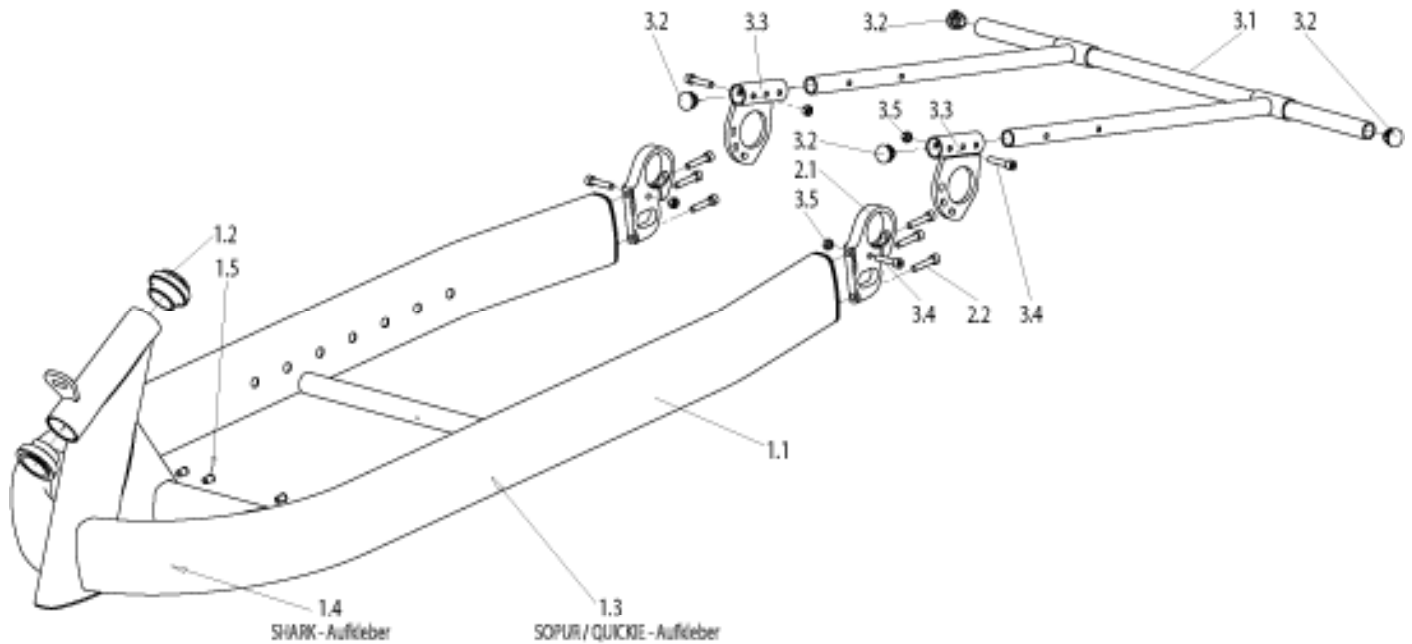


Pos.	Item Number	Description	Remarks
1.1-1.5	471 812 200S	Frame SHARK, short	
1.1-1.5	471 812 209S	Frame SHARK - S, short	
1.1-1.5	471 812 201S	Frame SHARK - S, long	
1.2	000 228 300S	Headset	2 Pcs.
1.3	000 280 200	SOPUR - Sticker, large, Silver	
1.3	000 280 201	SOPUR - Sticker, large, Black	
1.3	000 280 205	SOPUR - Sticker, large, Gold	
1.3	000 280 202	QUICKIE - Sticker, large, Silver	
1.3	000 280 203	QUICKIE - Sticker, large, Black	
1.3	000 280 206	QUICKIE - Sticker, large, Gold	
1.4	000 280 204	SHARK - Sticker, Silver	
1.4	000 280 211	SHARK - Sticker, Black	
1.4	000 280 212	SHARK - Sticker, Gold	
2.1	471 812 013	Axle bracket	
2.2	000 102 012-ET	Soc. screw M 6 x 30 DIN 912	25 Pcs.
3.1-3.5	471 812 301	Bumper 20" cpl.	
3.1-3.5	471 812 308	Bumper 26" cpl.	
3.1	471 812 302	Bumper	
3.2	000 282 038-ET	Louvered cap 20 mm	25 St./Pkg.
3.3	471 853 011	Bumper bracket	right
3.3	471 753 011	Bumper bracket	left
3.4	000 102 012-ET	Soc. screw M 6 x 30 DIN 912	25 Pcs.
3.5	000 128 007-ET	Hex nut M 6 DIN 985	25 Pcs.

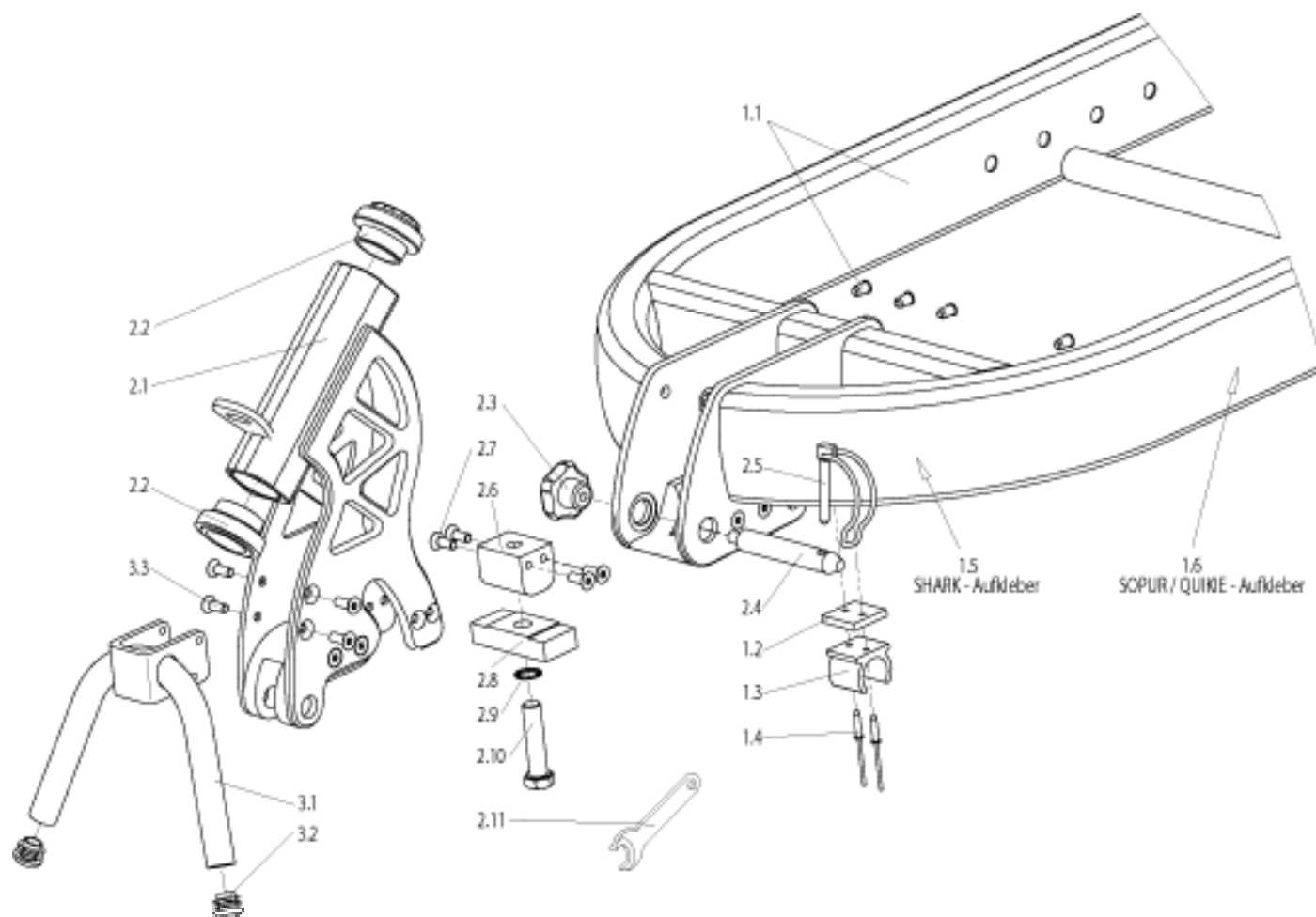


Pos.	Item Number	Description	Remarks
1.1-1.4	471 812 204S	Frame short, w. X-Link	
1.1-1.4	471 812 202S	Frame large, w. X-Link	
1.2	471 812 041	Spacer plate	
1.3	300 833 005	Clamp	
1.4	000 144 013-ET	Riv nut	50 Pcs.
1.5	000 280 204	SHARK - Sticker	
1.4	000 280 211	SHARK - Sticker, black	
1.4	000 280 212	SHARK - Sticker, gold	
1.6	000 280 200	SOPUR - Sticker, silver, large	
1.6	000 280 201	SOPUR - Sticker, black, large	
1.6	000 280 205	SOPUR - Sticker, gold, large	
1.6	000 280 202	QUICKIE - Sticker, silver, large	
1.6	000 280 203	QUICKIE - Sticker, black, large	
1.6	000 280 203	QUICKIE - Sticker, gold, large	
2.1	471 812 040	Steering head removable, SHARK	
2.1	471 812 048	Steering head removable, SHARK S	
2.10	000 120 202	Hex screw M 12 x 50 DIN 933	
2.11	000 604 183	Wrench SW 19 mm	
2.2	000 228 300S	Fork bearing set cpl.	
2.3	000 233 034	Star handle DIN 6336	Form K, M 8
2.4	471 812 044	Pin	


Pos.	Item Number	Description	Remarks
2.5	000 243 003	Safety clip	
2.6	471 812 066	Receiver	
2.7	000 104 029-ET	Screw M x 20 DIN 7991	25 Pcs.
2.8	471 812 067	Bridge	
2.9	000 138 010-ET	Safetywasher M 12	10 Pcs.
3.1	471 812 107	Stand for X-Link fork, black	
3.2	000 282 042-ET	Cap 18 mm	10 Pcs.
3.3	000 104 026-ET	Screw M 6 x 12 DIN 7991	25 Pcs.

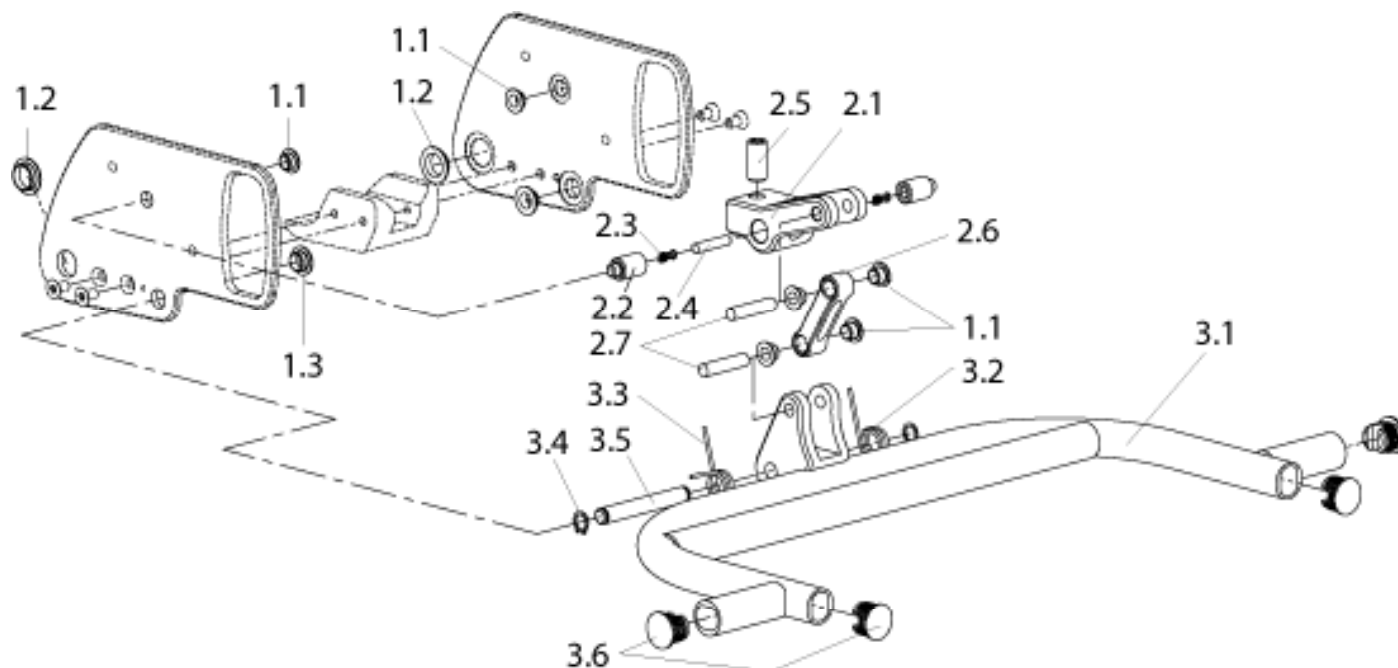

Pos.	Item Number	Description	Remarks
1.1	000 282 121	Bushing 8 x 10 x 5,5	
1.2	000 282 123	Bushing 10 x 12 x 7	
1.3	000 282 122	Bushing 15 x 17 x 9	
2.1	471 812 033	Hinge	
2.2	471 812 043	Spring housing	
2.3	000 242 011	Spring 0,7 x 4,3 x 12,1 x 5,5	
2.4	000 154 210	Steel pin 6h8 x 24 DIN 7	
2.5	000 233 022	Screw M 12 x 30 DIN 913	
2.6	471 812 034	Bone	
2.7	000 154 211	Steel pin 8h8 x 36 DIN 7	
3.1	471 812 045	Stand with hinge	
3.2	471 712 063	Spring left	
3.3	471 812 063	Spring right	
3.4	000 181 009	Safety ring 10 mm DIN 471	
3.5	471 812 042	Shaft	
3.6	760 827 007-ET	Cap 23 mm	10 Pcs.

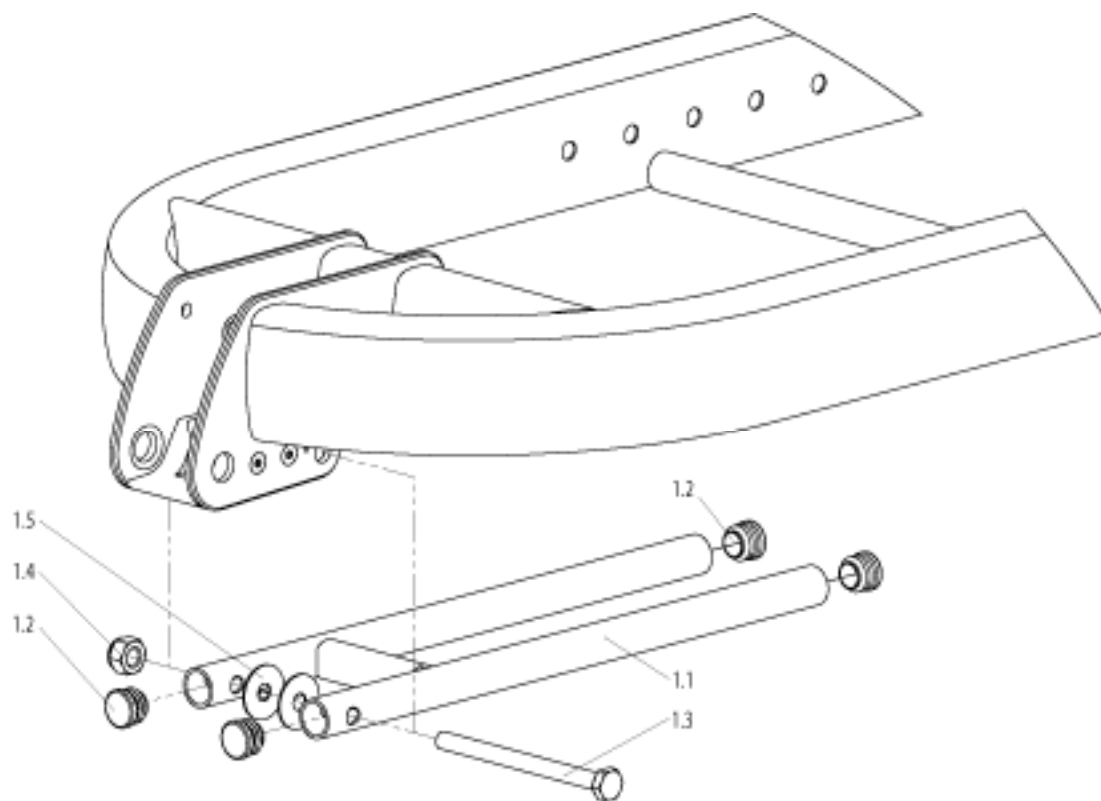


Pos.	Item Number	Description	Remarks
1.1-1.5	471 812 129	Stand for removable fork cpl.	
1.2	760 827 007-ET	Hex nut M 12 DIN 985	10 Pcs.
1.3	000 100 907	Hex screw M 10 x 130 DIN 933	6 Pcs.
1.4	000 128 016-ET	Hex nut M 10 DIN 985	10 Pcs.
1.5	000 130 154-ET	Washer 32 x 9,9 x 1,5 Plastic, black	25 Pcs.

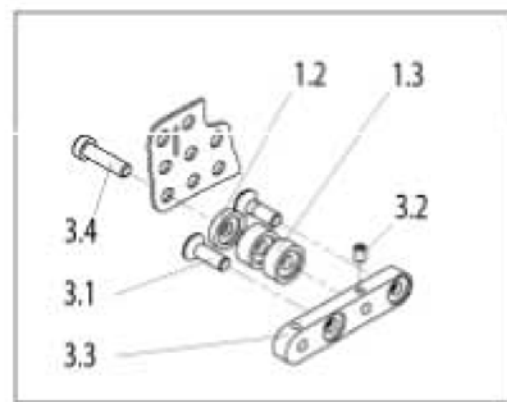


Pos.	Item Number	Description	Remarks
1.1-1.4,2.1-2.3	471 841 050	BACKREST HIGH, FIXED, CPL	
1.1-1.3,2.1-2.3,3.1-3.4	471 841 070	BACKREST HIGH, FOLD DOWN, CPL	
1.1	471 841 010S	BACKREST HIGH	
1.2	471 841 026-ET	SPACER 5 MM	4 Pcs.
1.3	471 841 016-ET	SPACER	12 Pcs.
1.4	000 102 172-ET	SOC. SREW M 8 X 35 DIN 912 F.	10 Pcs.
1.4	000 102 227-ET	SOC SC M8X50 DIN 912 F.W:40	6 Pcs.
1.4	000 102 173-ET	SOC. SREW M 8 X 55 DIN 912 F.	6 Pcs.
1.4	000 102 174-ET	SOC. SREW M 8 X 65 DIN 912 F.	6 Pcs.
1.4	000 102 176-ET	SOC. SREW M 8 X 75 DIN 912 F.	6 Pcs.
1.4	000 102 175-ET	SOC. SREW M 8 X 85 DIN 912 F.	6 Pcs.
2.1	471 841 005	BACK CUSHION STD	
2.2	471 843 001	BACK COVER	
2.3	471 843 002	SOFT STRIP 1515 MM	
3.1	000 104 116-ET	FHCS M8 X 25	8 Pcs.
3.2	000 102 231-ET	SET SC M 6 X 10 DIN 551	25 Pcs.
3.3	471 841 030	BACKREST BRACKET	
3.4	000 102 171-ET	SOC. SREW M 8 X 25 DIN 912 F.	10 Pcs.
3.4	000 102 172-ET	SOC. SREW M 8 X 35 DIN 912 F.	10 Pcs.
3.4	000 102 227-ET	SOC SC M8X50 DIN 912 F.W:40	6 Pcs.
3.4	000 102 173-ET	SOC. SREW M 8 X 55 DIN 912 F.	6 Pcs.
3.4	000 102 174-ET	SOC. SREW M 8 X 65 DIN 912 F.	6 Pcs.
3.4	000 102 176-ET	SOC. SREW M 8 X 75 DIN 912 F.	6 Pcs.


Pos.	Item Number	Description	Remarks
4.1-4.7	471 841 060S	NECK SUPPORT CPL.	
4.1	471 841 018	NECK SUPPORT	
4.2	471 841 031	NECK PAD	
4.3	471 843 008	NECK COVER	
4.4	000 102 140-ET	SOC SC M5X35 DIN 912 F.W:36	25 Pcs.
4.5	622 842 003-ET	WASHER	6 Pcs.
4.6	000 130 111-ET	WASHER A 5.3 DIN 125	25 Pcs.
4.7	000 128 007-ET	HEX NUT M6 DIN 985	25 Pcs.

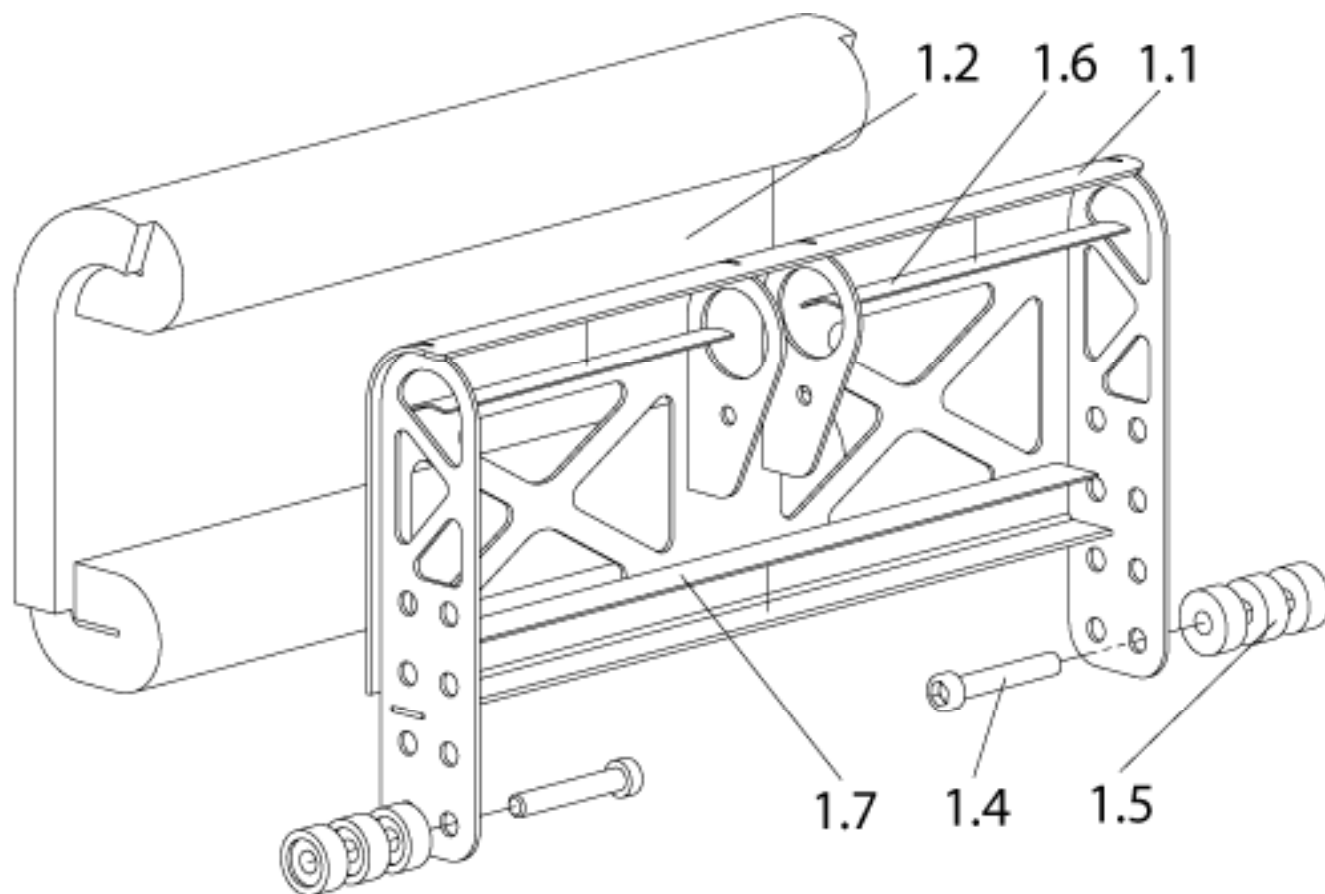


Pos.	Item Number	Description	Remarks
1.1	471 841 020S	Back rest low	
1.2	471 843 003	Back pad small	
w/o pic.	471 843 004	Back cover	
1.4	000 102 225-ET	Soc. screw M 8 x 30 DIN 912 f. W: 36	10 Pcs.
1.4	000 102 226-ET	Soc. screw M 8 x 40 DIN 912 f. W: 38	10 Pcs.
1.4	000 102 228-ET	Soc. screw M 8 x 50 DIN 912 f. W: 40	6 Pcs.
1.4	000 102 229-ET	Soc. screw M 8 x 60 DIN 912 f. W: 42	6 Pcs.
1.4	?	Soc. screw M 8 x 70 DIN 912 f. W: 44	6 Pcs.
1.5	471 841 016-ET	Spacer	10 Pcs.
1.7	471 843 005	Soft strip 995 mm	

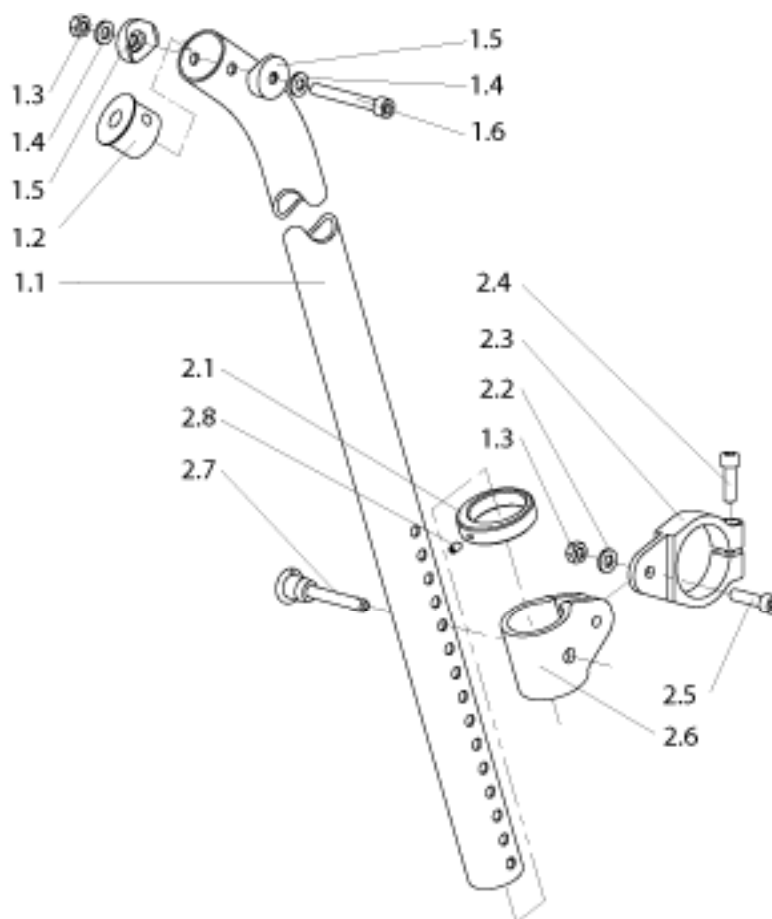


Pos.	Item Number	Description	Remarks
1.1	471 841 025.01	BACKTUBE	
1.2	471 841 017	Plug	25 Pcs.
1.3	000 128 007-ET	HEX NUT M6 DIN 985	25 Pcs.
1.4	000 130 116-ET	WASHER A 6.4 DIN 9021	25 Pcs.
1.5	000 831 232-ET	SADDLE	25 Pcs.
2.1	614 812 013	POSITIONING RING	10 Pcs.
2.2	000 130 107-ET	WASHER A 6.3 DIN 125	25 Pcs.
2.3	471 841 015	TUBE CLAMP	
2.4	000 102 029-ET	SOC SCREW M 6 X 20 DIN 912	25 Pcs.
2.5	000 102 006-ET	SOC SCREW M 6 X 25 DIN 912	25 Pcs.
2.6	614 812 007	CLAMP	
2.7	000 243 001	QR AXLE	
2.8	000 105 027-ET	SET SCREW M 4 X 6 DIN 914	25 Pcs.

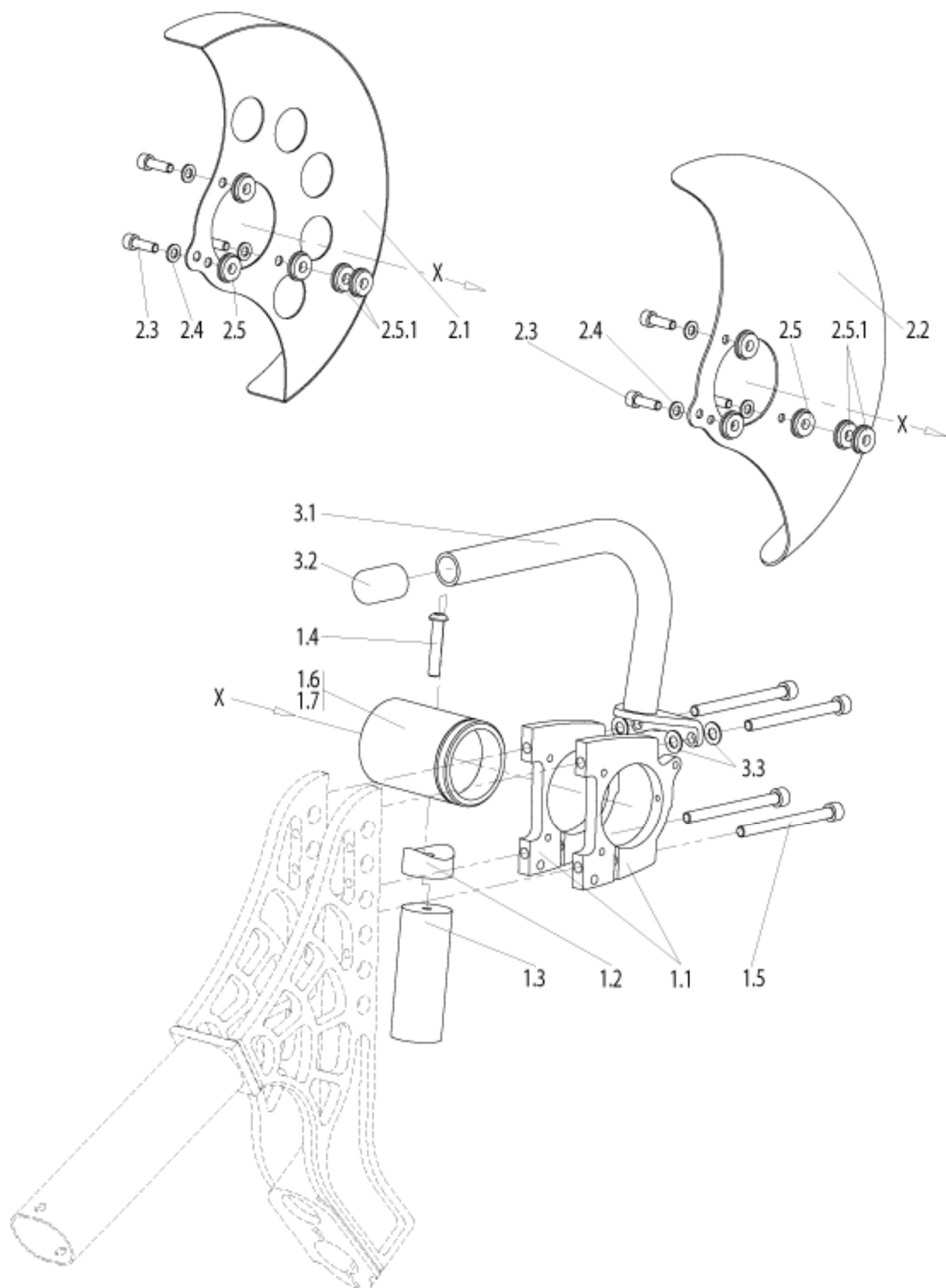


Pos.	Item Number	Description	Remarks
1.1	471 812 057	Spindal clamp large	
1.2	471 812 117	Saddle for derailleur tube	
1.3	471 812 118	Derailleur tube	
1.4	000 102 312.01	Screw M 6 x 32 Torx	25 Pcs.
1.5	000 102 013-ET	Soc. screw M 6 x 70 DIN 912	10 Pcs.
1.6	471 812 076.01	Bottom bracket threaded	
1.7	471 812 076.02	Bottom bracket for ball bearing	
2.1	471 812 087	Chain guard std.	
2.2	471 812 090	Chain guard carbon	
2.3	000 102 319-ET	Soc. screw M 5 x 16 DIN 912	25 Pcs.
2.4	000 130 111-ET	Washer A 5,3 DIN 125	25 Pcs.
2.5	000 138 001	Rubber washer	
2.5.1	000 138 001	Rubber washer, fork low only	
3.1-3.2	471 812 125	Handle cpl.	
o.B.	000 130 121-ET	Washer 6,4 DIN 7349 (for Handle)	25 Pcs.
3.2	000 282 041-ET	Cap	10 Pcs.

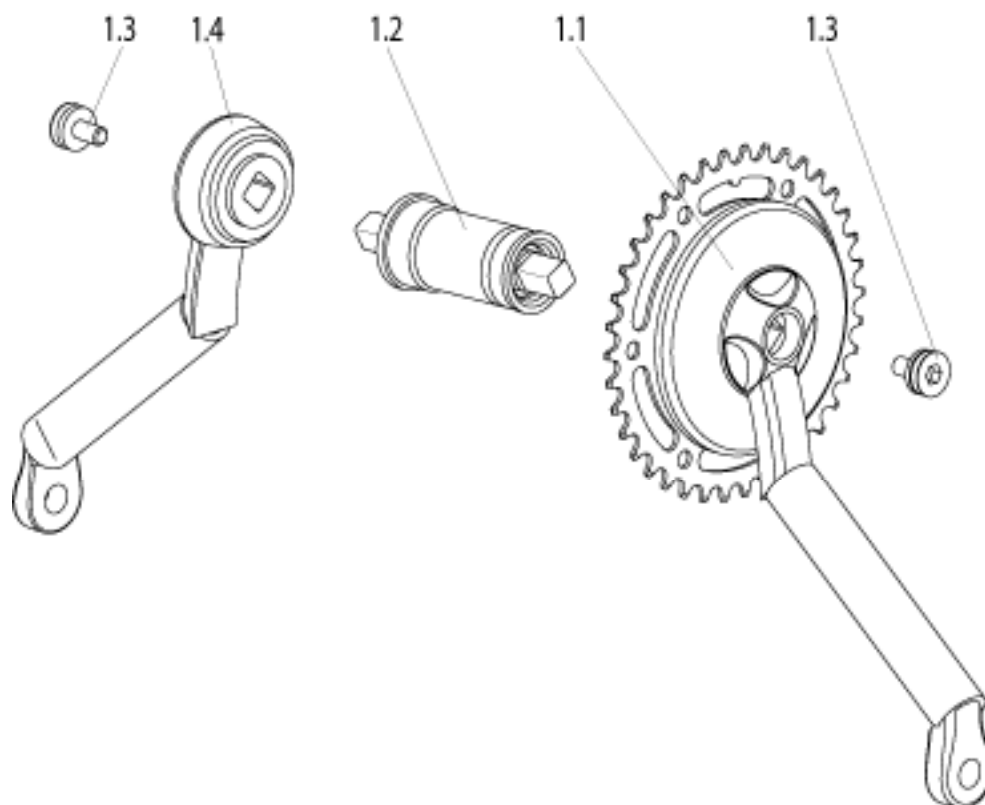


Pos.	Item Number	Description
1.1,1.3	000 228 102	Standard crank f. 7-Speed 175 mm
1.2	000 228 062	Spindal assy
1.3	na	Crank screw
1.1-1.4	000 228 103	STND CRANK F.3X9-SPD 175MM

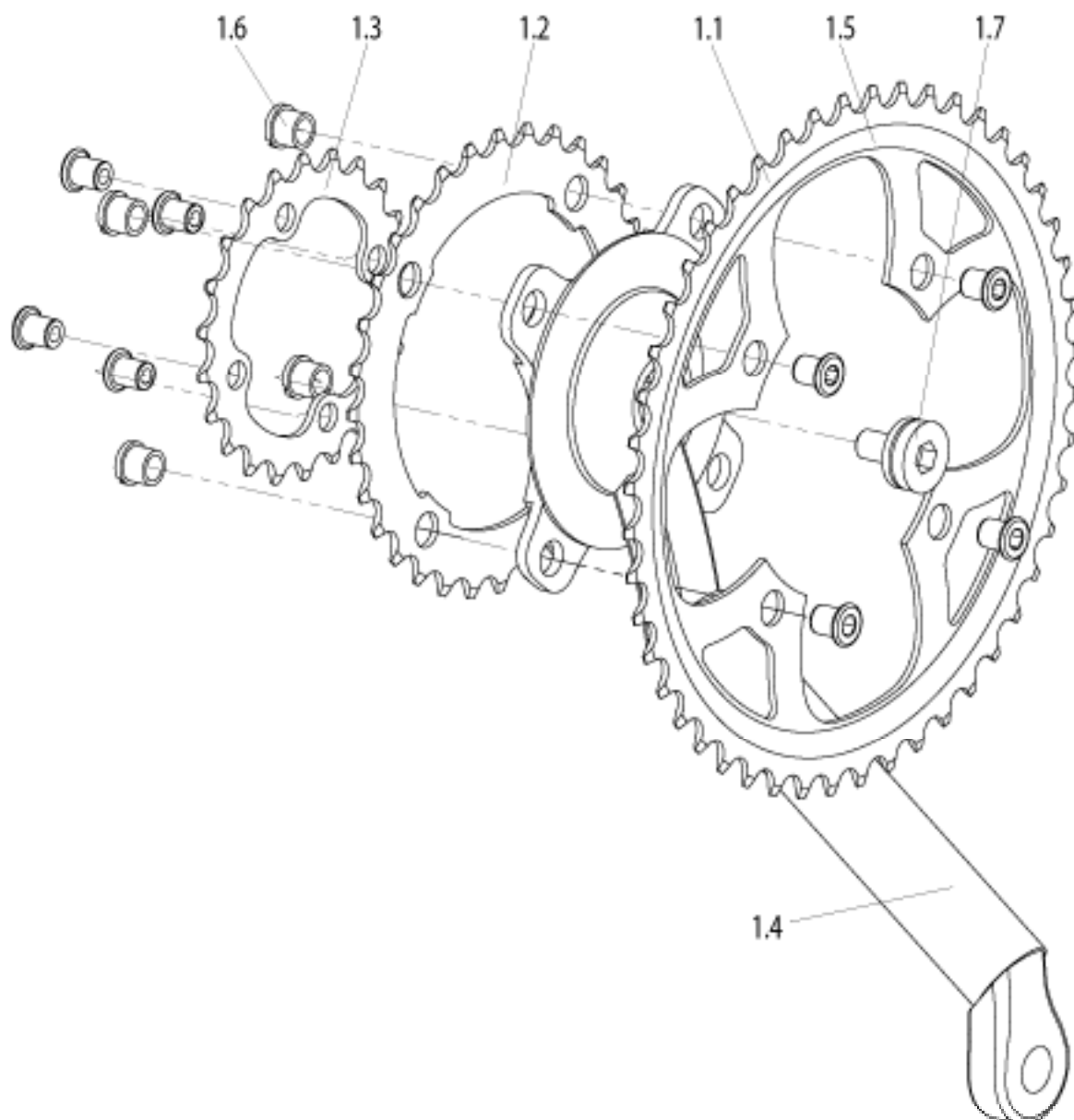


Pos.	Item Number	Description	Remarks
1.1-1.7	000 228 103	Standard crank f. 3 x 9-Speed 175 mm	
1.1	000 228 126	Chain ring 48 teeth	
1.2	000 228 124	Chain ring 36 teeth	
1.3	000 228 122	Chain ring 26 teeth	
1.4	000 228 103	Standard - V-Crank	Pair
1.5-1.6	000 228 128	Chain ring srew / nut - Kit	4 Pcs.
1.7	na	Crank - Screw	

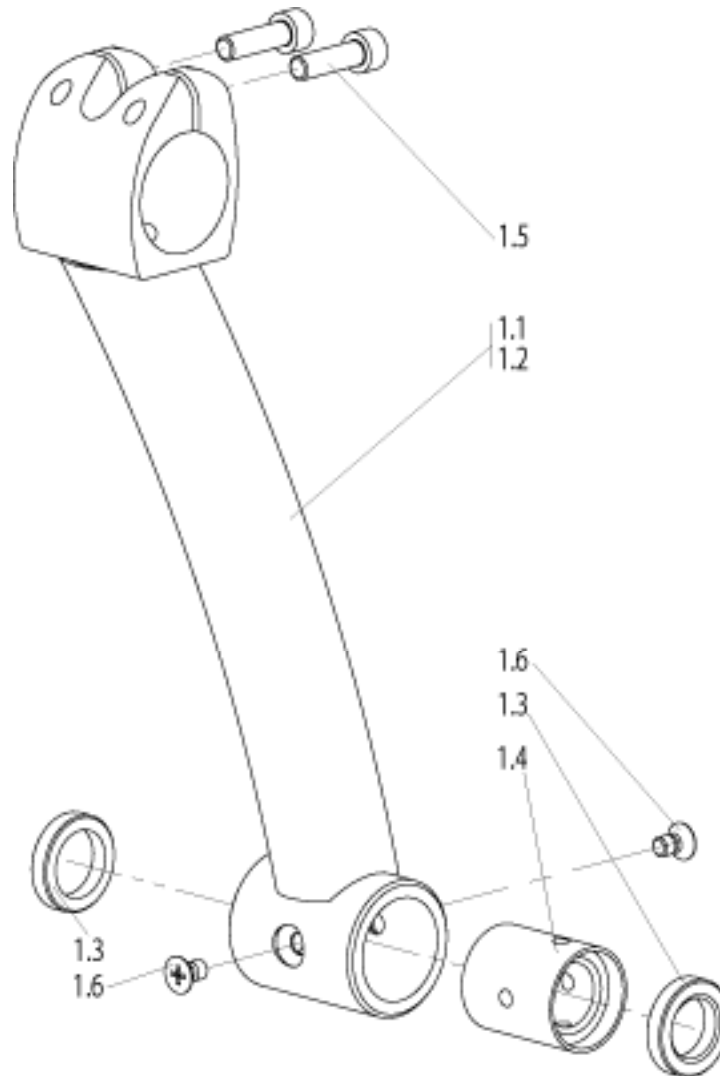


Pos.	Item Number	Description	Remarks
1.1,1.3-1.5	471 854 001	SPECIAL V-CRANK RIGHT	rechts
1.2-1.5	471 754 001	SPECIAL V-CRANK LEFT	links
1.3	000 210 016	BALL BEARING 24X15X5/61802 2RS	
1.5	000 121 029-ET	SOC SCREW M 6 X 20 DIN 912	25 Pcs.

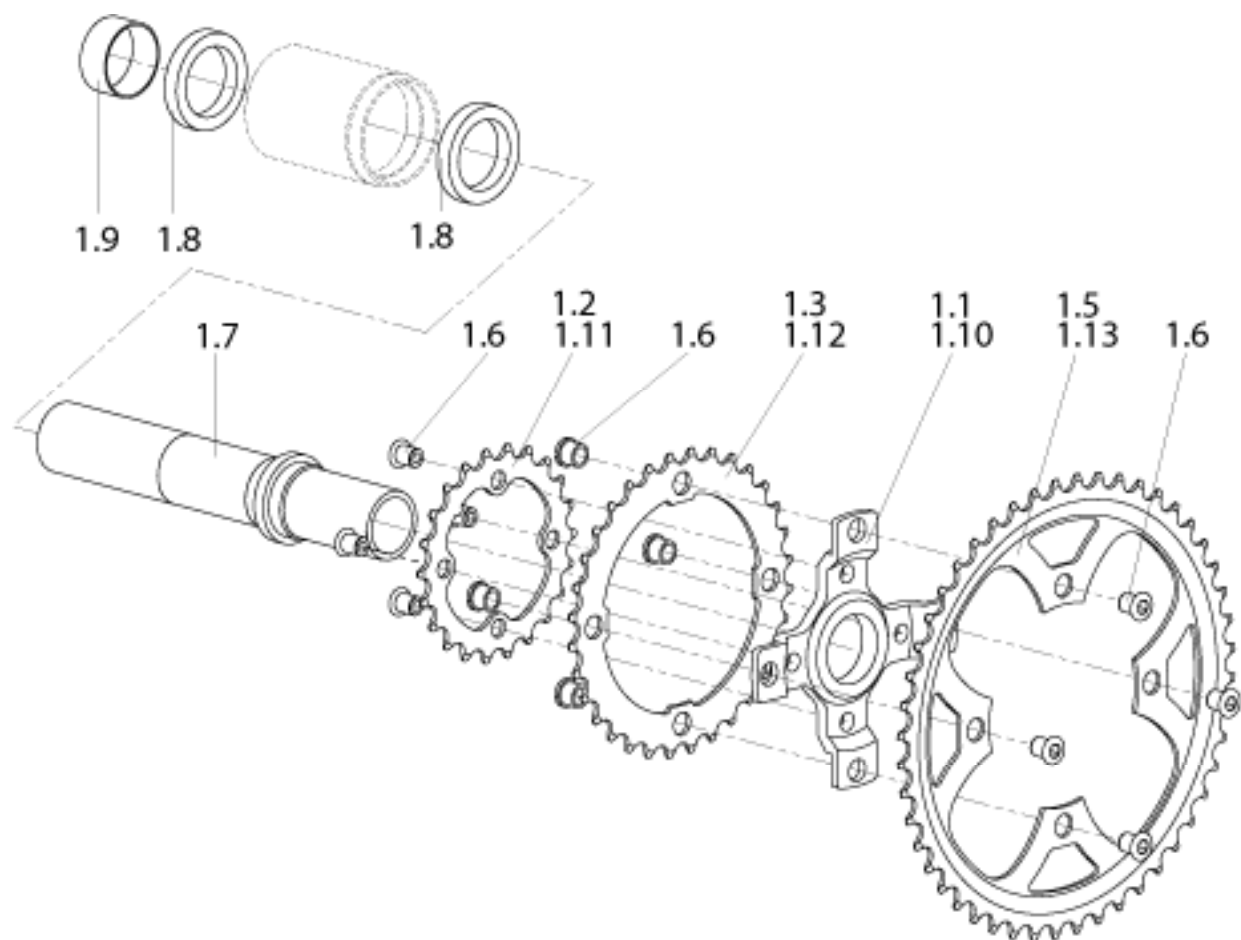

Pos.	Item Number	Description	Remarks
1.1	471 854 051	Chain rings bracket small (4 arms)	
1.2	000 228 122	Chain ring 26 teeth	
1.3	000 228 124	Chain ring 36 teeth	
1.5	000 228 126	Chain ring 48 teeth	
1.6	000 228 128	Chain ring srew / nut	4 Pcs.
1.7	471 854 040	Crank shaft	
1.8	000 210 015	Bearing 30 x 42 x 7 61806 2RS	
1.9	471 854 041	Spacer	
1.10	471 854 050	Chain rings bracket large (5 arms)	w/o pic.
1.11	000 228 123	Chain ring 30 teeth	
1.12	000 228 125	Chain ring 42 teeth	
1.13	000 228 127	Chain ring 52 teeth	

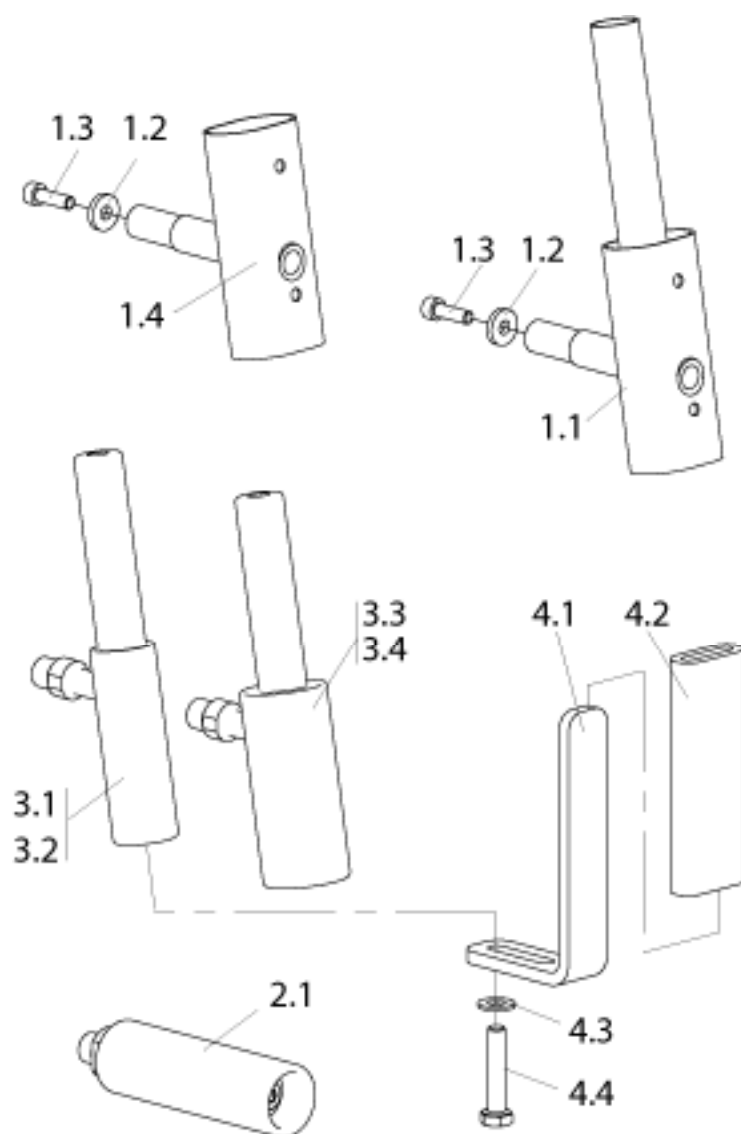


Pos.	Item Number	Description	Remarks
1.1-1.3	471 812 102	Vertical shifter handle cpl.	
1.2-1.4	471 812 103	Vertical handle cpl.	
1.1	471 854 006	Vertical shifter handle	
1.2	000 130 121-ET	Washer A 6,4 DIN 7349	25 Pcs.
1.3	000 102 029-ET	Soc. screw M 6 x 20 DIN 912	25 Pcs.
1.4	471 854 016	Vertical handle	
2.1	471 854 030	Standard handle	right
2.1	471 754 030	Standard handle	left
3.2	470 812 079	Ergo handle 30 mm, w. possibility of mounting quadlever	right
3.1	470 712 079	Ergo handle 30 mm, w. possibility of mounting quadlever	left
3.3	470 812 078	Ergo handle 40 mm, w/o possibility of mounting quadlever	right
3.4	470 712 078	Ergo handle 40 mm, w/o possibility of mounting quadlever	left
4.1-4.4	406 812 047	Quad lever cpl.	
4.1	406 812 055	Quad lever	
4.2	000 281 013	Soft handle	
4.3	000 130 108-ET	Washer A 8,4 DIN 125	25 Pcs.
4.4	000 100 029-ET	Hex. screw M 8 x 30 DIN 933	10 Pcs.
o-B.	000 228 068	MTB - Glocke	

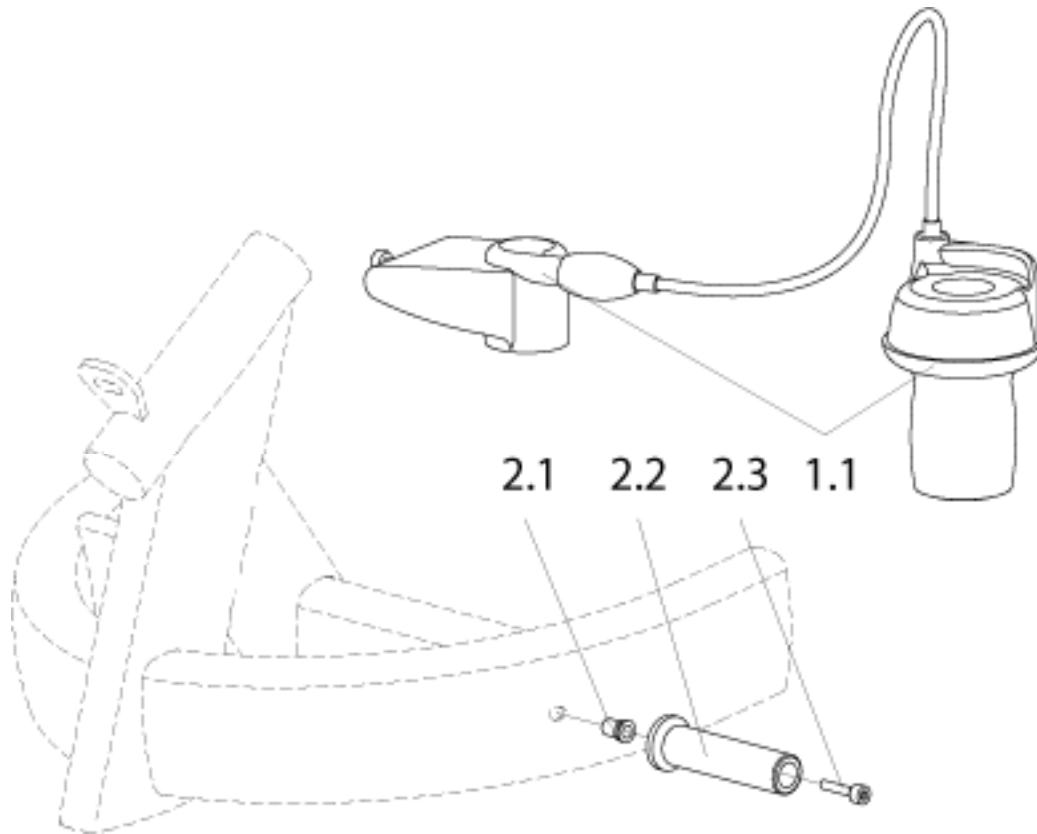


Pos.	Item Number	Description	Remarks
1.1	000 228 046	Shifter f. 7-Speed w. clickbox cpl.	
2.1	spare part n/a	Rivet nut FKO M 8 x 14 -300	
2.2	471 854 033	Shifter holder f. frame mounting	
2.3	000 102 137-ET	Soc. screw M 8 x 25 DIN 912	25 Pcs.

Pos.	Item Number	Description	Remarks
1.6	000 228 079-ET	Cable cap	10 Pcs.
1.7	471 858 008	Shifter cable housing for caster wheel fork high 760 mm	
1.7	471 858 009	Shifter cable housing for caster wheel fork low 650 mm	
1.8-1.12	471 858 016	Shifter mount at front fork	
1.8	615 821 003	Inside clamp	
1.9	615 821 002	Outside clamp	
1.10	000 102 137-ET	Soc. screw M 6 x 25 DIN 912	25 Pcs.
1.11	471 812 410	Shifter holder	
1.12	000 861 200	Adapter sleeve	
3.1	471 854 100	Drive chain for caster wheel fork high , 5-arm chain ring bracket 1795 mm	
3.1	471 854 101	Drive chain for caster wheel fork high , 4-arm chain ring bracket 1720 mm	
3.2	471 854 102	Drive chain for caster wheel fork high , 5-arm chain ring bracket 1545 mm	
3.2	471 854 103	Drive chain for caster wheel fork high , 4-arm chain ring bracket 1470 mm	
3.3	000 228 096-ET	Chain link	2 Pcs.
4.1	471 858 020	Rear derailleur SHIMANO for caster wheel fork high cpl.	
4.1	471 858 021	Rear derailleur SHIMANO LX for caster wheel fork low cpl.	

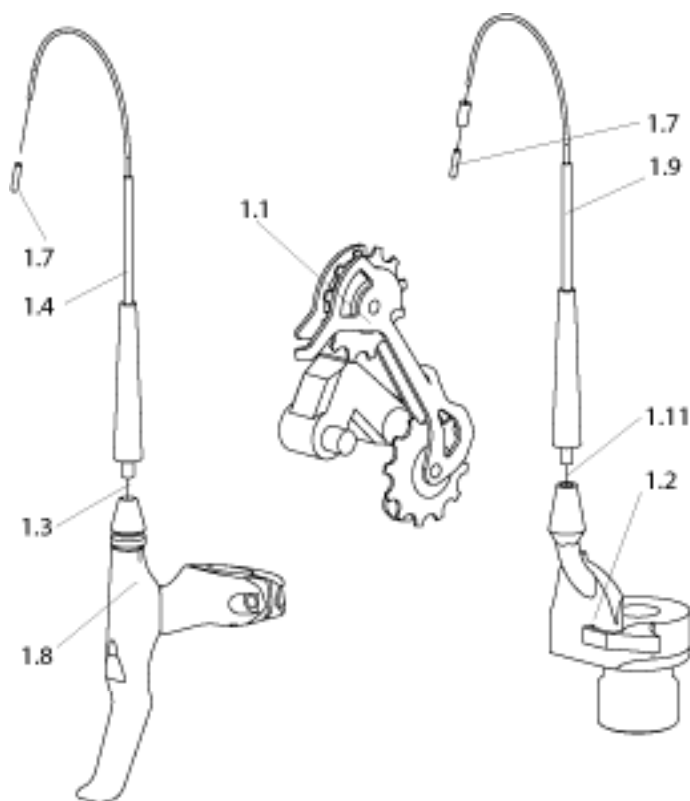


Pos.	Item Number	Description	Remarks
1.3-1.7,1.9-1.11	470 813 030	Spare kits: Shift / brake cables	
1.1	000 228 298	Rear derailleur, 9-speed, Racing	
1.1	000 228 101	Rear derailleur, 9-speed, Touring	
1.2	000 228 299	GRIP SHIFT - shifter, Touring	right
1.2	000 228 206	GRIP SHIFT - shifter, Racing	
1.3	000 228 076-ET	Brake cable	
1.4	471 858 003	SHIFTER CABLE HOUSING 1020MM	
1.7	000 228 079-ET	Cable cap	10 Pcs.
1.8	000 762 005	Brake lever Touring	
1.8	000 762 002	Brake lever Racing	
1.9	471 858 002	Shifter cable housing when handle mount 1570 mm / Fork low	
1.11	000 228 075-ET	Shifter cable	1 Pcs.


Gabel tief, Schaltung 3x9:	Länge in cm
SRAM Schaltung am Griff:	120
SRAM Schaltung am Rahmen:	169
SRAM Schaltung an der Gabel:	25
SRAM Bremse am Griff	83
SRAM Bremse am Rahmen:	75
SRAM Bremse an der Gabel:	83
SHIMANO Schaltung an der Gabel:	65
SHIMANO Bremse an der Gabel:	75
Parkbremse:	26
Gabel hoch, Schaltung 3x 9:	
SRAM Schaltung am Griff:	143
SRAM Schaltung am Rahmen:	169
SRAM Schaltung an der Gabel:	35
SRAM Bremse am Griff	100
SRAM Bremse am Rahmen:	123
SRAM Bremse an der Gabel:	63
SHIMANO Schaltung an der Gabel:	80
SHIMANO Bremse an der Gabel:	55
Parkbremse:	32
Gabel hoch, Schaltung 7 Gang:	
Schaltung am Griff:	148
Schaltung am Rahmen:	163
Schaltung an der Gabel:	42
Parkbremse:	48

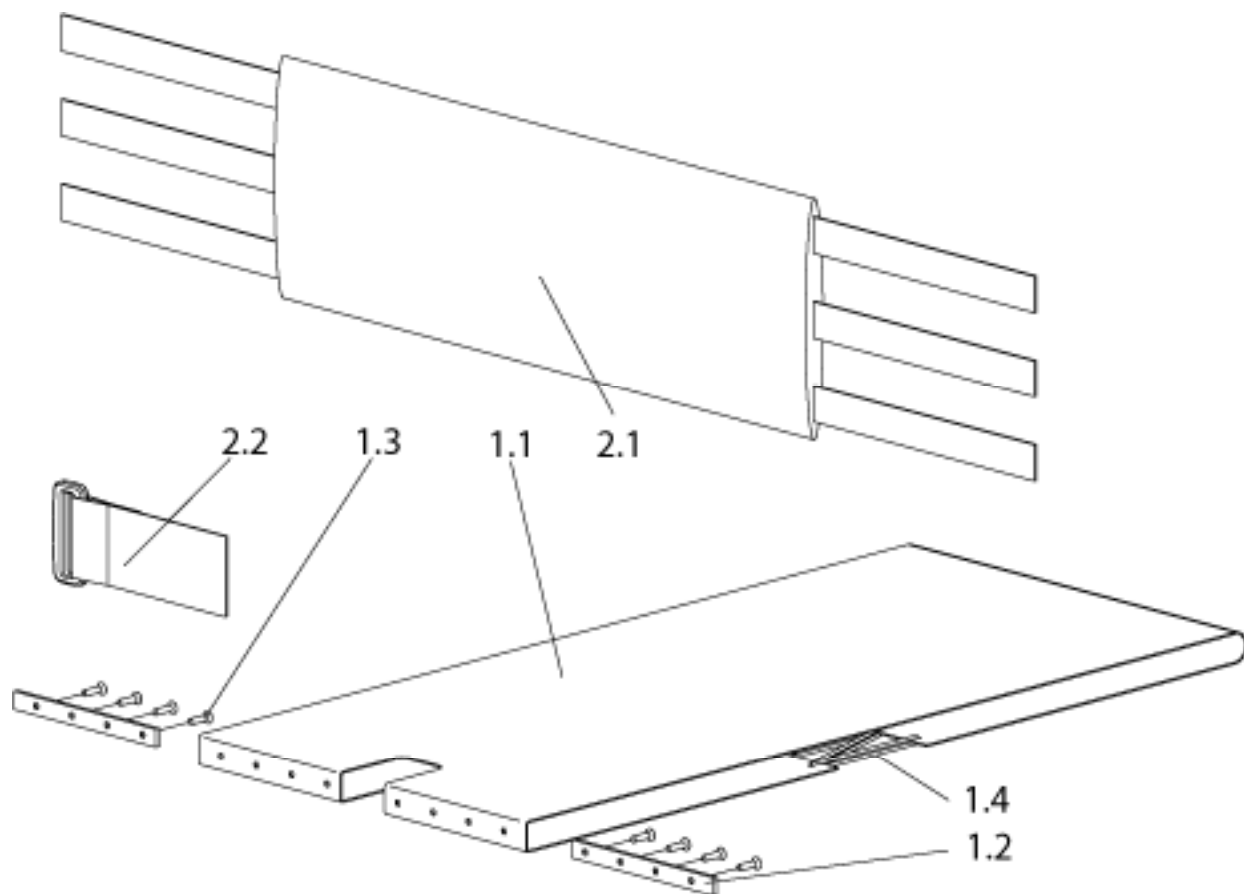


Pos.	Item Number	Description	Remarks
1.1	471 832 001S	Seat upholstery	
1.2	471 832 002	Seat rail	
1.3	000 104 083-ET	Set screw M 5 x 10	25 Pcs.
1.4	000 067 020	Cord	Running length
2.1-2.2	471 841 042	Hip belt cpl.	
2.1	471 841 041	Hip belt	
2.2	471 841 040	Hip belt holder	6 pcs.
3.1	000 835 002	Seat Cushion	

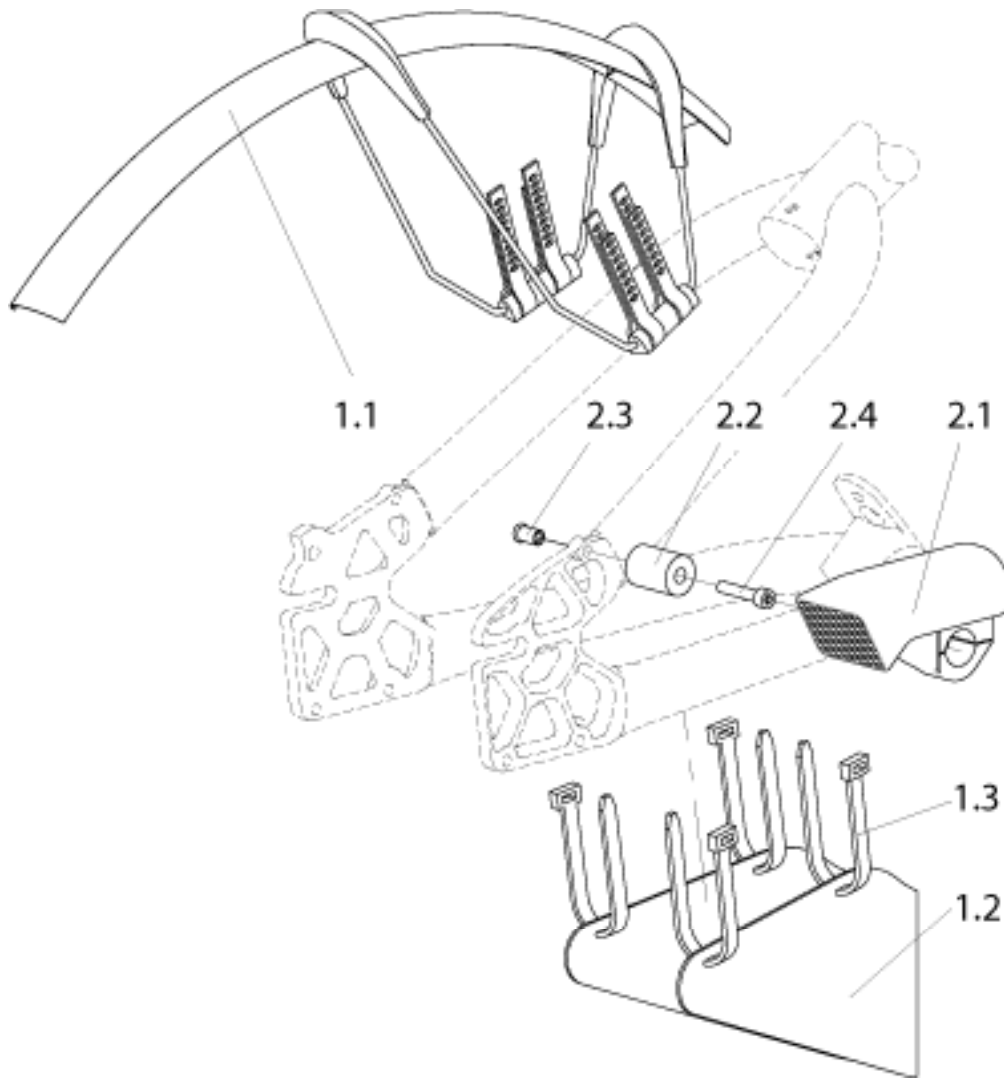

Pos.	Item Number	Description	Remarks
1.1	000 228 302	Fender front	
1.2	471 853 020	Fender rear	
1.3	000 280 010-ET	Zip tie 178 mm x 4,8 mm	5 Pcs.
2.1-2.4	471 812 304	Head light cpl.	
2.2	471 861 002	Clamping tube	
2.3	840 826 007	Socket nut	
2.4	000 102 012-ET	Soc. screw M 6 x 30 DIN 912	25 Pcs.

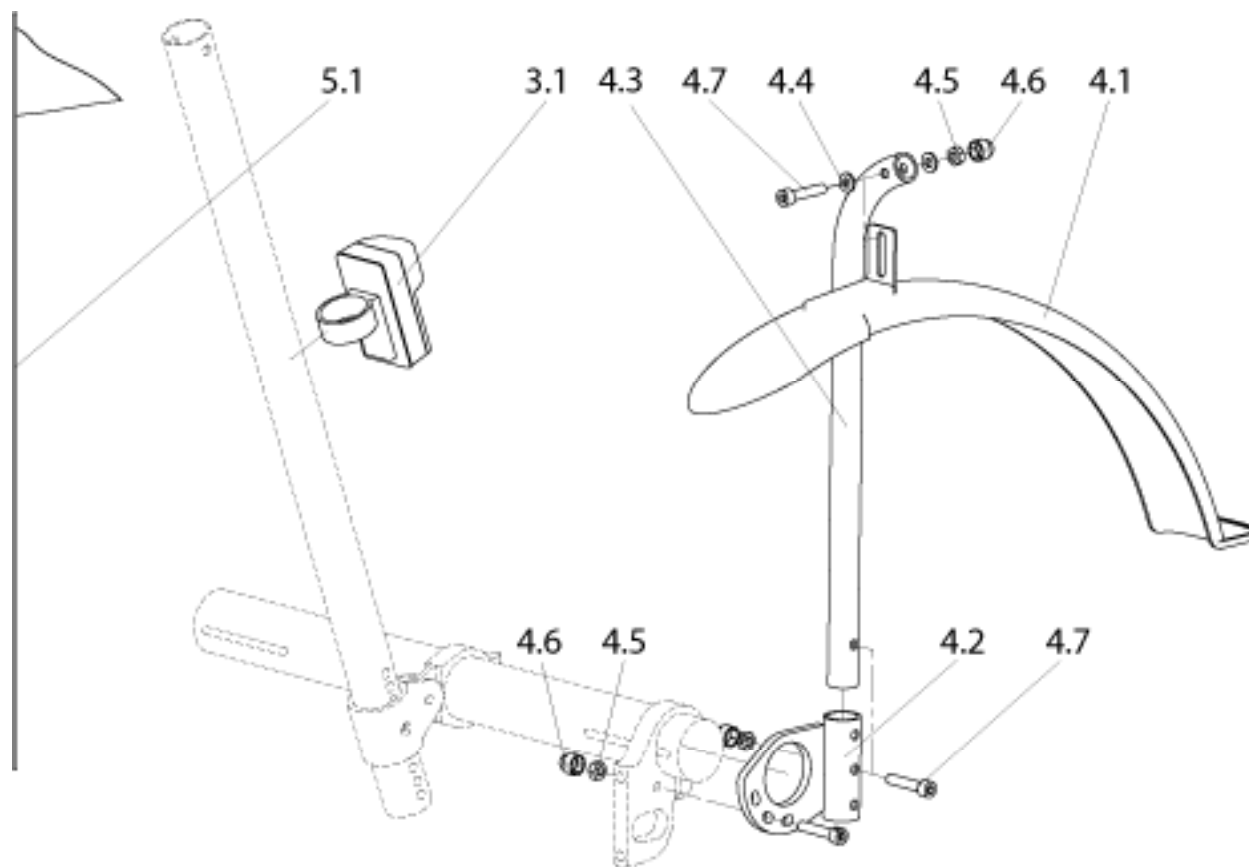

Pos.	Item Number	Description	Remarks
3.1	000 228 115	Rear light	
4.1-4.7	471 812 305	Fender rear 20" cpl.	Pair
4.1-4.7	471 812 306	Fender rear 26" cpl.	Pair
4.1	000 228 303	Fender rear 20"	
4.1	000 228 304	Fender rear 26"	
4.2	471 853 011	Bumper / fender bracket	right
4.2	471 753 011	Bumper / fender bracket	left
4.3	471 853 009.02	Fender holder 26"	
4.3	471 853 009.01	Fender holder 20"	
4.4	000 130 107-ET	Washer A 6,3 DIN 125	25 Pcs.
4.5	000 128 007-ET	Hex. nut M 6 DIN 985	25 Pcs.
4.6	000 282 063-ET	Nut cap f. M 6	10 Pcs.
4.7	000 102 012-ET	Soc. screw M 6 x 30 DIN 912	25 Pcs.
5.1	000 228 229	Signal flag	

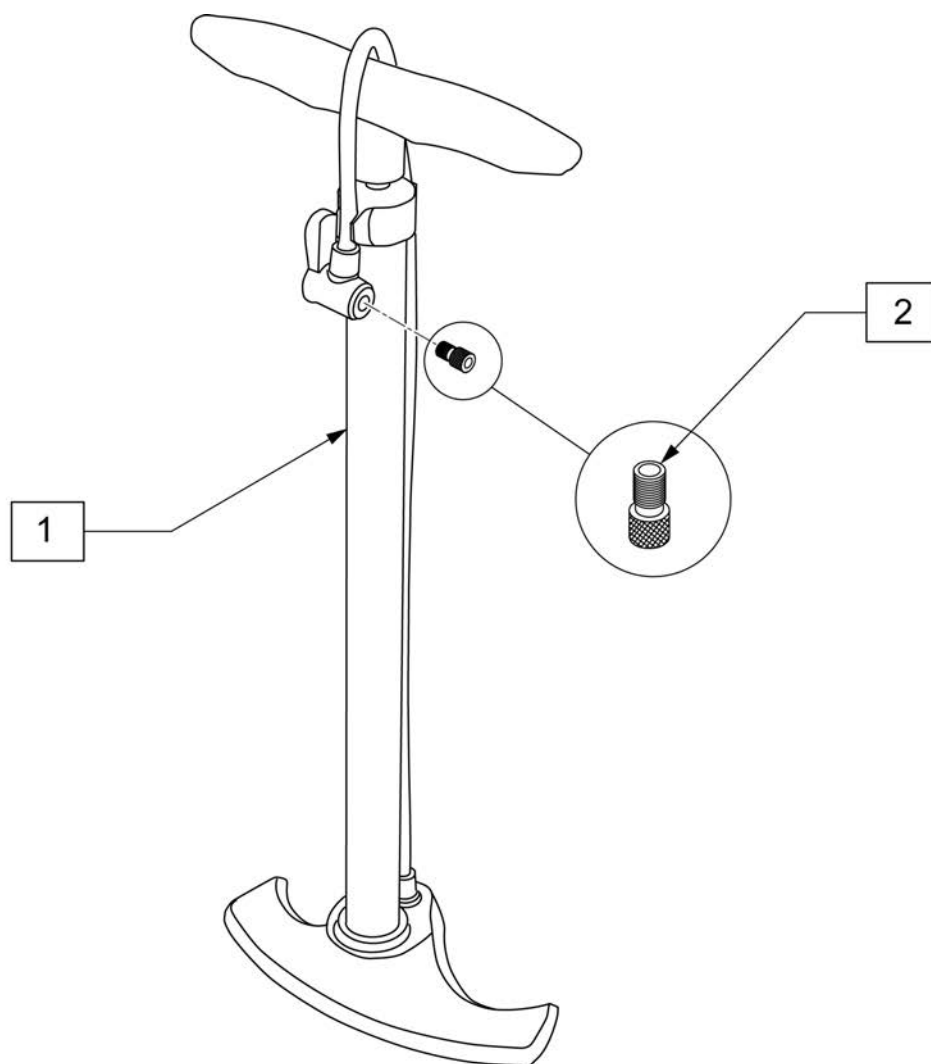

Pos.	Item Number	Description
1	389050	TIRE PUMP
2	381230	VALVE ADAPTER PRESTA



www.sunrisemedical.com

Sunrise Medical
7477 East Dry Creek Parkway
Longmont, Colorado
80503

800.333.4000 303.218.4600 In Canada: 800.263.3390

©2007 Sunrise Medical Inc.
021017