

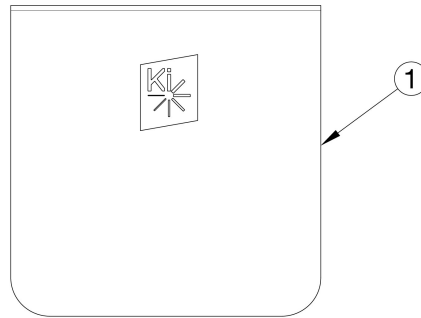


# Catalyst E Back Upholstery Padded Velcro Adjustable

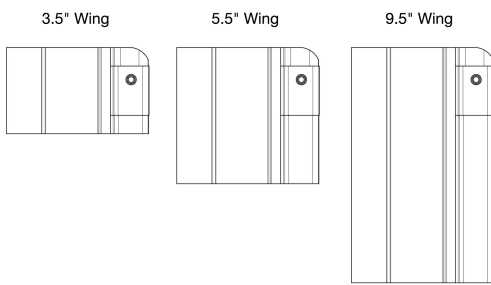

Backrest Range and Height	Wings 13" - 17" Wide	Wings 18" - 22" Wide	Strap Qty
Short 11" - 12"	003083 003088	002746 002749	1
Short 13" - 14" Medium 13" - 14"	002859 002867	002861 002869	1
Medium 15" - 16"	002859 002867	002861 002869	2
Medium 17"	002859 002867	002861 002869	3
Tall 17" - 18"	N/A	N/A	3
Tall 19" - 20"	N/A	N/A	4
Tall 17-18"	002875 002883	002877 002885	1
Tall 19"	002875 002883	002877 002885	2
Tall 20"	002875 002883	002877 002885	3

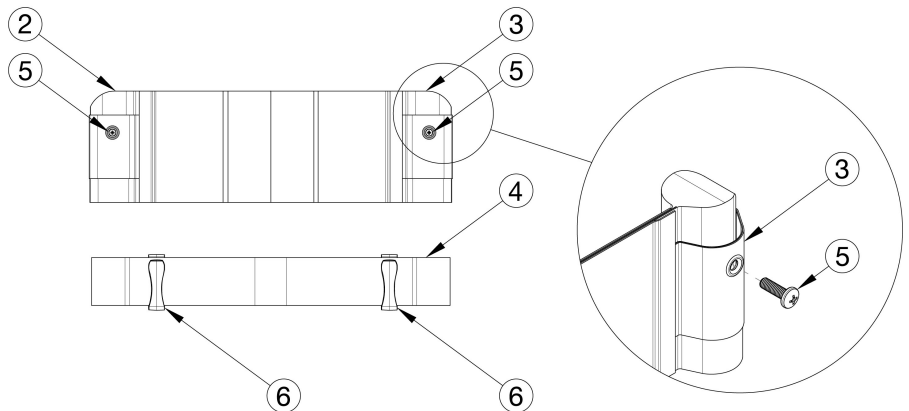

Item Number	Part Number	Description	Quantity
1a, 2a, 3a, 4a, 5, 6	115856	Back Upholstery Assembly Velcro Adjustable 14" Wide, 11-12 Height	1
1a, 2c, 3c, 4a, 5, 6	115857	Back Upholstery Assembly Velcro Adjustable 14" Wide, 13-14 Height	1
1b, 2c, 3c, 4a, 5, 6	115858	Back Upholstery Assembly Velcro Adjustable 14" Wide, 15-16 Height	1
1c, 2e, 3e, 4a, 5, 6	115859	Back Upholstery Assembly Velcro Adjustable 14" Wide, 17-20 Height	1
1d, 2a, 3a, 4b, 5, 6	115860	Back Upholstery Assembly Velcro Adjustable 16" Wide, 11-12 Height	1
1d, 2a, 3a, 4b, 5, 6	115861	Back Upholstery Assembly Velcro Adjustable 16" Wide, 13-14 Height	1
1e, 2c, 3c, 4b, 5, 6	115862	Back Upholstery Assembly Velcro Adjustable 16" Wide, 15-16 Height	1
1f, 2e, 3e, 4b, 5, 6	115863	Back Upholstery Assembly Velcro Adjustable 16" Wide, 17-20 Height	1
1g, 2b, 3b, 4c, 5, 6	115864	Back Upholstery Assembly Velcro Adjustable 18" Wide, 11-12 Height	1
1g, 2d, 3d, 4c, 5, 6	115865	Back Upholstery Assembly Velcro Adjustable 18" Wide, 13-14 Height	1
1h, 2d, 3d, 4c, 5, 6	115866	Back Upholstery Assembly Velcro Adjustable 18" Wide, 15-16 Height	1
1i, 2f, 3f, 4c, 5, 6	115867	Back Upholstery Assembly Velcro Adjustable 18" Wide, 17-20 Height	1
1j, 2b, 3b, 4d, 5, 6	115868	Back Upholstery Assembly Velcro Adjustable 20" Wide, 11-12 Height	1

Item Number	Part Number	Description	Quantity
1j, 2d, 3d, 4d, 5, 6	115869	Back Upholstery Assembly Velcro Adjustable 20" Wide, 13-14 Height	1
1k, 2d, 3d, 4d, 5, 6	115870	Back Upholstery Assembly Velcro Adjustable 20" Wide, 15-16 Height	1
1l, 2f, 3f, 4d, 5, 6	115871	Back Upholstery Assembly Velcro Adjustable 20" Wide, 17-20 Height	1
1m, 2d, 3d, 4e, 5, 6	115872	Back Upholstery Assembly Velcro Adjustable 22" Wide, 13-16 Height	1
1n, 2f, 3f, 4e, 5, 6	115873	Back Upholstery Assembly Velcro Adjustable 22" Wide, 17-20 Height	1
1a	002498	Backrest Cover 13" Rigid / 14" Cat x 11-14"	1
1b	002499	Backrest Cover 13" Rigid / 14" Cat x 13-16"	1
1c	003043	Backrest Cover 13" Rigid - 14" CAT/Liberty x 17-20"	1
1d	002506	Backrest Cover 15" Rigid / 16" Cat x 11-14"	1
1e	002507	Backrest Cover 15" Rigid / 16" Cat x 13-16"	1
1f	003045	Backrest Cover 15" Rigid - 16" CAT/Liberty x 17-20"	1
1g	003049	Backrest Cover 17" Rigid / 18" Cat x 11-14"	1
1h	003050	Backrest Cover 17" Rigid / 18" Cat x 13-16"	1
1i	003052	Backrest Cover 17" Rigid - 18" CAT/Liberty x 17-20"	1
1j	003061	Backrest Cover 19" Rigid / 20" Cat x 11-14"	1
1k	003062	Backrest Cover 19" Rigid / 20" Cat x 13-16"	1
1l	003064	Backrest Cover 19" Rigid - 20" CAT/Liberty x 17-20"	1
1m	003072	Backrest Cover 21" Rigid / 22" Cat x 13-16"	1
1n	003074	Backrest Cover 21" Rigid - 22" CAT/Liberty x 17-20"	1
2a	003088	Standard Upholstery Wing 3.5" Height 13" - 17" Wide, Left	1
2b	002749	Standard Upholstery Wing 3.5" Height 18" - 22" Wide, Left	1
2c	002867	Standard Upholstery Wing 5.5" Height 13-17" Wide Left	1
2d	002869	Standard Upholstery Wing 5.5" Height 18-22" Wide Left	1
2e	002883	Standard Upholstery Wing 9.5" Height 13-17" Wide Left	1
2f	002885	Standard Upholstery Wing 9.5" Height 18-22" Wide Left	1
3a	003083	Standard Upholstery Wing 3.5" Height 13" - 17" Wide, Right	1
3b	002746	Standard Upholstery Wing 3.5" Height 18" - 22" Wide, Right	1

Item Number	Part Number	Description	Quantity
3c	002859	Standard Upholstery Wing 5.5" Height 13-17" Wide Right	1
3d	002861	Standard Upholstery Wing 5.5" Height 18-22" Wide Right	1
3e	002875	Standard Upholstery Wing 9.5" Height 13-17" Wide Right	1
3f	002877	Standard Upholstery Wing 9.5" Height 18-22" Wide Right	1
4a	002537	Back Upholstery Strap 27.3" <i>Used on 14" chair widths</i>	1
4b	002539	Back Upholstery Strap 31.3" <i>Used on 16" chair widths</i>	1
4c	002741	Back Upholstery Strap 35.3" <i>Used on 18" chair widths</i>	1
4d	002743	Back Upholstery Strap 39.3" <i>Used on 20" chair widths</i>	1
4e	002745	Back Upholstery Strap 43.3" <i>Used on 22" chair widths</i>	1
5	100734	M5 x 16 PHMS ZC	2
6	102736	Quick-Slip Keeper	2