


Catalyst E Side Frame Assembly

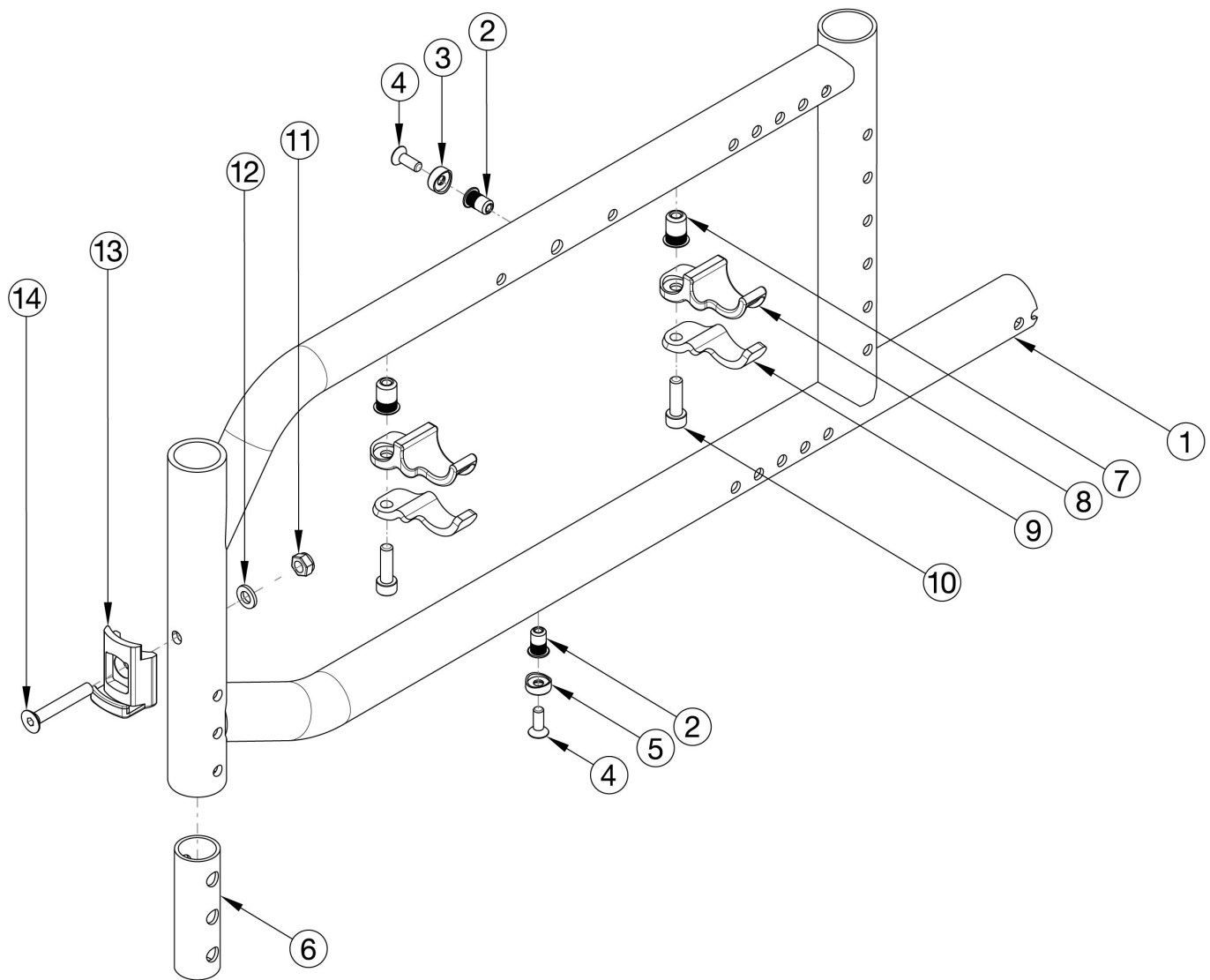

Frame assemblies are based on Seat Depth. If switching to new Seat Depth, new Height Adjustable T-Arms or Height Adjustable Flip Back T-Arms may be needed.

Item Number	Part Number	Description	Quantity	Retail
1b, 2, 6, 7	115926_ _ _	Frame Assembly, Left, 18, Catalyst E	1	\$55.33
1c, 2, 6, 7	115927_ _ _	Frame Assembly, Left, 20, Catalyst E	1	\$56.33
1a, 2, 6, 7	115928_ _ _	Frame Assembly, Right, 16, Catalyst E	1	\$54.13
1b, 2, 6, 7	115929_ _ _	Frame Assembly, Right, 18, Catalyst E	1	\$55.33
1c, 2, 6, 7	115930_ _ _	Frame Assembly, Right, 20, Catalyst E	1	\$56.33

Item Number	Part Number	Description	Quantity	Retail
3-5, 8-12	115924	Seat Hook and Support Hardware Kit, Catalyst E <i>For 1 side</i>	1	\$17.70
3	111940	X-Brace Top Clamp Retainer Saddle	1	\$0.25
4	102937	M5 X 14 FHCS, Blk Zn	2	\$1.66
5	111941	X-Brace Bottom Clamp Retainer Saddle	1	\$0.25
6	111942	Front Frame Insert Cat E	1	\$2.35
7	100661	M6 X 1.0 Rivnut	1	\$0.94
8	111937	Seat Hook	1	\$0.85
9	111938	Seat Hook Support, Steel	1	\$2.50
10	101180	M6 x 20 SHCS BLK ZC	1	\$2.21
11	100658	M6 Nylock Nut Blk Zn	1	\$1.10
12	100746	M6 Flat Washer Blk Zn	1	\$1.66
13	000336	Latch Block	1	\$39.40
14	101819	M6 x 40 FHCS Blk Zn	1	\$1.10

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
BRD	Red
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer