


# Liberty Caster Housing

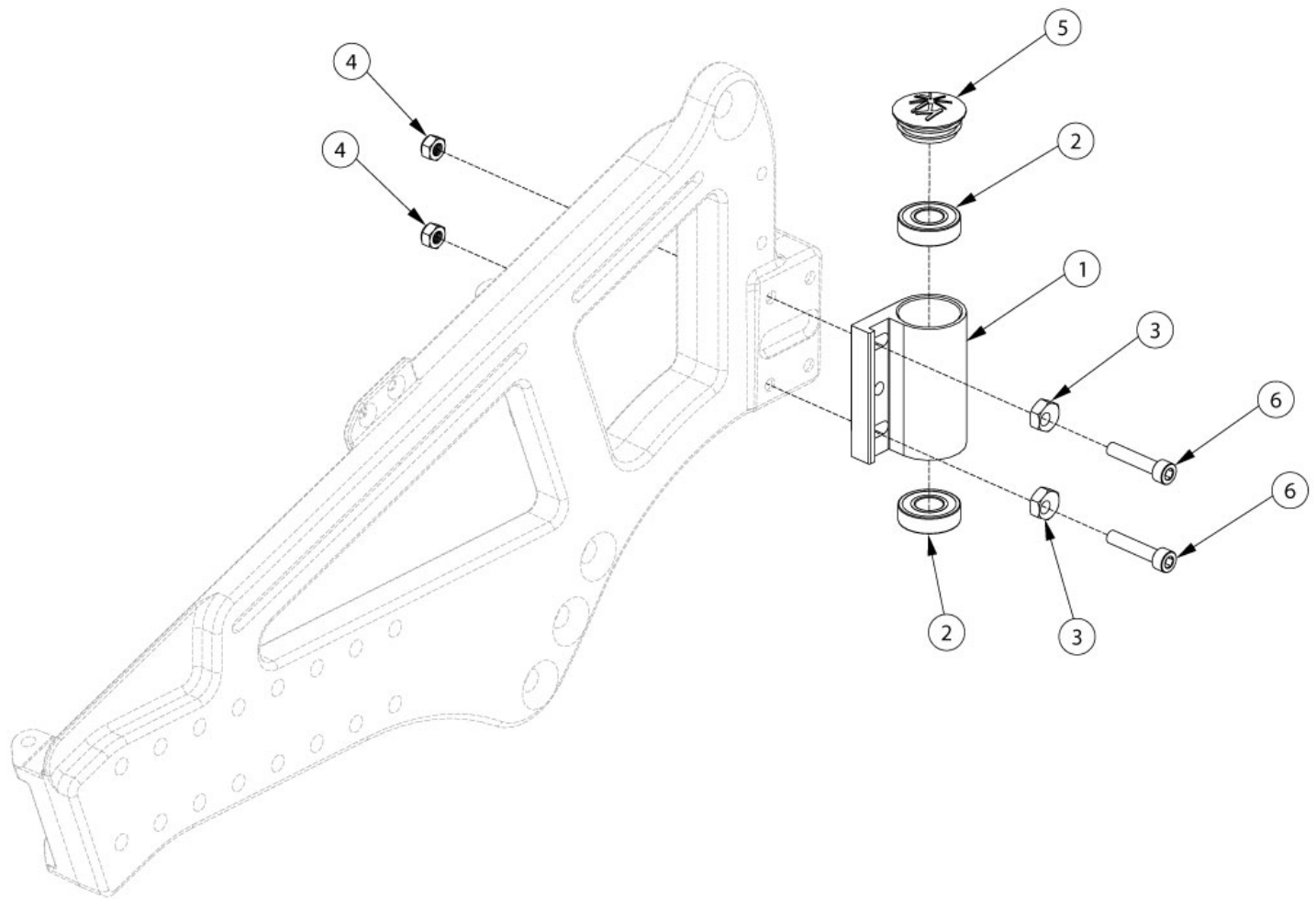

Item Number	Part Number	Description	Quantity
1a-6	506152	Caster Housing Assembly, Left, Liberty	1
1b-6	506151	Caster Housing Assembly, Right, Liberty	1
1a, 2	100526	Caster Housing Left, Black, CAT/Liberty	1
1b, 2	100527	Caster Housing Right, Black, CAT/Liberty	1
2	100902	Bearing 1 1/8" x 1/2" R8RS	2
3	100510	Eccentric Washer	2
4	100658	M6 Nylock Nut Blk Zn	2
5	102005	Ki Caster Housing Cap	1
6	102627	M6 X 30 SHCS BLK ZC	2