




# MALE URINAL KIT

## URINE COLLECTION SYSTEM

USE AND CARE INSTRUCTIONS  
FOR PRODUCT NO. 4401, 4400 AND 4399

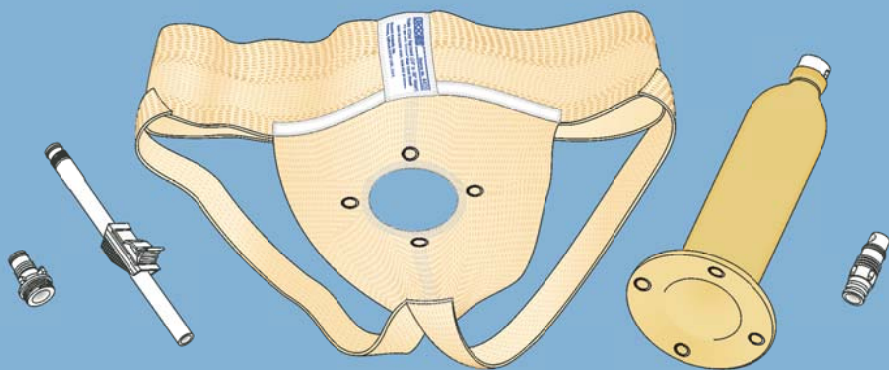

Examine this product carefully. If not entirely satisfied, do not accept.

**UNDER NO CONDITIONS WILL IT BE SUBJECT TO RETURN!**

Sanitary reasons make this ruling absolute.

### **KIT CONTENTS:**

- 1 Male Urinal Suspensory Garment (No. 4421 or 4420)
- 1 Male urinal Sheath (No. 4410 or 4409)
- 1 Standard Bottom Drain Valve (No. 6004)
- 1 Straight-thru Adaptor with Clamp (No. 6005)
- 1 Male Urinal to Latex Leg Bag Adaptor (No. 6022)
- 1 Illustrated Use and Care Instructions

## IMPORTANT! READ THIS FIRST:

For best results, certain precautions should be taken before each application. Among them, be certain that: all snaps are firmly fastened to both the garment and the urinal sheath before each use, there are no loose stitches and there are no sharp edges on any of the components.

**CAUTION:** Those with known allergic reactions to synthetic materials, especially latex rubber, should not wear this garment. Discontinue use immediately at first signs of any skin or other complications and consult a fitter or physician. If discomfort occurs or prolonged wear affects the performance of the garment, discontinue use. This product is intended for use in medical rehabilitation for urine collection, not as any form of protection during sports.

## DIRECTIONS FOR USE AS A DRIP URINAL

### STEP 1: Trimming the Inner Sheath

Pull out the funnel-shaped, inner sheath as shown in Figure 1. The opening of the sheath should be trimmed to fit snugly around the penis to prevent leakage. The sheath is funnel-shaped and may be trimmed to a number of different sizes. If the opening of the sheath is too small or too tight, trim off not more than an  $\frac{1}{8}$ " (3mm) at a time. Repeat if necessary until satisfied that you have a comfortable, snug fit. Cut the sheath with a pair of large, sharp scissors to prevent jagged or nicked edges before application.

**CAUTION:** For comfort, the inner sheath is made of a thinner, light-weight latex and may tear if cut or trimmed improperly. Be certain there are no signs of jagged or nicked edges before application.

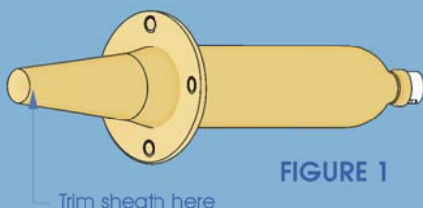


FIGURE 1

### STEP 2: Application of the Urinal Sheath

After the sheath has been trimmed, allow the penis to enter as far as possible so that the end of the inner sheath is as far as possible behind the head of the penis. Return the inner sheath with the penis inserted to its original position inside the out sheath. If the Standard Bottom Drain Valve (No. 6004) is not already in place, inset into the valve housing at the tip of the urinal sheath and while turning gently clockwise, screw it into position as shown in Figure 2. **CAUTION:** Do not over-tighten drain valve. To open the valve for drainage, twist it counter-clockwise  $1\frac{1}{2}$  turns.



FIGURE 2  
Enlarged View of the  
Urinal Sheath Valve Housing

### STEP 3: Application and Adjustment of the Suspensory

With the urinal sheath already applied to the penis, pull the suspensory garment over the legs passing the tip of the urinal sheath through the opening in the front of the garment. Then snap the urinal sheath firmly into position as shown in Figure 3.

Adjustment of the suspensory garment should be made while wearing with the use of the laced closure at the rear of the garment. Please note: The garment should fit snug around the waist, but not too tight.

**NOTE:** Do not over-tighten the Bottom Drain Valve—doing so may damage the valve housing. Remember to close the Standard Bottom Drain Valve before use.


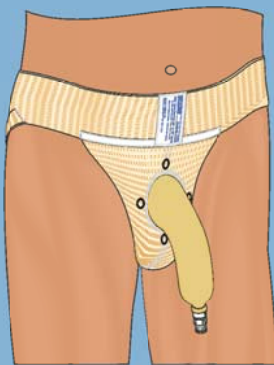


FIGURE 3  
Suspensory Garment and  
Sheath Fitted correctly

## DIRECTIONS FOR USE WITH A UROCARE® LATEX LEG BAG


### STEP 1: Deciding on a Position for the Latex Leg Bag

Follow steps one(1) through three(3) on the previous page. Once the suspensory and sheath are in place it's time to decide on a position for your leg bag. If your preferred leg bag position is either over or below the knee, the use of an extension tube (No.6005, supplied, or No. 6016, not supplied) may be necessary—see Figure 4.

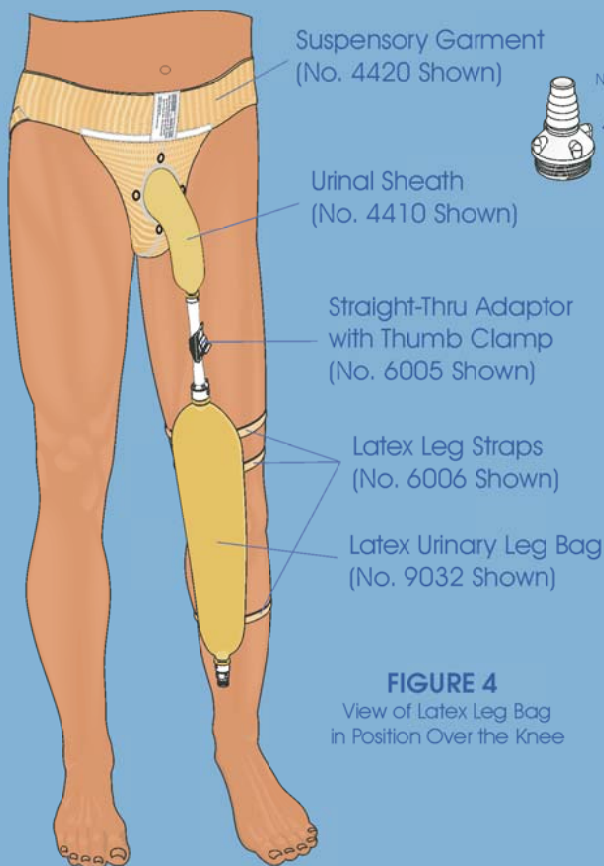

**FIGURE 4**

View of Latex Leg Bag  
in Position Over the Knee

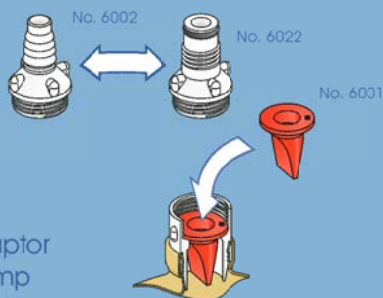

**FIGURE 5**

Enlarged View of  
Latex Leg Bag Top

**IMPORTANT:** To avoid drainage problems, before using this product, read all instructions and information carefully. Understand how it works and how to maintain it properly.

No. 6000  
Black Gasket-Ring used to  
replace the anti-reflux valve  
when not in use.

### STEP 2: Replacement of the Standard Bottom Drain Valve

If you prefer to connect your latex leg bag directly to the sheath, the Standard Bottom Drain Valve(No. 6004) will need to be replaced with the Male Urinal Adaptor(No. 6022). To do so, remove the drain valve from the sheath by turning it counter-clockwise until it separates from the valve housing.

### STEP 3: Replacement of the Standard Top Adaptor

Remove the Standard Top Adaptor from your latex leg bag in the same manner as above. Leaving the anti-reflux valve in position, attach the Male Urinal Adaptor to the valve housing—see Figure 5.

### STEP 4: Attaching the Leg Straps

Whether you are using the latex leg straps supplied with your leg bag or a set of fabric straps, you will need to attach them to your latex leg bag before it is attached to the urinal sheath by feeding the strap ties through the strap loops on the latex leg bag.



### STEP 5: Connecting the Male Urinal Adaptor to the Urinal Sheath

The Male Urinal Adaptor while already attached to the latex leg bag can be connected to the urinal sheath by inserting the tip of the adaptor into the valve housing of the sheath and turning both the adaptor and the bag clockwise until firmly in position—see Figure 4.

### STEP 6: Securing the Leg Straps and Latex Leg Bag to the Leg

Once the urinal adaptor has been properly attached to the sheath it's time to secure the leg straps to the leg. This is the final step. At this point you may wish to make some minor adjustments to the suspensory garment, sheath or leg bag position for comfort.

## DIRECTIONS FOR USE WITH OTHER LEG BAGS

### STEP 1: Deciding on a Position for the Latex Leg Bag


Follow steps one(1) through three(3) on the previous page. Once the suspensory and sheath are in place it's time to decide on a position for your leg bag. If your preferred leg bag position is either over or below the knee, an extension tube (No.6005, supplied, or No. 6016, not be supplied) may be necessary.

### STEP 2: Replacement of the Standard Bottom Drain Valve

If you prefer to connect your leg bag directly to the sheath, the Standard Bottom Drain Valve(No. 6004) will need to be replaced with the Straight-Thru Adaptor(No. 6005). To do so, remove the drain valve from the sheath by turning it counter-clockwise until it separates from the valve housing. Then, insert the Straight-Thru Adaptor into the valve housing on the urinal sheath and turn clockwise until firmly into position.

NOTE: Do not over-tighten the Straight-Thru Adaptor—doing so may damage the valve housing.

Remember to close the Clamp before use.

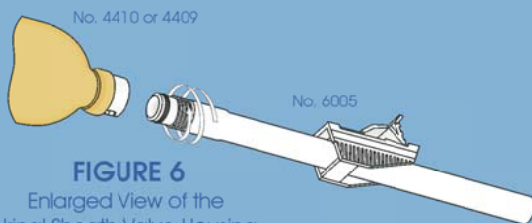

**FIGURE 6**  
Enlarged View of the  
Urinal Sheath Valve Housing

### STEP 3: Attaching the Leg Bag

You will need to attach your leg bag before it is attached to the urinal sheath by feeding the leg strap ends or the leg strap ties through the strap loops on the leg bag and then fastening the bag to your leg in the desired position.

### STEP 4: Connecting Your Leg Bag to the Urinal Sheath

At this point you can connect the Straight-Thru Adaptor to your leg bag by inserting the top of the leg bag adaptor into the end of the Straight-Thru Adaptor—the use of an extension tube between the Straight-Thru Adaptor and the leg bag may be necessary if your leg bag is positioned more than 6"(16cm) from the end of the urinal sheath—see Figure 6.

## CARE AND MAINTENANCE INSTRUCTIONS

To clean a urinal after use, remove the urinal sheath from the suspensory garment and wash the garment thoroughly with soap and water. If desired, the urinal sheath can be immersed in boiling water for one(1) to three(3) minutes maximum. If after continued use a deodorant is needed, soak for two(2) or three(3) minutes in a weak solution of Urolux® urinary & ostomy appliance cleanser/deodorant followed by a complete rinsing and drying—see back panel of Urolux® bottle for more details.

CAUTION: Avoid contact with ointments, oils or anything that will damage the delicate latex. Store in a cool, dark place. KEEP OUT OF DIRECT SUNLIGHT!


# DON'T SETTLE FOR ANYTHING LESS!

We've been providing security and confidence for over 22 years. Choose complete protection—Urocare latex reusable urinary leg bags.

**Superior**, self-cleaning, patented, anti-reflux valve which prevents back-flow and allows uninterrupted drainage for problem-free use.

**Improved**, strap-loop design ensures an accurate placement and a secure hold without glues which might allow the strap-loops to separate from the bag.

Each Urocare® latex urinary leg bag is packed on a printed skin-film board and includes adaptors, latex leg straps, use and care instructions that have been translated into four (4) languages: English, German, Spanish and French.

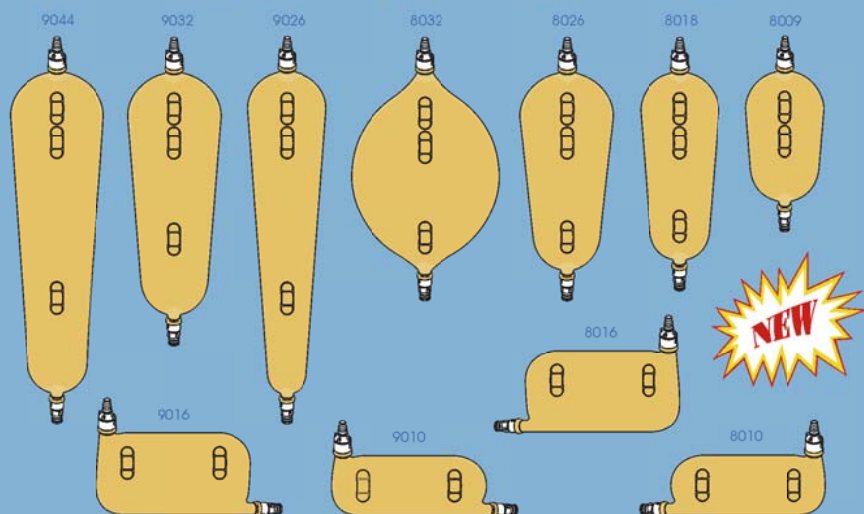

PRODUCT NUMBER	DESCRIPTION	DIMENSIONS <sup>1</sup>		CAPACITIES <sup>1</sup>	
		INCHES (")	CENTIMETERS (cm)	Fluid Ounces (fl. Oz.)	Milliliters (ml.)
9044	Extra-Large	20% x 4%	52.4 x 12.1	44	1,300
9032	Large	16% x 4%	42.5 x 12.1	32	1,000
9026	Long-Slim	20% x 3%	52.4 x 9.5	26	800
9016 <sup>‡</sup>	Sport-Right, Large	5% x 10	14.6 x 25.0	16	500
9010 <sup>‡</sup>	Sport-Right, Small	4% x 10	12.1 x 25.0	10	300
8032	Large-Oval	13 x 7	33.0 x 17.8	32	1,000
8026	Short-Wide	13% x 4%	34.0 x 12.1	26	800
8018	Medium	13 x 3%	33.0 x 9.5	18	550
8016 <sup>‡</sup>	Sport-Left, Large	5% x 10	14.6 x 25.0	16	500
8010 <sup>‡</sup>	Sport-Left, Small	4% x 10	12.1 x 25.0	10	300
8009	Small	9% x 3%	24.8 x 9.2	9	275

<sup>1</sup> Approximate dimensions include adaptors and drain valves. Approximate volumes in fluid ounces and milliliters.

<sup>‡</sup> These leg bags are designed to be worn horizontally above the hemlines of skirts, dresses and swimsuits.



## RELATED PRODUCTS

Urocare® Reusable Latex Leg Bags

Uro-Safe® Vinyl Leg Bags

Night Drainage Systems

Itch•ender!® Spray-on Lotion

Uro-Wash® Skin Cleanser

Uro-Con® & Uro-Cath® Male External Catheters

Drain Valves & Adaptors

Drainage Bottles

Uro-Prep® Protective Skin Wipes

Fabric Leg Bag Holders

Fabric Leg Straps

Uro-Bond® II Brush-on Silicone Adhesive

Urofoam® Adhesive Foam Strips

Adhesive Remover Pads

Urolux® Appliance Cleanser & Deodorant

Connectors & Clamps

Extension & Drainage Tubing

Dri-Sleeper™ Nocturnal Enuretic Alarm

For more information on any of Urocare's other products, contact your nearest Urocare dealer or write to:

Urocare Products, Inc.  
Customer Service Department  
2735 Melbourne Avenue  
Pomona, CA 91767-1931, U.S.A.

Tel: (909) 621-6013 • Fax: (909) 621-4436

Web: [www.urocare.com](http://www.urocare.com)



PLEASE RECYCLE  
Printed on  
Recycled Paper