


Catalyst Half Folding Backrest

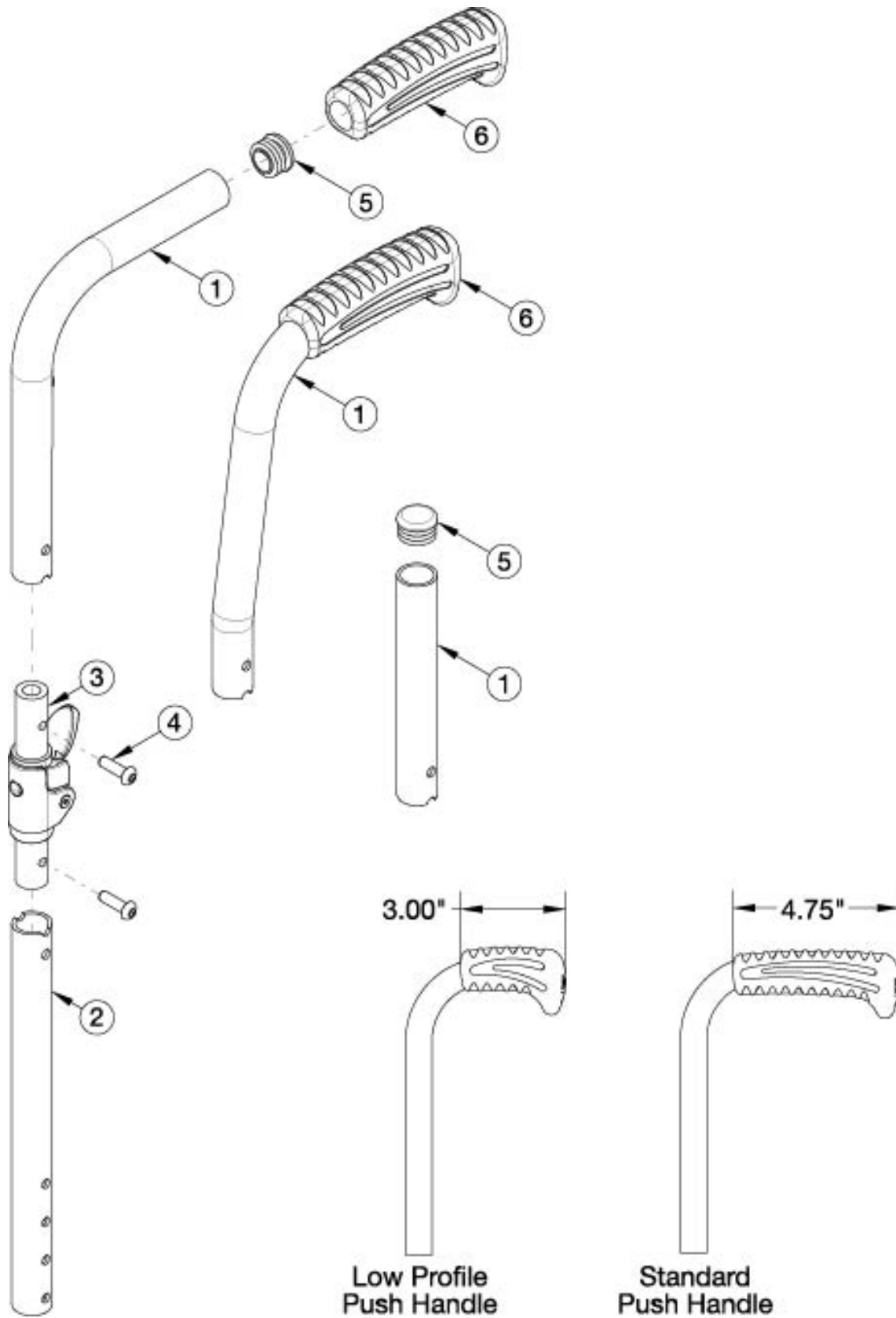

New Style as of 9/6/23 Not available with Transit option, Heavy Duty option, Flip Back Armrest or Angle Adjustable Locking Extendable Flip Up Armrest Low Profile Push Handles are not available with Drum Brake If changing from another style backrest and the Drum Brake is present, a new Drum Brake Cable Kit may be required If Anti-Rollback is present, see Back Tubes for Anti-Rollback System

Item Number	Part Number	Description	Quantity
3, 4	121007	Half Folding Backrest Hinge Latch Assembly	1
3, 4	121008	Half Folding Backrest Hinge Latch Assembly Pair	1
1a, 2, 3, 4, 5, 6a	121009	Half Folding Back Post Assembly Short Straight w/ Standard Push Handle	1
1a, 2, 3, 4, 5, 6a	121010	Half Folding Back Post Assembly Short Straight w/ Standard Push Handle Pair	1
1b, 2, 3, 4, 5, 6a	121011	Half Folding Back Post Assembly Tall Straight w/ Standard Push Handle	1
1b, 2, 3, 4, 5, 6a	121012	Half Folding Back Post Assembly Tall Straight w/ Standard Push Handle Pair	1
1c, 2, 3, 4, 5, 6b	121013	Half Folding Back Post Assembly Short Straight w/ Low Profile Push Handle	1
1c, 2, 3, 4, 5, 6b	121014	Half Folding Back Post Assembly Short Straight w/ Low Profile Push Handle Pair	1
1d, 2, 3, 4, 5, 6b	121015	Half Folding Back Post Assembly Tall Straight w/ Low Profile Push Handle	1
1d, 2, 3, 4, 5, 6b	121016	Half Folding Back Post Assembly Tall Straight w/ Low Profile Push Handle Pair	1
1e, 2, 3, 4, 5	121017	Half Folding Back Post Assembly Short Straight	1
1e, 2, 3, 4, 5	121018	Half Folding Back Post Assembly Short Straight Pair	1
1f, 2, 3, 4, 5	121019	Half Folding Back Post Assembly Tall Straight	1
1f, 2, 3, 4, 5	121020	Half Folding Back Post Assembly Tall Straight Pair	1
1g, 2, 3, 4, 5, 6a	121021	Half Folding Back Post Assembly Short 8 Degree Bend with Standard Push Handle	1
1g, 2, 3, 4, 5, 6a	121022	Half Folding Back Post Assembly Short 8 Degree Bend with Standard Push Handle Pair	1
1h, 2, 3, 4, 5, 6a	121023	Half Folding Back Post Assembly Tall 8 Degree Bend with Standard Push Handle	1
1h, 2, 3, 4, 5, 6a	121024	Half Folding Back Post Assembly Tall 8 Degree Bend with Standard Push Handle Pair	1
1i, 2, 3, 4, 5, 6b	121025	Half Folding Back Post Assembly Short 8 Degree Bend with Low Profile Push Handle	1
1i, 2, 3, 4, 5, 6b	121026	Half Folding Back Post Assembly Short 8 Degree Bend with Low Profile Push Handle Pair	1
1j, 2, 3, 4, 5, 6b	121027	Half Folding Back Post Assembly Tall 8 Degree Bend with Low Profile Push Handle	1
1j, 2, 3, 4, 5, 6b	121028	Half Folding Back Post Assembly Tall 8 Degree Bend with Low Profile Push Handle Pair	1