


(Discontinued 2) Rogue Backrest Mount and Hardware

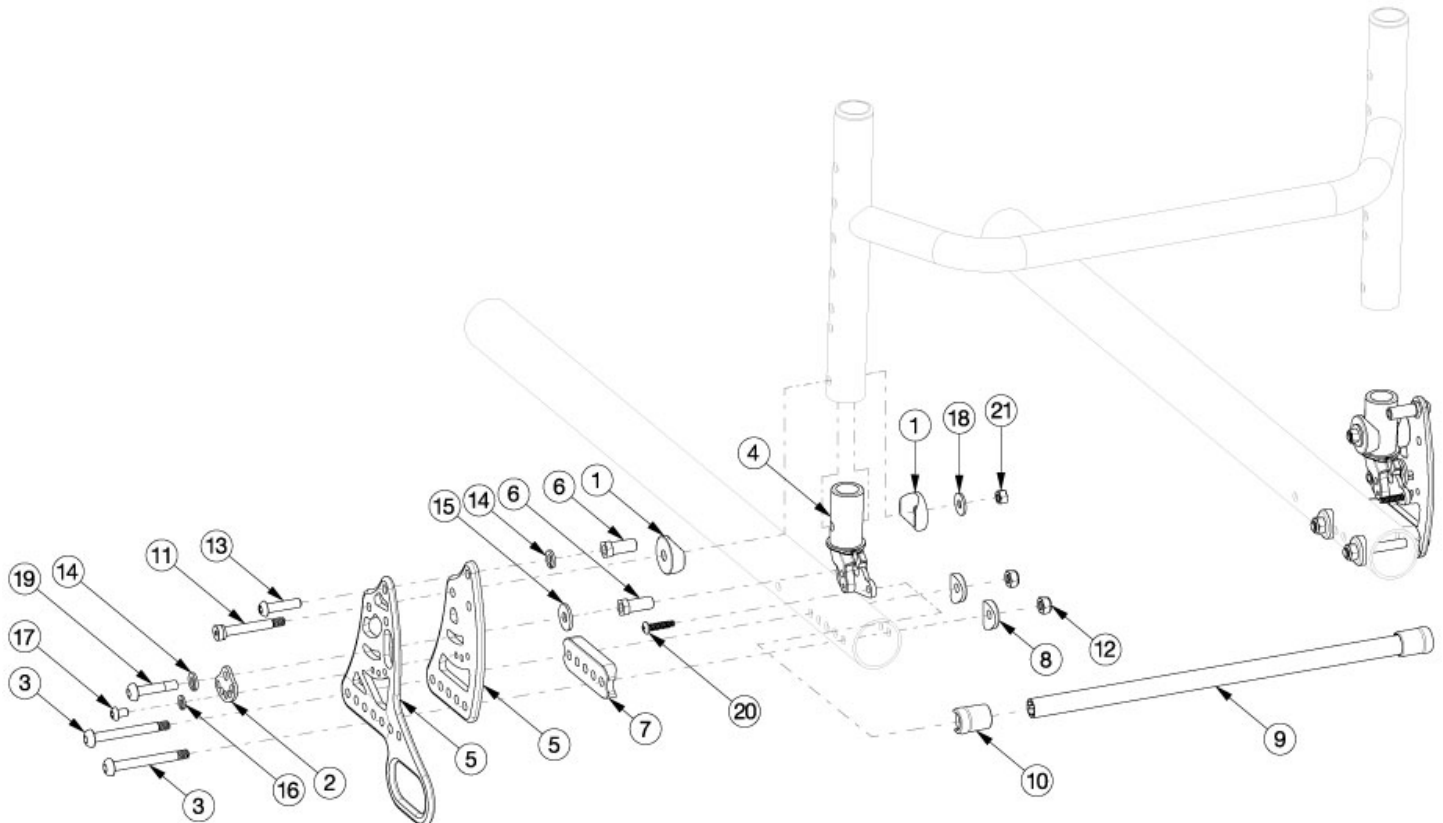

Please note the following: ****Backrest Bracket Mount Kits (parts 107199 and 107200) and the Backrest Transit Bracket Mount Kit (part 107245) contain complete assemblies for the left and right sides. **This Rogue Backrest Mount and Hardware is effective starting with serial number RG0016612. **If retrofitting to this backrest style, new back frames and backrest tubes are required. **If replacing backrest parts for a Discontinued Rogue with XP option, see Discontinued Rogue Backrest Mount and Hardware page. If you have a Rogue XP, see the Rogue XP Backrest Mount pages.**

Item Number	Part Number	Description	Quantity
1	000055	Saddle 1"	4
2	001762	Arc Adjustment Cam	2
3a	003232	M6 x 55 BHCS Blk Zc <i>Used on chairs with 0" or positive center of gravity</i>	4
3b	003241	M6 x 60 BHCS P/THD Custom Blk Zc <i>Used on chairs with negative center of gravity</i>	4
5a	003711	Back Bracket Right, Black Anodized	1
5b	003711_ _ _	Color Anodized Back Bracket, Right	1
5c	003712	Back Bracket Left, Black Anodized	1
5d	003712_ _ _	Color Anodized Back Bracket, Left	1

Item Number	Part Number	Description	Quantity
5e	004434	Back Bracket with Transit, Black, Rigid	1
6	003713	Strike Hex	4
7	003715	Back Saddle 1 3/8" Tube	2
8	003716	Saddle 1 3/8" Tube	4
9a	003739	Backrest Release Bar 12" Wide, Black Anodized	1
9b	003740	Backrest Release Bar 13" Wide, Black Anodized	1
9c	003741	Backrest Release Bar 14" Wide, Black Anodized	1
9d	003742	Backrest Release Bar 15" Wide, Black Anodized	1
9e	003743	Backrest Release Bar 16" Wide, Black Anodized	1
9f	003744	Backrest Release Bar 17" Wide, Black Anodized	1
9h	003746	Backrest Release Bar 19" Wide, Black Anodized	1
9i	003747	Backrest Release Bar 20" Wide, Black Anodized	1
11	103284	M6 x 35 Shoulder Bolt BZ	2
12	100658	M6 Nylock Nut Blk Zn	4
13	101593	M6 x 25 BTN SHCS BLKZC	2
14	101835	M6 Lock Washer Blk Zn	4
15	101853	M6 Washer Blk Zn	2
16	102614	M5 Split Lock Washer BLK ZC	2
17	102621	M5 x 8 BHCS BLK ZC	2
18	103022	M5 Washer, Blk Zn	2
19	103227	M6 x 30 Long Shoulder BHCS BZ	2
20	103279	#8-16 Tri-Lobular Plastic Screw BZ	2
21	103283	M5 Top Lock Nut BZ	2

Color Anodizing Samples

