Monster Bag

DIESTCO Monster Bag mounts to the seatback of your scooter or wheelchair providing storage area for shopping, laundry, groceries, etc. Made from high quality, water resistant polyester with reflective safety tape, visible up to 300 yards.


Fits seatbacks 7" - 24" tall.

Mounting Instructions:

- 1) Remove the strap from the metal loop.
- 2) Pass the strap through the opening between the seat and the seatback.

 Be sure to pull the strap through completely and that it is not twisted.
- 3) Re-insert the strap through the metal loop and pull through.
- 4) Pull strap down tightly and fasten the loop end to the hook portion of the strap.
- 5) Excess strap can be cut once you have your Monster Bag mounted.

If you have any issues mounting the bag, please call us at (800)795-2392 for assistance.

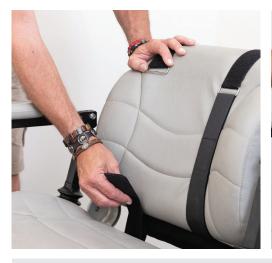







DIESTCO Mfg. Corp. P.O. Box 6504 Chico, CA 95927 (800) 795-2392 www.diestco.com We strongly encourage you to call us at (800)795-2392 if you have any problems, questions, or comments. We are always developing new products for wheelchairs, scooters, crutch, and walker users...so let us know if there's something special you need.