


GUARDIAN ALERT 911™

Enjoy the Freedom of Independent Living

OPERATING INSTRUCTIONS Model No. 30911

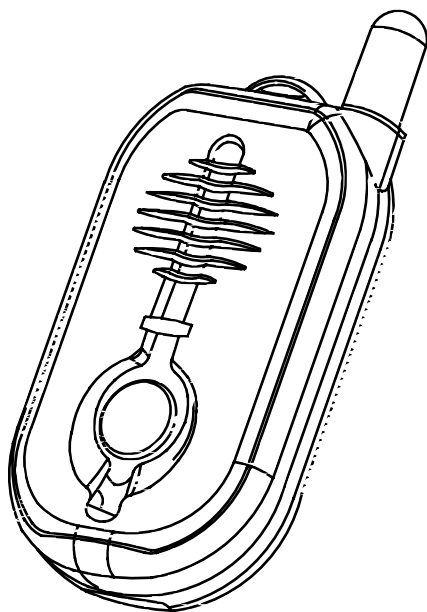


Note: Battery must be installed before use.

Thank you for choosing the Guardian Alert 911 – your personal link to the 911 emergency operator when you need it.


PLEASE READ THIS MANUAL COMPLETELY AND SAVE IT FOR REFERENCE.

Contents:


The items below are included with your Guardian Alert 911 . Additional parts or replacements can be ordered from LogicMark, LLC by calling our Customer Service department at 703-934-7934.

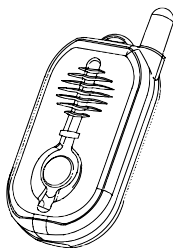


Pendant

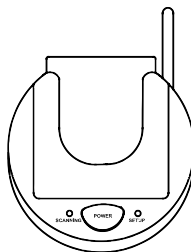


Base Unit

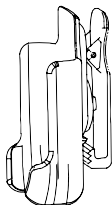

Belt Clip

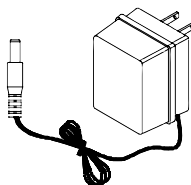

AC Adapter

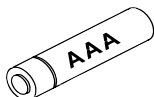


AAA Alkaline Battery




Phone Cord



Screwdriver

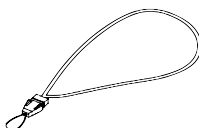

Lanyard

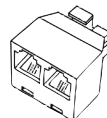

Line Splitter

Optional Accessories:



Battery Backup #30912

TABLE OF CONTENTS

Overview of the Guardian Alert 9114

Installing the Battery4

Locating the Base Unit.....5

Pendant Features5

Base Unit Features6

Connecting the Guardian Alert 911

 The Standard Connection.....6

 Connecting a Second Phone.....7

 If Your Home Has DSL.....7

Operating the Guardian Alert 911

 About the Pendant.....7

 Attaching the Lanyard.....8

 Water Resistance.....8

 Battery Check.....9

 System Check.....10

 Emergency Dialing Using the Panic Button11

 Hanging Up.....11

Troubleshooting Guide12

Overview Of The Guardian Alert 911:

The Guardian Alert 911 is a tiny cordless speaker phone, small enough to be worn around your neck. It is smaller than most cell phones.

By pressing a single button, the Guardian Alert 911 automatically dials the universal 911 emergency number. You will then be able to speak directly to the 911 operator about your emergency.

It has enough range to cover most homes and even into the yard. It is NOT a cell phone, and therefore will not work far from your home. It is designed to be used primarily inside the home. You can use the System Check feature (see page 10) to confirm that the PENDANT is within range of the BASE UNIT.


The Guardian Alert 911 PENDANT has its own replaceable battery. The battery must be installed prior to use. It will last at least 6 to 12 months in standby mode. Because the PENDANT never needs charging, it can be kept with you at all times. We recommend that the PENDANT battery be replaced every six months.

Like most cordless phones, the Guardian Alert 911 must be plugged into a power outlet using an AC adapter (included) to work. An optional battery backup is available which can keep the Guardian Alert 911 BASE UNIT working for up to 24 hours in case of a power failure. It can be ordered from the Customer Service number on the back of this manual. Ask for part number 30912.

Installing the Battery:

The PENDANT requires one AAA battery. We recommend using a fresh, high quality alkaline or lithium cell from a recognized brand. Under normal use, the unit will remain on standby and ready for use for at least six months. When activated during an emergency, you can talk for a minimum of 30 minutes. Follow the diagram to the right to install the battery which came with your kit. Insert positive end of battery into compartment first.


To insure that your Guardian Alert 911 will be ready when you need it, replace the battery every 6 months.

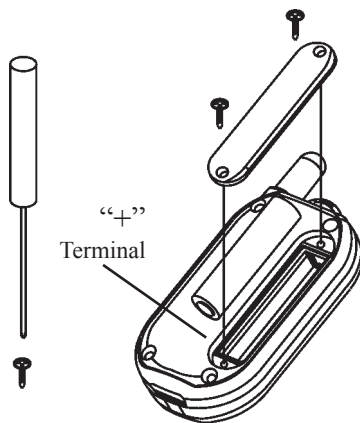


NOTE: The battery cover will only fit easily on the case one direction.

Locating the Base Unit:

For best results and longest range, locate the BASE UNIT in an area away from household appliances which could present electrical interference. These include microwave ovens, televisions, electric mixers or blenders, hairdryers, or other cordless phones.

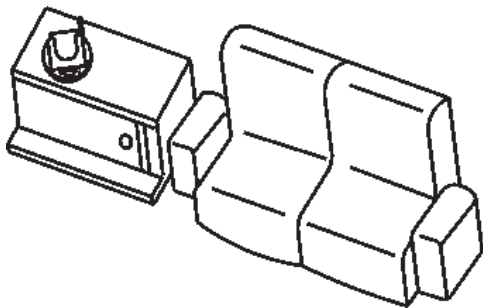


Good Location

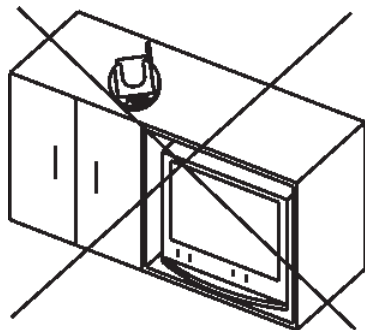


Bad Location

If more than one Guardian Alert 911 is to be used in one home, it is recommended that the BASE UNITS be located at least 10' apart.

Pendant Features:

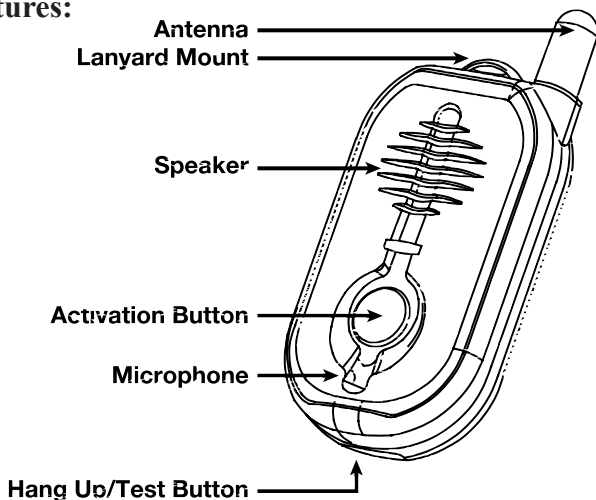


Base Unit Features:

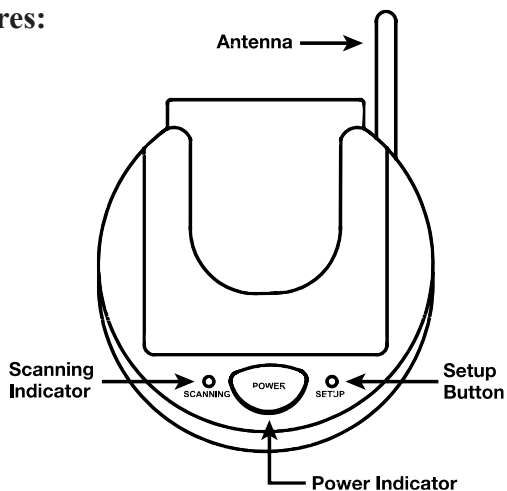

Connecting The Guardian Alert 911:

The Standard Connection:

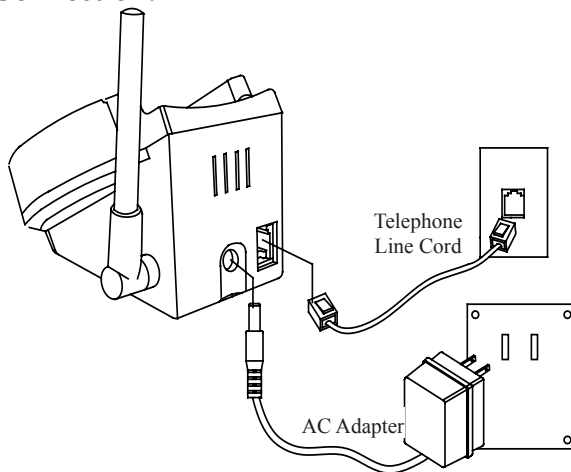


No special phone line or special service is required to use your Guardian Alert 911. Your phone will work with your existing touch-tone phone line and existing service. It works the same way as any cordless phone.

Simply plug in the AC ADAPTER and TELEPHONE LINE CORD as shown.

Caution: Do not plug the AC ADAPTER into an outlet which is controlled by a switch. The switch could accidentally be turned off, thus rendering the BASE UNIT inoperable.


Connecting A Second Phone

You may need to plug a conventional telephone into same outlet used by the Guardian Alert 911. This is done very simply by using the telephone line splitter (included). See illustration on right.

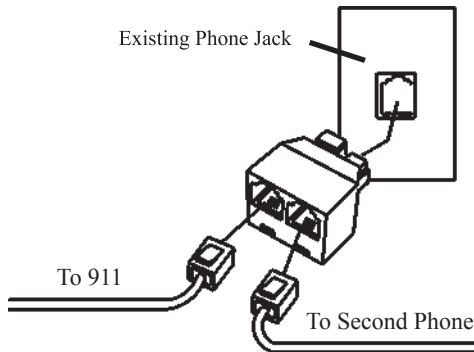

If Your Home Has DSL

If your home has DSL service for your computer, you will need to use a DSL filter in line with the Guardian Alert 911 OR IT WILL NOT WORK. This filter can be obtained from a local electronics supplier or your DSL service provider. See illustration to the right.

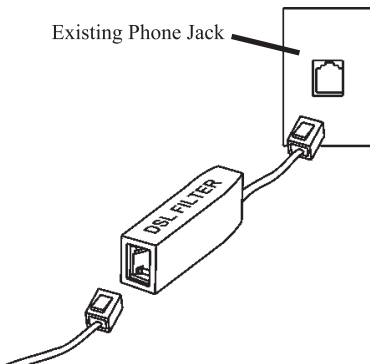

Operating the Guardian Alert 911

About The Pendant:


The PENDANT is a tiny, cordless speaker phone. It may be worn around the neck, on a belt using the supplied clip, or kept in your pocket.

There is a large blue button on the front of the PENDANT which activates the system and automatically dials the 911 emergency number when pressed for more than ½ second.

Once connected, you can speak directly with the 911 operator and explain the nature of your emergency.

Attaching The Lanyard

If the neck strap is to be used, attach it as shown to the right.

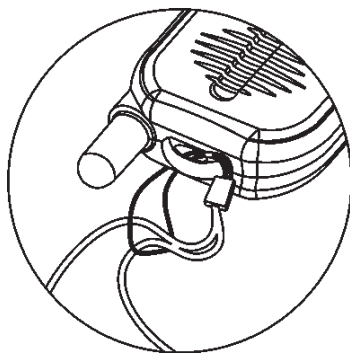


Water Resistance

The Guardian Alert 911 will resist occasional splashes from water. However, it should never be submerged or exposed to water continuously.

Battery Check

The Guardian Alert 911 has a built-in battery tester. When you press the button on the bottom of the PENDANT, a voice announcement will inform you of the battery condition. One of the following messages will be heard:

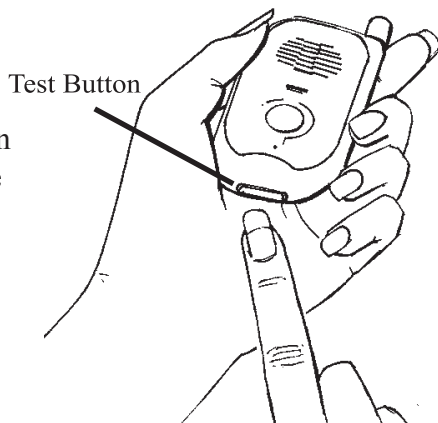

“Battery is OK.”

Check the battery condition once per week.

“Replace battery soon.”

The battery should be replaced within the next few days.

“Replace battery now.”


The battery should be replaced the same day.

Please note: If there is no response after pressing the button for one second or more, the battery is dead and must be replaced immediately.

The battery should be tested twice a month along with the System Check outlined on the next page. If the battery is tested every day, it will be drained much more quickly and will not last as long.

System Check

The button on the bottom of the case which is used to perform the BATTERY CHECK, will also perform a SYSTEM CHECK. Press and hold this button for more than four (4) seconds. One of the following announcements will be made.

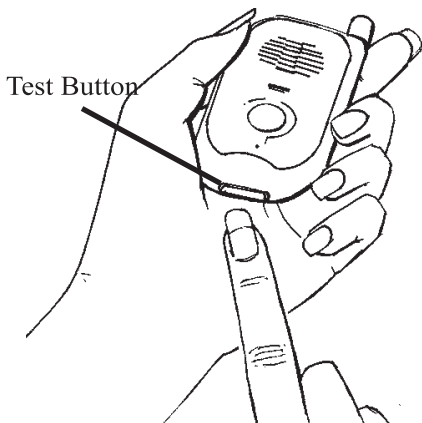

“System check, OK.”

This confirms the following:

Battery status.

The PENDANT is working.

The BASE UNIT is working.

The BASE UNIT is connected to a working phone line.

The PENDANT is within range of the BASE UNIT.

“System check failed.”

This confirms one or more of the following:

The PENDANT has malfunctioned.

The BASE UNIT has malfunctioned.

The PENDANT and BASE UNIT are not communicating with each other or are out of range.

The BASE UNIT is not connected to the phone line or can not get a dial tone for some reason.


If you receive a “System check failed.” notification, please refer to the TROUBLESHOOTING guide at the back of this manual to determine the cause.

This feature is very useful - allowing you to confirm that the system has coverage from all parts of your home. Simply walk to the various parts of your home and push the Battery Check / System Check button until you hear the dial tone and the “SYSTEM CHECK OK” voice prompt.

We highly recommend that you perform this test every 2 to 4 weeks to confirm that your system is working properly and that the pendant and base station are in good communication and that the base station can get a dial tone. It is also recommended to perform this system check after a lightening storm that could damage the base station which is plugged into the phone line.


Emergency Dialing Using the Panic Button

In the event of an emergency, press the blue panic button on the front of the PENDANT. When this button is pressed for more than ½ second, the Guardian Alert 911 will automatically dial 911. When the emergency response operator answers, speak in a normal voice and you will be heard. There is no need to put the PENDANT up to your ear unless you are having trouble hearing. There is no need to hold the PENDANT up to your mouth. You will be clearly heard, even if the PENDANT is an arm's length away.

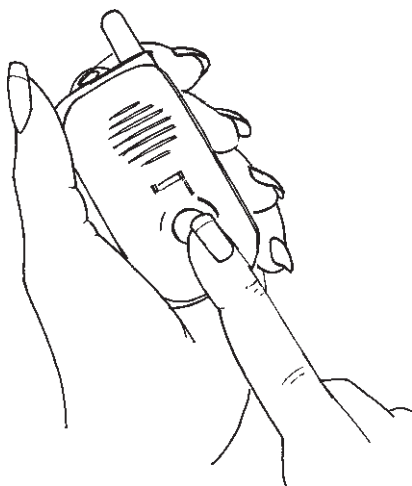

Hanging Up

You can hang up any time by pressing the TEST BUTTON on the bottom of the case.

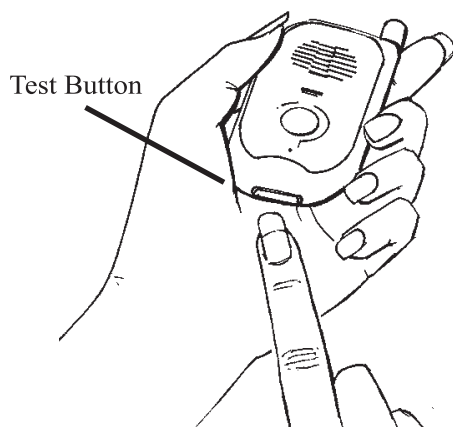

TROUBLESHOOTING GUIDE

What if I can't get the Guardian Alert 911 to work?

Step 1 – Check the PENDANT.

What to do:

Press the BATTERY CHECK/SYSTEM CHECK button on the bottom of the PENDANT.

What to expect:

The a voice should announce the condition of the battery.

A light on the front of the PENDANT will come on.

If nothing happens, the most likely cause is that the battery is dead or has been installed backwards.

If the PENDANT seems to be working, go on to step 2.

Step 2 – Check the connections on the BASE UNIT

What to do:

Make sure the AC ADAPTER is plugged into a live AC outlet and into the BASE UNIT.

What to expect:

The blue POWER light will be on if the BASE UNIT is connected to power.

If the POWER light is on and the BASE UNIT is connected to a working phone line, go on to step 3.

Step 3 – Make sure the PENDANT and the BASE UNIT are talking to each other.

Background information: The PENDANT and the BASE UNIT are connected by a wireless radio link. They must be synchronized to work together. They should have already been synchronized at the factory,

but resetting this is simple.

What to do:

1. Press the “SET UP” button on the BASE UNIT until the red “SCANNING” light comes on. Release the SET UP button.
2. Now press and hold the PANIC BUTTON and the TEST BUTTON buttons (on the pendant) together until the “SCANNING” light on the BASE UNIT turns off.

What to expect:

The red light will go out on the BASE UNIT. This indicates the PENDANT and the BASE UNIT are synchronized. At this point, everything should be working fine.

If your Guardian Alert 911 is still not working, call our TECH SUPPORT department at 703-934-7934.

Information The FCC Wants You To Know:

FCC ID: TYD30911

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Privacy of communications may not be ensured when using this phone.

Operating Range:

The operating range of your Guardian Alert 911 will depend on many factors including intervening walls, electrical interference or various appliances such as vacuum cleaners, microwave ovens, mixers, coffee grinders, hair dryers and other sources of electrical noise around the house.

General Warnings And Precautions!

Warning!

- LogicMark, LLC DOES NOT represent this unit to be completely waterproof. To reduce the risk of damage to the unit, DO NOT expose the pendant or base unit to prolonged water, rain or moisture.
- The Guardian Alert 911 is intended for residential use only. It may not work properly if connected to some commercial phone systems.
- The Guardian Alert 911 works with tone dialing systems only. Do not use if you have pulse dialing only.
- This product and/or accessories may contain lead, a chemical known to the State of California to cause birth defects or other reproductive harm.

Disclaimer:

LogicMark, LLC. will not be responsible for the improper use of the Guardian Alert 911, nor will it be responsible for failure resulting from the use of other equipment connected to the same phone line. We will not be responsible for the quality of the phone line or the reliability or quality of the phone service with which the Guardian Alert 911 is used.

LogicMark, LLC. will not be responsible for the installation of the 911 Guardian Phone. It will not be responsible for the improper use or abuse of this unit.

Important Safety Instructions:

When using your Guardian Alert 911 , please observe the following instructions in order to reduce the risk of personal injury, electrical shock, or fire.

1. Save these instructions for future reference.
2. Read all instructions carefully and make sure you understand them.
3. Unplug this product from the electrical outlet before attempting to clean it. Do not use any liquids for cleaning either the Base Unit or the Pendant. Use only a soft damp cloth.
4. Do not locate the Base Unit near any source of water, such as in the bathroom, near a sink in the kitchen, or the like.
5. Do not open the unit, (except the battery compartment as directed in these instructions) on the Pendant for any reason. Any attempt to open the case of the Base Unit or the Pendant will void the warranty and may damage

the unit or prevent it from working properly.

6. Do not use any power source other than that supplied from LogicMark, LLC specifically for this product. Use of any other power source could damage your unit or make it unsafe.
7. If your unit becomes damaged do not attempt to repair it. Please return it to LogicMark, LLC for service or replacement as stipulated below under “ONE YEAR LIMITED WARRANTY”.

ONE YEAR LIMITED WARRANTY

a. **What Is Covered:** Logic Mark, LLC warrants to the original consumer/purchaser that this product will be free of defects in material and workmanship under normal use and circumstances for a period of one (1) year from the date of original purchase for use. During the term of this limited warranty we will, at our option, repair or replace any part of the product we deem to be defective without charge to you. We may use new or used parts of the same quality, and keep all replaced parts.

b. **How To Get Service:** Return the product, postage prepaid to LogicMark, LLC, 8625 Hampton Way, Fairfax Station, Virginia, 22039, USA. Include a note describing what is wrong with the product. If the product is defective we will repair or replace it and return it to you. If the product is not defective, we will not repair or replace it, but will return it to you.

c. **What Is Not Included:** Repair or replacement of the product is our only duty. This warranty does not include replaceable batteries which you must provide. We make no other express warranty. All implied warranties including any warranty of merchantability of the product or its fitness for any special purpose. are limited in duration to the one-year term of this expressed limited warranty. We do not warrant that the product will help prevent any emergency event. We make no representation or warranty as to the response time or capability of any agency or person who may be notified of an emergency by your use of the product. This warranty does not cover repairs that are needed because of an accident, acts of god, your failure to properly use the product, or if someone other than us attempts to repair or change the product. We are not liable for consequential or incidental damages of any kind. You agree that this is our only warranty and we have given you no other warranty for the product. All implied warranties are limited in duration to the one-year term of this expressed limited warranty.

d. **STATE LAW:** Some states do not allow the exclusion or the limitation of consequential or incidental damages, or a limitation on the duration of implied warranties, so the above limitations or exclusions may not apply to you. The warranty gives you specific legal rights and you may also have other rights which may vary from state to state.

OUT OF WARRANTY REPAIRS

If the warranty period has expired or if you are not the original owner of the product, LogicMark, LLC will at its option either (1) replace this product with a functionally similar (but not necessarily identical) refurbished product or (2) repair the original product and return it to the original consumer/purchaser after payment of repair/replacement charges has been received.

LogicMark, LLC

8625 Hampton Way;
Fairfax Station, Virginia 22039

Tel: 703-934-7934

Tel: 800-519-2419

Fax: 703-934-7935

www.LogicMark.com