



(Discontinued 1) Rogue Style Backrest with Adjustable Height Rigidizer Bar on Rogue ALX - Growth (Formerly Tsunami)

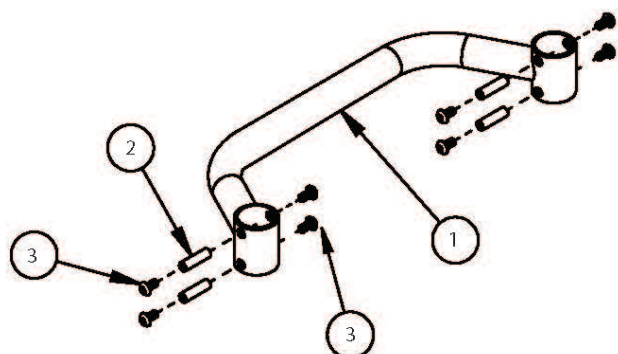

Height Adjustable Backrest w/Adjustable Rigidizer Bar

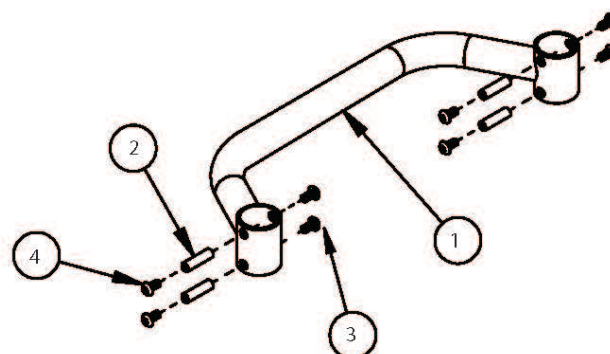

Fixed Height Backrest w/Adjustable Rigidizer Bar

This Rogue Style Backrest is not available with the Tsunami Heavy Duty option

Item Number	Part Number	Description	Quantity
1a	000185	Adjustable Height Rigidizer Bar 14" Wide	1
1b	000186	Adjustable Height Rigidizer Bar 15" Wide	1
1c	000187	Adjustable Height Rigidizer Bar 16" Wide	1
1d	000188	Adjustable Height Rigidizer Bar 17" Wide	1
1e	000189	Adjustable Height Rigidizer Bar 18" Wide	1
1f	000190	Adjustable Height Rigidizer Bar 19" Wide	1
1g	000191	Adjustable Height Rigidizer Bar 20" Wide	1
2	000073	M5 x 24 Threaded Barrel	6
3	100669	M5 x 10 BTN SHCS BLKZC w/Patch <i>Quantity 8 used with Rogue Style Height Adjustable Backrest; Quantity 4 used with Rogue Style Fixed Height Backrest</i>	8
4	103524	M5 x 10 BTN SHCS BLK ZN <i>Only used with Rogue Style Fixed Height Backrest</i>	4