


# Liberty Folding Base Cross Brace - Growth

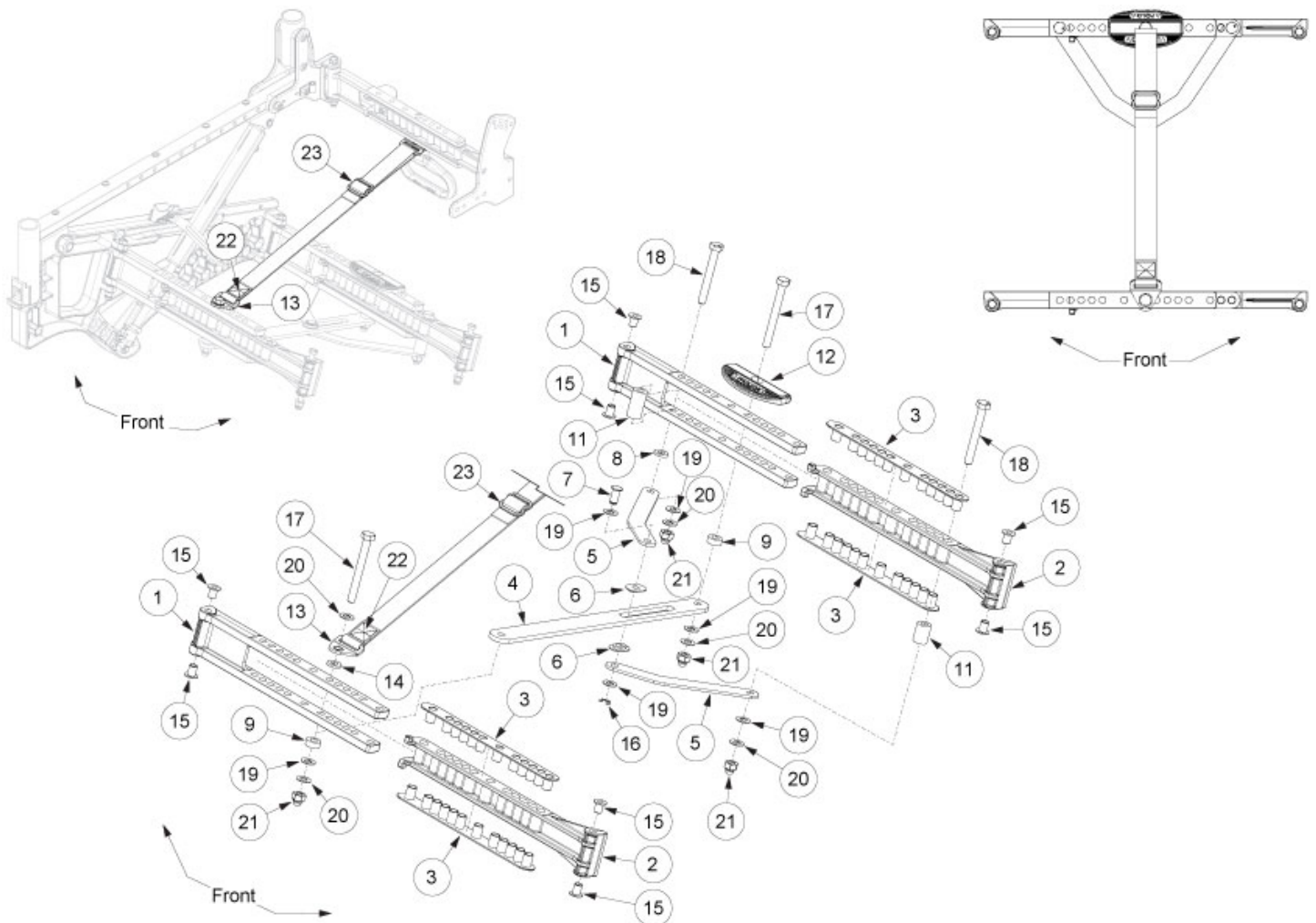

Can you achieve your new width within the existing width range? If no, please choose the appropriate kit below.

Item Number	Part Number	Description	Quantity
1a, 2a, 3-23	110071	Cross Brace Assembly, Base, Small, 14W with Instruction <i>Used on 14 - 15" wide with folding frame</i>	1
1a, 2a, 3-23	110072	Cross Brace Assembly, Base, Small, 16W with Instruction <i>Used on 16 - 18" wide with a small width range and folding frame</i>	1
1b, 2b, 3-23	110073	Cross Brace Assembly, Base, Medium, 16W with Instruction <i>Used on 16 - 17" wide with medium width range and folding frame</i>	1
1b, 2b, 3-23	110074	Cross Brace Assembly, Base, Medium, 18W with Instruction <i>Used on 18 - 20" wide with medium width range and folding frame</i>	1
1c, 2c, 3-23	110075	Cross Brace Assembly, Base, Large, 18W with Instruction	1

Item Number	Part Number	Description	Quantity
		<i>Used on 18 - 19" wide with large width range and folding frame</i>	
1c, 2c, 3-23	110076	Cross Brace Assembly, Base, Large, 20W with Instruction <i>Used on 20 - 22" wide with large width range and folding frame</i>	1
1a, 2a, 3-8, 10, 11, 15, 16, 18-21	110077	Cross Brace Assembly Non-Folding, Base, Small, 14W with Instruction <i>Used on 14 - 15" wide with non-folding frame. Includes non-folding frame hardware.</i>	1
1a, 2a, 3-8, 10, 11, 15, 16, 18-21	110078	Cross Brace Assembly Non-Folding, Base, Small, 16W with Instruction <i>Used on 16 - 18" wide with a small width range and non-folding frame. Includes non-folding frame hardware.</i>	1
1b, 2b, 3-8, 10, 11, 15, 16, 18-21	110079	Cross Brace Assembly Non-Folding, Base, Medium, 16W with Instruction <i>Used on 16 - 17" wide with medium width range and non-folding frame. Includes non-folding frame hardware.</i>	1
1b, 2b, 3-8, 10, 11, 15, 16, 18-21	110080	Cross Brace Assembly Non-Folding, Base, Medium, 18W with Instruction <i>Used on 18 - 20" wide with medium width range and non-folding frame. Includes non-folding frame hardware.</i>	1
1c, 2c, 3-8, 10, 11, 15, 16, 18-21	110081	Cross Brace Assembly Non-Folding, Base, Large, 18W with Instruction <i>Used on 18 - 19" wide with large width range and non-folding frame. Includes non-folding frame hardware.</i>	1
1c, 2c, 3-8, 10, 11, 15, 16, 18-21	110082	Cross Brace Assembly Non-Folding, Base, Large, 20W with Instruction <i>Used on 20 - 22" wide with large width range and non-folding frame. Includes non-folding frame hardware.</i>	1
13, 22, 23	111130	Folding Strap with Mount and Strap Slide, Liberty	1