


# Rogue2 / Rogue XP 20 One Arm Drive Quick Release Aluminum Drive Handrim

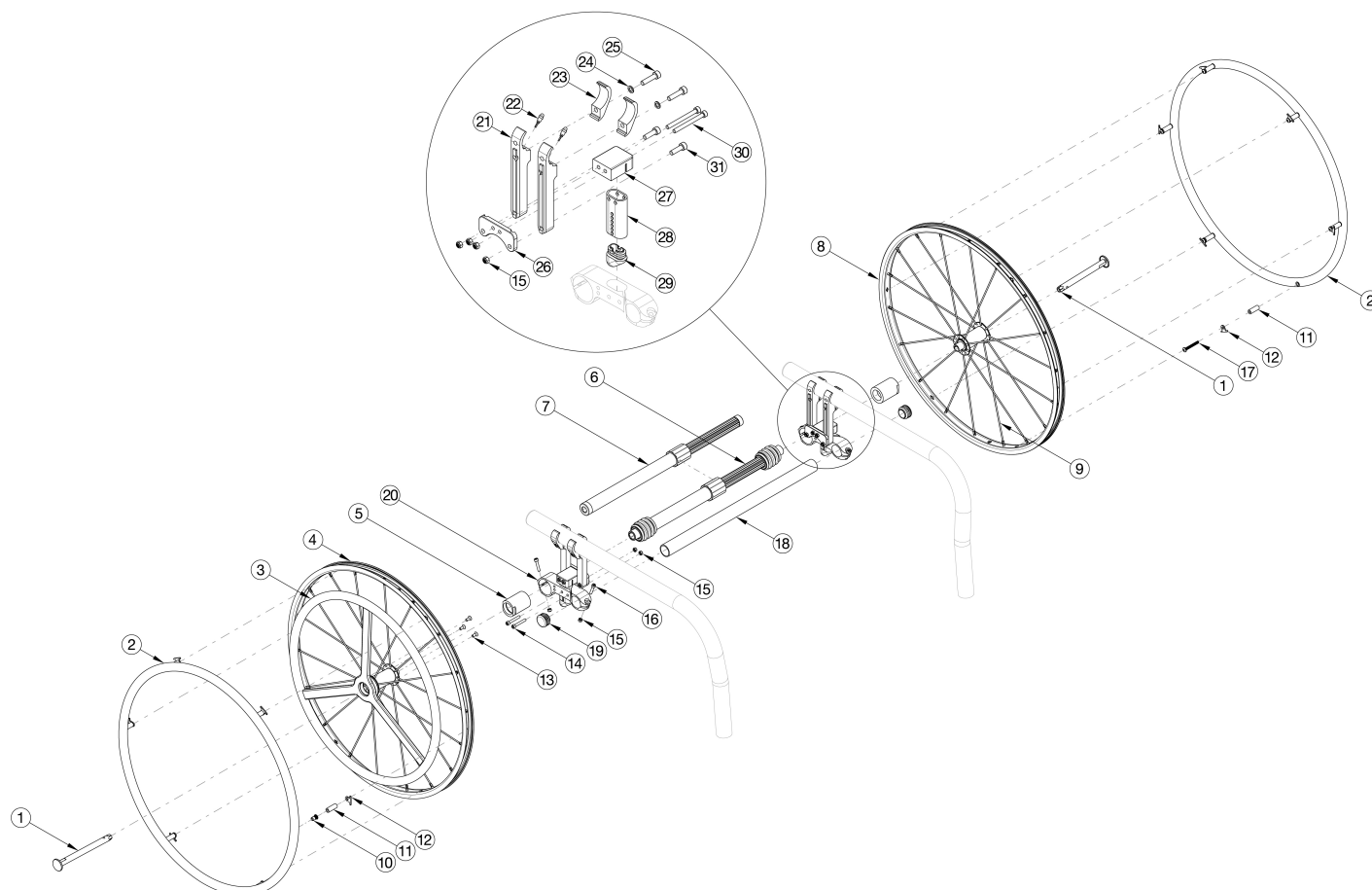

If retrofitting to add the One Arm Drive, new rear towers and kit 124265 are required in addition to the complete One Arm Drive system and camber tube kit. Effective 5/13/2022, all One Arm Drives will be shipped with a close mounted handrim on the non-drive wheel. Most One Arm Drive kits do not include rear tires. Tires need to be quoted separately. Used on Rogue XP chairs invoiced starting on 3/12/25

| Item Number                                | Part Number | Description                                                                                                                                         | Quantity |
|--------------------------------------------|-------------|-----------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| 1, 2, 3, 4, 5a, 7a, 8, 10, 11a, 11b, 12-17 | 112663      | Rogue Quick Release One Arm Drive Retrofit Kit, 0° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 177-255mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5a, 7b, 8, 10, 11a, 11b, 12-17 | 112664      | Rogue Quick Release One Arm Drive Retrofit Kit, 0° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 225-295mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5a, 7c, 8, 10, 11a, 11b, 12-17 | 112665      | Rogue Quick Release One Arm Drive Retrofit Kit, 0° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 260-370mm Drive Shaft | 1        |

| Item Number                                | Part Number | Description                                                                                                                                         | Quantity |
|--------------------------------------------|-------------|-----------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| 1, 2, 3, 4, 5a, 7d, 8, 10, 11a, 11b, 12-17 | 112666      | Rogue Quick Release One Arm Drive Retrofit Kit, 0° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 330-440mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5a, 7e, 8, 10, 11a, 11b, 12-17 | 112667      | Rogue Quick Release One Arm Drive Retrofit Kit, 0° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 421-491mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b, 6a, 8, 10, 11a, 11b, 12-17 | 112668      | Rogue Quick Release One Arm Drive Retrofit Kit, 2° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 170-195mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b, 6b, 8, 10, 11a, 11b, 12-17 | 112669      | Rogue Quick Release One Arm Drive Retrofit Kit, 2° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 180-210mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b, 6c, 8, 10, 11a, 11b, 12-17 | 112670      | Rogue Quick Release One Arm Drive Retrofit Kit, 2° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 195-235mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b, 6d, 8, 10, 11a, 11b, 12-17 | 112671      | Rogue Quick Release One Arm Drive Retrofit Kit, 2° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 220-280mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b, 6e, 8, 10, 11a, 11b, 12-17 | 112672      | Rogue Quick Release One Arm Drive Retrofit Kit, 2° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 270-330mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b, 6f, 8, 10, 11a, 11b, 12-17 | 112673      | Rogue Quick Release One Arm Drive Retrofit Kit, 2° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 295-370mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b, 6g, 8, 10, 11a, 11b, 12-17 | 112674      | Rogue Quick Release One Arm Drive Retrofit Kit, 2° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 350-460mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b, 6h, 8, 10, 11a, 11b, 12-17 | 112675      | Rogue Quick Release One Arm Drive Retrofit Kit, 2° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 420-530mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5c, 6c, 8, 10, 11a, 11b, 12-17 | 112676      | Rogue Quick Release One Arm Drive Retrofit Kit, 4° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 195-235mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5c, 6d, 8, 10, 11a, 11b, 12-17 | 112677      | Rogue Quick Release One Arm Drive Retrofit Kit, 4° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 220-280mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5c, 6e, 8, 10, 11a, 11b, 12-17 | 112678      | Rogue Quick Release One Arm Drive Retrofit Kit, 4° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 270-330mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5c, 6f, 8, 10, 11a, 11b, 12-17 | 112679      | Rogue Quick Release One Arm Drive Retrofit Kit, 4° Camber, 20" Spoke with Aluminum Drive Handrim, Aluminum Non Drive Handrim, 295-370mm Drive Shaft | 1        |

| Item Number                                      | Part Number | Description                                                                                                                                                        | Quantity |
|--------------------------------------------------|-------------|--------------------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| 1, 2, 3, 4, 5c,<br>6g, 8, 10, 11a,<br>11b, 12-17 | 112680      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>4° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Aluminum Non Drive Handrim, 350-<br>460mm Drive Shaft      | 1        |
| 1, 2, 3, 4, 5a,<br>7a, 8, 10, 11a,<br>11b, 12-17 | 121425      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>0° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>177-255mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5a,<br>7b, 8, 10, 11a,<br>11b, 12-17 | 121428      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>0° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>225-295mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5a,<br>7c, 8, 10, 11a,<br>11b, 12-17 | 121429      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>0° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>260-370mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5a,<br>7d, 8, 10, 11a,<br>11b, 12-17 | 121431      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>0° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>330-440mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5a,<br>7e, 8, 10, 11a,<br>11b, 12-17 | 121432      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>0° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>421-491mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b,<br>6a, 8, 10, 11a,<br>11b, 12-17 | 121433      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>2° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>170-195mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b,<br>6b, 8, 10, 11a,<br>11b, 12-17 | 121434      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>2° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>180-210mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b,<br>6c, 8, 10, 11a,<br>11b, 12-17 | 121435      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>2° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>195-235mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b,<br>6d, 8, 10, 11a,<br>11b, 12-17 | 121436      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>2° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>220-280mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b,<br>6e, 8, 10, 11a,<br>11b, 12-17 | 121437      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>2° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>270-330mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b,<br>6f, 8, 10, 11a,<br>11b, 12-17 | 121438      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>2° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>295-370mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b,<br>6g, 8, 10, 11a,<br>11b, 12-17 | 121439      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>2° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>350-460mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5b,<br>6h, 8, 10, 11a,<br>11b, 12-17 | 121440      | Rogue Quick Release One Arm Drive Retrofit Kit,<br>2° Camber, 20" Spoke with Aluminum Drive<br>Handrim, Plastic Coated Non Drive Handrim,<br>420-530mm Drive Shaft | 1        |

| Item Number                                | Part Number | Description                                                                                                                                               | Quantity |
|--------------------------------------------|-------------|-----------------------------------------------------------------------------------------------------------------------------------------------------------|----------|
| 1, 2, 3, 4, 5c, 6c, 8, 10, 11a, 11b, 12-17 | 121441      | Rogue Quick Release One Arm Drive Retrofit Kit, 4° Camber, 20" Spoke with Aluminum Drive Handrim, Plastic Coated Non Drive Handrim, 195-235mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5c, 6d, 8, 10, 11a, 11b, 12-17 | 121442      | Rogue Quick Release One Arm Drive Retrofit Kit, 4° Camber, 20" Spoke with Aluminum Drive Handrim, Plastic Coated Non Drive Handrim, 220-280mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5c, 6e, 8, 10, 11a, 11b, 12-17 | 121443      | Rogue Quick Release One Arm Drive Retrofit Kit, 4° Camber, 20" Spoke with Aluminum Drive Handrim, Plastic Coated Non Drive Handrim, 270-330mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5c, 6f, 8, 10, 11a, 11b, 12-17 | 121444      | Rogue Quick Release One Arm Drive Retrofit Kit, 4° Camber, 20" Spoke with Aluminum Drive Handrim, Plastic Coated Non Drive Handrim, 295-370mm Drive Shaft | 1        |
| 1, 2, 3, 4, 5c, 6g, 8, 10, 11a, 11b, 12-17 | 121445      | Rogue Quick Release One Arm Drive Retrofit Kit, 4° Camber, 20" Spoke with Aluminum Drive Handrim, Plastic Coated Non Drive Handrim, 350-460mm Drive Shaft | 1        |
| 1, 2, 8, 11b, 12                           | 119587      | Quick Release One Arm Drive, 20" Spoke Wheel with Axle (Driven Side) and Close Mounted Aluminum Handrim                                                   | 1        |
| 1, 3, 4, 13                                | 112899      | Quick Release One Arm Drive Drive, 20" Spoke Wheel with Axle and Aluminum Drive Handrim                                                                   | 1        |
| 10, 11a, 12                                | 502168      | Handrim Hardware Kit, 6 Rivet Nut, EACH                                                                                                                   | 1        |
| 2a, 11b, 12, 17                            | 116447      | Aluminum Anodized Handrim 6 Rivet Nut 20" with Close Mount Hardware, Non-Drive Side                                                                       | 1        |
| 2b, 11b, 12, 17                            | 121850      | Plastic Coated Handrim 6 Rivet Nut 20" with Close Mount Hardware, Non-Drive Side                                                                          | 1        |
| 15, 23a, 24-31                             | 124265      | Camber Tower Clamp Black Anodized w/ Hardware Assembly, One Side Rogue2 / Rogue XP                                                                        | 1        |
| 15, 23b, 24-31                             | 124265_ _ _ | Color Anodized/Color Accent Camber Tower Clamp w/ Hardware Assembly, One Side Rogue2 / Rogue XP                                                           | 1        |
| 2a                                         | 200536      | Handrim Aluminum Anodized 6 Rivet Nut 20"<br><i>Does not include mounting hardware</i>                                                                    | 1        |
| 2b                                         | 112819      | Handrim Plastic Coated 6 Rivet Nut 20"<br><i>Does not include mounting hardware</i>                                                                       | 1        |
| 3                                          | 111830      | Quick Release One Arm Drive, Aluminum Drive Side Handrim for 20" Wheel<br><i>Does not include mounting hardware</i>                                       | 1        |
| 3, 32                                      | 118246      | Quick Release One Arm Drive, Aluminum Drive Side Handrim for 20" Wheel with Micro Grip<br><i>Does not include mounting hardware</i>                       | 1        |
| 4                                          | 111484      | Quick Release One Arm Drive Drive, 20" Spoke Wheel with Axle (Drive Side)                                                                                 | 0        |
| 5a                                         | 107031      | One Arm Drive Quick Release Camber Plug w/Bearings - 0° Camber                                                                                            | 2        |
| 5b                                         | 003170      | One Arm Drive Quick Release Camber Plug w/Bearings - 2° Camber                                                                                            | 2        |

| Item Number | Part Number | Description                                                                                         | Quantity |
|-------------|-------------|-----------------------------------------------------------------------------------------------------|----------|
| 5c          | 003171      | One Arm Drive Quick Release Camber Plug w/Bearings - 4° Camber                                      | 2        |
| 6a          | 111858      | One Arm Drive - Quick Release Camber Drive Shaft 170-195mm                                          | 1        |
| 6b          | 111627      | One Arm Drive - Quick Release Camber Drive Shaft 180-210mm                                          | 1        |
| 6c          | 111628      | One Arm Drive - Quick Release Camber Drive Shaft 195-235mm                                          | 1        |
| 6d          | 111498      | One Arm Drive - Quick Release Camber Drive Shaft 220-280mm                                          | 1        |
| 6e          | 111499      | One Arm Drive - Quick Release Camber Drive Shaft 270-330mm                                          | 1        |
| 6f          | 111500      | One Arm Drive - Quick Release Camber Drive Shaft 295-370mm                                          | 1        |
| 6g          | 111501      | One Arm Drive - Quick Release Camber Drive Shaft 350-460mm                                          | 1        |
| 6h          | 111502      | One Arm Drive - Quick Release Camber Drive Shaft 420-530mm                                          | 1        |
| 7a          | 111503      | One Arm Drive - Quick Release Zero Camber Drive Shaft 177-255mm                                     | 1        |
| 7b          | 111504      | One Arm Drive - Quick Release Zero Camber Drive Shaft 225-295mm                                     | 1        |
| 7c          | 111505      | One Arm Drive - Quick Release Zero Camber Drive Shaft 260-370mm                                     | 1        |
| 7d          | 111506      | One Arm Drive - Quick Release Zero Camber Drive Shaft 330-440mm                                     | 1        |
| 7e          | 111507      | One Arm Drive - Quick Release Zero Camber Drive Shaft 421-491mm                                     | 1        |
| 8           | 111508      | Quick Release One Arm Drive, 20" Spoke Wheel with Axle (Driven Side)                                | 1        |
| 9           | 111513      | Replacement Spoke, Quick Release One Arm Drive, 20"                                                 | 1        |
| 10          | 122063      | PHMS, M5x0.8x40, CLS4.8, PHL, BLZ, NYL, DIN7985, Custom<br><i>Used with Standard Mount handrims</i> | 6        |
| 11a         | 100653      | Handrim Spacer Spoke<br><i>Used with Close Mount handrims</i>                                       | 6        |
| 11b         | 100792      | Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer                                          | 6        |
| 12          | 100698      | Handrim Tab                                                                                         | 6        |
| 13          | 103522      | M5 x 0.8 x 12 FHCS CLS 10.9 Nyl Blk Zn DIN7991                                                      | 3        |
| 14          | 101734      | M5 x 35 SHCS BLK ZC                                                                                 | 2        |
| 15          | 100657      | M5 Nylock Nut Blk Zn                                                                                | 4        |
| 16          | 101717      | M5 x 25 SHCS BLK ZC                                                                                 | 2        |
| 17          | 122065      | BHCS, M5x0.8x25, CLS12.9, BLZ, NYL, ISO7380, Custom                                                 | 1        |
| 18a         | 003369      | One Arm Drive Camber Tube 11" Kit, Klik / Rogue XP                                                  | 1        |

| Item Number | Part Number | Description                                                                                             | Quantity |
|-------------|-------------|---------------------------------------------------------------------------------------------------------|----------|
|             |             | <i>Does not include tube end plugs</i>                                                                  |          |
| 18b         | 003370      | One Arm Drive Camber Tube 12" Kit, Klik /<br>Rogue / Rogue XP<br><i>Does not include tube end plugs</i> | 1        |
| 18c         | 003371      | One Arm Drive Camber Tube 13" Kit, Klik /<br>Rogue / Rogue XP<br><i>Does not include tube end plugs</i> | 1        |
| 18d         | 003372      | One Arm Drive Camber Tube 14" Kit, Klik /<br>Rogue / Rogue XP<br><i>Does not include tube end plugs</i> | 1        |
| 18e         | 003373      | One Arm Drive Camber Tube 15" Kit, Klik /<br>Rogue / Rogue XP<br><i>Does not include tube end plugs</i> | 1        |
| 18f         | 003374      | One Arm Drive Camber Tube 16" Kit, Klik /<br>Rogue / Rogue XP<br><i>Does not include tube end plugs</i> | 1        |
| 18d         | 003708      | One Arm Drive Camber Tube 17" Kit, Klik /<br>Rogue / Rogue XP<br><i>Does not include tube end plugs</i> | 1        |
| 18e         | 003709      | One Arm Drive Camber Tube 18" Kit, Klik /<br>Rogue / Rogue XP<br><i>Does not include tube end plugs</i> | 1        |
| 18f         | 116857      | One Arm Drive Camber Tube 19" Kit, Rogue<br><i>Does not include tube end plugs</i>                      | 1        |
| 18g         | 116858      | One Arm Drive Camber Tube 20" Kit, Rogue<br><i>Does not include tube end plugs</i>                      | 1        |
| 19          | 100532      | Tube End Plug 1 1/8"                                                                                    | 1        |
| 20          | 108074      | One Arm Drive Rogue/Klik Axle Mount 0 Degree<br>Camber PAIR                                             | 4        |
| 21a         | 114004      | Rear Tower Inner Index Clamp 2.125", Black,<br>Right                                                    | 2        |
| 21b         | 114004_ _ _ | Rear Tower Inner Index Clamp 2.125",<br>Anodized/Color Accent, Right                                    | 2        |
| 21c         | 114005      | Rear Tower Inner Index Clamp 2.875", Black,<br>Right                                                    | 2        |
| 21d         | 114005_ _ _ | Rear Tower Inner Index Clamp 2.875",<br>Anodized/Color Accent, Right                                    | 2        |
| 21e         | 114006      | Rear Tower Inner Index Clamp 3.625, Black,<br>Right                                                     | 2        |
| 21f         | 114006_ _ _ | Rear Tower Inner Index Clamp 3.625,<br>Anodized/Color Accent, Right                                     | 2        |
| 21g         | 114007      | Rear Tower Inner Index Clamp 4.500", Black,<br>Right                                                    | 2        |
| 21h         | 114007_ _ _ | Rear Tower Inner Index Clamp 4.500",<br>Anodized/Color Accent, Right                                    | 2        |
| 21i         | 114008      | Rear Tower Inner Index Clamp 5.375", Black,<br>Right                                                    | 2        |
| 21j         | 114008_ _ _ | Rear Tower Inner Index Clamp 5.375",<br>Anodized/Color Accent, Right                                    | 2        |
| 21k         | 114009      | Rear Tower Inner Index Clamp 6.250", Black,<br>Right                                                    | 2        |

| Item Number | Part Number | Description                                                          | Quantity |
|-------------|-------------|----------------------------------------------------------------------|----------|
| 21l         | 114009_ _ _ | Rear Tower Inner Index Clamp 6.250",<br>Anodized/Color Accent, Right | 2        |
| 21m         | 114034      | Rear Tower Inner Index Clamp 2.125", Black, Left                     | 2        |
| 21n         | 114034_ _ _ | Rear Tower Inner Index Clamp 2.125",<br>Anodized/Color Accent, Left  | 2        |
| 21o         | 114035      | Rear Tower Inner Index Clamp 2.875", Black, Left                     | 2        |
| 21p         | 114035_ _ _ | Rear Tower Inner Index Clamp 2.875",<br>Anodized/Color Accent, Left  | 2        |
| 21q         | 114036      | Rear Tower Inner Index Clamp 3.625, Black, Left                      | 2        |
| 21r         | 114036_ _ _ | Rear Tower Inner Index Clamp 3.625,<br>Anodized/Color Accent, Left   | 2        |
| 21s         | 114037      | Rear Tower Inner Index Clamp 4.500", Black, Left                     | 2        |
| 21t         | 114037_ _ _ | Rear Tower Inner Index Clamp 4.500",<br>Anodized/Color Accent, Left  | 2        |
| 21u         | 114038      | Rear Tower Inner Index Clamp 5.375", Black, Left                     | 2        |
| 21v         | 114038_ _ _ | Rear Tower Inner Index Clamp 5.375",<br>Anodized/Color Accent, Left  | 2        |
| 21w         | 114039      | Rear Tower Inner Index Clamp 6.250", Black, Left                     | 2        |
| 21x         | 114039_ _ _ | Rear Tower Inner Index Clamp 6.250",<br>Anodized/Color Accent, Left  | 2        |
| 22          | 001788      | Button Assembly                                                      | 2        |
| 23          | 114003      | Rear Tower Outer Clamp, Black                                        | 4        |
| 24          | 102326      | Flat Washer ID .254" x OD .396" Thickness .047"<br>Zinc              | 4        |
| 25          | 100780      | M6 x 25 SHCS ZC                                                      | 4        |
| 27          | 119699      | Tower Spacer                                                         | 2        |
| 28          | 000923      | Camber Mount Tower Short Black Anodized Rigid                        | 2        |
| 29          | 000920      | Camber Mount Cap Rigid                                               | 2        |
| 31          | 101180      | M6 x 20 SHCS BLK ZC                                                  | 4        |

| Rogue2 and Rogue XP One Arm Drive Drive Shaft Utilization Chart |        |            |                        |                                                                          |
|-----------------------------------------------------------------|--------|------------|------------------------|--------------------------------------------------------------------------|
| Chair Width                                                     | Camber | Wheel Size | Wheel Spacing          | Drive Shaft                                                              |
| 11                                                              | 0      | 20" - 24"  | 1", 1.25", 1.5", 1.75" | 111503 - One Arm Drive - Quick Release Zero Camber Drive Shaft 177-255mm |
|                                                                 | 2      | 20" - 24"  | 1.25", 1.5", 1.75"     | 111498 - One Arm Drive - Quick Release Camber Drive Shaft 220-280mm      |
|                                                                 | 4      | 20"        | 0.75", 1", 1.25", 1.5" | 111498 - One Arm Drive - Quick Release Camber Drive Shaft 220-280mm      |
|                                                                 |        | 22" - 24"  | 0.75", 1", 1.25"       | 111498 - One Arm Drive - Quick Release Camber Drive Shaft 220-280mm      |
|                                                                 |        |            | 1.5"                   | 111499 - One Arm Drive - Quick Release Camber Drive Shaft 270-330mm      |
| 12                                                              | 0      | 20" - 24"  | 1"                     | 111503 - One Arm Drive - Quick Release Zero Camber Drive Shaft 177-255mm |
|                                                                 |        |            | 1.25", 1.5", 1.75"     | 111504 - One Arm Drive - Quick Release Zero Camber Drive Shaft 225-295mm |
|                                                                 | 2      | 20" - 24"  | 1.25"                  | 111498 - One Arm Drive - Quick Release Camber Drive Shaft 220-280mm      |
|                                                                 |        |            | 1.5", 1.75"            | 111499 - One Arm Drive - Quick Release Camber Drive Shaft 270-330mm      |
|                                                                 | 4      | 20"        | 0.75", 1"              | 111498 - One Arm Drive - Quick Release Camber Drive Shaft 220-280mm      |
| 13                                                              | 0      | 20" - 24"  | 1", 1.25"              | 111504 - One Arm Drive - Quick Release Zero Camber Drive Shaft 225-295mm |
|                                                                 |        | 20" - 24"  | 1.5", 1.75"            | 111505 - One Arm Drive - Quick Release Zero Camber Drive Shaft 260-370mm |
|                                                                 | 2      | 20" - 24"  | 1.25", 1.5", 1.75"     | 111499 - One Arm Drive - Quick Release Camber Drive Shaft 270-330mm      |
|                                                                 | 4      | 20"        | 0.75", 1", 1.25", 1.5" | 111499 - One Arm Drive - Quick Release Camber Drive Shaft 270-330mm      |
|                                                                 |        | 22" - 24"  | 0.75", 1", 1.25"       | 111499 - One Arm Drive - Quick Release Camber Drive Shaft 270-330mm      |
| 14                                                              | 0      | 20" - 24"  | 1", 1.25", 1.5", 1.75" | 111505 - One Arm Drive - Quick Release Zero Camber Drive Shaft 260-370mm |
|                                                                 | 2      | 20" - 24"  | 1.25"                  | 111499 - One Arm Drive - Quick Release Camber Drive Shaft 270-330mm      |
|                                                                 |        |            | 1.5", 1.75"            | 111500 - One Arm Drive - Quick Release Camber Drive Shaft 295-370mm      |
|                                                                 | 4      | 20"        | 0.75", 1"              | 111499 - One Arm Drive - Quick Release Camber Drive Shaft 270-330mm      |
|                                                                 |        | 22" - 24"  | 1.25", 1.5"            | 111500 - One Arm Drive - Quick Release Camber Drive Shaft 295-370mm      |
| 15                                                              | 0      | 20" - 24"  | 1", 1.25", 1.5", 1.75" | 111505 - One Arm Drive - Quick Release Zero Camber Drive Shaft 260-370mm |
|                                                                 |        |            | 1.25", 1.5"            | 111500 - One Arm Drive - Quick Release Camber Drive Shaft 295-370mm      |
|                                                                 | 2      | 20" - 24"  | 1.75"                  | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
|                                                                 | 4      | 20"        | 0.75", 1", 1.25"       | 111500 - One Arm Drive - Quick Release Camber Drive Shaft 295-370mm      |
|                                                                 |        | 22" - 24"  | 1.5"                   | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
| 16                                                              | 0      | 20" - 24"  | 0.75", 1"              | 111500 - One Arm Drive - Quick Release Camber Drive Shaft 295-370mm      |
|                                                                 |        |            | 1", 1.25", 1.5"        | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
|                                                                 | 2      | 20" - 24"  | 1.25", 1.5", 1.75"     | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
|                                                                 | 4      | 20"        | 0.75"                  | 111500 - One Arm Drive - Quick Release Camber Drive Shaft 295-370mm      |
|                                                                 |        | 22" - 24"  | 1", 1.25", 1.5"        | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
| 17                                                              | 0      | 20" - 24"  | 1", 1.25", 1.5", 1.75" | 111506 - One Arm Drive - Quick Release Zero Camber Drive Shaft 330-440mm |
|                                                                 | 2      | 20" - 24"  | 1.25", 1.5", 1.75"     | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
|                                                                 | 4      | 20" - 24"  | 0.75", 1", 1.25", 1.5" | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
| 18                                                              | 0      | 20" - 24"  | 1", 1.25", 1.5", 1.75" | 111506 - One Arm Drive - Quick Release Zero Camber Drive Shaft 330-440mm |
|                                                                 | 2      | 20" - 24"  | 1.25", 1.5", 1.75"     | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
|                                                                 | 4      | 20"        | 0.75", 1", 1.25", 1.5" | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
|                                                                 |        | 22" - 24"  | 0.75", 1", 1.25"       | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
| 19                                                              | 0      | 20" - 24"  | 1"                     | 111506 - One Arm Drive - Quick Release Zero Camber Drive Shaft 330-440mm |
|                                                                 |        |            | 1.5", 1.75"            | 111507 - One Arm Drive - Quick Release Zero Camber Drive Shaft 421-491mm |
|                                                                 | 2      | 20" - 24"  | 1.25"                  | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
|                                                                 |        |            | 1.5", 1.75"            | 111502 - One Arm Drive - Quick Release Camber Drive Shaft 420-530mm      |
|                                                                 | 4      | 20"        | 0.75", 1"              | 111501 - One Arm Drive - Quick Release Camber Drive Shaft 350-460mm      |
| 20                                                              | 0      | 20" - 24"  | 1", 1.25", 1.5", 1.75" | 111507 - One Arm Drive - Quick Release Zero Camber Drive Shaft 421-491mm |
|                                                                 | 2      | 20" - 24"  | 1.25", 1.5", 1.75"     | 111502 - One Arm Drive - Quick Release Camber Drive Shaft 420-530mm      |
|                                                                 | 4      | 20" - 24"  | 0.75", 1", 1.25", 1.5" | 111502 - One Arm Drive - Quick Release Camber Drive Shaft 420-530mm      |