


Catalyst Stroller Handle Back Post (Depth Adjustable)

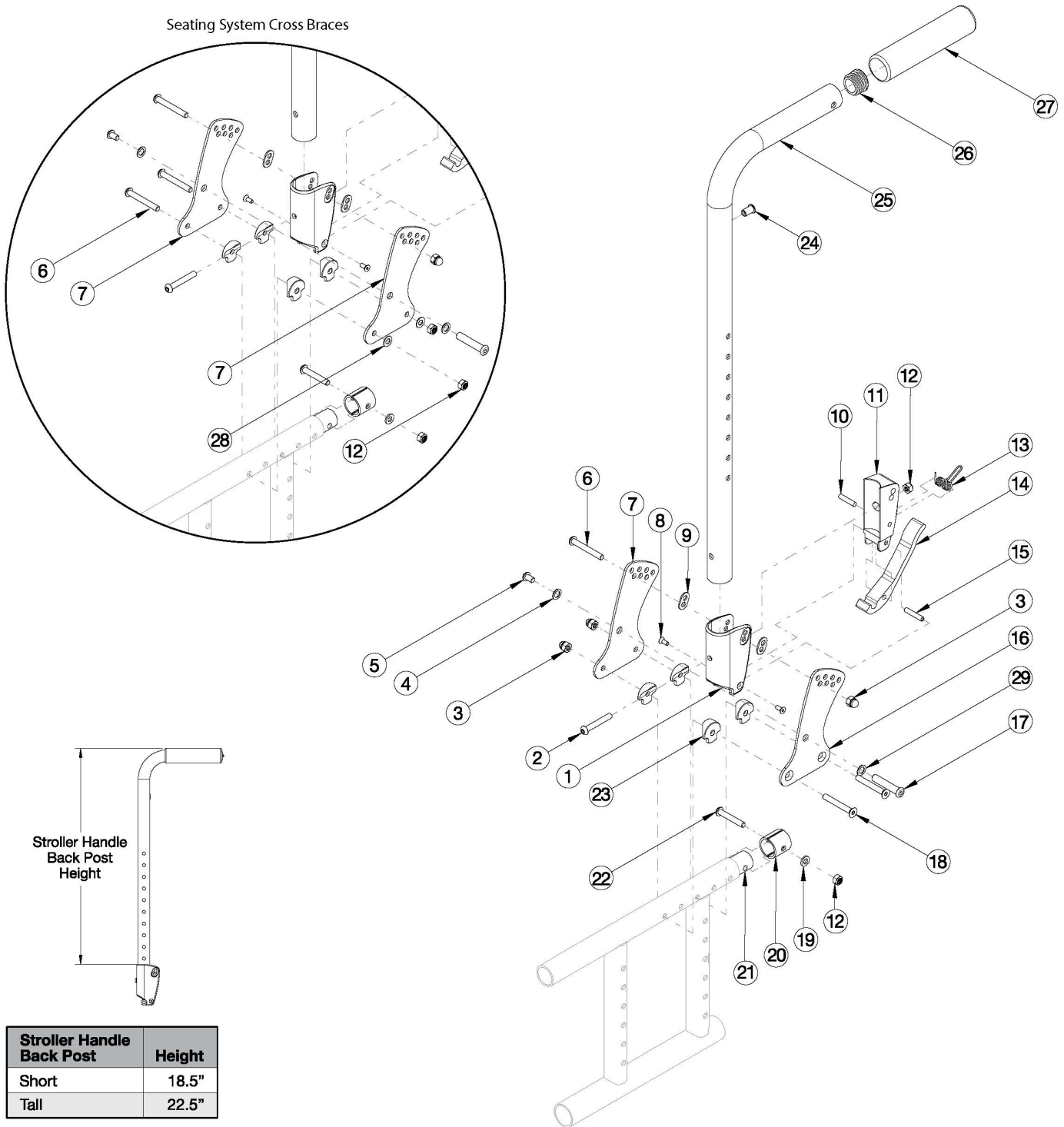

This backrest style was released February 8th, 2021 <p>

If changing from another style backrest and the Drum Brake is present, a new Drum Brake Cable Kit may be required

Item Number	Part Number	Description	Quantity	Retail
1-15, 16a, 17-20, 22-24, 25a, 26, 27, 29	112597	Stroller Handle Back Post with Grip Assembly with All Hardware, for Std Cross Braces, Short, Left Side <i>If chair was invoiced with backrest code 50BOP032, new backcanes are not required to retrofit the drum brake. If chair was invoiced with backrest code 50BOP010, a complete backrest retrofit to this Depth Adjustable Back design is required in order to retrofit the drum brake.</i>	1	\$177.10
1-15, 16b, 17-20, 22-24, 25a, 26, 27, 29	112598	Stroller Handle Back Post with Grip Assembly with All Hardware, for Std Cross Braces, Short, Right Side <i>If chair was invoiced with backrest code 50BOP032, new backcanes are not required to retrofit the drum brake. If chair was invoiced with backrest code 50BOP010, a complete backrest retrofit to this Depth Adjustable Back design is required in order to retrofit the drum brake.</i>	1	\$177.10
1-15, 16a, 17-20, 22-24, 25b, 26, 27, 29	112599	Stroller Handle Back Post with Grip Assembly with All Hardware, for Std Cross Braces, Tall, Left Side <i>If chair was invoiced with backrest code 50BOP032, new backcanes are not required to retrofit the drum brake. If chair was invoiced with backrest code 50BOP010, a complete backrest retrofit to this Depth Adjustable Back design is required in order to retrofit the drum brake.</i>	1	\$177.10
1-15, 16b, 17-20, 22-24, 25b, 26, 27, 29	112600	Stroller Handle Back Post with Grip Assembly with All Hardware, for Std Cross Braces, Tall, Right Side <i>If chair was invoiced with backrest code 50BOP032, new backcanes are not required to retrofit the drum brake. If chair was invoiced with backrest code 50BOP010, a complete backrest retrofit to this Depth Adjustable Back design is required in order to retrofit the drum brake.</i>	1	\$177.10
1-15, 17-20, 22-24, 25a, 26, 27, 28	112601	Stroller Handle Back Post with Grip Assembly with All Hardware, for Seating Cross Braces, Short <i>If chair was invoiced with backrest code 50BOP032, new backcanes are not required to retrofit the drum brake. If chair was invoiced with backrest code 50BOP010, a complete backrest retrofit to this Depth Adjustable Back design is required in order to retrofit the drum brake.</i>	1	\$181.84
1-15, 17-20, 22-24, 25b, 26, 27, 28	112602	Stroller Handle Back Post with Grip Assembly with All Hardware, for Seating Cross Braces, Tall <i>If chair was invoiced with backrest code 50BOP032, new backcanes are not required to retrofit the drum brake. If chair was invoiced with backrest code 50BOP010, a complete backrest retrofit to this Depth Adjustable Back design is required in order to retrofit the drum brake.</i>	1	\$181.84
24, 25a, 26, 27	506811	Stroller Handle Back Post with Grip Assembly, Short <i>If chair was invoiced with backrest code 50BOP032, new backcanes are not required to retrofit the drum brake. If chair was invoiced with backrest code 50BOP010, a complete backrest retrofit to this Depth Adjustable Back design is required in order to retrofit the drum brake.</i>	1	\$49.67
24, 25b, 26, 27	506812	Stroller Handle Back Post with Grip Assembly, Tall <i>If chair was invoiced with backrest code 50BOP032, new backcanes are not required to retrofit the drum brake. If chair was invoiced with backrest code 50BOP010, a complete backrest retrofit to this Depth Adjustable Back design is required in order to retrofit the drum brake.</i>	1	\$49.67
1-15, 16a, 17-20, 22-23, 29	112603	Depth Adjustable Backrest Hardware Assembly, for Std Cross Braces, Left Side	1	\$133.60
1-15, 16b, 17-20, 22-23, 29	112604	Depth Adjustable Backrest Hardware Assembly, for Std Cross Braces, Right Side	1	\$133.60

Item Number	Part Number	Description	Quantity	Retail
1-15, 17-20, 22-23, 28	112605	Depth Adjustable Backrest Hardware Assembly, for Seating Cross Braces, One Side	1	\$138.33
1-15, 17-20, 22-23, 28	112606	Depth Adjustable Backrest Hardware Assembly, for Seating Cross Braces, PAIR	1	\$276.66
1, 9	503834	Cover Plate, Latch Insert Kit	1	\$25.33
2	100797	M6 x 40 BHCS BLK ZC	1	\$1.05
3	100558	M6 NYL INS Domed Cap L/N ZC	3	\$1.05
4	101701	Washer 3.315/.350 x .425/.525 x .059/.067 18-8 F/W	1	\$2.63
5	101184	M6 x 10 BTN SHCS BLKZC w/Patch	1	\$1.58
6	100670	M6 x 45 BTN SHCS BLK ZC	1	\$1.58
7	111833	Depth Adjustable Back Plate, Folding <i>Use quantity 1 per side if standard cross braces are present. Use quantity 2 per side if seating system cross braces are present.</i>	1	\$23.21
8	101818	M4 x 10 FHCS BLKZC w/Patch	2	\$1.05
9	001897	Washer, Dual, PSA	2	\$2.09
10	102694	Dowel Pin, M5 X 24, Blk Zn	1	\$1.58
11	001895	Latch Insert, Folding Backrest	1	\$6.60
12	100658	M6 Nylock Nut Blk Zn <i>Use quantity 2 per side if standard cross braces are present. Use quantity 4 per side if seating system cross braces are present.</i>	2	\$1.05
13	001896	Spring, Backrest Latch	1	\$0.53
14	001893	Latch Lever, Backrest	1	\$13.34
15	002513	Threaded Barrel, 5mm OD x M4	1	\$1.94
16a	111835	Depth Adjustable Back Plate, Inner, Folding, Left <i>Only used when standard cross braces are present</i>	1	\$23.21
16b	111834	Depth Adjustable Back Plate, Inner, Folding, Right <i>Only used when standard cross braces are present</i>	1	\$23.21
17	003284	Binding Post, 8mm x 40mm x M6	1	\$1.46
18	101592	M6 x 45 CLS10.9 DIN7991 FHCS BLKZC	2	\$2.63
19	100746	M6 Flat Washer Blk Zn	1	\$1.58
20	113140	Tube Spacer .88 x .083 <i>Replacement for part number 001230</i>	2	\$3.37

Item Number	Part Number	Description	Quantity	Retail
21	000605	Depth Adjustable Frame Extension Tube, Catalyst/Spark	1	\$37.14
22	101811	M6 x 35 BHCS Blk Zn	1	\$1.05
23	000612	1" Half Saddle Washer	4	\$1.92
26	100804	Tube End Plug 1"	1	\$1.58
28	102803	Washer Flat .250 x .500 x .027 HX BLZ <i>Only used when seating system cross braces are present</i>	2	\$1.58
29	101861	Nylon Washer 5/16 x 1/2 x .03 <i>Only used when standard cross braces are present</i>	1	\$0.53