


Rogue ALX / Little Wave XP Luggage Carrier (Formerly Tsunami)

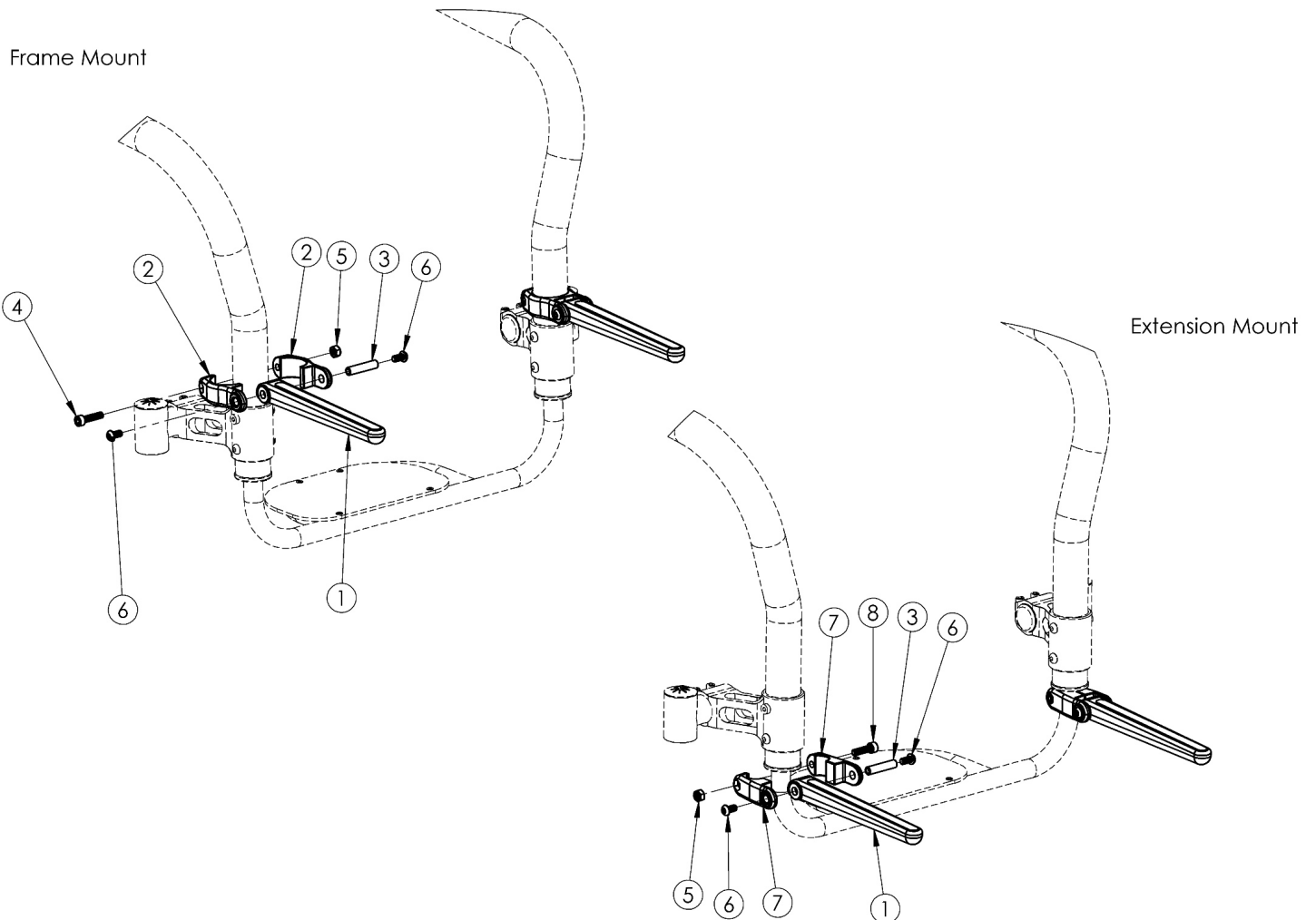

Item Number	Part Number	Description	Quantity
1 - 8	504465	Luggage Carrier Kit, Rogue/RG ALX/LW XP	1
1	003430	Luggage Carrier Fork Assembly	2
2	003412	Luggage Carrier Clamp 1.375 <i>Used to mount Luggage Carrier to the frame</i>	4
3	003413	Luggage Carrier Barrel	2
4	100780	M6 x 25 SHCS ZC <i>Only used when Luggage Carrier is mounted to the frame</i>	2
5	100658	M6 Nylock Nut Blk Zn	2
6	100667	M6 x 12 BTN SHCS BLKZC w/Patch	4

Item Number	Part Number	Description	Quantity
7	003409	Luggage Carrier Clamp .75 <i>Used to mount Luggage Carrier to the footrest tubes/extensions</i>	4
8	101845	M6 x 22 SHCS Blk Zn <i>Only used when Luggage Carrier is mounted to the footrest tubes/extensions</i>	2