


CR45 Caster Forks and Stems

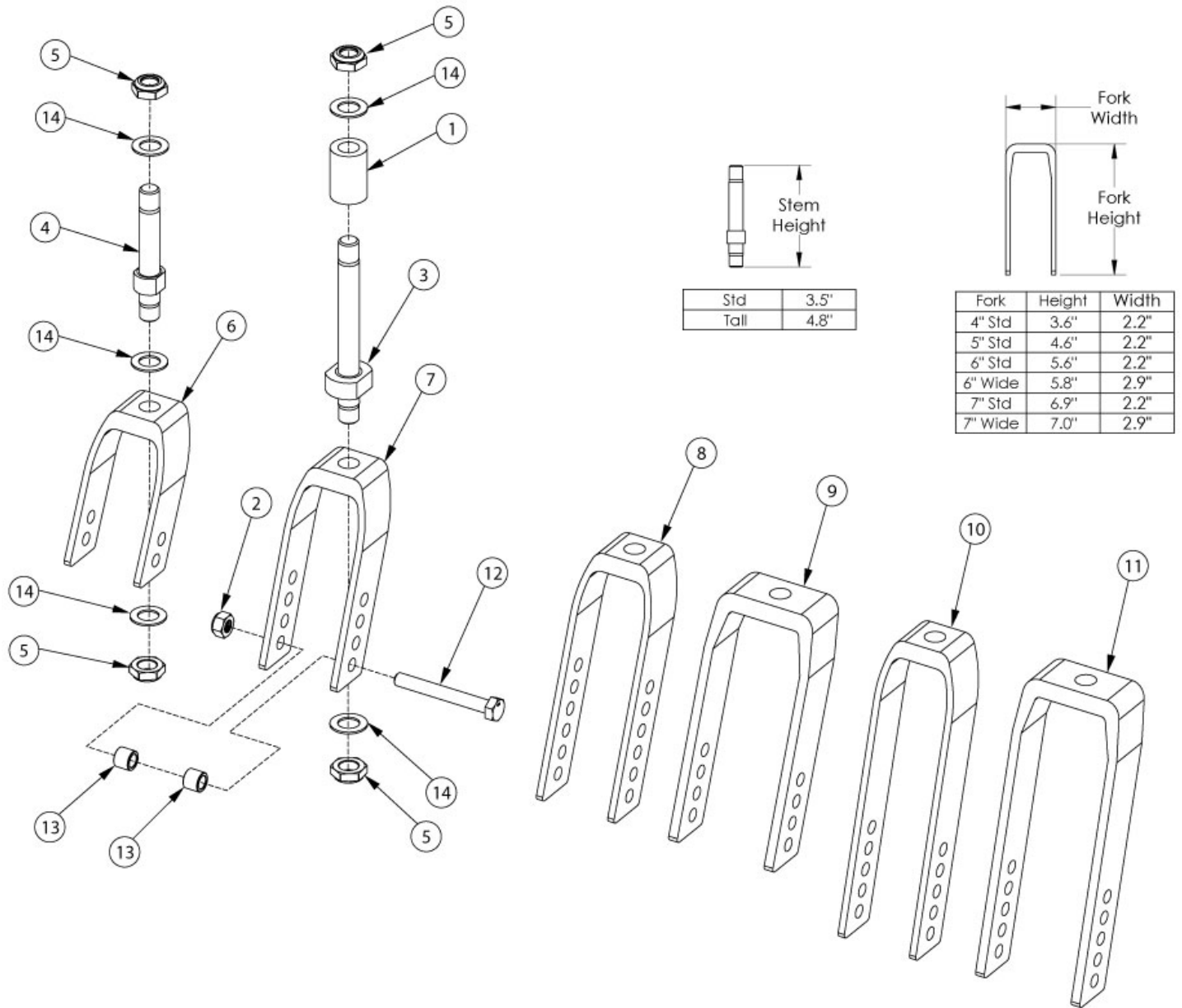

If changing to 2" casters, wide forks and hardware needed.

Item Number	Part Number	Description	Quantity
4, 5, 6, 14	112904	Fork Assembly, Black, 4" Std Stem, Tilt	1
4, 5, 7, 14	503816	Fork Assembly 5" Std Stem, Tilt	1
4, 5, 8, 14	503814	Fork Assembly 6" Std Stem, Tilt	1
4, 5, 9, 14	503817	Fork Assembly 6" Wide Std Stem, Tilt <i>Only used with 2" wide casters.</i>	1

Item Number	Part Number	Description	Quantity
4, 5, 10, 14	503815	Fork Assembly 7" Std Stem, Tilt	1
4, 5, 11, 14	503818	Fork Assembly 7" Wide Std Stem, Tilt <i>Only used with 2" wide casters.</i>	1
1, 3, 5, 6, 14	506153	Fork Assembly 4" Tall Stem, Tilt	1
1, 3, 5, 7, 14	506369	Fork Assembly 5" Tall Stem Tilt	1
1, 3, 5, 8, 14	506370	Fork Assembly 6" Tall Stem Tilt	1
1, 3, 5, 9, 14	506371	Fork Assembly 6" Wide Tall Stem Tilt <i>Only used with 2" wide casters.</i>	1
1, 3, 5, 10, 14	506372	Fork Assembly 7" Tall Stem Tilt	1
2, 12a, 13a	502653	Caster Attachment Hardware Kit CAT/FCS	1
2, 12b, 13b	502654	Caster Attachment Hardware Kit for 6" and 7" Wide Fork, CAT/FCS <i>Only used with 2" wide casters</i>	1
1	002476	Spacer, 3/4" OD x 1/2" ID x 1.23	1
2	100659	M8 Nylock Nut Blk Zn	1
1, 3, 5, 14	506374	Caster Stem Tall, Adult Tilt	1
4, 5, 14	503819	Caster Stem Standard, Tilt	1
5	100529	M12 x 1.25mm Lock Nut Black	2
5, 6	506492	Caster Fork 4", Tilt	1
5, 7	503823	Caster Fork 5" Tilt	1
5, 8	503824	Caster Fork 6" Tilt	1
5, 9	503825	Caster Fork 6" Wide Tilt <i>Only used with 2" wide casters</i>	1
5, 10	503826	Caster Fork 7" Tilt	1
5, 11	503827	Caster Fork 7" Wide Tilt <i>Only used with 2" wide casters</i>	1
12a	101829	M8 x 65 HHCS Blk Zn	1
12b	101831	M8 x 80 HHCS Blk Zn <i>Only used with 2" wide casters</i>	1
13a	100792	Caster Spacer D8 x D12 x 10mm / Close Mount Handrim Spacer <i>Used with all casters except 6x2</i>	2
13b	001246	Caster Spacer Black 6 x 2 Poly .472ODX.319IDX.370LG	2

Item Number	Part Number	Description	Quantity
		<i>Only used with 6" x 2" casters</i>	
14	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	3