


(Discontinued) Catalyst 5 Curved Axle Plate

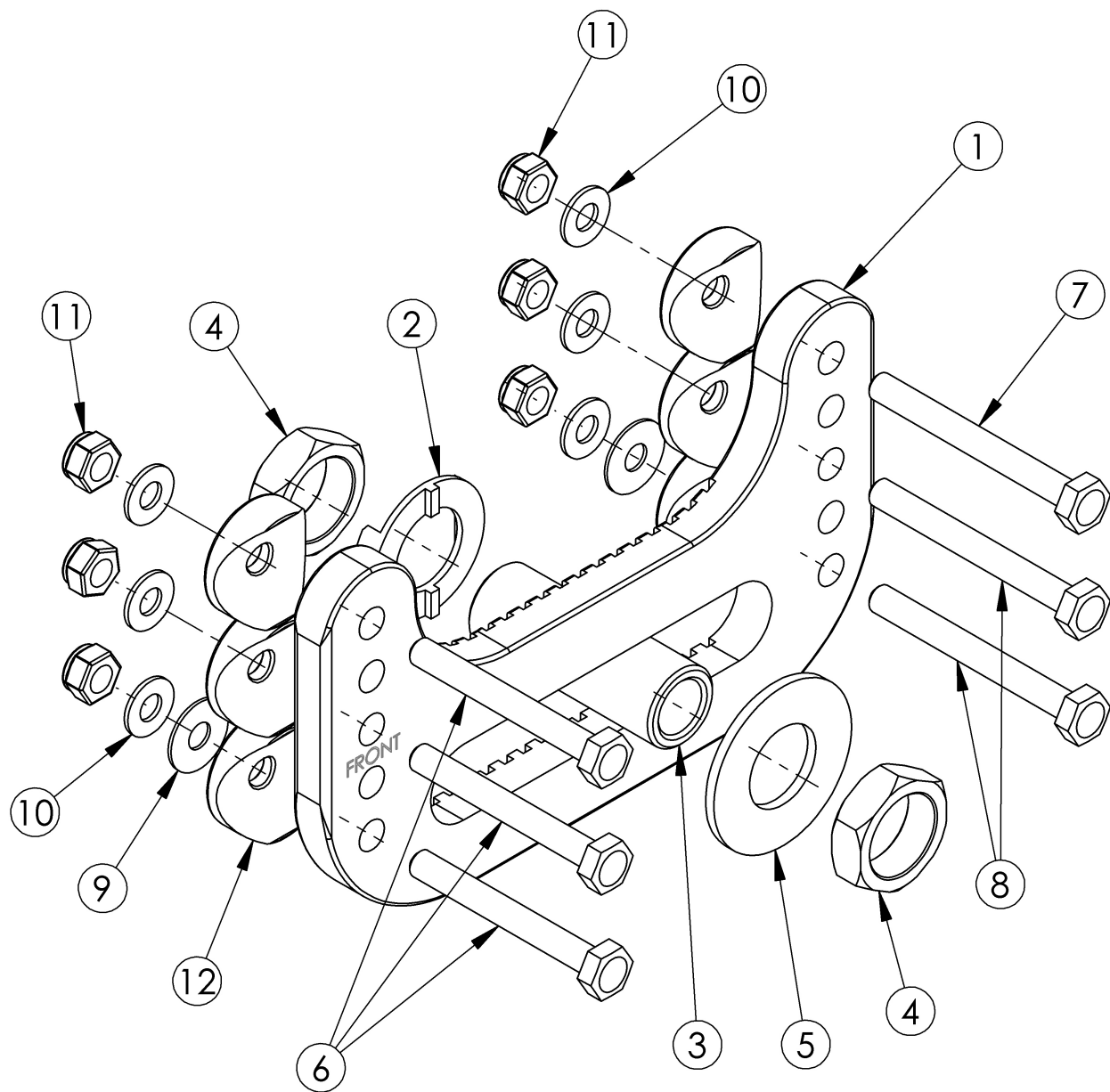

Number of washers (Item 9) per bottom screw	Degrees of Camber
3 (6 total)	2°
5 (10 total)	4°

Item Number	Part Number	Description	Quantity
6a, 7b, 8b, 10, 11	503275	Axle Plate Hardware Kit Cat 5 for Anti Tips (Pair)	1
6a, 7c, 8c, 8d, 10, 11, 12	503276	Axle Plate Hardware Kit Cat 5 for Anti Tips and Transit and Swing Away Arms (Pair)	1
6b, 7a, 7b, 10, 11	503278	Axle Plate Hardware Kit Cat 5 Depth Adj or Spark for Anti Tips (Pair)	1
6b, 7b, 7c, 8c, 10, 11	503279	Axle Plate Hardware Kit Cat 5 Depth Adj or Spark for Anti Tips and Transit	1
2 - 5	500504	Axle Receiver Assembly 1/2"	1
2	004444	Axle Plate Index Washer <i>Replaced 000051 and 004678</i>	1
3	100756	Axle Receiver 1/2" Diameter	1
4	100514	Axle Receiver Nut M18 x D12 x 8mm	2
5	004445	Axle Plate Flat Washer <i>Replaced 000465</i>	1
6a	101826	M6 x 50 HHCS Blk Zn <i>Used with Curved Rear Frames</i>	3
6b	100674	M6 x 45 CLS10.9 DIN931 HHCS ZC <i>Used with Depth Adjustable and Spark Rear Frames</i>	3
7a	101826	M6 x 50 HHCS Blk Zn <i>Used with Axle Plate Only on Curved Rear Frames</i>	1
7b	100676	M6 x 55 CLS10.9 DIN931 HHCS ZC <i>Used with Anti-Tip or Transit or Swing Away Arm Bracket</i>	1
7c	100677	M6 x 60 CLS10.9 DIN931 HHCS ZC <i>Used with Anti-Tip and Transit or Swing Away Arm Bracket</i>	1
8a	101826	M6 x 50 HHCS Blk Zn <i>Used with Axle Plate Only</i>	2
8b	100677	M6 x 60 CLS10.9 DIN931 HHCS ZC <i>Used with Anti-Tip or Transit or Swing Away Arm Bracket</i>	2
8c	100679	M6 x 65 CLS 8.8 DIN931 HHCS ZC <i>Used with Anti-Tip and Transit or Swing Away Arm Bracket</i>	2
8d	100680	M6 x 70 CLS 8.8 DIN931 HHCS ZC <i>Used with Anti-Tip and Transit and Swing Away Arm Bracket</i>	2
9	100763	Washer #12 Thin	10
10	100746	M6 Flat Washer Blk Zn	6
11	100658	M6 Nylock Nut Blk Zn	6

Item Number	Part Number	Description	Quantity
12	000056	Saddle 1 1/8" <i>Used with Transit or Swing Away Arm Bracket</i>	6

Wheel Size / Tire Profile / Rear Seat Height

Select Rear Seat Height in row corresponding to Wheel Size and Tire Profile. 14" Seat Depth not available with 24", 25" or 26" Rear Wheels.

☐ Requires curved axle plate ☐ Any axle plate can be used (Curved, Standard, Amputee, One Arm Drive)

Wheel Size	Tire Profile	Rear Seat Height									
20"	Low	☐ 11.5	☐ 12 ¹	☐ 12.5	☐ 13 ¹	☐ 13.5	☐ 14	☐ 14.5	☐ 15	☐ 15.5	☐ 16
	Full	☐ 12	☐ 12.5 ¹	☐ 13	☐ 13.5 ¹	☐ 14	☐ 14.5	☐ 15	☐ 15.5	☐ 16	☐ 16.5
22"	Low	☐ 12.5	☐ 13 ¹	☐ 13.5	☐ 14 ¹	☐ 14.5	☐ 15	☐ 15.5	☐ 16	☐ 16.5	☐ 17
	Full	☐ 13	☐ 13.5 ¹	☐ 14	☐ 14.5 ¹	☐ 15	☐ 15.5	☐ 16	☐ 16.5	☐ 17	☐ 17.5
24"	Low	☐ 13.5	☐ 14 ¹	☐ 14.5	☐ 15 ¹	☐ 15.5	☐ 16	☐ 16.5	☐ 17	☐ 17.5	☐ 18
	Full	☐ 14	☐ 14.5 ¹	☐ 15	☐ 15.5 ¹	☐ 16	☐ 16.5	☐ 17	☐ 17.5	☐ 18	☐ 18.5
25"	Low	☐ 14	☐ 14.5 ¹	☐ 15	☐ 15.5 ¹	☐ 16	☐ 16.5	☐ 17	☐ 17.5	☐ 18	☐ 18.5
26"	Low	☐ 14.5	☐ 15 ¹	☐ 15.5	☐ 16 ¹	☐ 16.5	☐ 17	☐ 17.5	☐ 18	☐ 18.5	☐ 19
	Full	☐ 15	☐ 15.5 ¹	☐ 16	☐ 16.5 ¹	☐ 17	☐ 17.5	☐ 18	☐ 18.5	☐ 19	☐ 19.5

¹ Not available with Amputee Axle Plate