


Arc O2 Holder

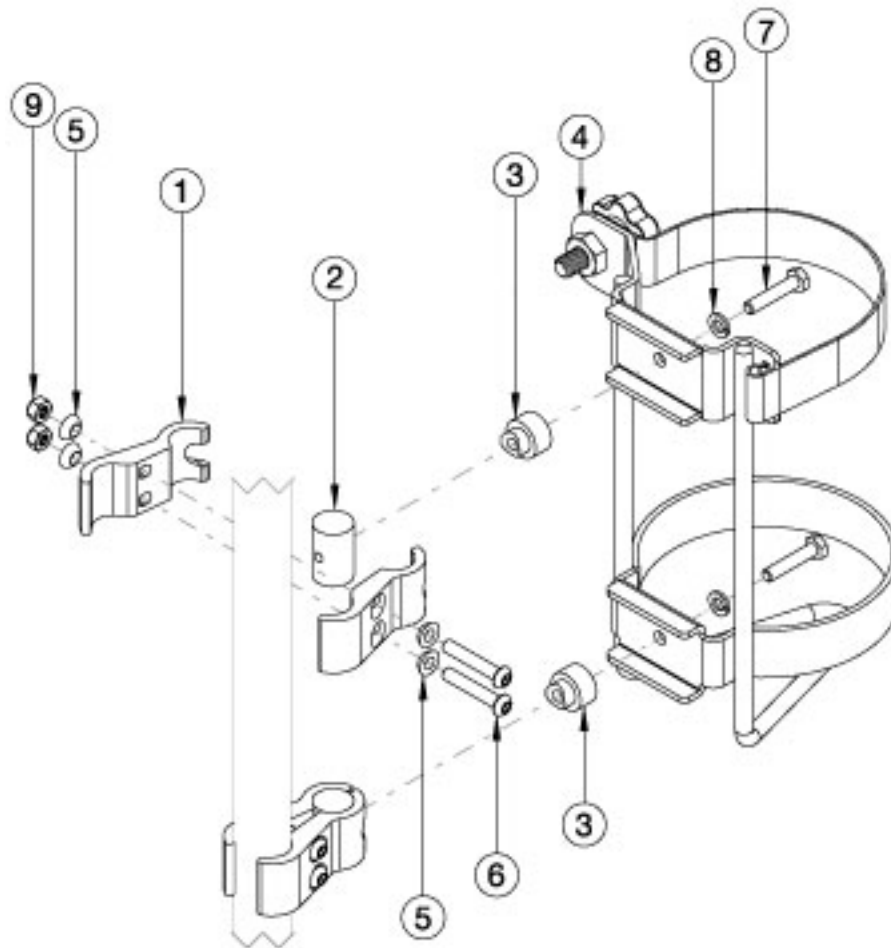

Item Number	Part Number	Description	Quantity
1-9	113514	Complete O2 Holder Assembly, Arc	1
2-4, 7, 8	113515	O2 Holder Assembly, Arc	1
1, 5, 6, 9	113516	O2 Holder Clamp Kit, Each, Arc	1
1	111589	Clamp, Mount, Accessory	4
2	111595	Barrel Nut, 3/4" OD x M6	2
3	111596	Spacer, Accessory	2
4	001416	O2 Holder	1
5	003810	Washer, Spher, 10Mm	8
6	101811	M6 x 35 BHCS Blk Zn	4
7	101824	M6 X 30 Hhcs Blk Zn	2

Item Number	Part Number	Description	Quantity
8	101835	M6 Lock Washer Blk Zn	2
9	100658	M6 Nylock Nut Blk Zn	4