


Liberty Seat Frame - Growth

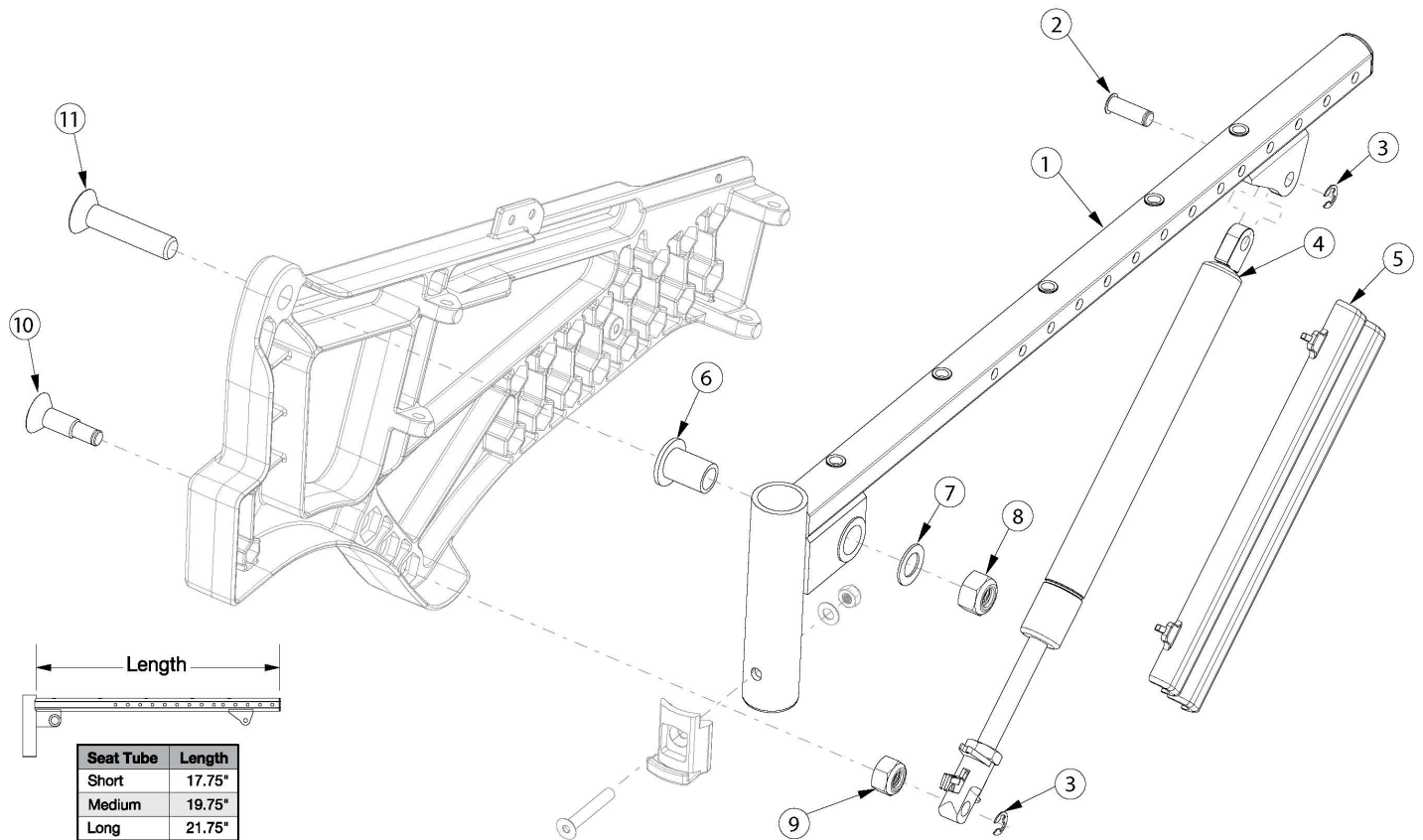

Can you achieve your new depth within the existing depth range? Short = 14" - 16" Medium = 16" - 18" Long = 18" - 20" If no, please choose the appropriate kit below.

Item Number	Part Number	Description	Quantity
2, 3, 4b, 5, 9, 10	506149	Gas Spring Kit, 300N, Liberty	1
2, 3, 4a, 5, 9, 10	506150	Gas Spring Kit, 450N, Liberty	1
1a, 6, 7, 8, 11	506146	Seat Tube Assembly, Short, Liberty	1
1b, 6, 7, 8, 11	506147	Seat Tube Assembly, Medium, Liberty	1
1c, 6, 7, 8, 11	506148	Seat Tube Assembly, Long, Liberty <i>Updated to longer clevis on seat tube 10/21/2020. Not compatible with Transit Wheels.</i>	1
4a, 5	506508	Gas Spring, 95mm Stroke, 450N with Cover, Liberty <i>Used on Widths 18" and Greater (Default Setting)</i>	1
4b, 5	506509	Gas Spring, 95mm Stroke, 300N with Cover, Liberty <i>Used on Widths Less than 18" (Default Setting)</i>	1