


Flip Drum Brake (Hand Tilt)

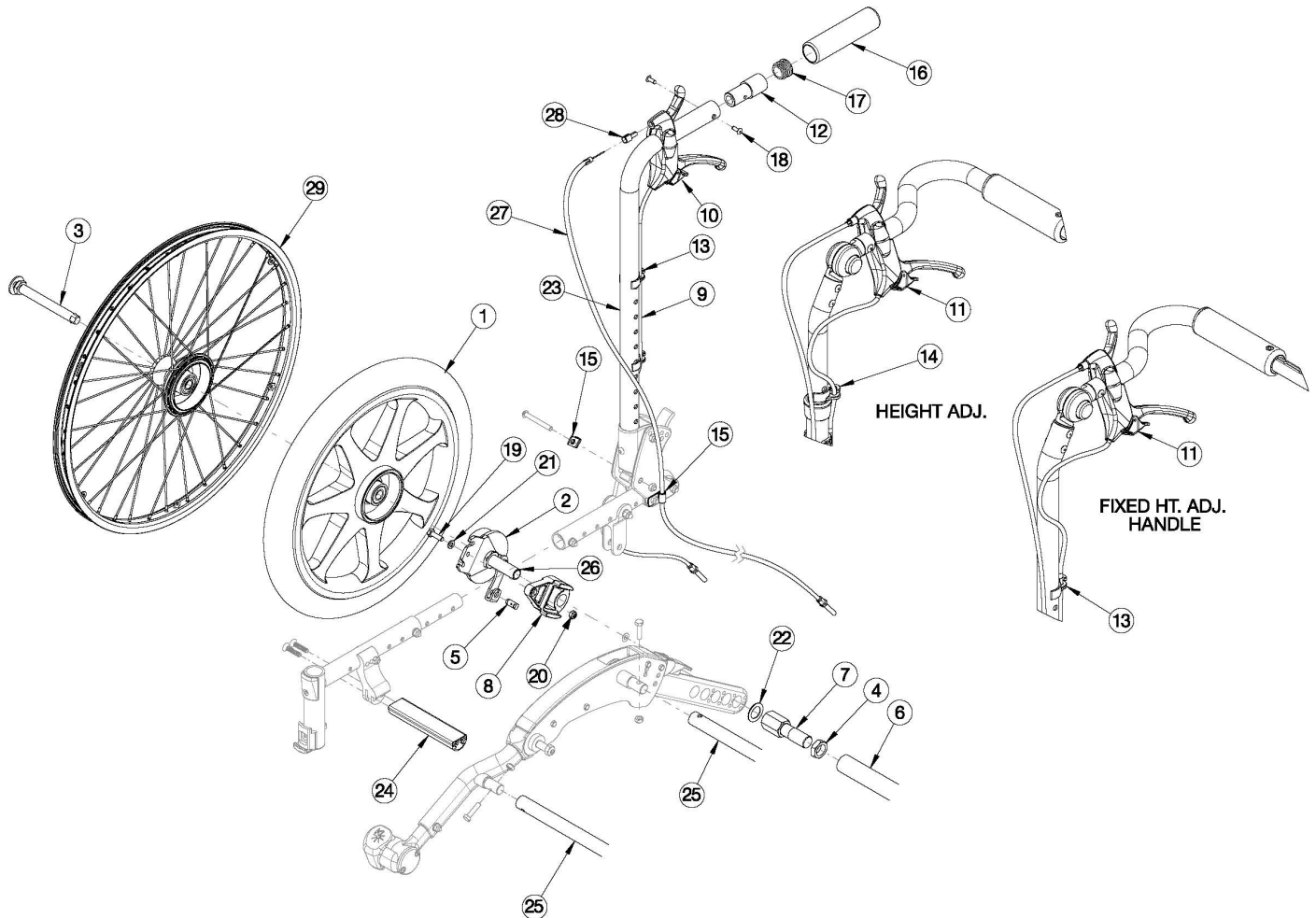

For complete retrofit, quote both drum brake retrofit kit and drum brake cable kit. Must also quote rear tires and handdrims if quoting MAXX Spoke wheels. If changing wheel type and the drum brake is already present, the drum brake rotor kit with hardware is required in addition to the wheels.

Item Number	Part Number	Description	Quantity
1a, 2a, 2b, 3a, 4, 5, 6a, 7, 8, 19-22, 26a	113376	Drum Brake, 12" Mag, Pneumatic Tire, 10W Retrofit Kit FLIP	1
1a, 2a, 2b, 3a, 4, 5, 6b, 7, 8, 19-22, 26a	113377	Drum Brake, 12" Mag, Pneumatic Tire, 11W Retrofit Kit FLIP	1
1a, 2a, 2b, 3a, 4, 5, 6c, 7, 8, 19-22, 26a	113378	Drum Brake, 12" Mag, Pneumatic Tire, 12W Retrofit Kit FLIP	1
1a, 2a, 2b, 3a, 4, 5, 6d, 7, 8, 19-22, 26a	113379	Drum Brake, 12" Mag, Pneumatic Tire, 13W Retrofit Kit FLIP	1
1a, 2a, 2b, 3a, 4, 5, 6e, 7, 8, 19-22, 26a	113380	Drum Brake, 12" Mag, Pneumatic Tire, 14W Retrofit Kit FLIP	1

Item Number	Part Number	Description	Quantity
1a, 2a, 2b, 3a, 4, 5, 6f, 7, 8, 19-22, 26a	113381	Drum Brake, 12" Mag, Pneumatic Tire, 15W Retrofit Kit FLIP	1
1a, 2a, 2b, 3a, 4, 5, 6g, 7, 8, 19-22, 26a	113382	Drum Brake, 12" Mag, Pneumatic Tire, 16W Retrofit Kit FLIP	1
1a, 2a, 2b, 3a, 4, 5, 6h, 7, 8, 19-22, 26a	113383	Drum Brake, 12" Mag, Pneumatic Tire, 17W Retrofit Kit FLIP	1
1a, 2a, 2b, 3a, 4, 5, 6i, 7, 8, 19-22, 26a	113384	Drum Brake, 12" Mag, Pneumatic Tire, 18W Retrofit Kit FLIP	1
1b, 2a, 2b, 3a, 4, 5, 6a, 7, 8, 19-22, 26a	113385	Drum Brake, 12" Mag, Full Poly Tire, 10W Retrofit Kit FLIP	1
1b, 2a, 2b, 3a, 4, 5, 6b, 7, 8, 19-22, 26a	113386	Drum Brake, 12" Mag, Full Poly Tire, 11W Retrofit Kit FLIP	1
1b, 2a, 2b, 3a, 4, 5, 6c, 7, 8, 19-22, 26a	113387	Drum Brake, 12" Mag, Full Poly Tire, 12W Retrofit Kit FLIP	1
1b, 2a, 2b, 3a, 4, 5, 6d, 7, 8, 19-22, 26a	113388	Drum Brake, 12" Mag, Full Poly Tire, 13W Retrofit Kit FLIP	1
1b, 2a, 2b, 3a, 4, 5, 6e, 7, 8, 19-22, 26a	113389	Drum Brake, 12" Mag, Full Poly Tire, 14W Retrofit Kit FLIP	1
1b, 2a, 2b, 3a, 4, 5, 6f, 7, 8, 19-22, 26a	113390	Drum Brake, 12" Mag, Full Poly Tire, 15W Retrofit Kit FLIP	1
1b, 2a, 2b, 3a, 4, 5, 6g, 7, 8, 19-22, 26a	113391	Drum Brake, 12" Mag, Full Poly Tire, 16W Retrofit Kit FLIP	1
1b, 2a, 2b, 3a, 4, 5, 6h, 7, 8, 19-22, 26a	113392	Drum Brake, 12" Mag, Full Poly Tire, 17W Retrofit Kit FLIP	1
1b, 2a, 2b, 3a, 4, 5, 6i, 7, 8, 19-22, 26a	113393	Drum Brake, 12" Mag, Full Poly Tire, 18W Retrofit Kit FLIP	1
1c, 2a, 2b, 3a, 4, 5, 6a, 7, 8, 19-22, 26a	502897	Drum Brake, 16" Mag, Pneumatic Tire, 10W Retrofit Kit FLIP	1
1c, 2a, 2b, 3a, 4, 5, 6b, 7, 8, 19-22, 26a	502898	Drum Brake, 16" Mag, Pneumatic Tire, 11W Retrofit Kit FLIP	1
1c, 2a, 2b, 3a, 4, 5, 6c, 7, 8, 19-22, 26a	502899	Drum Brake, 16" Mag, Pneumatic Tire, 12W Retrofit Kit FLIP	1
1c, 2a, 2b, 3a, 4, 5, 6d, 7, 8, 19-22, 26a	502900	Drum Brake, 16" Mag, Pneumatic Tire, 13W Retrofit Kit FLIP	1
1c, 2a, 2b, 3a, 4, 5, 6e, 7, 8, 19-22, 26a	502901	Drum Brake, 16" Mag, Pneumatic Tire, 14W Retrofit Kit FLIP	1
1c, 2a, 2b, 3a, 4, 5, 6f, 7, 8, 19-22, 26a	502902	Drum Brake, 16" Mag, Pneumatic Tire, 15W Retrofit Kit FLIP	1

Item Number	Part Number	Description	Quantity
1c, 2a, 2b, 3a, 4, 5, 6g, 7, 8, 19-22, 26a	502903	Drum Brake, 16" Mag, Pneumatic Tire, 16W Retrofit Kit FLIP	1
1c, 2a, 2b, 3a, 4, 5, 6h, 7, 8, 19-22, 26a	502904	Drum Brake, 16" Mag, Pneumatic Tire, 17W Retrofit Kit FLIP	1
1c, 2a, 2b, 3a, 4, 5, 6i, 7, 8, 19-22, 26a	502905	Drum Brake, 16" Mag, Pneumatic Tire, 18W Retrofit Kit FLIP	1
1d, 2a, 2b, 3a, 4, 5, 6a, 7, 8, 19-22, 26a	502906	Drum Brake, 16" Mag, Pneumatic w/ Airless Insert Tire, 10W Retrofit Kit FLIP	1
1d, 2a, 2b, 3a, 4, 5, 6b, 7, 8, 19-22, 26a	502907	Drum Brake, 16" Mag, Pneumatic w/ Airless Insert Tire, 11W Retrofit Kit FLIP	1
1d, 2a, 2b, 3a, 4, 5, 6c, 7, 8, 19-22, 26a	502908	Drum Brake, 16" Mag, Pneumatic w/ Airless Insert Tire, 12W Retrofit Kit FLIP	1
1d, 2a, 2b, 3a, 4, 5, 6d, 7, 8, 19-22, 26a	502909	Drum Brake, 16" Mag, Pneumatic w/ Airless Insert Tire, 13W Retrofit Kit FLIP	1
1d, 2a, 2b, 3a, 4, 5, 6e, 7, 8, 19-22, 26a	502910	Drum Brake, 16" Mag, Pneumatic w/ Airless Insert Tire, 14W Retrofit Kit FLIP	1
1d, 2a, 2b, 3a, 4, 5, 6f, 7, 8, 19-22, 26a	502911	Drum Brake, 16" Mag, Pneumatic w/ Airless Insert Tire, 15W Retrofit Kit FLIP	1
1d, 2a, 2b, 3a, 4, 5, 6g, 7, 8, 19-22, 26a	502912	Drum Brake, 16" Mag, Pneumatic w/ Airless Insert Tire, 16W Retrofit Kit FLIP	1
1d, 2a, 2b, 3a, 4, 5, 6h, 7, 8, 19-22, 26a	502913	Drum Brake, 16" Mag, Pneumatic w/ Airless Insert Tire, 17W Retrofit Kit FLIP	1
1d, 2a, 2b, 3a, 4, 5, 6i, 7, 8, 19-22, 26a	502914	Drum Brake, 16" Mag, Pneumatic w/ Airless Insert Tire, 18W Retrofit Kit FLIP	1
1e, 2a, 2b, 3a, 4, 5, 6a, 7, 8, 19-22, 26a	502915	Drum Brake, 16" Mag, Full Poly Tire, 10W Retrofit Kit FLIP	1
1e, 2a, 2b, 3a, 4, 5, 6b, 7, 8, 19-22, 26a	502916	Drum Brake, 16" Mag, Full Poly Tire, 11W Retrofit Kit FLIP	1
1e, 2a, 2b, 3a, 4, 5, 6c, 7, 8, 19-22, 26a	502917	Drum Brake, 16" Mag, Full Poly Tire, 12W Retrofit Kit FLIP	1
1e, 2a, 2b, 3a, 4, 5, 6d, 7, 8, 19-22, 26a	502918	Drum Brake, 16" Mag, Full Poly Tire, 13W Retrofit Kit FLIP	1
1e, 2a, 2b, 3a, 4, 5, 6e, 7, 8, 19-22, 26a	502919	Drum Brake, 16" Mag, Full Poly Tire, 14W Retrofit Kit FLIP	1
1e, 2a, 2b, 3a, 4, 5, 6f, 7, 8, 19-22, 26a	502920	Drum Brake, 16" Mag, Full Poly Tire, 15W Retrofit Kit FLIP	1
1e, 2a, 2b, 3a, 4, 5, 6g, 7, 8, 19-22, 26a	502921	Drum Brake, 16" Mag, Full Poly Tire, 16W Retrofit Kit FLIP	1

Item Number	Part Number	Description	Quantity
1e, 2a, 2b, 3a, 4, 5, 6h, 7, 8, 19-22, 26a	502922	Drum Brake, 16" Mag, Full Poly Tire, 17W Retrofit Kit FLIP	1
1e, 2a, 2b, 3a, 4, 5, 6i, 7, 8, 19-22, 26a	502923	Drum Brake, 16" Mag, Full Poly Tire, 18W Retrofit Kit FLIP	1
2a, 2b, 3b, 4, 5, 6a, 7, 8, 19-22, 26b, 29a	113394	Drum Brake, 20" MAXX Spoke, 10W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6b, 7, 8, 19-22, 26b, 29a	113395	Drum Brake, 20" MAXX Spoke, 11W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6c, 7, 8, 19-22, 26b, 29a	113396	Drum Brake, 20" MAXX Spoke, 12W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6d, 7, 8, 19-22, 26b, 29a	113397	Drum Brake, 20" MAXX Spoke, 13W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6e, 7, 8, 19-22, 26b, 29a	113398	Drum Brake, 20" MAXX Spoke, 14W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6f, 7, 8, 19-22, 26b, 29a	113399	Drum Brake, 20" MAXX Spoke, 15W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6g, 7, 8, 19-22, 26b, 29a	113400	Drum Brake, 20" MAXX Spoke, 16W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6h, 7, 8, 19-22, 26b, 29a	113401	Drum Brake, 20" MAXX Spoke, 17W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6i, 7, 8, 19-22, 26b, 29a	113402	Drum Brake, 20" MAXX Spoke, 18W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6a, 7, 8, 19-22, 26b, 29b	113403	Drum Brake, 22" MAXX Spoke, 10W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6b, 7, 8, 19-22, 26b, 29b	113404	Drum Brake, 22" MAXX Spoke, 11W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6c, 7, 8, 19-22, 26b, 29b	113405	Drum Brake, 22" MAXX Spoke, 12W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6d, 7, 8, 19-22, 26b, 29b	113406	Drum Brake, 22" MAXX Spoke, 13W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1

Item Number	Part Number	Description	Quantity
2a, 2b, 3b, 4, 5, 6e, 7, 8, 19-22, 26b, 29b	113407	Drum Brake, 22" MAXX Spoke, 14W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6f, 7, 8, 19-22, 26b, 29b	113408	Drum Brake, 22" MAXX Spoke, 15W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6g, 7, 8, 19-22, 26b, 29b	113409	Drum Brake, 22" MAXX Spoke, 16W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6h, 7, 8, 19-22, 26b, 29b	113410	Drum Brake, 22" MAXX Spoke, 17W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6i, 7, 8, 19-22, 26b, 29b	113411	Drum Brake, 22" MAXX Spoke, 18W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6a, 7, 8, 19-22, 26b, 29c	113412	Drum Brake, 24" MAXX Spoke, 10W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6b, 7, 8, 19-22, 26b, 29c	113413	Drum Brake, 24" MAXX Spoke, 11W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6c, 7, 8, 19-22, 26b, 29c	113414	Drum Brake, 24" MAXX Spoke, 12W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6d, 7, 8, 19-22, 26b, 29c	113415	Drum Brake, 24" MAXX Spoke, 13W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6e, 7, 8, 19-22, 26b, 29c	113416	Drum Brake, 24" MAXX Spoke, 14W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6f, 7, 8, 19-22, 26b, 29c	113417	Drum Brake, 24" MAXX Spoke, 15W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6g, 7, 8, 19-22, 26b, 29c	113418	Drum Brake, 24" MAXX Spoke, 16W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6h, 7, 8, 19-22, 26b, 29c	113419	Drum Brake, 24" MAXX Spoke, 17W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6i, 7, 8, 19-22, 26b, 29c	113420	Drum Brake, 24" MAXX Spoke, 18W Retrofit Kit FLIP <i>This kit does not contain rear tires or handrims</i>	1
2a, 2b, 3b, 4, 5, 6a, 7, 8, 19-22, 26b, 29c	118250	Drum Brake, 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 10W Retrofit Kit FLIP	1


Item Number	Part Number	Description	Quantity
2a, 2b, 3b, 4, 5, 6b, 7, 8, 19-22, 26b, 29c	118251	Drum Brake, 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 11W Retrofit Kit, FLIP	1
2a, 2b, 3b, 4, 5, 6c, 7, 8, 19-22, 26b, 29c	118252	Drum Brake, 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 12W Retrofit Kit FLIP	1
2a, 2b, 3b, 4, 5, 6d, 7, 8, 19-22, 26b, 29c	118253	Drum Brake, 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 13W Retrofit Kit FLIP	1
2a, 2b, 3b, 4, 5, 6e, 7, 8, 19-22, 26b, 29c	118254	Drum Brake, 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 14W Retrofit Kit FLIP	1
2a, 2b, 3b, 4, 5, 6f, 7, 8, 19-22, 26b, 29c	118255	Drum Brake, 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 15W Retrofit Kit FLIP	1
2a, 2b, 3b, 4, 5, 6g, 7, 8, 19-22, 26b, 29c	118256	Drum Brake, 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 16W Retrofit Kit FLIP	1
2a, 2b, 3b, 4, 5, 6h, 7, 8, 19-22, 26b, 29c	118257	Drum Brake, 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 17W Retrofit Kit FLIP	1
2a, 2b, 3b, 4, 5, 6i, 7, 8, 19-22, 26b, 29c	118258	Drum Brake, 24" MAXX Spoke, Pneumatic w/ Airless Insert Tire (Black), 18W Retrofit Kit FLIP	1
5, 9a, 10, 12, 13, 15-18, 20, 23a, 27a, 28	114164	Drum Brake Cable Kit, Hand Tilt, Short Stroller Handle, 12" - 16" Deep, Pair, Flip	1
5, 9a, 10, 12, 13, 15-18, 20, 23a, 27b, 28	114165	Drum Brake Cable Kit, Hand Tilt, Short Stroller Handle, 17" - 20" Deep, Pair, Flip	1
5, 9b, 10, 12, 13, 15-18, 20, 23b, 27b, 28	114166	Drum Brake Cable Kit, Hand Tilt, Tall Stroller Handle, 12" - 16" Deep, Pair, Flip	1
5, 9b, 10, 12, 13, 15-18, 20, 23b, 27c, 28	114167	Drum Brake Cable Kit, Hand Tilt, Tall Stroller Handle, 17" - 20" Deep, Pair, Flip	1
5, 9c, 11, 13-15, 27d, 28	114168	Drum Brake Cable Kit, Hand Tilt, Short Adjustable Height w/Adjustable Handle, Pair, Flip	1
5, 9d, 11, 13-15, 27e, 28	114169	Drum Brake Cable Kit, Hand Tilt, Tall Adjustable Height w/Adjustable Handle, Pair, Flip	1
5, 9c, 11, 13, 15, 27d, 28	114170	Drum Brake Cable Kit, Hand Tilt, Short Fixed Height with Adjustable Handle, Pair, Flip	1
5, 9d, 11, 13, 15, 27e, 28	114171	Drum Brake Cable Kit, Hand Tilt, Tall Fixed Height with Adjustable Handle, Pair, Flip	1
6a, 24a, 25a, 25d	502868	Strut Kit 10" Wide Flip w/ Drum Brake	1
6b, 24b, 25b, 25e	502869	Strut Kit 11" Wide Flip w/ Drum Brake	1
6c, 24c, 25c, 25f	502870	Strut Kit 12" Wide Flip w/ Drum Brake	1
6d, 24d, 25d, 25g	502871	Strut Kit 13" Wide Flip w/ Drum Brake	1
6e, 24e, 25e, 25h	502872	Strut Kit 14" Wide Flip w/ Drum Brake	1
6f, 24f, 25f, 25i	502873	Strut Kit 15" Wide Flip w/ Drum Brake	1

Item Number	Part Number	Description	Quantity
6g, 24g, 25g, 25j	502874	Strut Kit 16" Wide Flip w/ Drum Brake	1
6h, 24h, 25h, 25k	502875	Strut Kit 17" Wide Flip w/ Drum Brake	1
6i, 24i, 25i, 25l	502876	Strut Kit 18" Wide Flip w/ Drum Brake	1
2a, 2b, 3a, 4, 5, 8, 19, 20, 21, 22, 26a	113421	Drum Brake Rotor Kit for Mag Wheels with Hardware PAIR, Flip	1
2a, 2b, 3b, 4, 5, 8, 19, 20, 21, 22, 26b	113422	Drum Brake Rotor Kit for MAXX Spoke Wheels with Hardware PAIR, Flip	1
3a, 4, 5, 8, 19, 20, 21, 22, 26a	113423	Hardware Kit for Drum Brake Rotor for Mag Wheels PAIR, Flip	1
3b, 4, 5, 8, 19, 20, 21, 22, 26b	113424	Hardware Kit for Drum Brake Rotor for MAXX Spoke Wheels PAIR, Flip	1
2a, 3a, 4, 5, 8, 19, 20, 21, 22, 26a	113425	Drum Brake Rotor Kit for Mag Wheels with Hardware, Side B, Flip	1
2b, 3a, 4, 5, 8, 19, 20, 21, 22, 26a	113426	Drum Brake Rotor Kit for Mag Wheels with Hardware, Side A, Flip	1
2a, 3b, 4, 5, 8, 19, 20, 21, 22, 26b	113427	Drum Brake Rotor Kit for MAXX Spoke Wheels with Hardware, Side B, Flip	1
2b, 3b, 4, 5, 8, 19, 20, 21, 22, 26b	113428	Drum Brake Rotor Kit for MAXX Spoke Wheels with Hardware, Side A, Flip	1
3a, 4, 5, 8, 19, 20, 21, 22, 26a	113429	Hardware Kit for Drum Brake Rotor for Mag Wheels, One Side, Flip	1
3b, 4, 5, 8, 19, 20, 21, 22, 26b	113430	Hardware Kit for Drum Brake Rotor for MAXX Spoke Wheels, One Side, Flip	1
1a	111804	12" Mag Wheel with Pneumatic Tire, Drum Brake Hub	2
1b	111805	12" Mag Wheel with Full Poly Tire, Drum Brake Hub	2
1c, 2-5	002678	Drum Brake Kit, 16" Mag, Pneumatic Tire PAIR	1
1d, 2-5	002679	Drum Brake Kit, 16" Mag, Pneumatic Tire w/ Airless Insert PAIR	1
1e, 2-5	002680	Drum Brake Kit, 16" Mag, Full Poly Tire PAIR	1
2a, 26a	113431	Drum Brake Rotor with Axle Sleeve, Side B, for Mag Wheel	1
2b, 26a	113432	Drum Brake Rotor with Axle Sleeve, Side A, for Mag Wheel	1
2a, 26b	113433	Drum Brake Rotor with Axle Sleeve, Side B, for MAXX Spoke Wheel	1
2b, 26b	113434	Drum Brake Rotor with Axle Sleeve, Side A, for MAXX Spoke Wheel	1
3a	101108	Axle Quick Release w/ Button 1/2" x 99MM <i>Used with 12" and 16" Mag wheels</i>	2
3b	101191	Axle Quick Release w/ Button 1/2" x 110MM	2


Item Number	Part Number	Description	Quantity
		<i>Used with MAXX Spoke wheels</i>	
5	111598	Pinch Bolt	2
6a	002565	Strut, Drum Brake, Ped, 10W	1
6b	002566	Strut, Drum Brake, Ped, 11W	1
6c	002567	Strut, Drum Brake, Ped, 12W	1
6d	002568	Strut, Drum Brake, Ped, 13W	1
6e	002569	Strut, Drum Brake, Ped, 14W	1
6f	002570	Strut, Drum Brake, Ped, 15W	1
6g	002571	Strut, Drum Brake, Ped, 16W	1
6h	002572	Strut, Drum Brake, Ped, 17W	1
6i	002573	Strut, Drum Brake, Ped, 18W	1
7	002675	Adapter, Strut, Drum Brake	2
8	002563	Adapter, Drum Brake, Flip	2
5, 9a	122727	Drum Brake / Hemi Wheel Lock Medium Cable <i>Used with short stroller handle back post</i>	2
5, 9b	122728	Drum Brake / Hemi Wheel Lock Long Cable <i>Used with tall stroller handle back post</i>	2
5, 9c	122729	Drum Brake / Hemi Wheel Lock XL Cable <i>Used with short height adjustable back post and short fixed height with adjustable handle back post</i>	2
5, 9d	122730	Cable, Drum Brake, XXL <i>Used with tall height adjustable back post and tall fixed height with adjustable handle back post</i>	2
10, 28	114147	Tilt and Drum Brake Lever, 25MM, with Dual Lever Cable End Adapter <i>Used with stroller handle back posts</i>	2
11, 28	114148	Tilt and Drum Brake Lever, 22MM, with Dual Lever Cable End Adapter <i>Used with height adjustable back posts and fixed height with adjustable handle back post.</i>	2
13	001117	Cable Clip 1" <i>Use quantity 2 per each stroller handle back post and fixed height with adjustable handle back post; use quantity 1 per each height adjustable back post</i>	4
14	001118	Cable Clip 7/8" <i>Use quantity 1 per each height adjustable back post</i>	2
15	001239	P-Clip 3/16"	2

Item Number	Part Number	Description	Quantity
19	101823	M6 X 20 Hhcs Blk Zn	2
20	100658	M6 Nylock Nut Blk Zn	2
21	100746	M6 Flat Washer Blk Zn	2
22	100583	Axle Plate Washer 18mm	2
12, 13, 16-18, 23a	503235	Stroller Handle Back Post for Drum Brake, Short, Flip	2
12, 13, 16-18, 23b	503236	Stroller Handle Back Post for Drum Brake, Tall, Flip	2
24a	001879	Strut Tube Seat 10 Wide	1
24b	001880	Strut Tube Seat 11 Wide	1
24c	001881	Strut Tube Seat 12 Wide	1
24d	001882	Strut Tube Seat 13 Wide	1
24e	001883	Strut Tube Seat 14 Wide	1
24f	001884	Strut Tube Seat 15 Wide	1
24g	001885	Strut Tube Seat 16 Wide	1
24h	001886	Strut Tube Seat 17 Wide	1
24i	001887	Strut Tube Seat 18 Wide	1
25a	002719	Frame Growth Strut 6.5" <i>Used on 10" wide chairs</i>	1
25b	002629	Frame Growth Strut 7.5" <i>Used on 11" wide chairs</i>	1
25c	002630	Frame Growth Strut 8.5" <i>Used on 12" wide chairs</i>	1
25d	002631	Frame Growth Strut 9.5" <i>Used on 10" and 13" wide chairs</i>	1
25e	002632	Frame Growth Strut 10.5" <i>Used on 11" and 14" wide chairs</i>	1
25f	002633	Frame Growth Strut 11.5" <i>Used on 12" and 15" wide chairs</i>	1
25g	002634	Frame Growth Strut 12.5" <i>Used on 13" and 16" wide chairs</i>	1
25h	002635	Frame Growth Strut 13.5" <i>Used on 14" and 17" wide chairs</i>	1

Item Number	Part Number	Description	Quantity
25i	002636	Frame Growth Strut 14.5" <i>Used on 15" and 18" wide chairs</i>	1
25j	002637	Frame Growth Strut 15.5" <i>Used on 16" wide chairs</i>	1
25k	002638	Frame Growth Strut 16.5" <i>Used on 17" wide chairs</i>	1
25l	002639	Frame Growth Strut 17.5" <i>Used on 18" wide chairs</i>	1
27a	112935	Cable, Tilt w/Drum, Medium <i>Used with short stroller handle back post with 12" - 16" seat depths</i>	2
27b	112936	Cable, Tilt w/Drum, Long, Flip <i>Used with short stroller handle back post with 17" - 20" seat depths; used with tall stroller handle back post with 12" - 16" seat depths</i>	2
27c	112937	Cable, Tilt w/Drum, XL <i>Used with tall stroller handle back post with 17" - 20" seat depths</i>	2
27d	112571	Cable, Tilt w/Drum, XXXL <i>Used with short adjustable height back posts and short fixed height with adjustable handle back post</i>	2
27e	112572	Cable, Tilt w/ Drum, 4XL <i>Used with tall adjustable height back posts and tall fixed height with adjustable handle back post</i>	2
29a	111731	Wheel 20" MAXX Spoke, Drum Brake	2
29b	111732	Wheel 22" MAXX Spoke, Drum Brake	2
29c	111733	Wheel 24" MAXX Spoke, Drum Brake	2
29d	118358	24" Spoke for Drum Brake, Pneumatic w/ Airless Insert Tire (Black) <i>Replaced kit number 117432. Does not include the handrim. Quote handrim hardware if needed.</i>	2

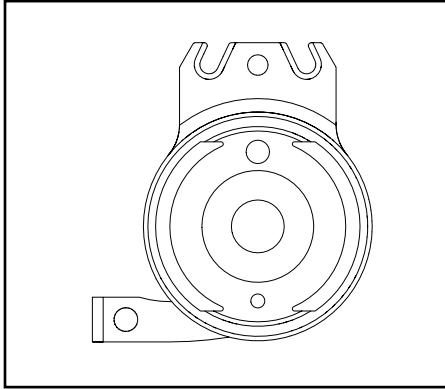



Rotor A - Part No. 111641

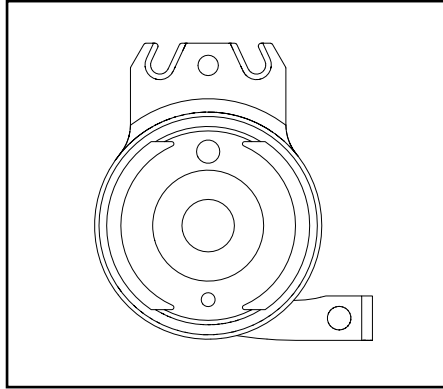

Rotor B - Part No. 111640



Catalyst 5		
Configuration	Left	Right
Axle plate curve up	B	A
Axle plate curve down	A	B

Left: Axle Plate Curve Up
Right: Axle Plate Curve Down

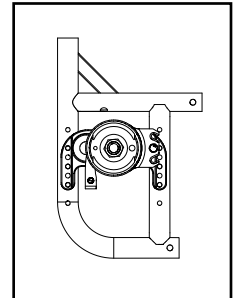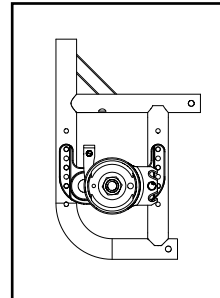


Catalyst 5VX		
Configuration	Left	Right
Axle Position holes 1-5 (top)	A	B
Axle Position holes 6-7 (bottom)	B	A


Left: Axle Position Top
Right: Axle Position Bottom

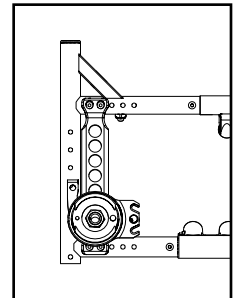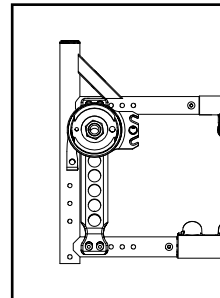


Liberty		
Configuration	Left	Right
All	B	A

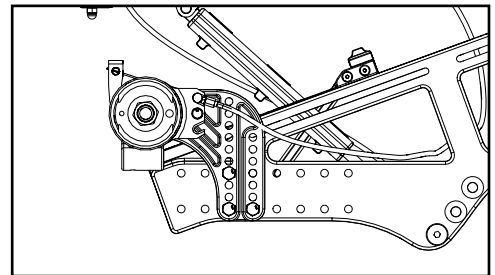


Flip		
Configuration	Left	Right
All	A	B

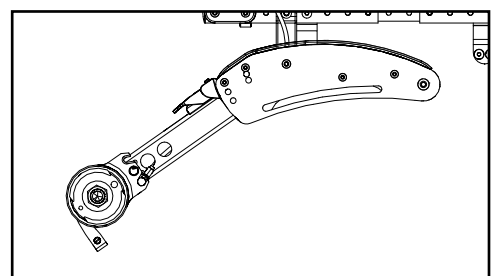

Focus		
Configuration	Left	Right
Standard (mounts above base frame)	A	B
Inverted (mounts below base frame)	B	A

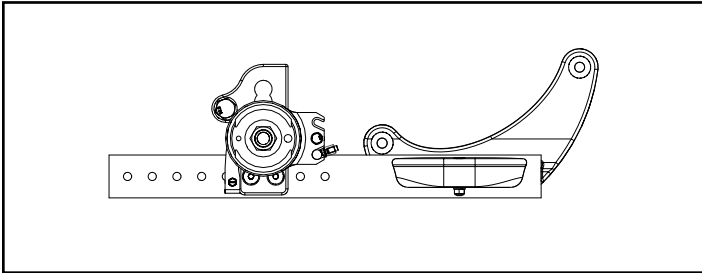


Standard Configuration

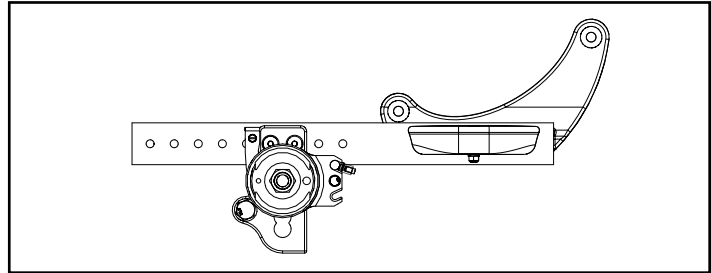


Inverted Configuration


Rigid		
Configuration	Left	Right
Tower 000925 - upper holes (14 or more exposed below clamp)	B	A
Tower 000925 - lower holes (13 or fewer holes exposed below clamp)	A	B
Tower 000926 (all hole locations)	A	B

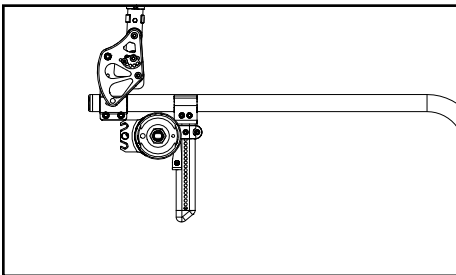

Tower 000925 - Upper Holes

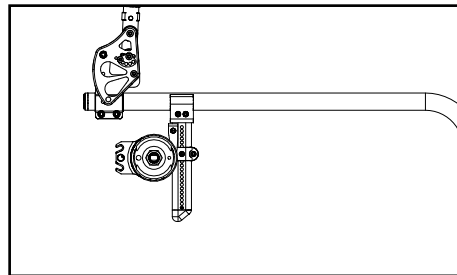

Tower 000925 - Lower Holes

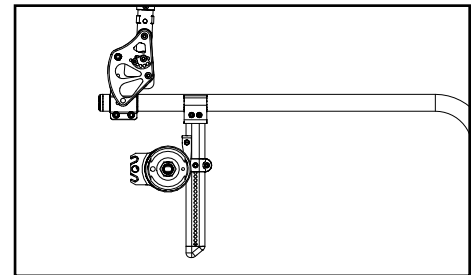

Tower 000926 - All Holes

Arc		
Configuration	Left	Right
All	B	A

