


Catalyst 5Ti Swing Away Front Frame

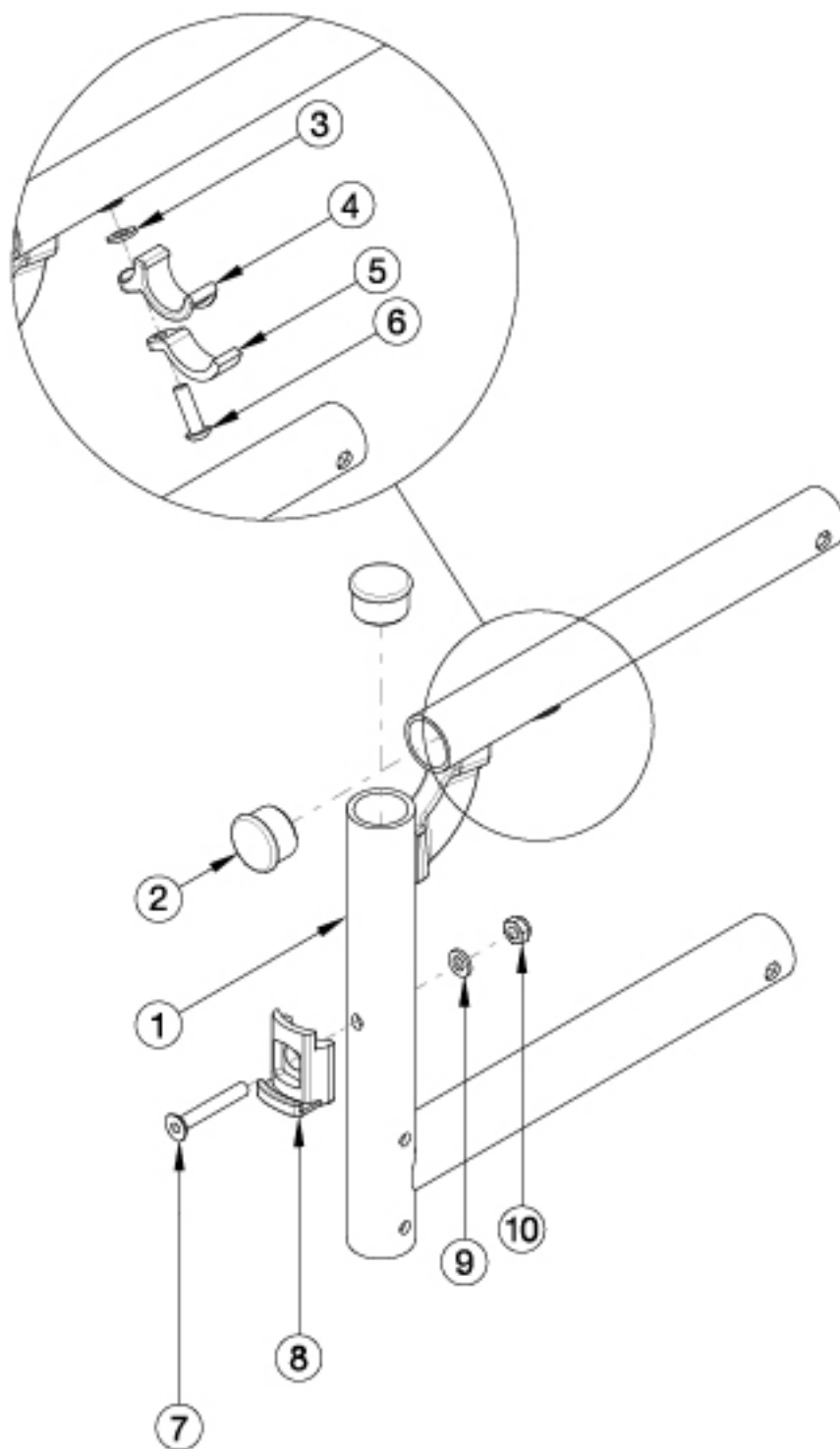

16" and 18" seat depths use the same front frames. 17" and 19" seat depths use the same front frames. Frame assemblies are based on Seat Depth. If switching to a new Seat Depth, new Height Adjustable T-Arms or Height Adjustable Flip Back T-Arms may be needed. If chair has Flip Back or the discontinued Height Adjustable Flip Back armrests, select a frame assembly with the Flip Back Arm Modification. This modification does not apply to Height Adjustable Flip Back T-Arms. If retrofitting from a Fixed Front Frame, new castor housings and attaching hardware is required.

Item Number	Part Number	Description	Quantity
3, 4, 5, 6	501005	Seat Hook and Support Hardware Kit, Standard, Catalyst <i>This kit contains quantity of 1 each</i>	1
1a, 2	100270	Frame Assembly, Catalyst Ti Front, Left 14	1
1b, 2	100271	Frame Assembly, Catalyst Ti Front, Left 15	1
1c, 2	100272	Frame Assembly, Catalyst Ti Front, Left 16	1
1d, 2	100273	Frame Assembly, Catalyst Ti Front, Left 17	1
1e, 2	501639	Frame Assembly, Catalyst Ti Front, Left 18	1
1f, 2	501640	Frame Assembly, Catalyst Ti Front, Left 19	1
1g, 2	100274	Frame Assembly, Catalyst Ti Front, Left 20	1
1h, 2	100279	Frame Assembly, Catalyst Ti Front, Right 14	1
1i, 2	100280	Frame Assembly, Catalyst Ti Front, Right 15	1
1j, 2	100281	Frame Assembly, Catalyst Ti Front, Right 16	1
1k, 2	100282	Frame Assembly, Catalyst Ti Front, Right 17	1
1l, 2	501641	Frame Assembly, Catalyst Ti Front, Right 18	1
1m, 2	501642	Frame Assembly, Catalyst Ti Front, Right 19	1
1n, 2	100283	Frame Assembly, Catalyst Ti Front, Right 20	1
1c, 2	114361	Frame Assembly, Catalyst Ti Front with Flip Back Arm Modification, Left 16	1
1d, 2	114362	Frame Assembly, Catalyst Ti Front with Flip Back Arm Modification, Left 17	1
1e, 2	114389	Frame Assembly, Catalyst Ti Front with Flip Back Arm Modification, Left 18	1
1f, 2	114390	Frame Assembly, Catalyst Ti Front with Flip Back Arm Modification, Left 19	1
1g, 2	114363	Frame Assembly, Catalyst Ti Front with Flip Back Arm Modification, Left 20	1
1j, 2	114366	Frame Assembly, Catalyst Ti Front with Flip Back Arm Modification, Right 16	1
1k, 2	114367	Frame Assembly, Catalyst Ti Front with Flip Back Arm Modification, Right 17	1
1l, 2	114391	Frame Assembly, Catalyst Ti Front with Flip Back Arm Modification, Right 18	1
1m, 2	114392	Frame Assembly, Catalyst Ti Front with Flip Back Arm Modification, Right 19	1
1n, 2	114368	Frame Assembly, Catalyst Ti Front with Flip Back Arm Modification, Right 20	1
2	100804	Tube End Plug 1"	1
3	100831	1/2" Washer OD .040" Thick	1

Item Number	Part Number	Description	Quantity
4	000003	Seat Hook, Catalyst	1
5	000004	Seat Hook Support, Catalyst	1
6	100668	M6 x 20 CLS10.9 BTN SHCS ZC w/Patch	1
7	101819	M6 x 40 FHCS Blk Zn	1
8	000336	Latch Block	1
9	100746	M6 Flat Washer Blk Zn	1
10	100658	M6 Nylock Nut Blk Zn	1

Code	Color Name
Matte	
AGM	Army Green
EGM	Emerald Green
GGM	Graphite Gray
IBM	Iridescent Burgundy
INM	Iridescent Navy
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White
Gloss	
BLK	Black
SKY	Blue Skies
PNK	Bubblegum Pink
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
FUS	Fuchsia
GSM	Granny Smith
ORG	Orange
FLM	Graphic Image Transfer
Discontinued	
BLU	Blue
BUR	Burgundy
EBL	Electric Blue
GRN	Forest Green
SAP	Sapphire
GRY	Slate Gray