


# Ethos Rear Anti-Tip

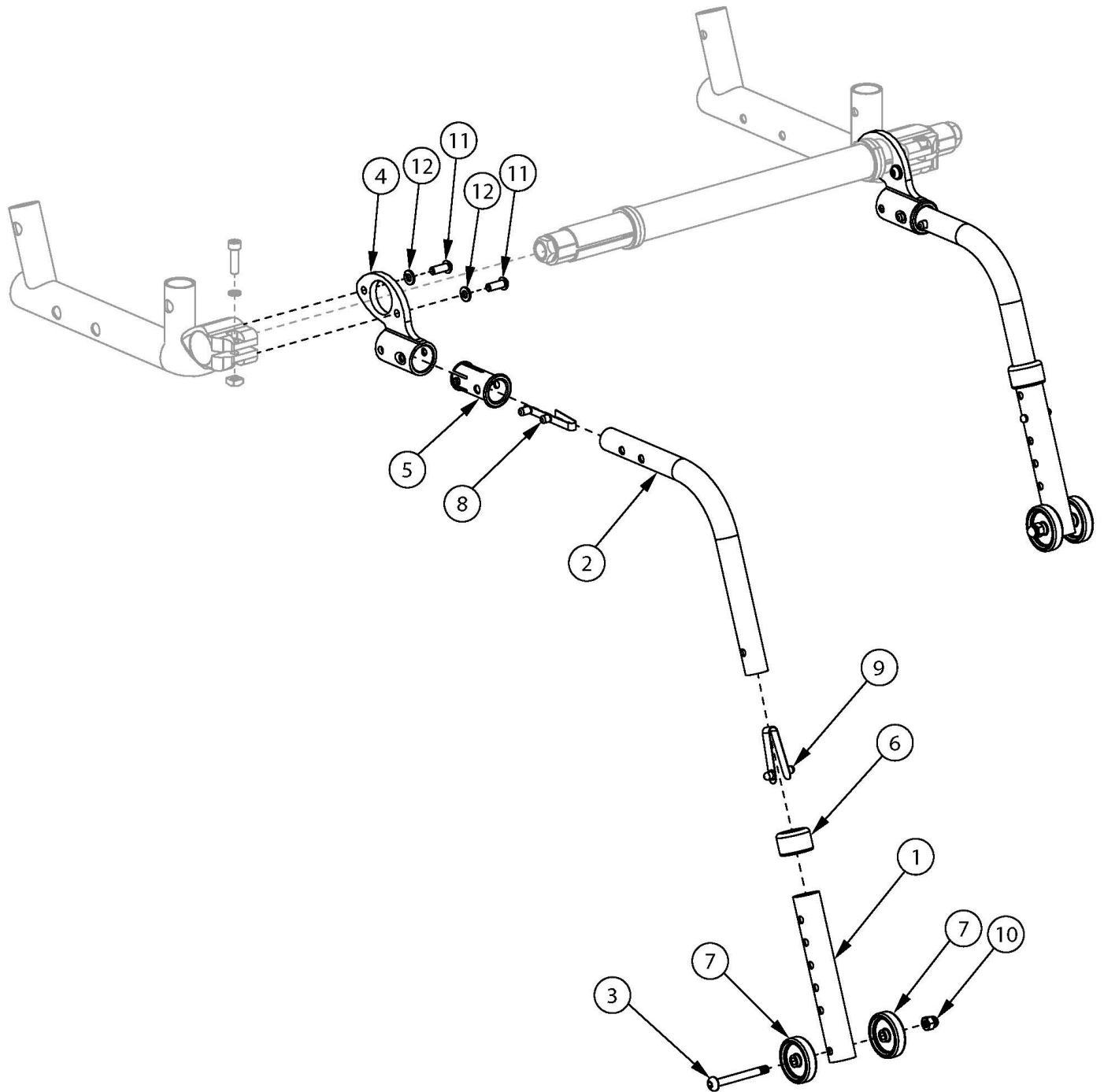

Item Number	Part Number	Description	Quantity	Retail
1, 2a, 3-12	108105	Rear Anti-Tip with Receiver Retrofit Kit, 22"-25" Rear Wheel, Ethos <i>This kit contains a pair of rear Anti-Tips with receivers and attaching hardware</i>	1	\$125.40
1, 2b, 3-12	108106	Rear Anti-Tip with Receiver Retrofit Kit, 26" Rear Wheel, Ethos	1	\$125.40

Item Number	Part Number	Description	Quantity	Retail
		<i>This kit contains a pair of rear Anti-Tips with receivers and attaching hardware</i>		
1, 2a, 3, 6-10	108102	Anti-Tip Assembly 22"-25" Wheel, Ethos	2	\$51.94
1, 2b, 3, 6-10	108103	Anti-Tip Assembly 26" Wheel, Ethos	2	\$51.94
1, 3, 7, 10	004691	Anti-Tip Lower Assembly	2	\$30.49
4, 5, 11, 12	108104	Rear Anti-Tip Receiver Kit, Ethos	2	\$10.76
2a	000077	Anti Tip Tube Upper Medium <i>Commonly used with 22" - 25" rear wheels</i>	2	\$18.20
2b	000078	Anti Tip Tube Upper Long <i>Commonly used with 26" rear wheels</i>	2	\$18.20
4	003982	Standard Anti-Tip Receiver	2	\$3.29
5	004187	Standard Anti Tip Sleeve	2	\$2.31
6	004306	Anti-Tip Anti Rattle Bushing	2	\$4.55
8	100533	Detent Button	2	\$3.52
9	100534	Detent Button Lower	2	\$3.52
11	101721	M6 x 16 Btn SHCS w/ Patch	4	\$2.06
12	101835	M6 Lock Washer Blk Zn	4	\$0.52