


Ethos Transit

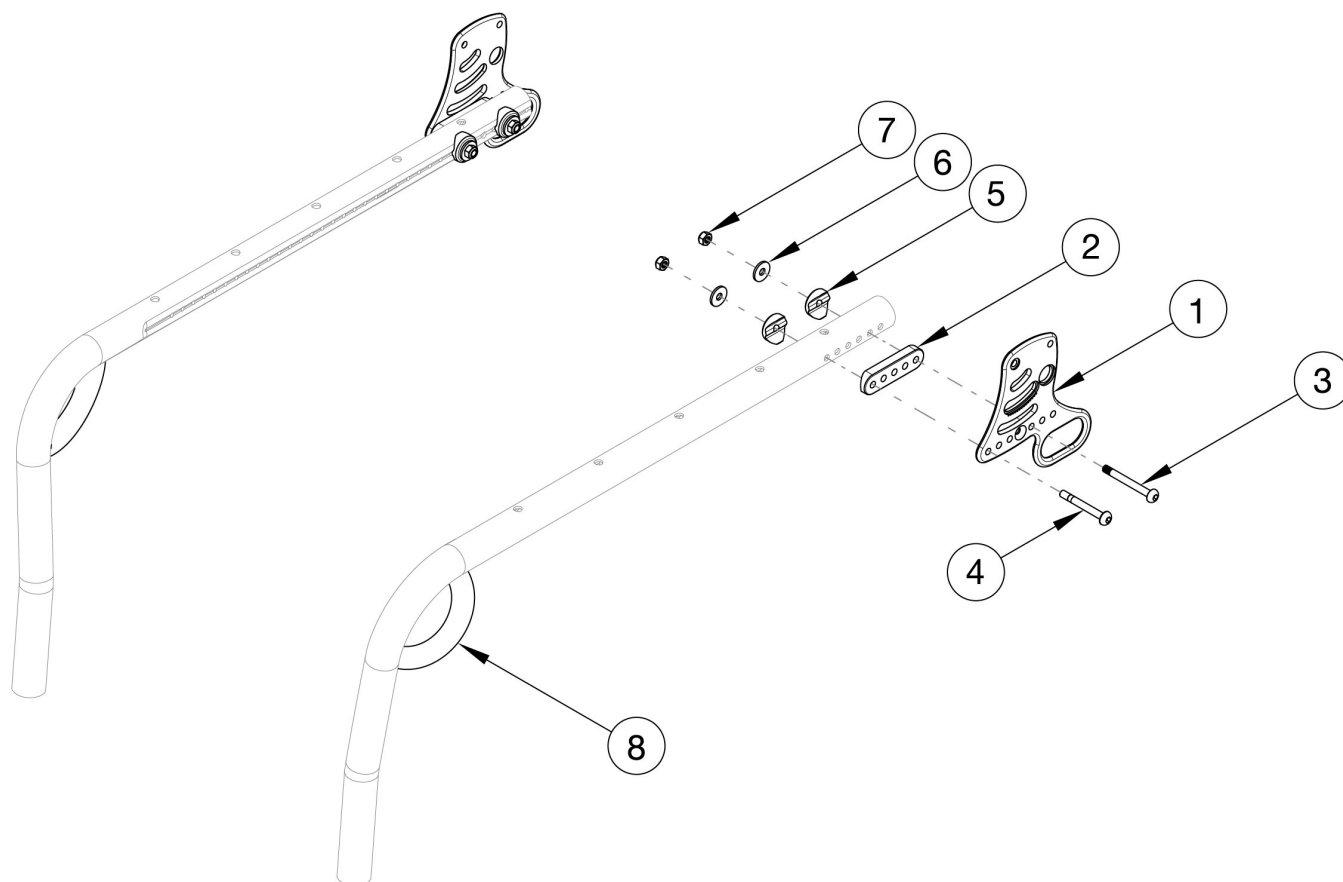

Ethos Transit cannot be retrofitted. If needing to add transit, please contact Customer Service for a Replacement Seat Frame. Utilize if chair has Rigid Locking Fold Down Back.

Item Number	Part Number	Description	Quantity
1a	115552	Back Plate, Transit, Right, Black	1
1b	115553	Back Plate, Transit, Left, Black	1
2	003848	Back Bracket Saddle 1 1/8" <i>Used if Swing Away Arms are not present</i>	2
3	003232	M6 x 55 BHCS Blk Zc <i>Used if Swing Away Arms are present</i>	4
4	003846	M6 x 50 BHCS, Custom, Blk Zc <i>Used if Swing Away Arms are not present</i>	4
5	003849	Saddle For Dimple 1 1/8"	4
6	101853	M6 Washer Blk Zn	4

Item Number	Part Number	Description	Quantity
7	100658	M6 Nylock Nut Blk Zn	4