


Catalyst E Anti-tips

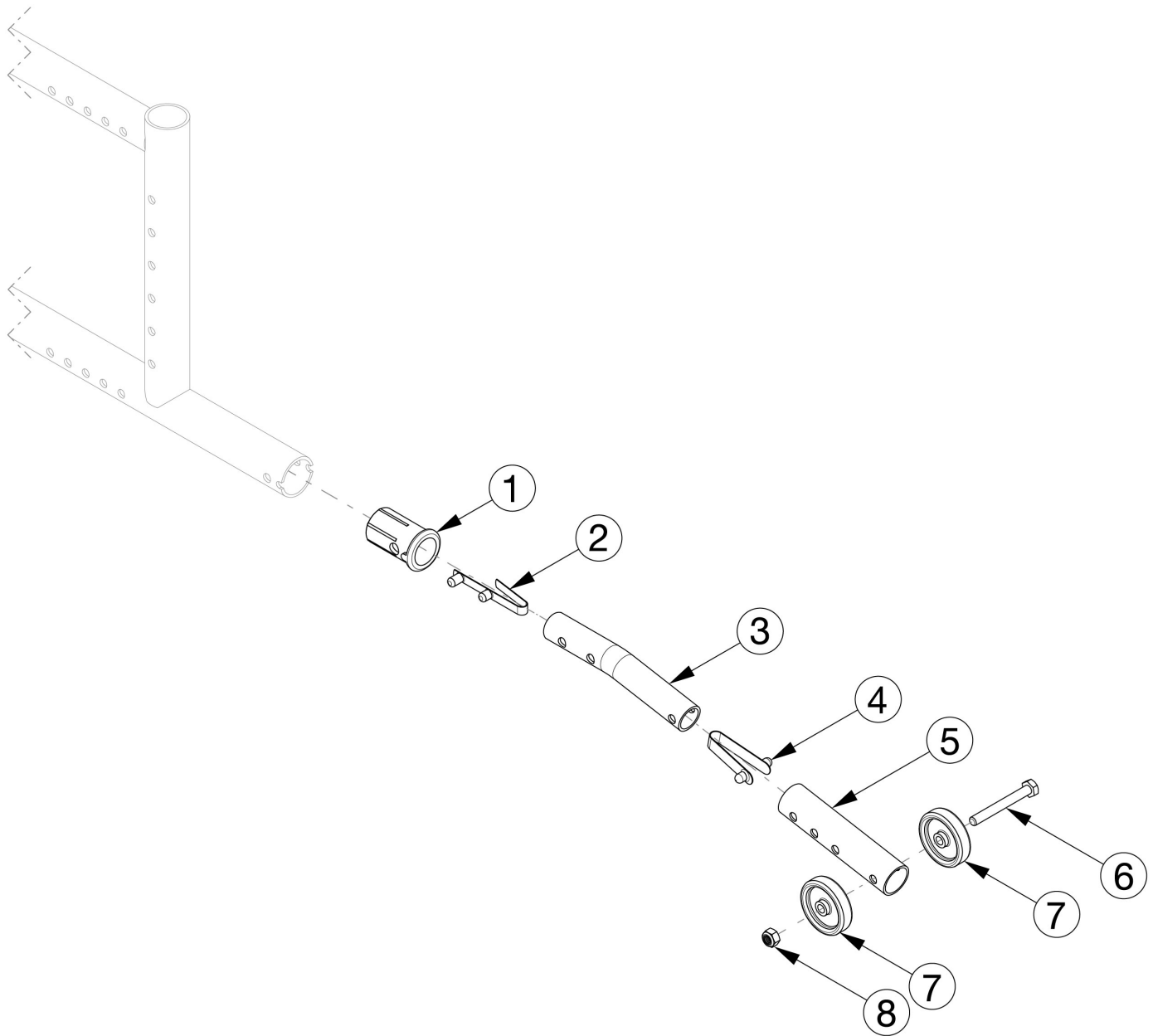

Item Number	Part Number	Description	Quantity
1-2, 3a, 4, 5a, 6-8	115302	Anti Tip Assembly Extra Short, Catalyst E <i>Used with 11.5" - 12.5" RSH</i>	1
1-2, 3b, 4, 5a, 6-8	115303	Anti Tip Assembly Short, Catalyst E <i>Used with 13" - 14" RSH</i>	1
1-2, 3c, 4, 5a, 6-8	115304	Anti Tip Assembly, Medium, Catalyst E <i>Used with 14.5" - 16" RSH</i>	1
1-2, 3d, 4, 5a, 6-8	115922	Anti Tip Assembly, Long, Catalyst E	1

Item Number	Part Number	Description	Quantity
		<i>Used with 16.5" - 18" RSH and 18" and 20" Wheels</i>	
1-2, 3e, 4, 5b, 6-8	115923	Anti Tip Assembly Extra Long, Catalyst E <i>Used with 18.5" - 20" RSH and 22" and 24" Wheels</i>	1
1	111943	Anti-Tip Sleeve, CAT E	1
2	100533	Detent Button	1
3a	112552	Anti-Tip Tube Upper Short, CAT E <i>Used with 11.5" - 12.5" RSH</i>	1
3b	112553	Anti-Tip Tube Upper Medium, CAT E <i>Used with 13" - 14" RSH</i>	1
3c	112554	Anti-Tip Tube Upper Long, CAT E <i>Used with 14.5" - 16" RSH</i>	1
3d	000991	Anti Tip Tube Upper Short <i>Used with 16.5" - 18" RSH and 18" and 20" Wheels</i>	1
3e	000077	Anti Tip Tube Upper Medium <i>Used with 18.5" - 20" RSH and 22" and 24" Wheels</i>	1
4	100534	Detent Button Lower	1
5a	000992	Anti-Tip Extension Short	1
5b	000076	Anti-Tip Extension Long, Anodized	1
6	101826	M6 x 50 HHCS Blk Zn	1
7	100506	Anti-Tip Tube Wheel	2
8	100658	M6 Nylock Nut Blk Zn	1