



Catalyst 5 Standard Caster Housing for Fixed Front Frame

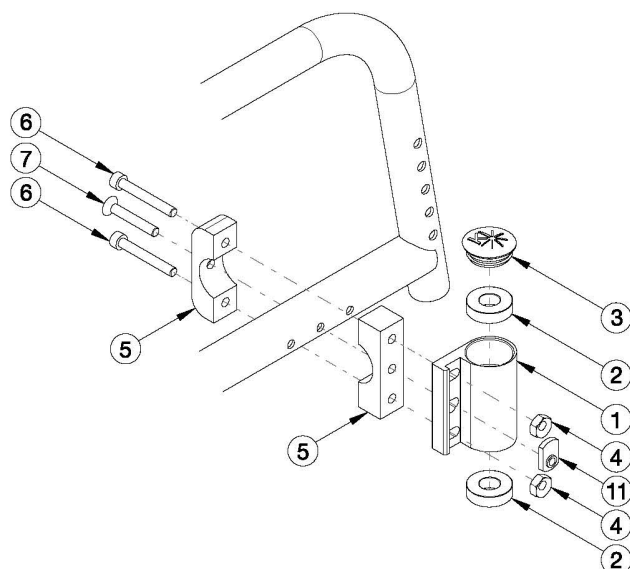

Fixed Front without Transit

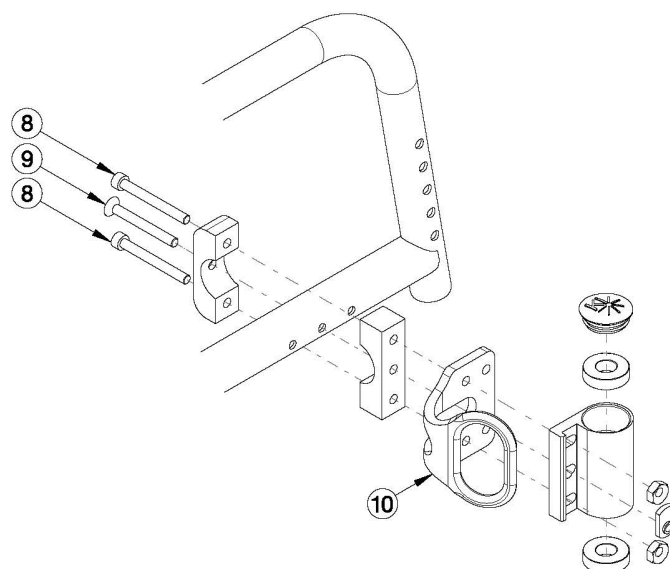

Fixed Front with Transit

<p>This design effective July 12th, 2021

Item Number	Part Number	Description	Quantity	Retail
1a, 2-7, 11	114444	Caster Housing Assembly, Fixed Front, Black, Left, Catalyst 5	1	\$73.37
1b, 2-7, 11	114444_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, Left, Catalyst 5	1	\$141.66
1c, 2-7, 11	114445	Caster Housing Assembly, Fixed Front, Black, Right, Catalyst 5	1	\$73.37
1d, 2-7, 11	114445_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, Right, Catalyst 5	1	\$141.66
1a, 1c, 2-7, 11	114446	Caster Housing Assembly, Fixed Front, Black, PAIR, Catalyst 5	1	\$146.74
1b, 1d, 2-7, 11	114446_ _ _	Color Anodized Caster Housing Assembly, Fixed Front, PAIR, Catalyst 5	1	\$283.32
1a, 2-5, 8, 9, 11	114447	Caster Housing Assembly, Fixed Front with Transit, Black, Left, Catalyst 5 <i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	1	\$78.61
1b, 2-5, 8, 9, 11	114447_ _ _	Color Anodized Caster Housing Assembly, Fixed Front with Transit, Left, Catalyst 5 <i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	1	\$146.90
1c, 2-5, 8, 9, 11	114448	Caster Housing Assembly, Fixed Front with Transit, Black, Right, Catalyst 5 <i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	1	\$78.61
1d, 2-5, 8, 9, 11	114448_ _ _	Color Anodized Caster Housing Assembly, Fixed Front with Transit, Right, Catalyst 5	1	\$146.90

Item Number	Part Number	Description	Quantity	Retail
		<i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>		
1a, 1c, 2-5, 8, 9, 11	114449	Caster Housing Assembly, Fixed Front with Transit, Black, PAIR, Catalyst 5 <i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	1	\$157.22
1b, 1d, 2-5, 8, 9, 11	114449_ _ _	Color Anodized Caster Housing Assembly, Fixed Front with Transit, PAIR, Catalyst 5 <i>Front transit bracket shown for reference only. Transit bracket is not included in this assembly.</i>	1	\$293.80
1a, 2	001429	Caster Housing Fixed Front Left, Black, Catalyst	1	\$37.66
1b, 2	001429_ _ _	Color Anodized Caster Housing Fixed Front, Left, Catalyst	1	\$105.95
1c, 2	001430	Caster Housing Fixed Front Right, Black, Catalyst	1	\$37.66
1d, 2	001430_ _ _	Color Anodized Caster Housing Fixed Front, Right, Catalyst	1	\$105.95
2	100530	Bearing M28 x 1/2" 6001RS	2	\$12.02
3	102005	Ki Caster Housing Cap	1	\$1.17
4	100745	Eccentric nut (threaded)	2	\$1.33
5	100528	Caster Housing Clamp	1	\$23.47
6	101472	M6 x 45 SHCS BLK ZC	2	\$1.58
7	101130	M6 x 55 FSHCS P/THD BLKZC	1	\$2.10
8	102473	M6 X 50 CLS12.9 SHCS BLK ZC	2	\$4.20
9	103068	M6 x 60mm FHCS Blk Zn	1	\$2.10
11	102092	M6 Weld Nut Offset	1	\$3.15

Color Anodizing Samples

