


Superlight Handrim

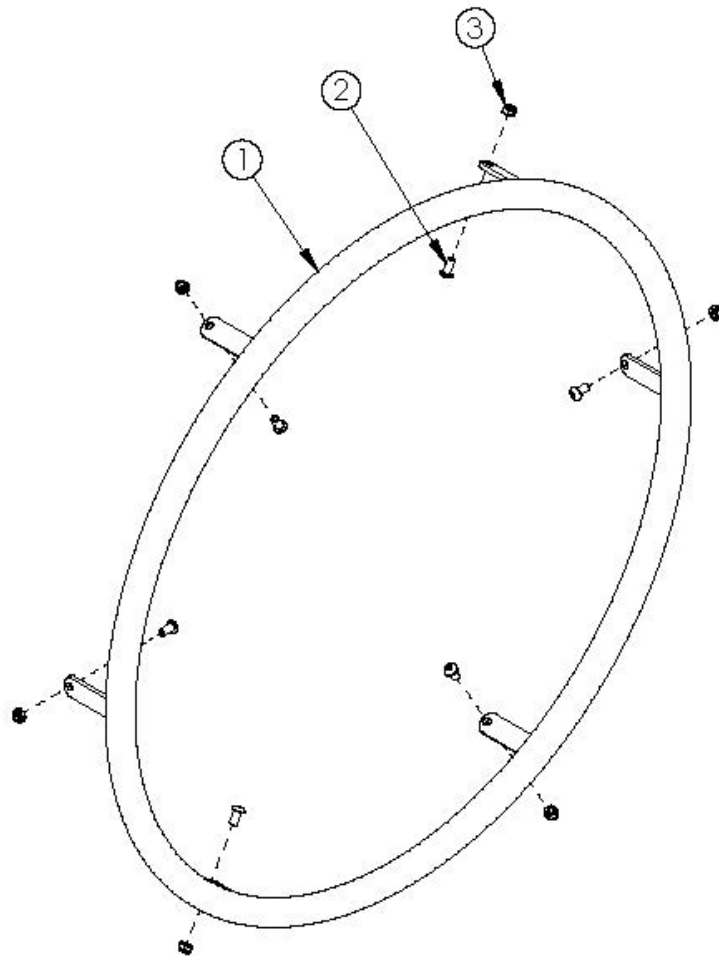

Superlight Handrim with Micro Grip



For more information see Handrim Hardware Chart. Quantity in Handrim Hardware Kits are for one wheel. Not available with Heavy Duty Option Tab mounted handrims must be installed prior to the tire being installed.

Item Number	Part Number	Description	Quantity
2b, 3	502174	Handrim Hardware Kit, 6 Tab, EACH <i>Used with Spinergy, Topolino, MAXX Spoke, and Spoke wheel</i>	1
2a	502170	Handrim Hardware Kit, Superlight, EACH <i>Used with Superlight Spoke</i>	1
1a	101161	Handrim Superlight Aluminum Anodized 24", 6 Tab	1
1b	505585	Handrim Superlight Aluminum Anodized 24" w/ Micro Grip	1
1c	101160	Handrim Superlight Aluminum Anodized 25", 6 Tab	1
1d	505586	Handrim Superlight Aluminum Anodized 25" w/ Micro Grip	1

Item Number	Part Number	Description	Quantity
2a	100536	M5 x 12 BTN SHCS ZC <i>Used with Superlight Spoke</i>	6
2b	100669	M5 x 10 BTN SHCS BLKZC w/Patch <i>Used with Spinergy, Topolino, MAXX Spoke, and Spoke wheel</i>	6
3	100657	M5 Nylock Nut Blk Zn <i>Used with Spinergy, Topolino, MAXX Spoke, and Spoke wheel</i>	6

Handrim Hardware Chart

Wheel		Handrim Connection				Handrim									
Wheel	Wheel Part No.	Connection Points	Rivet/Tab	Spacer*	Screw*	Aluminum Anodized	Superlight	Plastic Coated	Projection	Ergonomic Standard	Ergonomic LT	Natural Fit Standard	Natural Fit LT	Flex Rim	
22" Spoke	116730	5 or 6	116732 (Used with 5 Rivet) 100698 (Used with 6 Rivet)	100653 (Not used w/ Superlight Handrim)	Aluminum, Plastic Ctd, Projection Ergonomic: 100654 Natural Fit: 100835 Superlight: Screw: 100669 Nut: 100657	101898 (5 Rivet)		101964 (5 Rivet)	112824 (6 Rivet)	113082 (6 Rivet)	113085 (6 Rivet)	200538 (6 Rivet)	200201 (6 Rivet)		
24" Spoke	116728	5 or 6	Not used w/ Superlight Handrims			100975 (5 Rivet)	101161 (6 Tab)	100976 (5 Rivet)	112825 (6 Rivet)	113083 (6 Rivet)	113806 (6 Rivet)	100793 (6 Rivet)	200202 (6 Rivet)		
18" MAXX Spoke	109285	3	100698 (Not used w/ Superlight Handrim)	100653 (Not used w/ Superlight Handrim)	Aluminum, Plastic Ctd, Projection, Ergonomic: 100654 Natural Fit: 100835 Superlight: Screw: 100669 Nut: 100657	100206									
20" MAXX Spoke	108244	6				200536		112819							
22" MAXX Spoke	105135					100560		112820	112824	113082	113085	200538	200201		
24" MAXX Spoke	105136					200349	101161	112821	112825	113083	113086	100793	200202		
25" MAXX Spoke	107436					200350		112822	200548	113084	113087	200539	200540		
26" MAXX Spoke	107437					200351		112823	200549			100907	101454		
20" MAXX Mag	111853	6		Aluminum, Plastic Coated, Projection: 100629 Natural Fit & Ergonomic: 101756	Aluminum, Plastic Coated, Projection, Ergonomic: 103545 Natural Fit: 101893	200536		112819							
22" MAXX Mag	111854		100560				112820	112824	113082	113085	200538	200201			
24" MAXX Mag	111855		105137				112821	112825	113083	113086	100793	200202			
24" Superlight	101159	6			100536	100754	101161	100836		113080	113081	100830	100828		
25" Superlight	101160							101160	101091			101464	101460		

* Spacer and screw part numbers listed in the chart are for standard handrim mounting. If using close mount handrim mounting on aluminum anodized, plastic coated or projection handrims, use part number 100792 for the spacer, part number 102616 for the screw on Ki Spoke and MAXX Spoke wheels, part number 100666 for the screw on 5-Spoke X Core wheels, or part number 100654 for the screw on MAXX Mag wheels.

Effective 3/17/2023

Handrim Hardware Chart (Continued)													
20" Spinergy Spox	See Spinergy Spox Page	6			Screw: 100669	103125		103179					
22" Spinergy Spox						100827		100808			100889	100888	
24" Spinergy Spox					Nut: 100657	100766	101161	100615			100830	100828	
25" Spinergy Spox						100767	101160	100765			101464	101460	
26" Spinergy Spox						101477		101148			200200	100950	
22" Spinergy LX	See Spinergy LX Page	6			Screw: 100669	100827		100808			100889	100888	
24" Spinergy LX						100766	101161	100615		113080	113081	100830	100828
25" Spinergy LX					Nut: 100657	100767	101160	100765			101464	101460	See Spinergy LX Page
26" Spinergy LX						101477		101148			200200	100950	

Handrim Hardware Chart (Discontinued Wheels)													
Wheel		Handrim Connection				Handrim							
Wheel	Wheel Part No.	Connection Points	Rivet/Tab	Spacer*	Screw*	Aluminum Anodized	Superlight	Plastic Coated	Projection	Natural Fit Standard	Natural Fit LT	Flex Rim	
18" Ki Spoke	200529	3	100698	100653	Aluminum, Plastic Ctd, Projection: 100654	100206		101106					
20" Ki Spoke	200530	6				200536		200542					
22" Ki Spoke	200531					100560		100576	100569	200538	200201		
24" Ki Spoke	200532					200349		100577	200547	100793	200202		
25" Ki Spoke	200533					200350		101870	200548	200539	200540		
26" Ki Spoke	200534					200351		100578	200549	100907	101454		
20" 5-Spoke X Core	101961	5		Aluminum, Projection, Natural Fit: 100629	100724	101897		101963					
22" 5-Spoke X Core	101962					101898		101964					
24" 5-Spoke X Core	100960					100975		100976	200546				
24" 5-Spoke X Core	100960			Plastic Ctd: 101756	101893				100768	100769			

* Spacer and screw part numbers listed in the chart are for standard handrim mounting. If using close mount handrim mounting on aluminum anodized, plastic coated or projection handrims, use part number 100792 for the spacer, part number 102616 for the screw on Ki Spoke and MAXX Spoke wheels, part number 100666 for the screw on 5-Spoke X Core wheels, or part number 100654 for the screw on MAXX Mag wheels.

Effective 3/17/2023