


Catalyst E Aluminum Cross Braces

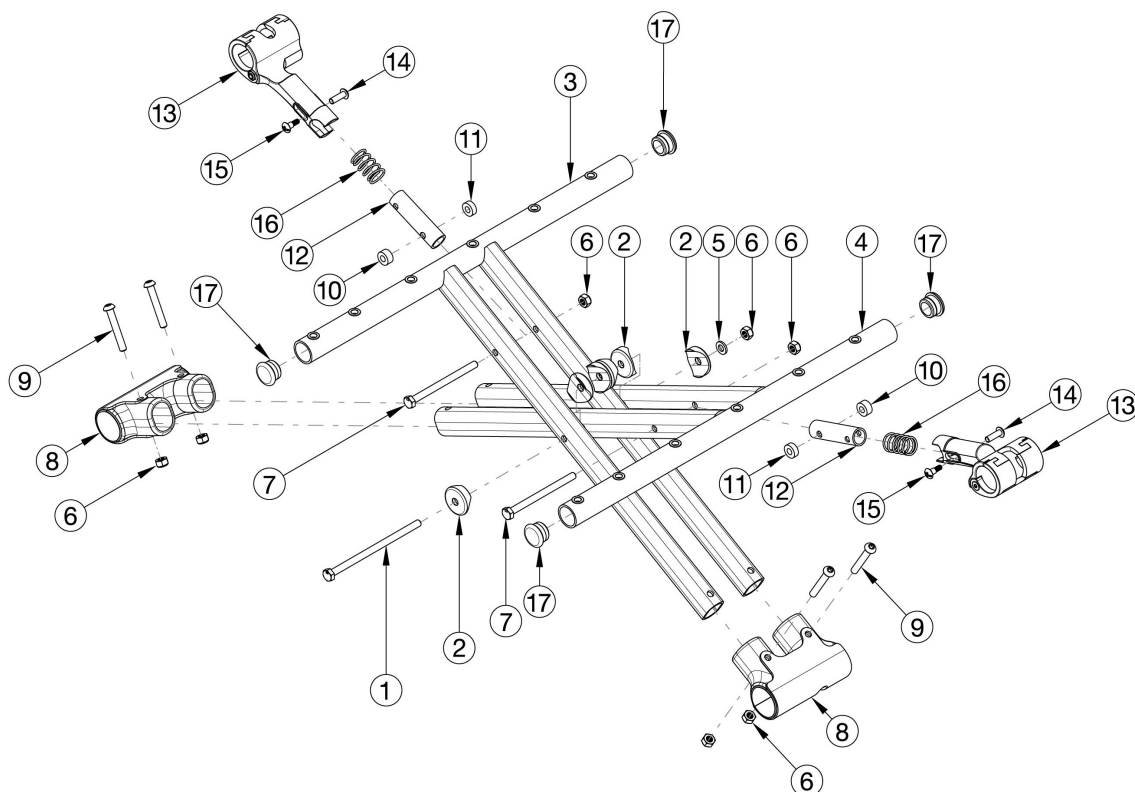

Cross Brace Overall Width	
Seat Width	Cross Brace Measurement
14"-16"	Approx. 1" less than seat width
17"	Approx. 1.25" less than seat width
18"-21"	Approx. 1.5" less than seat width
22"	Approx. 1.75" less than seat width

Cross Brace Overall Depth	
Seat Depth	Cross Brace Measurement
14"-20"	Approx. 0.375" less than seat depth

Assembled cross brace kits do not include upper pivots, lower pivots, or seat upholstery.

Item Number	Part Number	Description	Quantity
1-7, 9-11, 17	122563	Assembled Cross Brace Kit, 14 W x 15 D, Catalyst E	1
1-7, 9-11, 17	121987	Assembled Cross Brace Kit, 14 W x 16 D, Catalyst E	1
1-7, 9-11, 17	122564	Assembled Cross Brace Kit, 14 W x 17 D, Catalyst E	1

Item Number	Part Number	Description	Quantity
1-7, 9-11, 17	121988	Assembled Cross Brace Kit, 14 W x 18 D, Catalyst E	1
1-7, 9-11, 17	122565	Assembled Cross Brace Kit, 14 W x 19 D, Catalyst E	1
1-7, 9-11, 17	121989	Assembled Cross Brace Kit, 14 W x 20 D, Catalyst E	1
1-7, 9-11, 17	122566	Assembled Cross Brace Kit, 15 W x 15 D, Catalyst E	1
1-7, 9-11, 17	121991	Assembled Cross Brace Kit, 15 W x 16 D, Catalyst E	1
1-7, 9-11, 17	125130	Assembled Cross Brace Kit, 15 W x 17 D, Catalyst E	1
1-7, 9-11, 17	121992	Assembled Cross Brace Kit, 15 W x 18 D, Catalyst E	1
1-7, 9-11, 17	125131	Assembled Cross Brace Kit, 15 W x 19 D, Catalyst E	1
1-7, 9-11, 17	121993	Assembled Cross Brace Kit, 15 W x 20 D, Catalyst E	1
1-7, 9-11, 17	125132	Assembled Cross Brace Kit, 16 W x 15 D, Catalyst E	1
1-7, 9-11, 17	121994	Assembled Cross Brace Kit, 16 W x 16 D, Catalyst E	1
1-7, 9-11, 17	125133	Assembled Cross Brace Kit, 16 W x 17 D, Catalyst E	1
1-7, 9-11, 17	121995	Assembled Cross Brace Kit, 16 W x 18 D, Catalyst E	1
1-7, 9-11, 17	125254	Assembled Cross Brace Kit, 16 W x 19 D, Catalyst E	1
1-7, 9-11, 17	121996	Assembled Cross Brace Kit, 16 W x 20 D, Catalyst E	1
1-7, 9-11, 17	125134	Assembled Cross Brace Kit, 17 W x 15 D, Catalyst E	1
1-7, 9-11, 17	121997	Assembled Cross Brace Kit, 17 W x 16 D, Catalyst E	1
1-7, 9-11, 17	121998	Assembled Cross Brace Kit, 17 W x 18 D, Catalyst E	1
1-7, 9-11, 17	125135	Assembled Cross Brace Kit, 17 W x 19 D, Catalyst E	1
1-7, 9-11, 17	121999	Assembled Cross Brace Kit, 17 W x 20 D, Catalyst E	1
1-7, 9-11, 17	125137	Assembled Cross Brace Kit, 18 W x 15 D, Catalyst E	1
1-7, 9-11, 17	122000	Assembled Cross Brace Kit, 18 W x 16 D, Catalyst E	1
1-7, 9-11, 17	125138	Assembled Cross Brace Kit, 18 W x 17 D, Catalyst E	1
1-7, 9-11, 17	122001	Assembled Cross Brace Kit, 18 W x 18 D, Catalyst E	1
1-7, 9-11, 17	125140	Assembled Cross Brace Kit, 18 W x 19 D, Catalyst E	1
1-7, 9-11, 17	122002	Assembled Cross Brace Kit, 18 W x 20 D, Catalyst E	1
1-7, 9-11, 17	125141	Assembled Cross Brace Kit, 19 W x 15 D, Catalyst E	1

Item Number	Part Number	Description	Quantity
1-7, 9-11, 17	122003	Assembled Cross Brace Kit, 19 W x 16 D, Catalyst E	1
1-7, 9-11, 17	125142	Assembled Cross Brace Kit, 19 W x 17 D, Catalyst E	1
1-7, 9-11, 17	122004	Assembled Cross Brace Kit, 19 W x 18 D, Catalyst E	1
1-7, 9-11, 17	125143	Assembled Cross Brace Kit, 19 W x 19 D, Catalyst E	1
1-7, 9-11, 17	122005	Assembled Cross Brace Kit, 19 W x 20 D, Catalyst E	1
1-7, 9-11, 17	125144	Assembled Cross Brace Kit, 20 W x 15 D, Catalyst E	1
1-7, 9-11, 17	122006	Assembled Cross Brace Kit, 20 W x 16 D, Catalyst E	1
1-7, 9-11, 17	125145	Assembled Cross Brace Kit, 20 W x 17 D, Catalyst E	1
1-7, 9-11, 17	122007	Assembled Cross Brace Kit, 20 W x 18 D, Catalyst E	1
1-7, 9-11, 17	125146	Assembled Cross Brace Kit, 20 W x 19 D, Catalyst E	1
1-7, 9-11, 17	122008	Assembled Cross Brace Kit, 20 W x 20 D, Catalyst E	1
1-7, 9-11, 17	125147	Assembled Cross Brace Kit, 21 W x 15 D, Catalyst E	1
1-7, 9-11, 17	122009	Assembled Cross Brace Kit, 21 W x 16 D, Catalyst E	1
1-7, 9-11, 17	125148	Assembled Cross Brace Kit, 21 W x 17 D, Catalyst E	1
1-7, 9-11, 17	122010	Assembled Cross Brace Kit, 21 W x 18 D, Catalyst E	1
1-7, 9-11, 17	125255	Assembled Cross Brace Kit, 21 W x 19 D, Catalyst E	1
1-7, 9-11, 17	122011	Assembled Cross Brace Kit, 21 W x 20 D, Catalyst E	1
1-7, 9-11, 17	125256	Assembled Cross Brace Kit, 22 W x 15 D, Catalyst E	1
1-7, 9-11, 18	122215	Assembled Cross Brace Kit, 22 W x 16 D, Catalyst E	1
1-7, 9-11, 17	125257	Assembled Cross Brace Kit, 22 W x 17 D, Catalyst E	1
1-7, 9-11, 19	122216	Assembled Cross Brace Kit, 22 W x 18 D, Catalyst E	1
1-7, 9-11, 17	125258	Assembled Cross Brace Kit, 22 W x 19 D, Catalyst E	1
1-7, 9-11, 20	122217	Assembled Cross Brace Kit, 22 W x 20 D, Catalyst E	1
12a, 13 - 16	115770	Upper Pivot Assembly, 14" Wide	1
12b, 13-16	117402	Upper Pivot Assembly, 15" Wide	1
12c, 13-16	115771	Upper Pivot Assembly, 16" Wide	1
12d, 13-16	117403	Upper Pivot Assembly, 17" Wide	1

Item Number	Part Number	Description	Quantity
12e, 13-16	115772	Upper Pivot Assembly, 18" Wide	1
12f, 13-16	117404	Upper Pivot Assembly, 19" Wide	1
12g, 13-16	115773	Upper Pivot Assembly, 20" Wide	1
12h, 13-16	115774	Upper Pivot Assembly, 21" / 22" Wide	1
6, 8, 9	115775	Lower Pivot Assembly	1
1 - 2, 5 - 7, 10 - 11	115776	Cross Brace Hardware Kit	1
3a, 4a, 17	125259	Catalyst E Cross Brace, Aluminum, 14 x 15, PAIR	1
3b, 4b, 17	122012	Catalyst E Cross Brace, Aluminum, 14 x 16, PAIR	1
3c, 4c, 17	122013	Catalyst E Cross Brace, Aluminum, 14 x 18, PAIR	1
3d, 4d, 17	122014	Catalyst E Cross Brace, Aluminum, 14 x 20, PAIR	1
3e, 4e, 17	125294	Catalyst E Cross Brace, Aluminum, 15 x 15, PAIR	1
3f, 4f, 17	122015	Catalyst E Cross Brace, Aluminum, 15 x 16, PAIR	1
3g, 4g, 17	125295	Catalyst E Cross Brace, Aluminum, 15 x 17, PAIR	1
3h, 4h, 17	122016	Catalyst E Cross Brace, Aluminum, 15 x 18, PAIR	1
3i, 4i, 17	125296	Catalyst E Cross Brace, Aluminum, 15 x 19, PAIR	1
3j, 4j, 17	122017	Catalyst E Cross Brace, Aluminum, 15 x 20, PAIR	1
3k, 4k, 17	125297	Catalyst E Cross Brace, Aluminum, 16 x 15, PAIR	1
3l, 4l, 17	122018	Catalyst E Cross Brace, Aluminum, 16 x 16, PAIR	1
3m, 4m, 17	125298	Catalyst E Cross Brace, Aluminum, 16 x 17, PAIR	1
3n, 4n, 17	122019	Catalyst E Cross Brace, Aluminum, 16 x 18, PAIR	1
3o, 4o, 17	125260	Catalyst E Cross Brace, Aluminum, 16 x 19, PAIR	1
3p, 4p, 17	122020	Catalyst E Cross Brace, Aluminum, 16 x 20, PAIR	1
3q, 4q, 17	125299	Catalyst E Cross Brace, Aluminum, 17 x 15, PAIR	1
3r, 4r, 17	122021	Catalyst E Cross Brace, Aluminum, 17 x 16, PAIR	1
3s, 4s, 17	125300	Catalyst E Cross Brace, Aluminum, 17 x 17, PAIR	1
3t, 4t, 17	122022	Catalyst E Cross Brace, Aluminum, 17 x 18, PAIR	1
3u, 4u, 17	125301	Catalyst E Cross Brace, Aluminum, 17 x 19, PAIR	1
3v, 4v, 17	122023	Catalyst E Cross Brace, Aluminum, 17 x 20, PAIR	1

Item Number	Part Number	Description	Quantity
3w, 4w, 17	125302	Catalyst E Cross Brace, Aluminum, 18 x 15, PAIR	1
3x, 4x, 17	122024	Catalyst E Cross Brace, Aluminum, 18 x 16, PAIR	1
3y, 4y, 17	125303	Catalyst E Cross Brace, Aluminum, 18 x 17, PAIR	1
3z, 4z, 17	122025	Catalyst E Cross Brace, Aluminum, 18 x 18, PAIR	1
3ab, 4ab, 17	125304	Catalyst E Cross Brace, Aluminum, 18 x 19, PAIR	1
3ac, 4ac, 17	122026	Catalyst E Cross Brace, Aluminum, 18 x 20, PAIR	1
3ad, 4ad, 17	125305	Catalyst E Cross Brace, Aluminum, 19 x 15, PAIR	1
3ae, 4ae, 17	122027	Catalyst E Cross Brace, Aluminum, 19 x 16, PAIR	1
3af, 4af, 17	125306	Catalyst E Cross Brace, Aluminum, 19 x 17, PAIR	1
3ag, 4ag, 17	122028	Catalyst E Cross Brace, Aluminum, 19 x 18, PAIR	1
3ah, 4ah, 17	125307	Catalyst E Cross Brace, Aluminum, 19 x 19, PAIR	1
3ai, 4ai, 17	122029	Catalyst E Cross Brace, Aluminum, 19 x 20, PAIR	1
3aj, 4aj, 17	125308	Catalyst E Cross Brace, Aluminum, 20 x 15, PAIR	1
3ak, 4ak, 17	122030	Catalyst E Cross Brace, Aluminum, 20 x 16, PAIR	1
3al, 4al, 17	125309	Catalyst E Cross Brace, Aluminum, 20 x 17, PAIR	1
3am, 4am, 17	122031	Catalyst E Cross Brace, Aluminum, 20 x 18, PAIR	1
3an, 4an, 17	125310	Catalyst E Cross Brace, Aluminum, 20 x 19, PAIR	1
3ao, 4ao, 17	122032	Catalyst E Cross Brace, Aluminum, 20 x 20, PAIR	1
3ap, 4ap, 17	125311	Catalyst E Cross Brace, Aluminum, 21 x 15, PAIR	1
3aq, 4aq, 17	122033	Catalyst E Cross Brace, Aluminum, 21 x 16, PAIR	1
3ar, 4ar, 17	125312	Catalyst E Cross Brace, Aluminum, 21 x 17, PAIR	1
3as, 4as, 17	122034	Catalyst E Cross Brace, Aluminum, 21 x 18, PAIR	1
3at, 4at, 17	125261	Catalyst E Cross Brace, Aluminum, 21 x 19, PAIR	1
3au, 4au, 17	122035	Catalyst E Cross Brace, Aluminum, 21 x 20, PAIR	1
3av, 4av, 17	125262	Catalyst E Cross Brace, Aluminum, 22 x 15, PAIR	1
3aw, 4aw, 17	122036	Catalyst E Cross Brace, Aluminum, 22 x 16, PAIR	1
3ax, 4ax, 17	125263	Catalyst E Cross Brace, Aluminum, 22 x 17, PAIR	1
3ay, 4ay, 17	122037	Catalyst E Cross Brace, Aluminum, 22 x 18, PAIR	1

Item Number	Part Number	Description	Quantity
3az, 4az, 17	125264	Catalyst E Cross Brace, Aluminum, 22 x 19, PAIR	1
3ba, 4ba, 17	122038	Catalyst E Cross Brace, Aluminum, 22 x 20, PAIR	1
1	100678	M6 x 110 CLS8.8 DIN931 HHCS ZC	1
2	002807	Cross Tube Saddle CAT	6
5	100746	M6 Flat Washer Blk Zn	1
7	100660	M6 x 80mm CLS8.8 DIN931 HHCS ZC	2
10	100880	1/4" Plastic Spacer	2
11	105019	3/16" Plastic Spacer	2