


## Catalyst E Armrests - Height Adjustable T-Arm

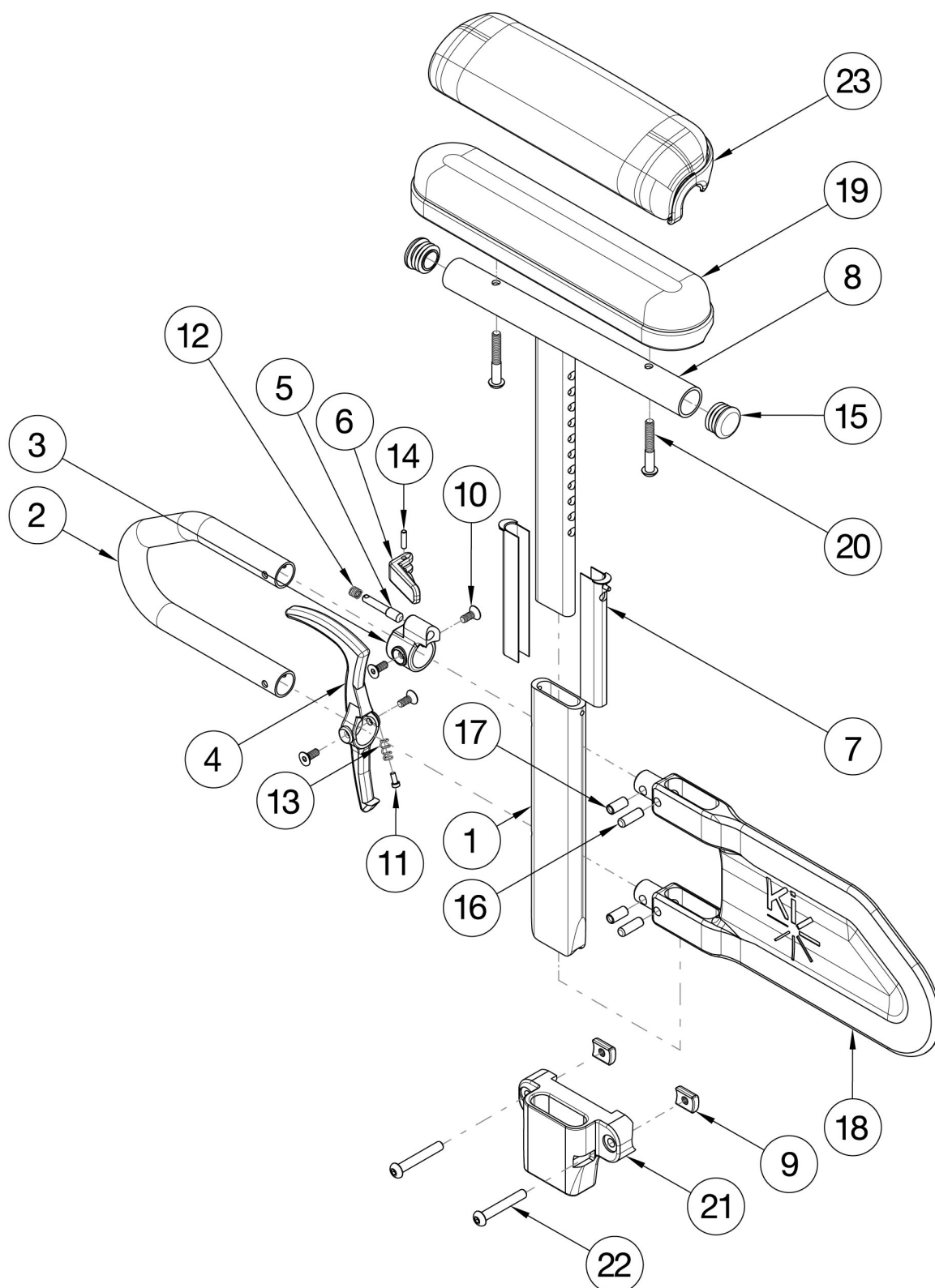

Short assemblies used with 14 - 16 Seat Depths. Long assemblies used with 17" - 20" Seat Depths. If retrofitting, Receivers are needed.

| Item Number                              | Part Number | Description                                                                                      | Quantity |
|------------------------------------------|-------------|--------------------------------------------------------------------------------------------------|----------|
| 1-7, 8a, 9-18a, 19a, 20a, 21, 22, 24, 25 | 115892      | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, 1" Receiver, Catalyst E         | 1        |
| 1-7, 8a, 9-18a, 19a, 20a, 21, 22, 24, 25 | 115893      | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, 1" Receiver, Catalyst E, PAIR   | 1        |
| 1-7, 8a, 9-18b, 19a, 20a, 21, 22, 24, 25 | 115894      | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1" Receiver, Catalyst E        | 1        |
| 1-7, 8a, 9-18b, 19a, 20a, 21, 22, 24, 25 | 115895      | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Full, 1" Receiver, Catalyst E, PAIR  | 1        |
| 1-7, 8a, 9-18a, 19b, 20a, 21, 22, 24, 25 | 115896      | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, 1" Receiver, Catalyst E         | 1        |
| 1-7, 8a, 9-18a, 19b, 20a, 21, 22, 24, 25 | 115897      | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, 1" Receiver, Catalyst E, PAIR   | 1        |
| 1-7, 8a, 9-18b, 19b, 20a, 21, 22, 24, 25 | 115898      | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1" Receiver, Catalyst E        | 1        |
| 1-7, 8a, 9-18b, 19b, 20a, 21, 22, 24, 25 | 115899      | Height Adj T-Arm Assm w/ Short Side Shield and Standard Pad Desk, 1" Receiver, Catalyst E, PAIR  | 1        |
| 1-7, 8a, 9-18a, 20b, 21, 22, 23a, 24, 25 | 115900      | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, 1" Receiver, Catalyst E        | 1        |
| 1-7, 8a, 9-18a, 20b, 21, 22, 23a, 24, 25 | 115901      | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, 1" Receiver, Catalyst E, PAIR  | 1        |
| 1-7, 8a, 9-18b, 20b, 21, 22, 23a, 24, 25 | 115902      | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, 1" Receiver, Catalyst E       | 1        |
| 1-7, 8a, 9-18b, 20b, 21, 22, 23a, 24, 25 | 115903      | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Full, 1" Receiver, Catalyst E, PAIR | 1        |
| 1-7, 8a, 9-18a, 20b, 21, 22, 23b, 24, 25 | 115904      | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, 1" Receiver, Catalyst E        | 1        |
| 1-7, 8a, 9-18a, 20b, 21, 22, 23b, 24, 25 | 115905      | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, 1" Receiver, Catalyst E, PAIR  | 1        |
| 1-7, 8a, 9-18b, 20b, 21, 22, 23b, 24, 25 | 115906      | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, 1" Receiver, Catalyst E       | 1        |
| 1-7, 8a, 9-18b, 20b, 21, 22, 23b, 24, 25 | 115907      | Height Adj T-Arm Assm w/ Short Side Shield and Waterfall Pad Desk, 1" Receiver, Catalyst E, PAIR | 1        |
| 1-7, 8a, 10-18a, 19a, 20a                | 505491      | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full                                  | 1        |
| 1-7, 8a, 10-18a, 19a, 20a                | 505492      | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Full, PAIR                            | 1        |

| Item Number               | Part Number | Description                                                                                                    | Quantity |
|---------------------------|-------------|----------------------------------------------------------------------------------------------------------------|----------|
| 1-7, 8a, 10-18b, 19a, 20a | 505644      | Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Full                                    | 1        |
| 1-7, 8a, 10-18b, 19a, 20a | 505645      | Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Full, Pair                              | 1        |
| 1-7, 8a, 10-18a, 19b, 20a | 505493      | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk                                                | 1        |
| 1-7, 8a, 10-18a, 19b, 20a | 505494      | Height Adj T-Arm Assm w/ Long Side Shield and Standard Pad Desk, PAIR                                          | 1        |
| 1-7, 8a, 10-18b, 19b, 20a | 505646      | Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Desk                                    | 1        |
| 1-7, 8a, 10-18b, 19b, 20a | 505647      | Height Adjustable T-Arm Assembly w/ Short Side Shield and Standard Pad Desk, Pair                              | 1        |
| 1-7, 8a, 10-18a, 20b, 23a | 505495      | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full                                               | 1        |
| 1-7, 8a, 10-18a, 20b, 23a | 505496      | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Full, PAIR                                         | 1        |
| 1-7, 8a, 10-18b, 20b, 23a | 505648      | Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Full                                   | 1        |
| 1-7, 8a, 10-18b, 20b, 23a | 505649      | Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Full, Pair                             | 1        |
| 1-7, 8a, 10-18a, 20b, 23b | 505497      | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk                                               | 1        |
| 1-7, 8a, 10-18a, 20b, 23b | 505498      | Height Adj T-Arm Assm w/ Long Side Shield and Waterfall Pad Desk, PAIR                                         | 1        |
| 1-7, 8a, 10-18b, 20b, 23b | 505650      | Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Desk                                   | 1        |
| 1-7, 8a, 10-18b, 20b, 23b | 505651      | Height Adjustable T-Arm Assembly w/ Short Side Shield and Waterfall Pad Desk, Pair                             | 1        |
| 1-7, 10-14, 16, 17, 18b   | 004160      | Height Adjustable T-Arm Lower Assembly Short<br><i>Does not include t-arm upper, arm pad or arm pad screws</i> | 1        |
| 1-7, 10-14, 16, 17, 18a   | 004161      | Height Adjustable T-Arm Lower Assembly Long<br><i>Does not include t-arm upper, arm pad or arm pad screws</i>  | 1        |
| 1-7, 8a, 10-18a           | 003465      | Height Adjustable T-Arm Long Desk<br><i>Does not include arm pad or arm pad screws</i>                         | 1        |
| 1-7, 8b, 10-18a           | 003466      | Height Adjustable T-Arm Long Full<br><i>Does not include arm pad or arm pad screws</i>                         | 1        |
| 1-7, 8a, 10-18b           | 003463      | Height Adjustable T-Arm Short Desk<br><i>Does not include arm pad or arm pad screws</i>                        | 1        |
| 1-7, 8b, 10-18b           | 003464      | Height Adjustable T-Arm Short Full<br><i>Does not include arm pad or arm pad screws</i>                        | 1        |
| 4, 10, 11, 13             | 505507      | Height Adjustable T-Arm Release Lever Assembly                                                                 | 1        |
| 16, 17, 18a               | 505508      | Height Adj T-Arm Long Side Shield Assembly                                                                     | 1        |
| 16, 17, 18b               | 506082      | Height Adjustable T-Arm Short Shield Assembly                                                                  | 1        |
| 9, 21, 22                 | 115844      | T-Arm Receiver Assembly, Catalyst E                                                                            | 1        |

| Item Number         | Part Number | Description                                                             | Quantity |
|---------------------|-------------|-------------------------------------------------------------------------|----------|
| 1                   | 000089      | Height Adjustable T Armrest Post                                        | 1        |
| 2                   | 000095      | Height Adjustable T Armrest Transfer Tube                               | 1        |
| 3, 5, 6, 10, 12, 14 | 505490      | Height Adjustable T-Arm Upper Latch Assembly                            | 1        |
| 7                   | 000090      | Height Adjustable T Armrest Shim                                        | 2        |
| 8a, 15              | 112906      | Height Adjustable T Armrest Upper Desk w/ Tube End Plug                 | 1        |
| 8b, 15              | 112907      | Height Adjustable T Armrest Upper Full w/ Tube End Plug                 | 1        |
| 9                   | 111954      | Threaded Half Nut Plate                                                 | 2        |
| 10                  | 003458      | M5 X 12mm FHCS BLK ZN W/ Patch                                          | 4        |
| 11                  | 000152      | Height Adjustable T Armrest Release Spring Button                       | 1        |
| 13                  | 000135      | Height Adjustable T Armrest Release Lever Spring                        | 1        |
| 15                  | 100803      | Tube End Plug 7/8"                                                      | 2        |
| 16                  | 003641      | .25" x .75" Dowel Pin                                                   | 2        |
| 17                  | 003457      | M5 x 14mm Barrel Nut                                                    | 2        |
| 19a                 | 100500      | Arm Pad Full                                                            | 1        |
| 19b                 | 100501      | Arm Pad Desk                                                            | 1        |
| 20a                 | 100696      | M5 x 30mm PPH Capscrew<br><i>Used with Standard Arm Pads</i>            | 2        |
| 20b                 | 100654      | M5 x 40 DIN7985 PH Pan HD M/S ZC<br><i>Used with Waterfall Arm Pads</i> | 2        |
| 21                  | 112010      | T-Arm Receiver                                                          | 1        |
| 22                  | 100797      | M6 x 40 BHCS BLK ZC                                                     | 1        |
| 23a                 | 102452      | Waterfall Arm Pad Full                                                  | 1        |
| 23b                 | 102521      | Waterfall Arm Pad Desk                                                  | 1        |
| 24                  | 000065      | M6 x 23.68 Threaded Barrel                                              | 2        |
| 25                  | 100866      | M6 x 20 CLS12.9 DIN7991 FSHCS Blk Zc w/Patch                            | 2        |