


Standard Elevating Leg Rest

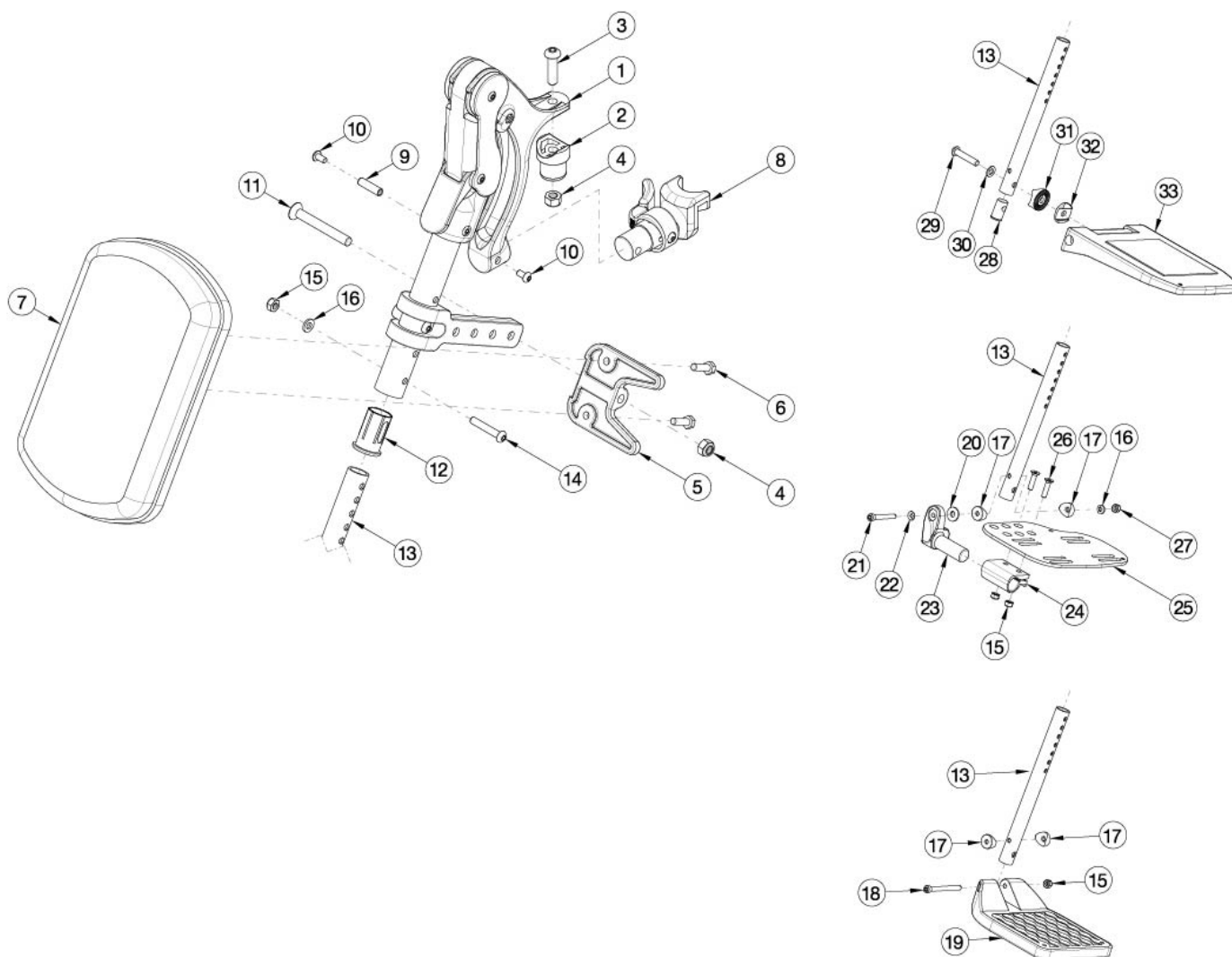

Item Number	Part Number	Description	Quantity
1-6, 7b, 8a, 8b, 9-12, 13a, 14-18, 19a	122621	Standard ELR Assembly 14, Composite Footplate, Short Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-18, 19a	122622	Standard ELR Assembly 15, Composite Footplate, Short Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-18, 19b	122623	Standard ELR Assembly 16-17, Composite Footplate, Short Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-18, 19c	122624	Standard ELR Assembly 18-24, Composite Footplate, Short Ext, Classic Release, Pair	1
1-6, 7b, 8a, 9-12, 13a, 14-18, 19a	122625	Standard ELR Assembly 14, Composite Footplate, Short Ext, Classic Release, Right	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8a, 9-12, 13a, 14-18, 19a	122626	Standard ELR Assembly 15, Composite Footplate, Short Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13a, 14-18, 19b	122627	Standard ELR Assembly 16-17, Composite Footplate, Short Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13a, 14-18, 19c	122628	Standard ELR Assembly 18-24, Composite Footplate, Short Ext, Classic Release, Right	1
1-6, 7b, 8b, 9-12, 13a, 14-18, 19a	122629	Standard ELR Assembly 14, Composite Footplate, Short Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13a, 14-18, 19a	122630	Standard ELR Assembly 15, Composite Footplate, Short Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13a, 14-18, 19b	122631	Standard ELR Assembly 16-17, Composite Footplate, Short Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13a, 14-18, 19c	122632	Standard ELR Assembly 18-24, Composite Footplate, Short Ext, Classic Release, Left	1
1-6, 7b, 8a, 8b, 9-12, 13b, 14-18, 19a	122633	Standard ELR Assembly 14, Composite Footplate, Medium Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-18, 19a	122634	Standard ELR Assembly 15, Composite Footplate, Medium Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-18, 19b	122635	Standard ELR Assembly 16-17, Composite Footplate, Medium Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-18, 19c	122636	Standard ELR Assembly 18-24, Composite Footplate, Medium Ext, Classic Release, Pair	1
1-6, 7b, 8a, 9-12, 13b, 14-18, 19a	122637	Standard ELR Assembly 14, Composite Footplate, Medium Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13b, 14-18, 19a	122638	Standard ELR Assembly 15, Composite Footplate, Medium Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13b, 14-18, 19b	122639	Standard ELR Assembly 16-17, Composite Footplate, Medium Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13b, 14-18, 19c	122640	Standard ELR Assembly 18-24, Composite Footplate, Medium Ext, Classic Release, Right	1
1-6, 7b, 8b, 9-12, 13b, 14-18, 19a	122641	Standard ELR Assembly 14, Composite Footplate, Medium Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13b, 14-18, 19a	122642	Standard ELR Assembly 15, Composite Footplate, Medium Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13b, 14-18, 19b	122643	Standard ELR Assembly 16-17, Composite Footplate, Medium Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13b, 14-18, 19c	122644	Standard ELR Assembly 18-24, Composite Footplate, Medium Ext, Classic Release, Left	1

Item Number	Part Number	Description	Quantity
1-6, 7b, 8a, 8b, 9-12, 13c, 14-18, 19a	122645	Standard ELR Assembly 14, Composite Footplate, Long Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-18, 19a	122646	Standard ELR Assembly 15, Composite Footplate, Long Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-18, 19b	122647	Standard ELR Assembly 16-17, Composite Footplate, Long Ext, Classic Release, Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-18, 19c	122648	Standard ELR Assembly 18-24, Composite Footplate, Long Ext, Classic Release, Pair	1
1-6, 7b, 8a, 9-12, 13c, 14-18, 19a	122649	Standard ELR Assembly 14, Composite Footplate, Long Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13c, 14-18, 19a	122650	Standard ELR Assembly 15, Composite Footplate, Long Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13c, 14-18, 19b	122651	Standard ELR Assembly 16-17, Composite Footplate, Long Ext, Classic Release, Right	1
1-6, 7a, 8a, 9-12, 13c, 14-18, 19c	122652	Standard ELR Assembly 18-24, Composite Footplate, Long Ext, Classic Release, Right	1
1-6, 7b, 8b, 9-12, 13c, 14-18, 19a	122653	Standard ELR Assembly 14, Composite Footplate, Long Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13c, 14-18, 19a	122654	Standard ELR Assembly 15, Composite Footplate, Long Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13c, 14-18, 19b	122655	Standard ELR Assembly 16-17, Composite Footplate, Long Ext, Classic Release, Left	1
1-6, 7a, 8b, 9-12, 13c, 14-18, 19c	122656	Standard ELR Assembly 18-24, Composite Footplate, Long Ext, Classic Release, Left	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-16, 28-32, 33a	122657	Standard ELR Assembly 15-16 Comp Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-16, 28-32, 33b	122658	Standard ELR Assembly 17-18 Comp Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-16, 28-32, 33c	122659	Standard ELR Assembly 19-22 Comp Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 9-12, 13a, 14-16, 28-32, 33a	122660	Standard ELR Assembly 15-16 Comp Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-16, 28-32, 33b	122661	Standard ELR Assembly 17-18 Comp Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-16, 28-32, 33c	122662	Standard ELR Assembly 19-22 Comp Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8b, 9-12, 13a, 14-16, 28-32, 33a	122663	Standard ELR Assembly 15-16 Comp Angle Adj FP Short Ext Classic Release Left	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8b, 9-12, 13a, 14-16, 28-32, 33b	122664	Standard ELR Assembly 17-18 Comp Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-16, 28-32, 33c	122665	Standard ELR Assembly 19-22 Comp Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-16, 28-32, 33a	122666	Standard ELR Assembly 15-16 Comp Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-16, 28-32, 33b	122667	Standard ELR Assembly 17-18 Comp Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-16, 28-32, 33c	122668	Standard ELR Assembly 19-22 Comp Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 9-12, 13b, 14-16, 28-32, 33a	122669	Standard ELR Assembly 15-16 Comp Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-16, 28-32, 33b	122670	Standard ELR Assembly 17-18 Comp Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-16, 28-32, 33c	122671	Standard ELR Assembly 19-22 Comp Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8b, 9-12, 13b, 14-16, 28-32, 33a	122672	Standard ELR Assembly 15-16 Comp Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-16, 28-32, 33b	122673	Standard ELR Assembly 17-18 Comp Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-16, 28-32, 33c	122674	Standard ELR Assembly 19-22 Comp Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-16, 28-32, 33a	122675	Standard ELR Assembly 15-16 Comp Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-16, 28-32, 33b	122676	Standard ELR Assembly 17-18 Comp Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-16, 28-32, 33c	122677	Standard ELR Assembly 19-22 Comp Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7a, 8a, 9-12, 13c, 14-16, 28-32, 33a	122678	Standard ELR Assembly 15-16 Comp Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-16, 28-32, 33b	122679	Standard ELR Assembly 17-18 Comp Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-16, 28-32, 33c	122680	Standard ELR Assembly 19-22 Comp Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8b, 9-12, 13c, 14-16, 28-32, 33a	122681	Standard ELR Assembly 15-16 Comp Angle Adj FP Long Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-16, 28-32, 33b	122682	Standard ELR Assembly 17-18 Comp Angle Adj FP Long Ext Classic Release Left	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8b, 9-12, 13c, 14-16, 28-32, 33c	122683	Standard ELR Assembly 19-22 Comp Angle Adj FP Long Ext Classic Release Left	1
1-6, 7b, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25a, 25b, 26, 27	122684	Standard ELR Assembly 14 Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25a, 25b, 26, 27	122685	Standard ELR Assembly 15 Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25c, 25d, 26, 27	122686	Standard ELR Assembly 16-17 Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13a, 14-17, 20-24, 25e, 25f, 26, 27	122687	Standard ELR Assembly 18-24 Angle Adj FP Short Ext Classic Release Pair	1
1-6, 7b, 8a, 9-12, 13a, 14-17, 20-24, 25b, 26, 27	122688	Standard ELR Assembly 14 Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-17, 20-24, 25b, 26, 27	122689	Standard ELR Assembly 15 Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-17, 20-24, 25d, 26, 27	122690	Standard ELR Assembly 16-17 Angle Adj FP Short Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13a, 14-17, 20-24, 25f, 26, 27	122691	Standard ELR Assembly 18-24 Angle Adj FP Short Ext Classic Release Right	1
1-6, 7b, 8b, 9-12, 13a, 14-17, 20-24, 25a, 26, 27	122692	Standard ELR Assembly 14 Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-17, 20-24, 25a, 26, 27	122693	Standard ELR Assembly 15 Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-17, 20-24, 25c, 26, 27	122694	Standard ELR Assembly 16-17 Angle Adj FP Short Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13a, 14-17, 20-24, 25e, 26, 27	122695	Standard ELR Assembly 18-24 Angle Adj FP Short Ext Classic Release Left	1
1-6, 7b, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25a, 25b, 26, 27	122696	Standard ELR Assembly 14 Angle Adj FP Medium Ext Classic Release Pair	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25a, 25b, 26, 27	122697	Standard ELR Assembly 15 Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25c, 25d, 26, 27	122698	Standard ELR Assembly 16-17 Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13b, 14-17, 20-24, 25e, 25f, 26, 27	122699	Standard ELR Assembly 18-24 Angle Adj FP Medium Ext Classic Release Pair	1
1-6, 7b, 8a, 9-12, 13b, 14-17, 20-24, 25b, 26, 27	122700	Standard ELR Assembly 14 Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-17, 20-24, 25b, 26, 27	122701	Standard ELR Assembly 15 Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-17, 20-24, 25d, 26, 27	122702	Standard ELR Assembly 16-17 Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13b, 14-17, 20-24, 25f, 26, 27	122703	Standard ELR Assembly 18-24 Angle Adj FP Medium Ext Classic Release Right	1
1-6, 7b, 8b, 9-12, 13b, 14-17, 20-24, 25a, 26, 27	122704	Standard ELR Assembly 14 Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-17, 20-24, 25a, 26, 27	122705	Standard ELR Assembly 15 Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-17, 20-24, 25c, 26, 27	122706	Standard ELR Assembly 16-17 Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13b, 14-17, 20-24, 25e, 26, 27	122707	Standard ELR Assembly 18-24 Angle Adj FP Medium Ext Classic Release Left	1
1-6, 7b, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25a, 25b, 26, 27	122708	Standard ELR Assembly 14 Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25a, 25b, 26, 27	122709	Standard ELR Assembly 15 Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7a, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25c, 25d, 26, 27	122710	Standard ELR Assembly 16-17 Angle Adj FP Long Ext Classic Release Pair	1

Item Number	Part Number	Description	Quantity
1-6, 7a, 8a, 8b, 9-12, 13c, 14-17, 20-24, 25e, 25f, 26, 27	122711	Standard ELR Assembly 18-24 Angle Adj FP Long Ext Classic Release Pair	1
1-6, 7b, 8a, 9-12, 13c, 14-17, 20-24, 25b, 26, 27	122712	Standard ELR Assembly 14 Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-17, 20-24, 25b, 26, 27	122713	Standard ELR Assembly 15 Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-17, 20-24, 25d, 26, 27	122714	Standard ELR Assembly 16-17 Angle Adj FP Long Ext Classic Release Right	1
1-6, 7a, 8a, 9-12, 13c, 14-17, 20-24, 25f, 26, 27	122715	Standard ELR Assembly 18-24 Angle Adj FP Long Ext Classic Release Right	1
1-6, 7b, 8b, 9-12, 13c, 14-17, 20-24, 25a, 26, 27	122716	Standard ELR Assembly 14 Angle Adj FP Long Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-17, 20-24, 25a, 26, 27	122717	Standard ELR Assembly 15 Angle Adj FP Long Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-17, 20-24, 25c, 26, 27	122718	Standard ELR Assembly 16-17 Angle Adj FP Long Ext Classic Release Left	1
1-6, 7a, 8b, 9-12, 13c, 14-17, 20-24, 25e, 26, 27	122719	Standard ELR Assembly 18-24 Angle Adj FP Long Ext Classic Release Left	1
1-6, 8a, 9-12, 14-16	122720	Standard ELR Upper Assembly with Classic Hanger Latch Release Right	1
1-6, 8b, 9-12, 14-17	122721	Standard ELR Upper Assembly with Classic Hanger Latch Release Left	1
2	000032	Footrest Hanger Pivot Saddle	2
3	103551	BHCS, M8 x 1.25 x 30 BLZ	2
4	100659	M8 Nylock Nut Blk Zn	2
6	101823	M6 X 20 Hhcs Blk Zn	4
7a	101585	Calf Pad	2
7b	101470	Pediatric Calf Pad	2
8a	003416	Classic Hanger Latch Assembly, Right	1
8b	003417	Classic Hanger Latch Assembly, Left	1

Item Number	Part Number	Description	Quantity
9	000073	M5 x 24 Threaded Barrel	2
10	100669	M5 x 10 BTN SHCS BLKZC w/Patch	4
12	000034	Sleeve 3/4" x 1 1/2"	2
13a	118966	Standard ELR Extension Tube Short	2
13b	118967	Standard ELR Extension Tube, Medium	2
13c	118968	Standard ELR Extension Tube, Long	2
14	101811	M6 x 35 BHCS Blk Zn	2
15	100658	M6 Nylock Nut Blk Zn	6
16	100746	M6 Flat Washer Blk Zn	4
17	000053	Saddle 3/4"	2
18	101849	M6 x 65 SHCS Blk Zn	2
19a	100537	Footplate Composite 14-15" Wide CAT/TILT	2
19b	100540	Footplate Composite 16-17" Wide CAT/TILT	2
19c	100542	Footplate Composite 18-24" Wide CAT/TILT	2
20	000428	Wear Washer	2
21	101472	M6 x 45 SHCS BLK ZC	2
22	101455	M6 Belleville Washer (.252 x .551 x .050)	2
23	000047	Footplate Angle Adjustable Casting 3/4 Right	2
24	100502	Angle Adj Footplate Clamp 2"	2
25a	000105	Footplate Angle Adjustable 14-15" Wide CAT/TILT Left (Std ELR Right)	2
25b	000106	Footplate Angle Adjustable 14-15" Wide CAT/TILT Right (Std ELR Left)	2
25c	000107	Footplate Angle Adjustable 16-17" Wide CAT/TILT Left (Std ELR Right)	2
25d	000108	Footplate Angle Adjustable 16-17" Wide CAT/TILT Right (Std ELR Left)	2
25e	000109	Footplate Angle Adjustable 18-24" Wide CAT/TILT Left (Std ELR Right)	2
25f	000110	Footplate Angle Adjustable 18-24" Wide CAT/TILT Right (Std ELR Left)	2
26	100662	M6 x 25 FSHCS BLK ZC	2
27	120030	M6 Top Lock Nut	2

Item Number	Part Number	Description	Quantity
28	003161	Footplate Plug for Composite Angle Adjustable Footplate Extension Tube	2
29	003189	Bolt M8 x 45MM, Galvanized Steel	2
30	003188	Washer D8.4 x 1.6 MM	2
31	003187	Locking Wheel w/Radius, R12.5 & R10, D30 x 11.3, Cast Steel, Black	2
32	003186	Locking Wheel w/Hexagon, D30 x 6.8/6, Cast Steel, Black	2
33a	003173	Composite Angle Adjustable Footplate Small 15-16" Wide	2
33b	003174	Composite Angle Adjustable Footplate Medium 17-18" Wide	2
33c	003175	Composite Angle Adjustable Footplate Large 19-24" Wide	2