


Flip for Leckey Seat Frame

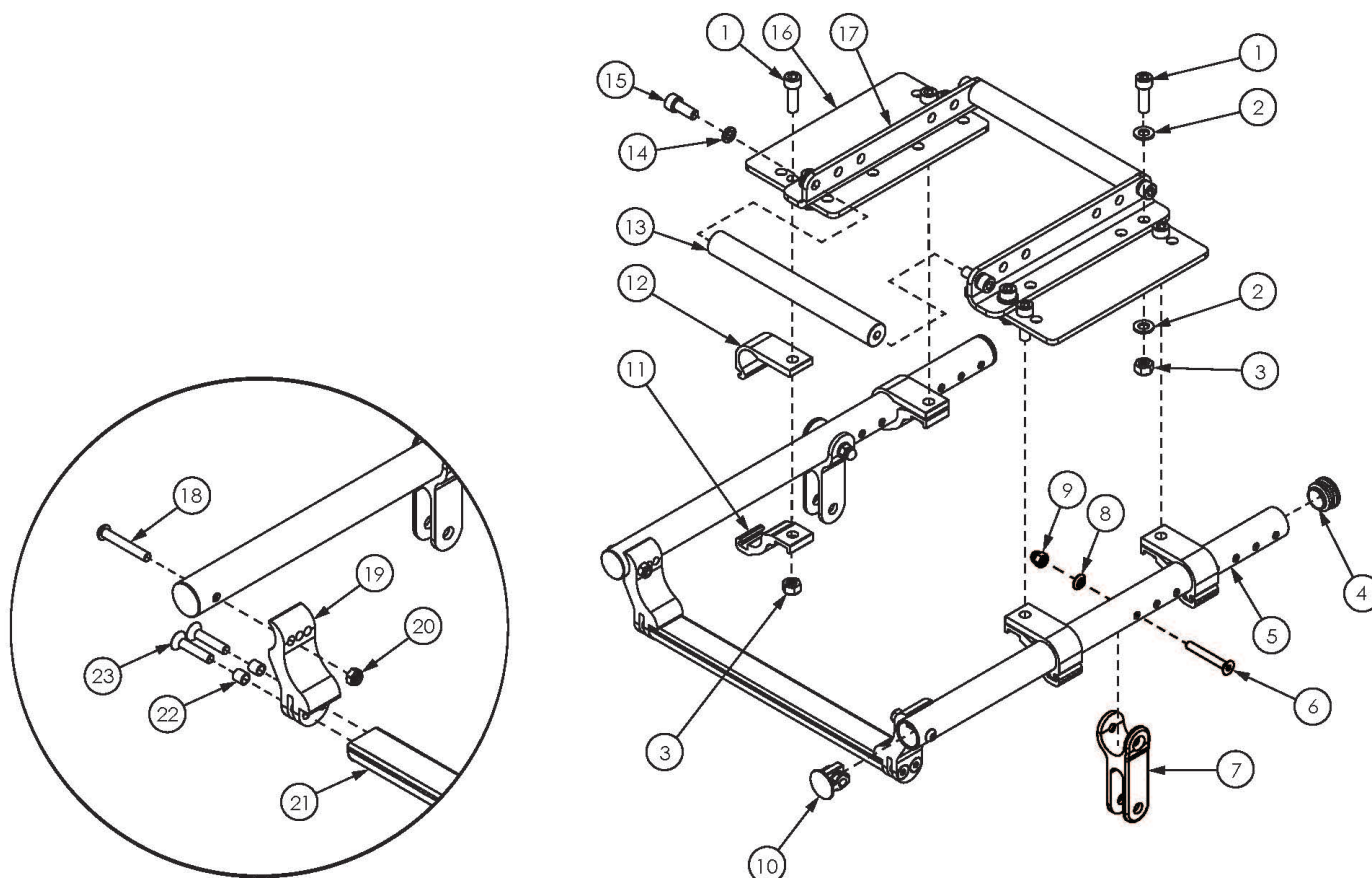

Flip for Leckey and Flip for x:panda are not eligible to be converted into another seating format

Item Number	Part Number	Description	Quantity
1 - 4, 5a, 10 - 12, 13a, 14 - 17	111260	15" Wide Flip Seat Frame Kit for Mygo	1
1 - 4, 5b, 10 - 12, 13b, 14 - 17	111261	14" Wide Flip Seat Frame Kit for Squiggles	1
1 - 4, 5a, 10 - 12, 13b, 14 - 17	111262	15" Wide Flip Seat Frame Kit for Squiggles	1
1 - 4, 5a, 10 - 12, 13a, 14 - 20, 21b, 22, 23	111263	15" Wide Flip Seat Frame Retrofit Kit for Mygo	1
1 - 4, 5b, 10 - 12, 13b, 14 - 20, 21a, 22, 23	111264	14" Wide Flip Seat Frame Retrofit Kit for Squiggles	1
1 - 4, 5a, 10 - 12, 13b, 14 - 20, 21b, 22, 23	111265	15" Wide Flip Seat Frame Retrofit Kit for Squiggles	1

Item Number	Part Number	Description	Quantity
1	102864	M8 x 25mm SHCS, BLK ZC	8
2	101858	Washer Flat M8 Blk Zn	8
3	100659	M8 Nylock Nut Blk Zn	8
4	100804	Tube End Plug 1"	2
5a, 4, 10	111256	Flip Seat Tube with Tube End Plugs for 15" Wide Mygo/Squiggles	2
5b, 4, 10	111257	Flip Seat Tube with Tube End Plugs for 14" Wide Squiggles	2
6	101592	M6 x 45 CLS10.9 DIN7991 FHCS BLKZC	2
7	001878	Tilt Yoke	2
8	003079	M6 Conical Washer	2
9	100558	M6 Nyl Ins Domed Cap L/N Zc	2
10	001028	Tube Plug 1" x .824"	2
11	001205	Seat Pan/Base Clamp TILT	4
12	001204	Seat Pan/Base Bracket TILT	4
13a	004575	Mounting Bar, Mygo, Flip	2
13b	004576	Mounting Bar, Squiggles, Flip	2
14	101834	M8 Hi-Collar Lockwasher Blk Zn	4
15	101850	M8 x 20 SHCS Blk Zn	4
13a, 14, 15	111258	Mounting Bar with Hardware Flip, Mygo	2
13b, 14, 15	111259	Mounting Bar with Hardware Flip, Squiggles	2
18	100797	M6 x 40 BHCS BLK ZC	2
19	001867	Mechlock Mount	2
20	100658	M6 Nylock Nut Blk Zn	2
21a	001883	Strut Tube Seat 14 Wide	1
21b	001884	Strut Tube Seat 15 Wide	1
22	001156	Pivot Bushing Tilt	4
23	102802	M6 x 30, FHCS, Blk Zn w/Patch	4