


Catalyst Titanium Cross Braces

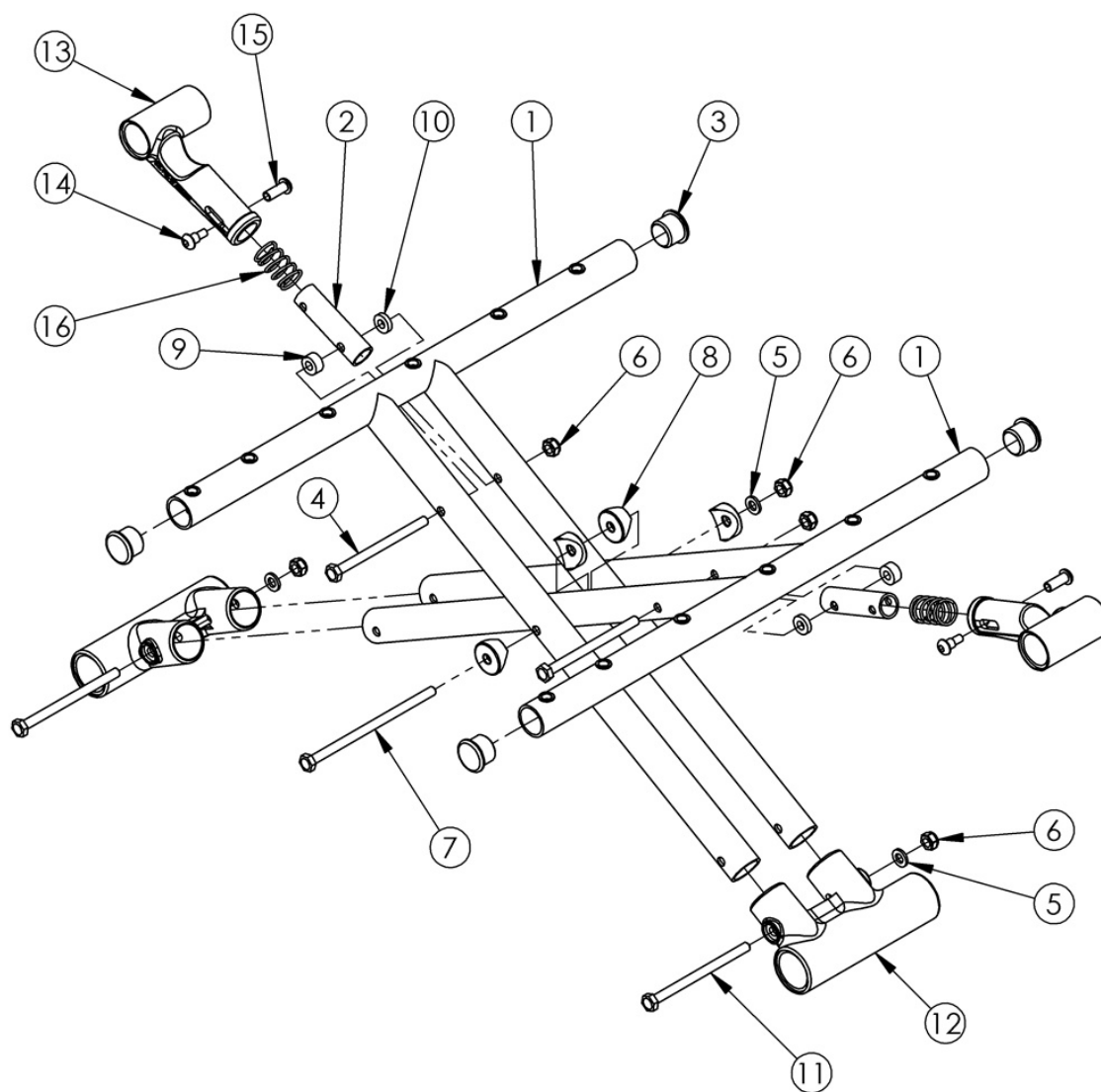

Round-style cross braces If changing from oval-style aluminum cross braces, lower frame pivots and cross brace hardware are required.

Item Number	Part Number	Description	Quantity
2a, 13a, 14-16	500024	Upper Pivot Assembly 14" Width	2
2b, 13b, 14-16	500025	Upper Pivot Assembly 15" Width	2
2c, 13b, 14-16	500026	Upper Pivot Assembly 16" Width	2
2d, 13b, 14-16	500027	Upper Pivot Assembly 17" Width	2
2e, 13b, 14-16	500028	Upper Pivot Assembly 18" Width	2

Item Number	Part Number	Description	Quantity
2f, 13b, 14-16	500029	Upper Pivot Assembly 19" Width	2
2g, 13b, 14-16	500030	Upper Pivot Assembly 20" Width	2
2h, 13b, 14-16	501479	Upper Pivot Assembly 21/22" Width	2
4-11	501398	Round-Style Cross Brace Hardware Kit CAT	1
1a, 3	100295	Cross Brace Titanium Right 14 x 14 CAT	1
1b, 3	100338	Cross Brace Titanium Left 14 x 14 CAT	1
1c, 3	100296	Cross Brace Titanium Right 14 x 15 CAT	1
1d, 3	100339	Cross Brace Titanium Left 14 x 15 CAT	1
1e, 3	100297	Cross Brace Titanium Right 14 x 16 CAT	1
1f, 3	100340	Cross Brace Titanium Left 14 x 16 CAT	1
1g, 3	100298	Cross Brace Titanium Right 14 x 17 CAT	1
1h, 3	100341	Cross Brace Titanium Left 14 x 17 CAT	1
1i, 3	100299	Cross Brace Titanium Right 14 x 18 CAT	1
1j, 3	100342	Cross Brace Titanium Left 14 x 18 CAT	1
1k, 3	124040	Cross Brace Titanium Left 14 x 19 CAT	1
1l, 3	124042	Cross Brace Titanium Right 14 x 19 CAT	1
1m, 3	124041	Cross Brace Titanium Left 14 x 20 CAT	1
1n, 3	124043	Cross Brace Titanium Right 14 x 20 CAT	1
1o, 3	100300	Cross Brace Titanium Right 15 x 14 CAT	1
1p, 3	100343	Cross Brace Titanium Left 15 x 14 CAT	1
1q, 3	100301	Cross Brace Titanium Right 15 x 15 CAT	1
1r, 3	100344	Cross Brace Titanium Left 15 x 15 CAT	1
1s, 3	100302	Cross Brace Titanium Right 15 x 16 CAT	1
1t, 3	100345	Cross Brace Titanium Left 15 x 16 CAT	1
1u, 3	100303	Cross Brace Titanium Right 15 x 17 CAT	1
1v, 3	100346	Cross Brace Titanium Left 15 x 17 CAT	1
1w, 3	100304	Cross Brace Titanium Right 15 x 18 CAT	1
1x, 3	100347	Cross Brace Titanium Left 15 x 18 CAT	1

Item Number	Part Number	Description	Quantity
1y, 3	100305	Cross Brace Titanium Right 15 x 19 CAT	1
1z, 3	100348	Cross Brace Titanium Left 15 x 19 CAT	1
1aa, 3	100306	Cross Brace Titanium Right 15 x 20 CAT	1
1ab, 3	100349	Cross Brace Titanium Left 15 x 20 CAT	1
1ac, 3	100307	Cross Brace Titanium Right 16 x 14 CAT	1
1ad, 3	100350	Cross Brace Titanium Left 16 x 14 CAT	1
1ae, 3	100308	Cross Brace Titanium Right 16 x 15 CAT	1
1af, 3	100351	Cross Brace Titanium Left 16 x 15 CAT	1
1ag, 3	100309	Cross Brace Titanium Right 16 x 16 CAT	1
1ah, 3	100352	Cross Brace Titanium Left 16 x 16 CAT	1
1ai, 3	100310	Cross Brace Titanium Right 16 x 17 CAT	1
1aj, 3	100353	Cross Brace Titanium Left 16 x 17 CAT	1
1ak, 3	100311	Cross Brace Titanium Right 16 x 18 CAT	1
1al, 3	100354	Cross Brace Titanium Left 16 x 18 CAT	1
1am, 3	100312	Cross Brace Titanium Right 16 x 19 CAT	1
1an, 3	100355	Cross Brace Titanium Left 16 x 19 CAT	1
1ao, 3	100313	Cross Brace Titanium Right 16 x 20 CAT	1
1ap, 3	100356	Cross Brace Titanium Left 16 x 20 CAT	1
1aq, 3	100314	Cross Brace Titanium Right 17 x 14 CAT	1
1ar, 3	100357	Cross Brace Titanium Left 17 x 14 CAT	1
1as, 3	100315	Cross Brace Titanium Right 17 x 15 CAT	1
1at, 3	100358	Cross Brace Titanium Left 17 x 15 CAT	1
1au, 3	100316	Cross Brace Titanium Right 17 x 16 CAT	1
1av, 3	100359	Cross Brace Titanium Left 17 x 16 CAT	1
1aw, 3	100317	Cross Brace Titanium Right 17 x 17 CAT	1
1ax, 3	100360	Cross Brace Titanium Left 17 x 17 CAT	1
1ay, 3	100318	Cross Brace Titanium Right 17 x 18 CAT	1
1az, 3	100361	Cross Brace Titanium Left 17 x 18 CAT	1

Item Number	Part Number	Description	Quantity
1ba, 3	100319	Cross Brace Titanium Right 17 x 19 CAT	1
1bb, 3	100362	Cross Brace Titanium Left 17 x 19 CAT	1
1bc, 3	100320	Cross Brace Titanium Right 17 x 20 CAT	1
1bd, 3	100363	Cross Brace Titanium Left 17 x 20 CAT	1
1be, 3	100321	Cross Brace Titanium Right 18 x 14 CAT	1
1bf, 3	100364	Cross Brace Titanium Left 18 x 14 CAT	1
1bg, 3	100322	Cross Brace Titanium Right 18 x 15 CAT	1
1bh, 3	100365	Cross Brace Titanium Left 18 x 15 CAT	1
1bi, 3	100323	Cross Brace Titanium Right 18 x 16 CAT	1
1bj, 3	100366	Cross Brace Titanium Left 18 x 16 CAT	1
1bk, 3	100324	Cross Brace Titanium Right 18 x 17 CAT	1
1bl, 3	100367	Cross Brace Titanium Left 18 x 17 CAT	1
1bm, 3	100325	Cross Brace Titanium Right 18 x 18 CAT	1
1bn, 3	100368	Cross Brace Titanium Left 18 x 18 CAT	1
1bo, 3	100326	Cross Brace Titanium Right 18 x 19 CAT	1
1bp, 3	100369	Cross Brace Titanium Left 18 x 19 CAT	1
1bq, 3	100327	Cross Brace Titanium Right 18 x 20 CAT	1
1br, 3	100370	Cross Brace Titanium Left 18 x 20 CAT	1
1bs, 3	100328	Cross Brace Titanium Right 19 x 16 CAT	1
1bt, 3	100371	Cross Brace Titanium Left 19 x 16 CAT	1
1bu, 3	100329	Cross Brace Titanium Right 19 x 17 CAT	1
1bv, 3	100372	Cross Brace Titanium Left 19 x 17 CAT	1
1bw, 3	100330	Cross Brace Titanium Right 19 x 18 CAT	1
1bx, 3	100373	Cross Brace Titanium Left 19 x 18 CAT	1
1by, 3	100331	Cross Brace Titanium Right 19 x 19 CAT	1
1bz, 3	100374	Cross Brace Titanium Left 19 x 19 CAT	1
1ca, 3	100332	Cross Brace Titanium Right 19 x 20 CAT	1
1cb, 3	100375	Cross Brace Titanium Left 19 x 20 CAT	1

Item Number	Part Number	Description	Quantity
1cc, 3	100333	Cross Brace Titanium Right 20 x 16 CAT	1
1cd, 3	100376	Cross Brace Titanium Left 20 x 16 CAT	1
1ce, 3	100334	Cross Brace Titanium Right 20 x 17 CAT	1
1cf, 3	100377	Cross Brace Titanium Left 20 x 17 CAT	1
1cg, 3	100335	Cross Brace Titanium Right 20 x 18 CAT	1
1ch, 3	100378	Cross Brace Titanium Left 20 x 18 CAT	1
	100336	Cross Brace Titanium Right 20 x 19 CAT	1
	100379	Cross Brace Titanium Left 20 x 19 CAT	1
	100337	Cross Brace Titanium Right 20 x 20 CAT	1
	100380	Cross Brace Titanium Left 20 x 20 CAT	1
3	100803	Tube End Plug 7/8"	4
4	100660	M6 x 80mm CLS8.8 DIN931 HHCS ZC	2
5	100746	M6 Flat Washer Blk Zn	1
6	100658	M6 Nylock Nut Blk Zn	3
7	100678	M6 x 110 CLS8.8 DIN931 HHCS ZC	1
8	000054	Saddle 7/8"	4
9	100880	1/4" Plastic Spacer <i>Not used on 14" wide chairs</i>	2
10	105019	3/16" Plastic Spacer <i>Not used on 14" wide chairs</i>	2
11	100704	M6 x 90 CLS8.8 DIN931 HHCS ZC	2
12	000001	Lower Frame Pivot CAT	2