


# Catalyst / Spark Hemi Wheel Lock

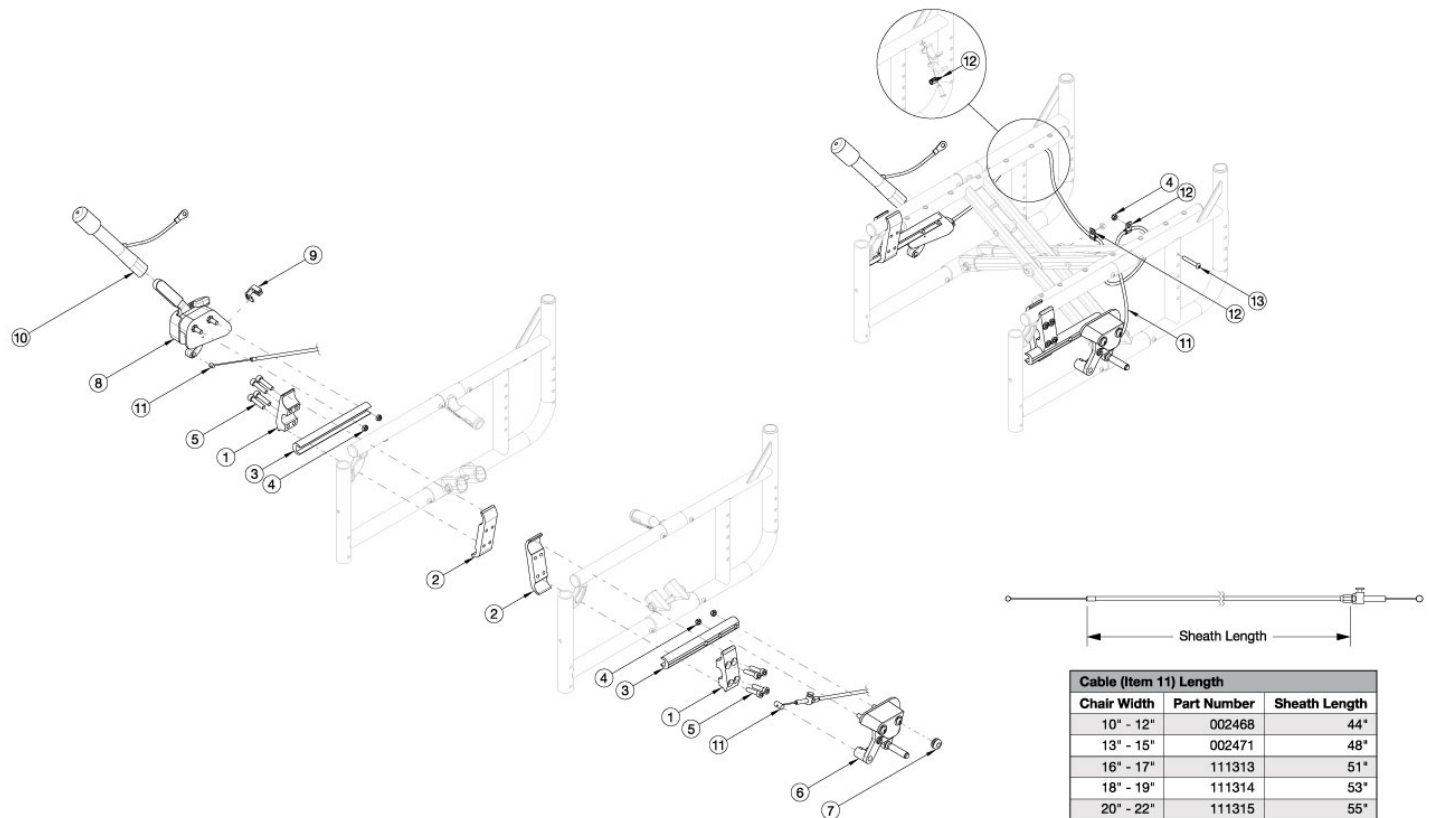

Item Number	Part Number	Description	Quantity
1-5, 6a, 7, 8a, 9, 10, 11a, 12, 13	116450	Hemi Wheel Lock Assembly, Width 10-12, Right Side Mounted, Catalyst	1
1-5, 6a, 7, 8a, 9, 10, 11b, 12, 13	116451	Hemi Wheel Lock Assembly, Width 13-15, Right Side Mounted, Catalyst	1
1-5, 6a, 7, 8a, 9, 10, 11c, 12, 13	116452	Hemi Wheel Lock Assembly, Width 16-17, Right Side Mounted, Catalyst	1
1-5, 6a, 7, 8a, 9, 10, 11d, 12, 13	116453	Hemi Wheel Lock Assembly, Width 18-19, Right Side Mounted, Catalyst	1
1-5, 6a, 7, 8a, 9, 10, 11e, 12, 13	116454	Hemi Wheel Lock Assembly, Width 20-22, Right Side Mounted, Catalyst	1
1-5, 6b, 7, 8b, 9, 10, 11a, 12, 13	116455	Hemi Wheel Lock Assembly, Width 10-12, Left Side Mounted, Catalyst	1
1-5, 6b, 7, 8b, 9, 10, 11b, 12, 13	116456	Hemi Wheel Lock Assembly, Width 13-15, Left Side Mounted, Catalyst	1
1-5, 6b, 7, 8b, 9, 10, 11c, 12, 13	116457	Hemi Wheel Lock Assembly, Width 16-17, Left Side Mounted, Catalyst	1

Item Number	Part Number	Description	Quantity
1-5, 6b, 7, 8b, 9, 10, 11d, 12, 13	116458	Hemi Wheel Lock Assembly, Width 18-19, Left Side Mounted, Catalyst	1
1-5, 6b, 7, 8b, 9, 10, 11e, 12, 13	116459	Hemi Wheel Lock Assembly, Width 20-22, Left Side Mounted, Catalyst	1
1	000143	Wheel Lock Clamp Extended Top CAT	1
2	000144	Wheel Lock Clamp Extended Bottom CAT	1
3	001321	Wheel Lock Mount Bar Extended	1
4	100658	M6 Nylock Nut Blk Zn	5
5	100780	M6 x 25 SHCS ZC	4
6a	111591	Wheel Lock Follower Assembly without Mount, Left	1
6b	111592	Wheel Lock Follower Assembly without Mount, Right	1
7	115743	Wheel Lock Retaining Grommet	1
8a	111593	Hemi Wheel Lock, Master Assembly, Right without Mount	1
8b	111594	Hemi Wheel Lock, Master Assembly, Left without Mount	1
9	115742	Hemi Wheel Lock Cable Retaining Clip	1
10	200369	Wheel Lock Extension	1
11a	116460	Hemi Wheel Lock Cable, 10-12" Widths, Catalyst <i>Used with Seat Widths 10 - 12"</i>	1
11b	116461	Hemi Wheel Lock Cable, 13-15" Widths, Catalyst <i>Used with Seat Widths 13 - 15"</i>	1
11c	111313	Hemi Wheel Lock Cable, Short <i>Used with Seat Widths 16 - 17"</i>	1
11d	111314	Hemi Wheel Lock Cable, Medium <i>Used with Seat Widths 18 - 19"</i>	1
11e	111315	Hemi Wheel Lock Cable, Long <i>Used with Seat Widths 20 - 22"</i>	1
12	001239	P-Clip 3/16"	3
13	100797	M6 x 40 BHCS BLK ZC	1