



# Catalyst 5Ti Adjustable Fender Side Guard

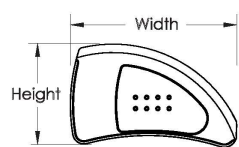

Side Guard	Height	Width
Small	6.6"	12.6"
Medium	7.7"	13.9"
Large	9.2"	14.9"

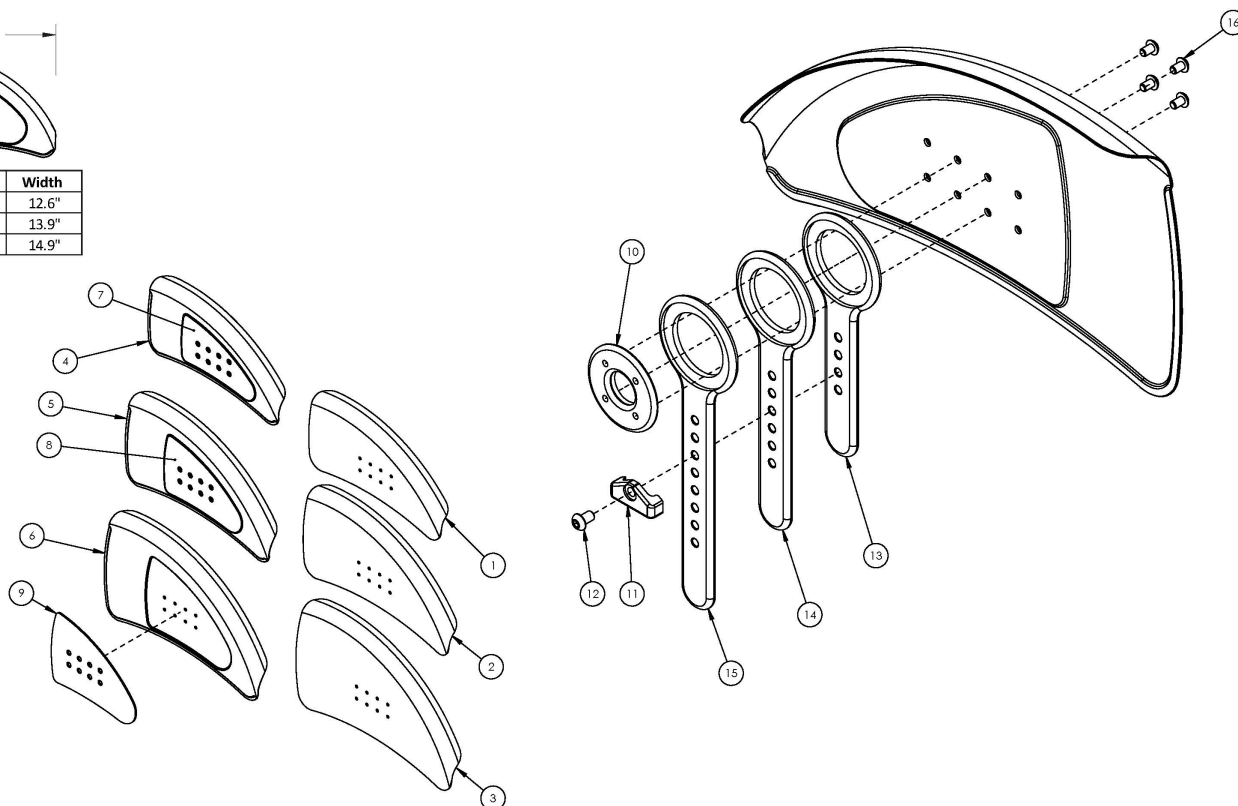

*If retrofitting, side guard receivers may be required*

Item Number	Part Number	Description	Quantity
1a, 1b, 10-13, 16	506715	Carbon Fiber Adjustable Fender Assembly, Small, PAIR	1
2a, 2b, 10-12, 14, 16	506717	Carbon Fiber Adjustable Fender Assembly, Medium, PAIR	1
3a, 3b, 10-12, 15, 16	506719	Carbon Fiber Adjustable Fender Assembly, Large, PAIR	1
4a, 7b, 10-13, 16	107576	Aluminum Adjustable Fender w/Foam Insert Assembly, Small, Right, Black Matte	1
4b, 7a, 10-13, 16	107577	Aluminum Adjustable Fender w/Foam Insert Assembly, Small, Left, Black Matte	1
4a, 4b, 7a, 7b, 10-13, 16	506703	Aluminum Adjustable Fender w/Foam Insert Assembly, Small, PAIR, Black Matte	1
5a, 8b, 10-12, 14, 16	107580	Aluminum Adjustable Fender w/Foam Insert Assembly, Medium, Right, Black Matte	1
5b, 8a, 10-12, 14, 16	107581	Aluminum Adjustable Fender w/Foam Insert Assembly, Medium, Left, Black Matte	1
5a, 5b, 8a, 8b, 10-12, 14, 16	506707	Aluminum Adjustable Fender w/Foam Insert Assembly, Medium, PAIR, Black Matte	1
6a, 9b, 10-12, 15, 16	107584	Aluminum Adjustable Fender w/Foam Insert Assembly, Large, Right, Black Matte	1
6b, 9a, 10-12, 15, 16	107585	Aluminum Adjustable Fender w/Foam Insert Assembly, Large, Left, Black Matte	1

Item Number	Part Number	Description	Quantity
6a, 6b, 9a, 9b, 10-12, 15, 16	506711	Aluminum Adjustable Fender w/Foam Insert Assembly, Large, PAIR, Black Matte	1
1a	003613	Carbon Fiber Fender Panel, Adjustable, Small, Right	1
1b	003614	Carbon Fiber Fender Panel, Adjustable, Small, Left	1
2a	003615	Carbon Fiber Fender Panel, Adjustable, Medium, Right	1
2b	003616	Carbon Fiber Fender Panel, Adjustable, Medium, Left	1
3a	003617	Carbon Fiber Fender Panel, Adjustable, Large, Right	1
3b	003618	Carbon Fiber Fender Panel, Adjustable, Large, Left	1
4a	003595	Aluminum Fender Panel, Adjustable, Small, Right, Black Matte	1
4b	003596	Aluminum Fender Panel, Adjustable, Small, Left, Black Matte	1
5a	003597	Aluminum Fender Panel, Adjustable, Medium, Right, Black Matte	1
5b	003598	Aluminum Fender Panel, Adjustable, Medium, Left, Black Matte	1
6a	003599	Aluminum Fender Panel, Adjustable, Large, Right, Black Matte	1
6b	003600	Aluminum Fender Panel, Adjustable, Large, Left, Black Matte	1
7a	003625	Fender Foam Insert, Adjustable, Small, Pair	1
8a	003627	Fender Foam Insert, Adjustable, Medium, Pair	1
9a	003629	Fender Foam Insert, Adjustable, Large, Pair	1
10	003590	Fender Post Mount Plate	1
11	001342	Side Guard Post Stop	1
12	101184	M6 x 10 BTN SHCS BLKZC w/Patch	1
13	003587	Post, Adjustable Fender, Short <i>Used with small side guard panels</i>	1
14	003588	Post, Adjustable Fender, Medium <i>Used with medium side guard panels</i>	1
15	003589	Post, Adjustable Fender, Tall <i>Used with large side guard panels</i>	1
16	103573	M5 x 8mm LP SHCS BLK	4
4a, 7b	506661	Aluminum Adjustable Fender Panel w/Foam Insert, Small, Right, Black Matte	1
4b, 7a	506662	Aluminum Adjustable Fender Panel w/Foam Insert, Small, Left, Black Matte	1

<b>Item Number</b>	<b>Part Number</b>	<b>Description</b>	<b>Quantity</b>
4a, 4b, 7a, 7b	506663	Aluminum Adjustable Fender Panel w/Foam Insert, Small, PAIR, Black Matte	1
5a, 8b	506673	Aluminum Adjustable Fender Panel w/Foam Insert, Medium, Right, Black Matte	1
5b, 8a	506674	Aluminum Adjustable Fender Panel w/Foam Insert, Medium, Left, Black Matte	1
5a, 5b, 8a, 8b	506675	Aluminum Adjustable Fender Panel w/Foam Insert, Medium, PAIR, Black Matte	1
6a, 9b	506685	Aluminum Adjustable Fender Panel w/Foam Insert, Large, Right, Black Matte	1
6b, 9a	506686	Aluminum Adjustable Fender Panel w/Foam Insert, Large, Left, Black Matte	1
6a, 6b, 9a, 9b	506687	Aluminum Adjustable Fender Panel w/Foam Insert, Large, PAIR, Black Matte	1
1a, 1b	506697	Carbon Fiber Adjustable Fender Panel, Small, PAIR	1
2a, 2b	506699	Carbon Fiber Adjustable Fender Panel, Medium, PAIR	1
3a, 3b	506701	Carbon Fiber Adjustable Fender Panel, Large, PAIR	1

# Determining Fender Panel & Post Size For Your Chair

## CATALYST

Fender side guard panel size needed to fit a Catalyst is driven by the height of the tire over the seat. To calculate this, use the formula listed below. Once you have the output of the formula use the reference matrix to find the appropriate panel and post sizes. For information on Wheel Outside Diameter reference the Wheel Outside Diameter chart in the parts manual.

**Formula:** Wheel Outside Diameter (OD) – Rear Seat Height = Tire Height over seat

### Reference Matrix:

Tire Height	Panel and Post Size
Tire Height = less than 6"	Small Fender/Short Post
Tire Height = 6-8"	Medium Fender/Medium Post
Tire Height = greater than 8"	Large Fender/Tall Post

## ADULT & PEDIATRIC RIGID

Fender side guard panel size and profile is driven by the height of the tire over the seat frame and Center of Gravity (CG) position. To calculate this, use the formula and CG rules listed below. Once you have the output of those use the reference matrix to find the appropriate panel and post size.

**Formula:** Wheel Outside Diameter (OD) – Rear Seat Height = Tire Height over seat

**CG Rules:** Axle CG position = distance from axle to backrest tube

*Ethos & Rogue:* CG position  $\leq 3.0$ " use Short Fenders & if CG is  $\geq 3.25$ " use Standard Fenders

*Tsunami:* CG position  $\leq 3.25$ " use Short Fenders & if CG is  $> 3.25$ " use Standard Fenders

*Pediatric Rigid:* CG position  $\leq 3.25$ " use Short Fenders & if CG is  $> 3.25$ " use Standard Fenders

### Reference Matrix:

Tire Height	Panel and Post Size
Tire Height = less than 7"	Small Fender/Short Post
Tire Height = 7-8.5"	Medium Fender/Medium Post
Tire Height = greater than 8.5"	Large Fender/Tall Post



# Wheel & Tire Outside Diameter Measurement Reference

Size	Tire Description	Outside Diameter
18"	Street Pneumatic 1-3/8"	18.9
18"	Full Profile Poly 1-3/8"	18.5
18"	Pneumatic w/Airless Insert 1-3/8"	18.8
18"	Low Profile Poly 1"	17.7
20"	Street Pneumatic 1-3/8"	20.7
20"	Full Profile Poly 1-3/8"	20.2
20"	Pneumatic w/Airless Insert 1-3/8"	20.5
20"	Low Profile Poly 1"	19.3
20"	Sentinel Solid 1"	19.6
20"	High Pressure 1"	20.3
20"	SHOX	19.6
22"	Street Pneumatic 1-3/8"	22.5
22"	Full Profile Poly 1-3/8"	22.3
22"	Pneumatic w/Airless Insert 1-3/8"	22.4
22"	Low Profile Poly 1"	21.4
22"	Sentinel Solid 1"	21.9
22"	High Pressure 1"	22.3
22"	SHOX	22
22"	Mountain 2"	24.1
24"	Street Pneumatic 1-3/8"	24
24"	Full Profile Poly 1-3/8"	23.6
24"	Pneumatic w/Airless Insert 1-3/8"	23.9
24"	Schwalbe HS-127 1-3/8"	24
24"	Euro Trek Knobby 1-3/8"	24.1
24"	Low Profile Poly 1"	22.8
24"	Sentinel Solid 1"	23.3
24"	High Pressure 1"	23.6
24"	Schwalbe One 1"	23.8
24"	Schwalbe Marathon Plus EVO 1"	24
24"	SHOX	23.3
24"	Schwalbe Downtown 1"	23.6
24"	Mountain 2"	25.6
25"	Sentinel Solid 1"	24
25"	High Pressure 1"	24.3
25"	Schwalbe One 1"	24.3
25"	Schwalbe Marathon Plus EVO 1	24.4
25"	SHOX	24.1
25"	Mountain 2"	26
26"	Street Pneumatic 1-3/8"	26
26"	Full Profile Poly 1-3/8"	25.8
26"	Pneumatic w/Airless Insert 1-3/8"	25.8
26"	Sentinel Solid 1"	25.1
26"	High Pressure 1"	25.5
26"	Schwalbe Marathon Plus EVO 1"	25.9
26"	SHOX	25.3

