


Focus CR Height Adjustable Low T-Arm

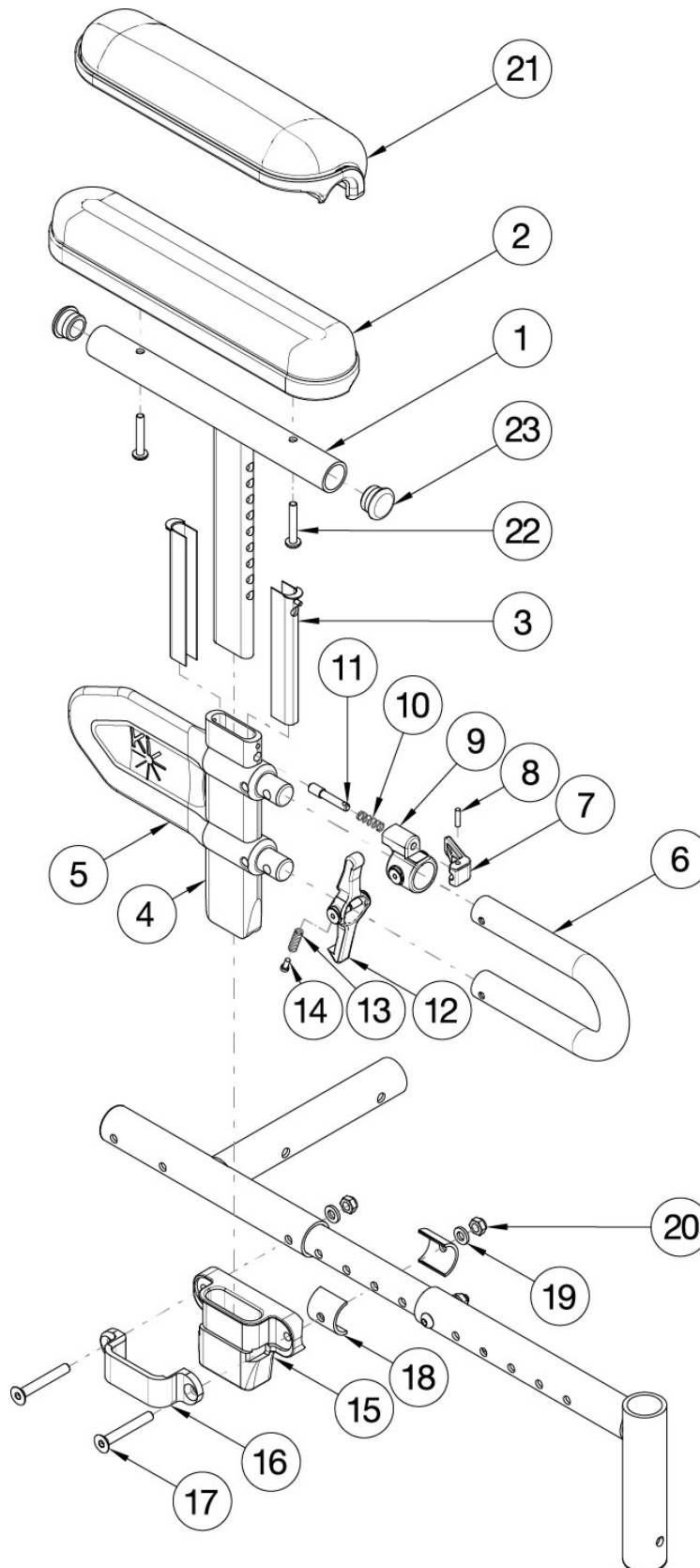

If retrofitting, Receivers are needed.

Item Number	Part Number	Description	Quantity
1a, 2b, 3-19, 22a	121213	Height Adj T-Arm Assm Low, Desk w/ Std Pad and Receiver PAIR Focus	1
1a, 2b, 3-19, 22a	121207	Height Adj T-Arm Assm Low, Desk w/ Std Pad and Receiver Focus	1
1b, 2a, 3-19, 22a	113453	Height Adj T-Arm Assm Low, Full w/ Std Pad and Receiver PAIR Focus	1
1b, 2a, 3-19, 22a	113349	Height Adj T-Arm Assm Low, Full w/ Std Pad and Receiver Focus	1
1a, 3-19, 21b, 22b	121215	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and Receiver PAIR Focus	1
1a, 3-19, 21b, 22b	121208	Height Adj T-Arm Assm Low, Desk w/ Waterfall Pad and Receiver Focus	1
1a, 3-19, 21a, 22b	113455	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and Receiver PAIR Focus	1
1a, 3-19, 21a, 22b	113353	Height Adj T-Arm Assm Low, Full w/ Waterfall Pad and Receiver Focus	1
1a, 2b, 3-14, 22a, 23	121241	Height Adjustable T-Arm Assembly Low, Desk w/ Std Pad	1
1b, 2a, 3-14, 22a, 23	503169	Height Adjustable T-Arm Assembly Low Full w/ Std Pad	1
1a, 3-14, 21b, 22b, 23	121243	Height Adjustable T-Arm Assembly Low, Desk w/ Waterfall Pad	1
1b, 3-14, 21a, 22b, 23	503185	Height Adjustable T-Arm Assembly Low Full w/ Waterfall Pad	1
1a, 3-14, 23	114488	Height Adjustable T-Arm, Desk, Low <i>Replaced 002681 / 002682</i>	1
1b, 3-14, 23	114489	Height Adjustable T-Arm, Full, Low <i>Replaced 002683 / 002684</i>	1
15-20	501730	Height Adj T-Arm Receiver Assm 1" CAT	1
7-11	500541	Arm Height Adjustable T Upper Lever Assembly	1
1a, 23	002648	Post Weldment, T-Arm, Low, Desk	1
1b, 23	002649	Post Weldment, T-Arm, Low, Full	1
2a	100500	Arm Pad Full	1
2b	100501	Arm Pad Desk	1
3	000090	Height Adjustable T Armrest Shim	2
4	002105	Tube, Outer, T-Arm, Low	1
6	002104	Transfer Tube, T-Arm, Low	1
8	200257	Dowel Pin 1/8 x 1/2	1
9	000134	Height Adjustable Armrest Lever Spring	1

Item Number	Part Number	Description	Quantity
10	000099	Height Adjustable Armrest Lever Pin	1
11	000096	Height Adjustable T Armrest Pin Lock Housing	1
12	002106	Lever, T-Arm, Low	1
13	000135	Height Adjustable T Armrest Release Lever Spring	1
14	000152	Height Adjustable T Armrest Release Spring Button	1
15	000087	Height Adjustable T-Arm Receiver 1"	1
16	000088	Height Adjustable T Armrest Bracket	1
17	100866	M6 x 20 CLS12.9 DIN7991 FSHCS Blk Zc w/Patch	2
22a	100696	M5 x 30mm PPH Capscrew <i>Used with Standard Arm Pads</i>	2
22b	100654	M5 x 40 DIN7985 PH Pan HD M/S ZC <i>Used with Waterfall Arm Pads</i>	2
23	100803	Tube End Plug 7/8"	2
18	114443	Tube Spacer, Black, 1" OD x .083" x .80 LG	0
19	100746	M6 Flat Washer Blk Zn	2
20	100658	M6 Nylock Nut Blk Zn	2
21	102452	Waterfall Arm Pad Full	1
21	102521	Waterfall Arm Pad Desk	1