




# Liberty Seat Pan

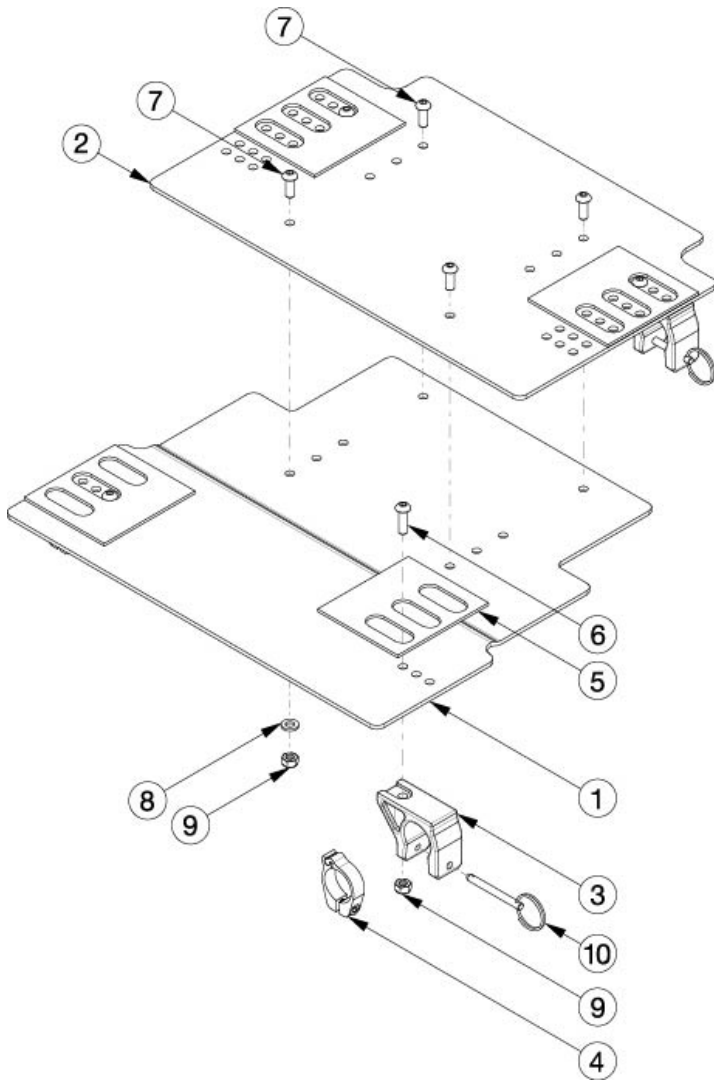

Front (Item 1)	
Frame Depth	Pan Depth
Short (14"-16")	4.5"
Medium (16"-18")	6.5"
Long (18"-20")	8.5"

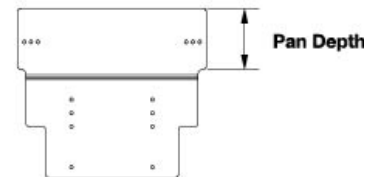

Rear (Item 2)	
Frame Depth	Pan Depth
All	8.75"

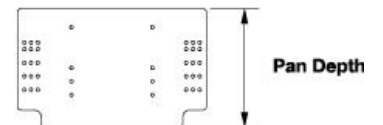

Item Number	Part Number	Description	Quantity
1a, 2a, 3-10	115951	Seat Pan Assembly, Short Depth, 14-16" Wide, Liberty	1
1b, 2b, 3-10	115952	Seat Pan Assembly, Short Depth, 16-18" Wide, Liberty	1
1c, 2c, 3-10	115953	Seat Pan Assembly, Short Depth, 18-20" Wide, Liberty	1
1d, 2d, 3-10	115954	Seat Pan Assembly, Short Depth, 20-22" Wide, Liberty <i>Used on 20" and 21" wide chairs</i>	1
1e, 2e, 3-10	117932	Seat Pan Assembly, Short Depth, 22" Wide, Liberty <i>Used on 22" wide chairs</i>	1
1f, 2a, 3-10	115955	Seat Pan Assembly, Medium Depth, 14-16" Wide, Liberty	1
1g, 2b, 3-10	115956	Seat Pan Assembly, Medium Depth, 16-18" Wide, Liberty	1

Item Number	Part Number	Description	Quantity
1h, 2c, 3-10	115957	Seat Pan Assembly, Medium Depth, 18-20" Wide, Liberty	1
1i, 2d, 3-10	115958	Seat Pan Assembly, Medium Depth, 20-22" Wide, Liberty <i>Used on 20" and 21" wide chairs</i>	1
1j, 2e, 3-10	117933	Seat Pan Assembly, Medium Depth, 22" Wide, Liberty <i>Used on 22" wide chairs</i>	1
1k, 2a, 3-10	115959	Seat Pan Assembly, Long Depth, 14-16" Wide, Liberty	1
1l, 2b, 3-10	115960	Seat Pan Assembly, Long Depth, 16-18" Wide, Liberty	1
1m, 2c, 3-10	115961	Seat Pan Assembly, Long Depth, 18-20" Wide, Liberty	1
1n, 2d, 3-10	115962	Seat Pan Assembly, Long Depth, 20-22" Wide, Liberty <i>Used on 20" and 21" wide chairs</i>	1
1o, 2e, 3-10	117934	Seat Pan Assembly, Long Depth, 22" Wide, Liberty <i>Used on 22" wide chairs</i>	1
1a, 5	115963	Seat Pan, Front, Short, 14-16" Wide, Liberty	1
1b, 5	115964	Seat Pan, Front, Short, 16-18" Wide, Liberty	1
1c, 5	115965	Seat Pan, Front, Short, 18-20" Wide, Liberty	1
1d, 5	115966	Seat Pan, Front, Short, 20-22" Wide, Liberty <i>Used on 20" and 21" wide chairs</i>	1
1e, 5	117935	Seat Pan, Front, Short, 22" Wide, Liberty <i>Used on 22" wide chairs</i>	1
1f, 5	115967	Seat Pan, Front, Medium, 14-16" Wide, Liberty	1
1g, 5	115968	Seat Pan, Front, Medium, 16-18" Wide, Liberty	1
1h, 5	115969	Seat Pan, Front, Medium, 18-20" Wide, Liberty	1
1i, 5	115970	Seat Pan, Front, Medium, 20-22" Wide, Liberty <i>Used on 20" and 21" wide chairs</i>	1
1j, 5	117936	Seat Pan, Front, Medium, 22" Wide, Liberty <i>Used on 22" wide chairs</i>	1
1k, 5	115971	Seat Pan, Front, Long, 14-16" Wide, Liberty	1
1l, 5	115972	Seat Pan, Front, Long, 16-18" Wide, Liberty	1
1m, 5	115973	Seat Pan, Front, Long, 18-20" Wide, Liberty	1
1n, 5	115974	Seat Pan, Front, Long, 20-22" Wide, Liberty <i>Used on 20" and 21" wide chairs</i>	1
1o, 5	117937	Seat Pan, Front, Long, 22" Wide, Liberty	1

Item Number	Part Number	Description	Quantity
		<i>Used on 22" wide chairs</i>	
2a, 5	115975	Seat Pan, Rear, 14-16" Wide, Liberty	1
2b, 5	115976	Seat Pan, Rear, 16-18" Wide, Liberty	1
2c, 5	115977	Seat Pan, Rear, 18-20" Wide, Liberty	1
2d, 5	115978	Seat Pan, Rear, 20-22" Wide, Liberty <i>Used on 20" and 21" wide chairs</i>	1
2e, 5	117938	Seat Pan, Rear, 22" Wide, Liberty <i>Used on 22" wide chairs</i>	1
3	113165	Seat Pan Mount, Removable, Liberty	4
4	102856	Seat Pan Stop Collar	2
5	001206	AB Tape 3x3 Die Cut Tilt	4
6	102623	M6 x 20 BHCS BLK ZC	4
7	102634	M6 X 1.0 X 16 BHCS BLK ZC	4
8	100746	M6 Flat Washer Blk Zn	4
9	100658	M6 Nylock Nut Blk Zn	8
10	111408	Pin with Lanyard	4