


Standard 5th Wheel

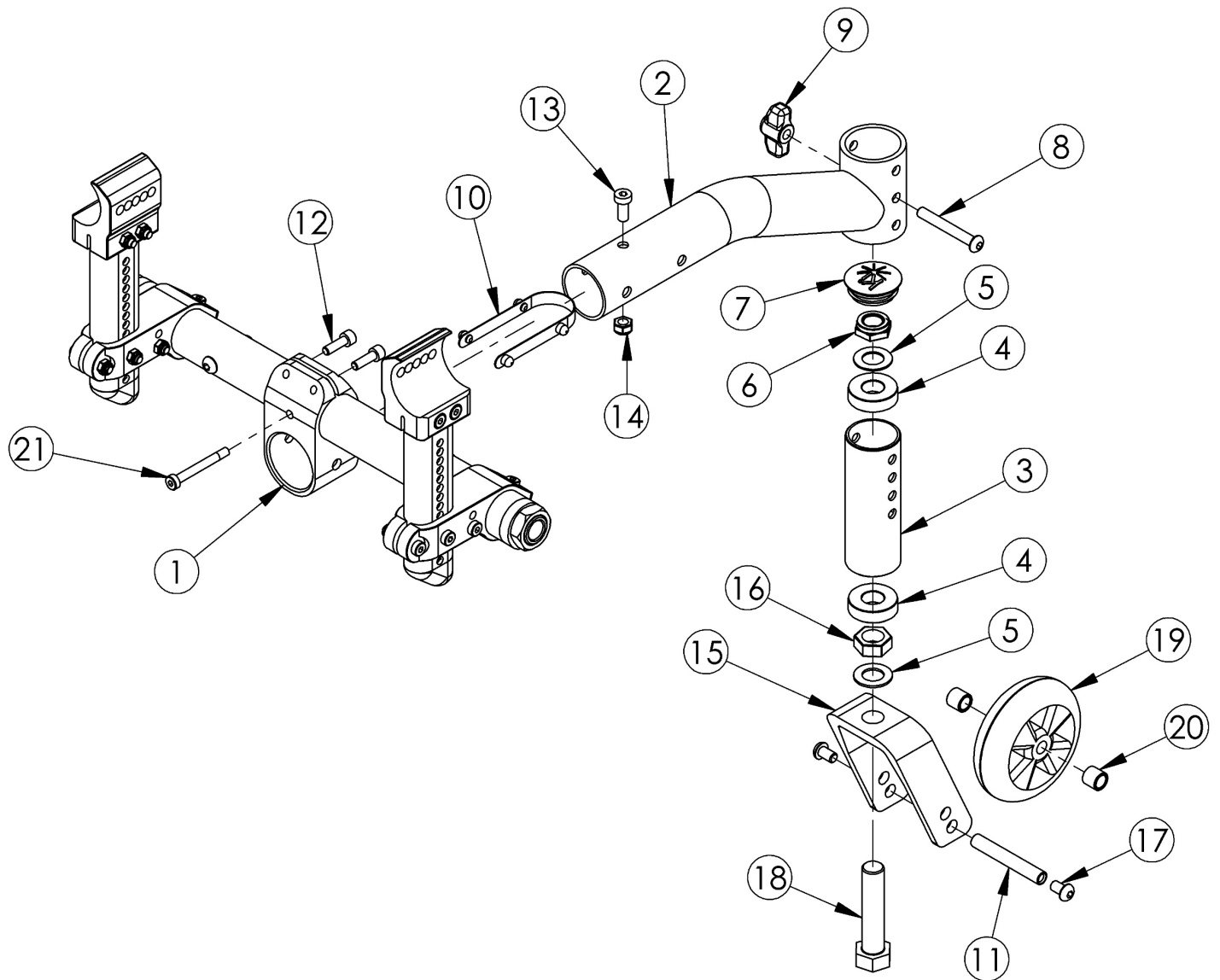

A new camber tube is needed if retrofitting to the Standard 5th Wheel

Item Number	Part Number	Description	Quantity	Retail
1, 2a, 3 - 10, 12 - 16, 18, 21	500589	5th Wheel Assembly Without Caster for 18" or 20" Rear Wheels	1	\$342.33
1, 2b, 3 - 10, 12 - 16, 18, 21	505262	5th Wheel Assembly Without Caster for 22" or 24" Rear Wheels	1	\$343.98
1, 2a, 3 - 18, 19a, 20, 21	500590	5th Wheel Assembly w/ 3" Rollerblade Caster for 18" or 20" Rear Wheels	1	\$394.55
1, 2b, 3 - 18, 19b, 20, 21	500600	5th Wheel Assembly w/ 4" Rollerblade Caster for 22" or 24" Rear Wheels	1	\$396.20
1, 2a, 3 - 18, 19c, 20, 21	505258	5th Wheel Assembly w/ 3" Lighted Rollerblade Caster for 18" or 20" Rear Wheels	1	\$394.55
1, 2b, 3 - 18, 19d, 20, 21	505259	5th Wheel Assembly w/ 4" Lighted Rollerblade Caster for 22" or 24" Rear Wheels	1	\$396.20

Item Number	Part Number	Description	Quantity	Retail
1, 2b, 3 - 18, 19e, 20, 21	505260	5th Wheel Assembly w/ 4 x 1" Poly Caster for 22" or 24" Rear Wheels	1	\$396.20
1, 2b, 3 - 18, 19f, 20, 21	505261	5th Wheel Assembly w/ 5 x 1" Poly Caster for 24" Rear Wheels	1	\$396.20
1	000593	5th Wheel Mounting Clamp	1	\$39.31
2a	000595	5th Wheel Mounting Arm <i>Used with 18" and 20" rear wheel sizes</i>	1	\$57.17
2b	001313	5th Wheel Mount Long <i>Used with 22" and 24" rear wheel sizes</i>	1	\$58.83
3	000594	5th Wheel Caster Housing	1	\$37.52
4	100530	Bearing M28 x 1/2" 6001RS	2	\$12.02
5	100682	Washer 1/2" .518 x .875 x .047 F/W Black Zinc	2	\$1.05
6	100900	1/2-13 NTE JAM NYL INS L/N HXFRE ZC	1	\$1.05
7	102005	Ki Caster Housing Cap	1	\$1.17
8	101123	M6 x 45 Low SHCS BLK ZC	1	\$1.58
9	101085	Mini-Wing M6 Tapped Knob	1	\$6.43
10	101081	5th Wheel Detent Button	1	\$18.77
11	000422	M6 x 56 Threaded Barrel	1	\$10.64
12	101120	M5 x 16 SHCS BLZ	2	\$1.58
13	101185	M6 x 14 CLS8.8 DIN7984 Low SHCS BLKZC	1	\$1.58
14	100658	M6 Nylock Nut Blk Zn	1	\$1.05
15	503822	Caster Fork 4", Std 5th Wheel <i>Contains 100545 fork with Removal and Installation Instruction</i>	1	\$44.95
16	101598	1/2-13 FIN HX JAM NUT ZC	1	\$1.58
17	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.58
18	101597	1/2-13 x 2-1/4 HH TAP BLT FULL THD ZC	1	\$1.58
19a	100790	Caster 3" Rollerblade <i>Used with 18" and 20" rear wheel sizes</i>	1	\$25.50
19b	100791	Caster 4" Rollerblade <i>Used with 22" and 24" rear wheel sizes</i>	1	\$26.27
19c	101140	Caster 3" Lighted Rollerblade <i>Used with 18" and 20" rear wheel sizes</i>	1	\$47.28

Item Number	Part Number	Description	Quantity	Retail
19d	100952	Caster 4" Lighted Rollerblade <i>Used with 22" and 24" rear wheel sizes</i>	1	\$52.53
19e	100809	Caster 4 x 1" Poly (1.25") <i>Used with 22" and 24" rear wheel sizes</i>	1	\$58.83
19f	100807	Caster 5 x 1" Poly (1.25") <i>Used only with 24" rear wheel sizes</i>	1	\$58.83
20	100792	Caster Spacer D8 x D12 x 10mm	2	\$2.14
21	101563	M5 x 42 Low SHCS PT BLKZC	1	\$1.58