


Rigid Swing Away Armrest

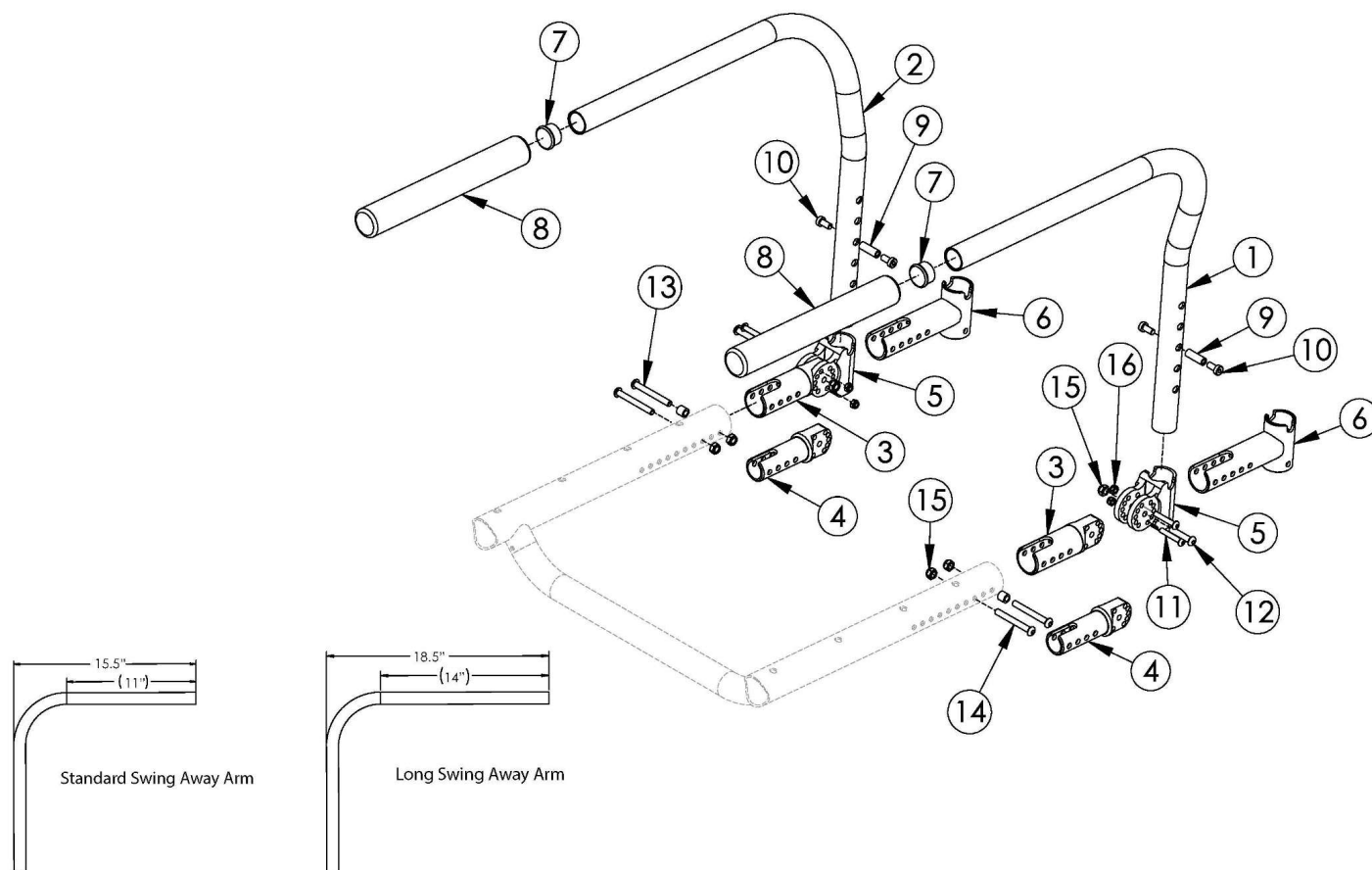

The following rules apply to the *Tsunami*, *Rogue ALX*, and *Little Wave*: Standard Swing Away Armrests not available with 3" adjustable depth in combination with transit. Standard Swing Away Armrests not available with 2" adjustable depth in combination with transit and center of gravity 0" or less. Long Swing Away Armrests not available with 3" adjustable depth in combination with transit and center of gravity 0" or less. Long Swing Away Armrests not available with 2" adjustable depth in combination with 20" seat depth, transit, and center of gravity 0" or less.

Item Number	Part Number	Description	Quantity
1a, 7 - 10	500986	Arm Assembly Swing Away Left Rigid	1
2a, 7 - 10	500987	Arm Assembly Swing Away Right Rigid	1
1a, 2a, 7 - 10	500551	Arm Assembly Swing Away Pair Rigid	1
1b, 7 - 10	501476	Arm Assembly Long Swing Away Left Rigid	1
2b, 7 - 10	501477	Arm Assembly Long Swing Away Right Rigid	1
1b, 2b, 7 - 10	501478	Arm Assembly Long Swing Away Pair Rigid	1
3, 5, 11, 12, 14, 15, 16	500991	Swing Away Arm Receiver Assembly AL Rigid <i>Used on Rogue, Tsunami Little Wave and Tsunami AL</i>	1

Item Number	Part Number	Description	Quantity
6, 14, 15	506101	Swing Away Arm Receiver Assembly ALX Rigid <i>Used on Rogue ALX and Tsunami ALX</i>	1
3	000441	Arm Swing Away Mount AL Rigid	1
4	000445	Arm Swing Away Mount TI	1
5	000446	Arm Swing Away Receiver Rigid	1
6	000386	Arm Swing Away Receiver for RG ALX	1
7	100804	Tube End Plug 1"	1
9	000065	M6 x 23.68 Threaded Barrel	1
10	101179	M6 X 10 Shcs Blk Zc W/Patch	2
11	101560	M5 x 30 BTN SHCS ZC	2
12	101119	M6 X 32 BHCS bright	1
13	101124	M6 x 50 Low SHCS BLK ZC	1
14	100797	M6 x 40 BHCS BLK ZC	1
15	100658	M6 Nylock Nut Blk Zn	2
16	100657	M5 Nylock Nut Blk Zn	2