



Focus CR Base Frame

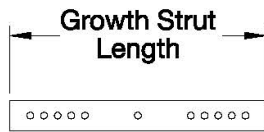

Growth Strut	Length
Short	6.6"
Medium	8.6"
Long	10.6"

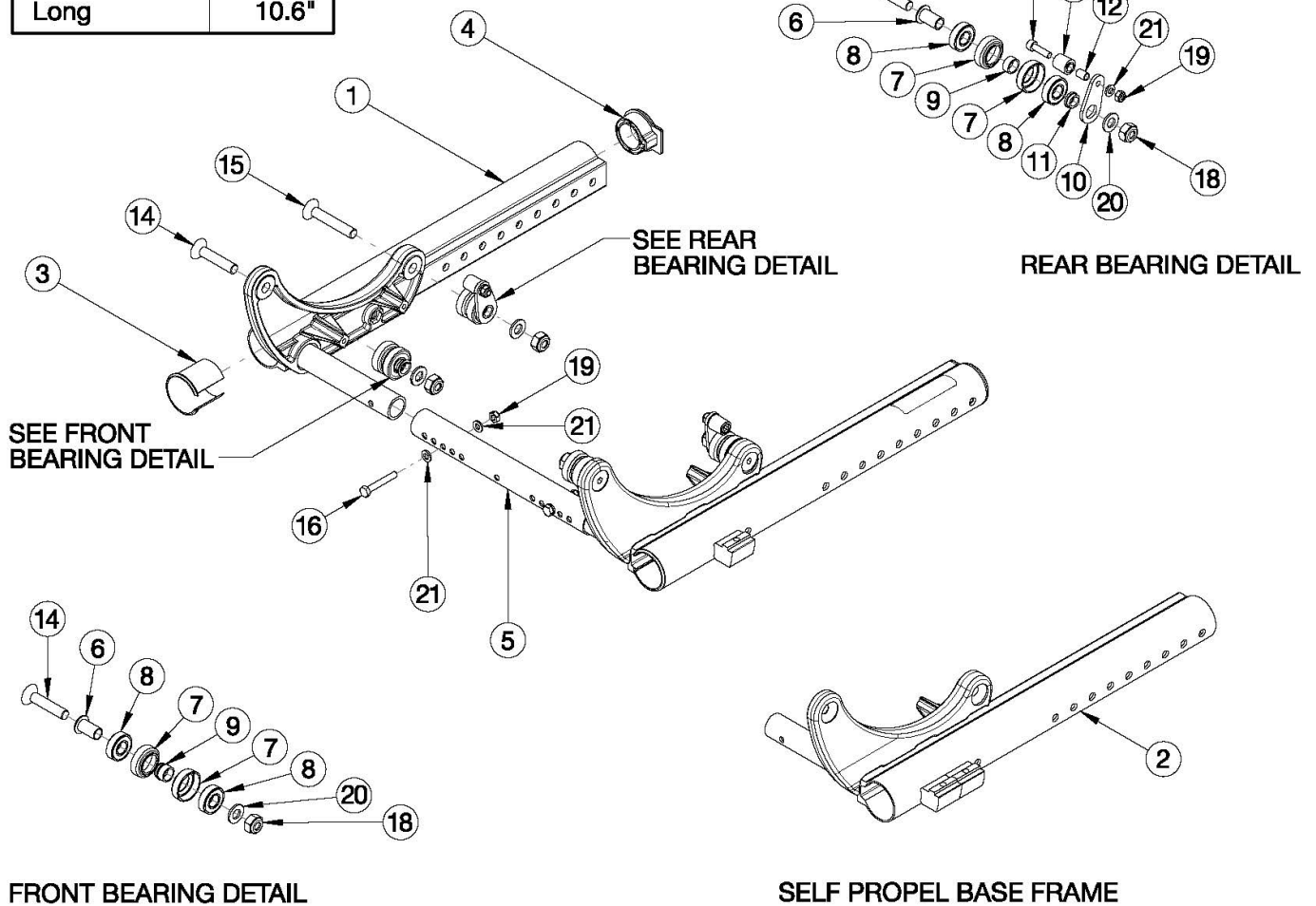

<p>
Heavy Duty Base Frame required for X-Large (20"-24") Frame Width Range

Item Number	Part Number	Description	Quantity	Retail
1a, 3, 4	501064_ _ _	Base Frame Assembly Right	1	\$405.73
1b, 3, 4	501065_ _ _	Base Frame Assembly Left	1	\$405.73
2a, 3, 4	503413_ _ _	Base Frame Self Propel Assembly Right	1	\$425.49

Item Number	Part Number	Description	Quantity	Retail
2b, 3, 4	503414_ _ _	Base Frame Self Propel Assembly Left	1	\$425.49
6-9, 14, 18, 20	502412	Front Bearing Kit Focus (1 Side)	1	\$70.47
6-13, 15, 17-21	502413	Rear Bearing Kit Focus (1 Side)	1	\$112.05
6-15, 17-21	501066	Bearing Replacement Kit Focus	1	\$364.71
3	001009	Sleeve Base Frame TILT	2	\$5.40
4	001010	Plug Base Frame TILT	2	\$10.12
5a	001011	Growth Strut, Base, Small Tilt <i>Used on Small Width Range</i>	1	\$18.91
5b	001012	Growth Strut, Base, Medium Tilt <i>Used on Medium Width Range</i>	1	\$18.91
5c	001013	Growth Strut, Base, Large Tilt <i>Used on Large Width Range</i>	1	\$18.91
6	001014	Bearing Shaft TILT	4	\$24.74
7	001015	Bearing Liner TILT	8	\$5.40
8	100902	Bearing 1 1/8" x 1/2" R8RS	8	\$6.30
9	001016	Bearing Spacer TILT	4	\$6.74
10	001017	Swivel Plate TILT	2	\$7.42
12	001019	Bearing Shaft Upper	2	\$5.62
13	001020	Bearing Upper TILT	2	\$5.40
14	101816	M10 x 50 FHCS Blk Zn	2	\$3.68
15	101817	M10 x 60 FHCS Blk Zn	2	\$4.73
16	101825	M6 x 40 HHCS Blk Zn	2	\$1.05
17	101845	M6 x 22 SHCS Blk Zn	2	\$1.58
18	101840	M10 Nylock Nut Blk Zn	4	\$1.05
19	100658	M6 Nylock Nut Blk Zn	4	\$1.05
20	101859	M10 Flat Washer Blk Zn	4	\$0.53
21	100746	M6 Flat Washer Blk Zn	6	\$1.58

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer