


Spark Rear Frame

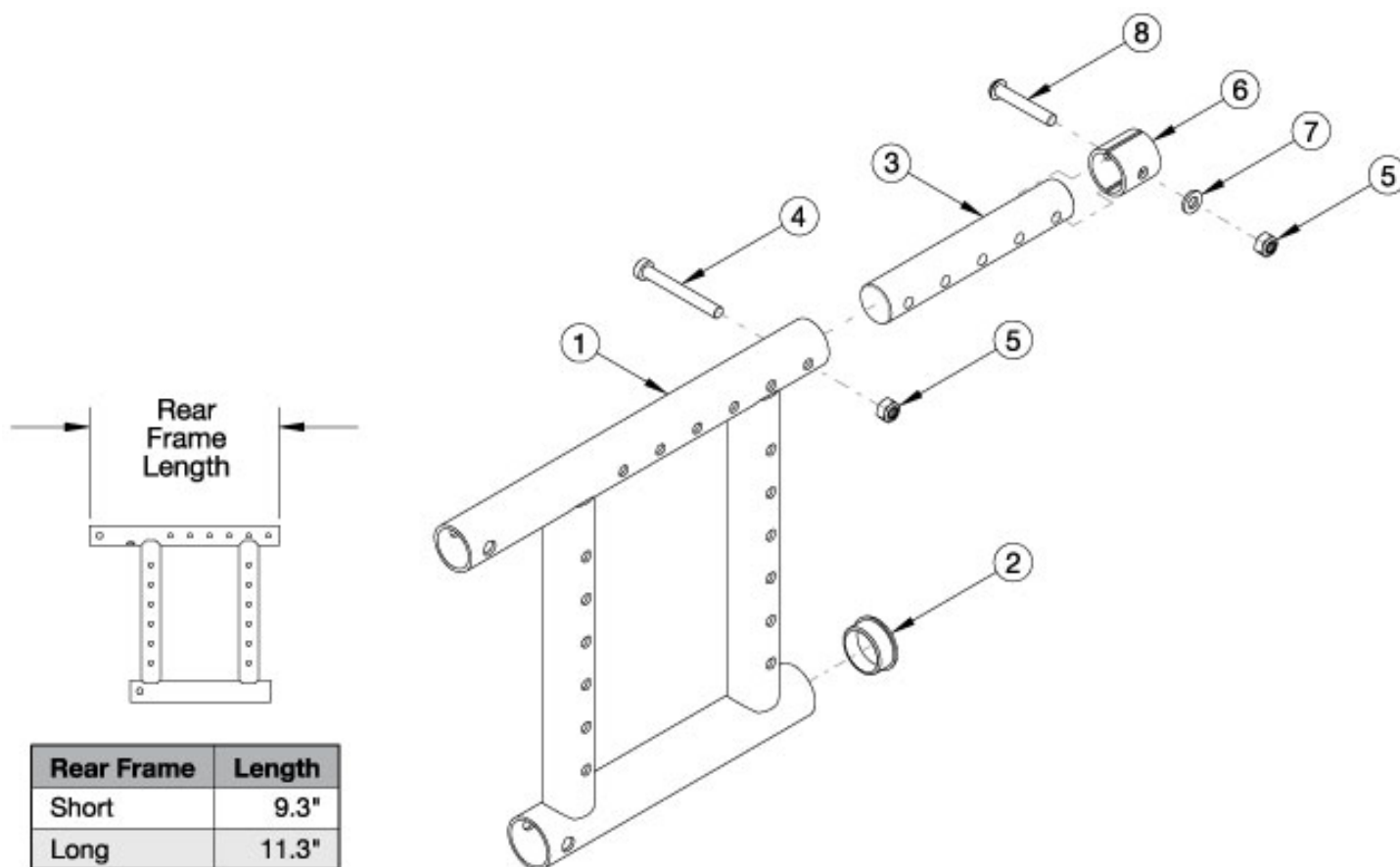

Use short frame for 12-13" Frame Depths. Use long frame for 14-19" Frame Depths If changing to a new seat depth, new Height Adjustable T-Arms may be required

Item Number	Part Number	Description	Quantity
1a, 2	101609_ _ _	Frame Assembly, Spark Rear, Left, Short <i>Use when Height Adjustable T-Arm is not present</i>	1
1b, 2	501637_ _ _	Frame Assembly, Spark Rear, Right, Short <i>Use when Height Adjustable T-Arm is not present</i>	1
1f, 2	111376_ _ _	Frame Assembly for Height Adjustable T-Arm, Spark Rear, Left, Short	1
1g, 2	111377_ _ _	Frame Assembly for Height Adjustable T-Arm, Spark Rear, Right, Short	1
1c, 2	101608_ _ _	Frame Assembly, Spark Rear, Left, Long <i>Use when Height Adjustable T-Arm is not present</i>	1
1d, 2	501638_ _ _	Frame Assembly, Spark Rear, Right, Long <i>Use when Height Adjustable T-Arm is not present</i>	1
1e, 2	111375_ _ _	Frame Assembly for Height Adjustable T-Arm, Spark Rear, Left, Long	1

Item Number	Part Number	Description	Quantity
1h. 2	111378_ _ _	Frame Assembly for Height Adjustable T-Arm, Spark Rear, Right, Long	1
3, 5-8	115765	Depth Adjustable Extension with Hardware, Catalyst/Spark	1
2	100532	Tube End Plug 1 1/8"	1
3	000605	Depth Adjustable Frame Extension Tube, Catalyst/Spark	1
4	101124	M6 x 50 Low SHCS BLK ZC	1
5	100658	M6 Nylock Nut Blk Zn	1
6	114443	Tube Spacer, Black, 1" OD x .083" x .80 LG <i>Used with Stroller Handle Backpost and Height Adjustable Straight with Push Handle Backpost designs released February 8th, 2021</i>	2
7	100746	M6 Flat Washer Blk Zn <i>Used with Stroller Handle Backpost and Height Adjustable Straight with Push Handle Backpost designs released February 8th, 2021</i>	1
8	101811	M6 x 35 BHCS Blk Zn <i>Used with Stroller Handle Backpost and Height Adjustable Straight with Push Handle Backpost designs released February 8th, 2021</i>	1

Matte Finish	
Code	Color Name
CGM	Charcoal Gray
EGM	Emerald Green
GGM	Graphite Gray*
IBM	Iridescent Burgundy
INM	Iridescent Navy
AGM	Military Green**
RRM	Retro Red
RBM	Royal Blue
SBM	Shadow Black
SWM	Smooth White

* Discontinued Color

** Formerly called Army Green

Gloss Finish	
Code	Color Name
BLK	Black
BLU	Blue
SKY	Blue Skies
BRO	Bright Orange*
PNK	Bubblegum Pink
BUR	Burgundy*
CBL	Candy Blue
PUR	Candy Purple
RED	Candy Red
COP	Copper
DPR	Deep Purple*
EBL	Electric Blue*
GRN	Forest Green*
FUS	Fuchsia*
GSM	Granny Smith
MTF	Metallic Fuchsia*
MOR	Metallic Orange
ORG	Orange*
SAP	Sapphire*
SIL	Silver
TEA	Teal
YLW	Yellow
FLM	Graphic Image Transfer