


Arc Dual Hand Tilt Mechanism

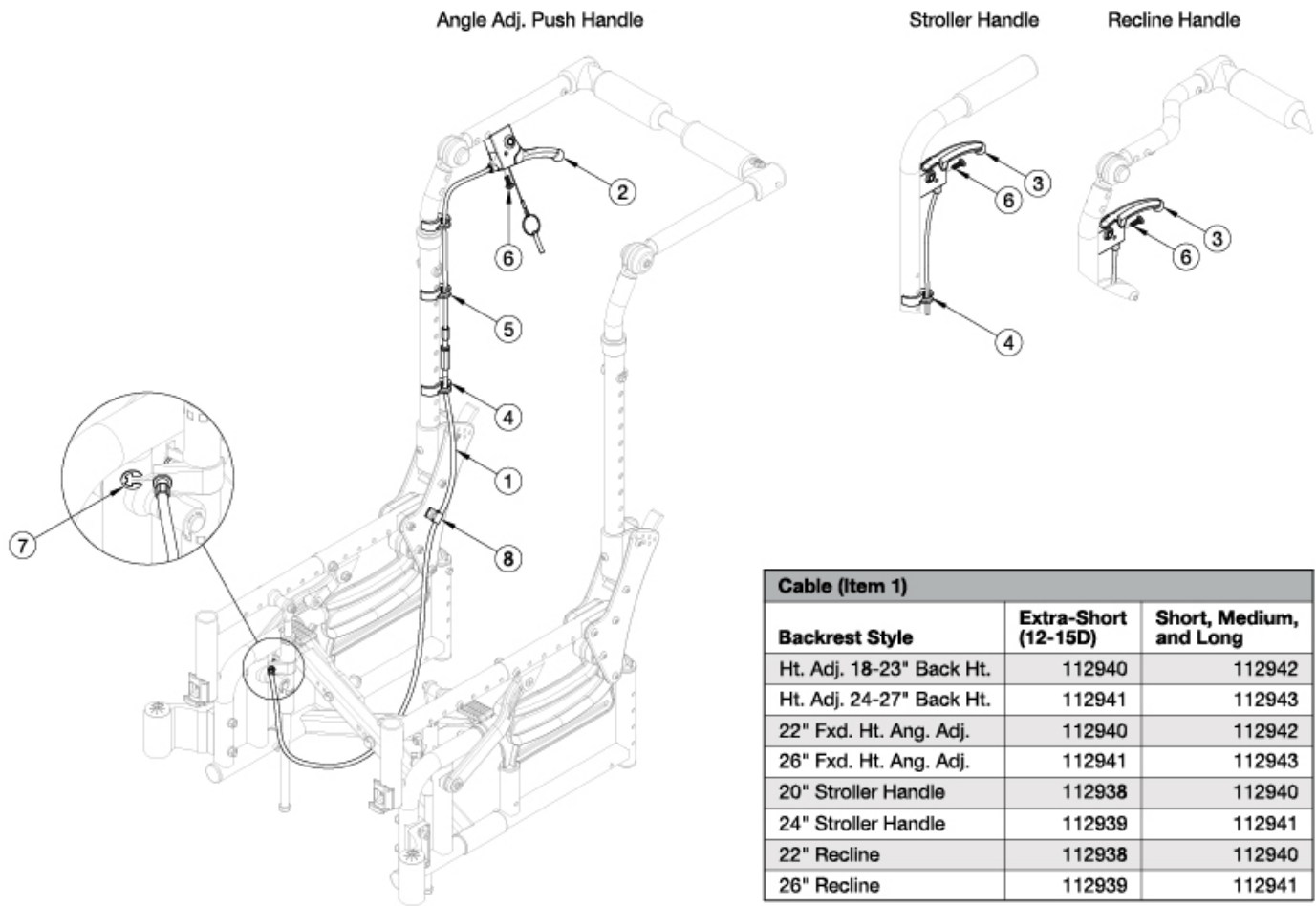

Cable (Item 1)		
Backrest Style	Extra-Short (12-15D)	Short, Medium, and Long
Ht. Adj. 18-23" Back Ht.	112940	112942
Ht. Adj. 24-27" Back Ht.	112941	112943
22" Fxd. Ht. Ang. Adj.	112940	112942
26" Fxd. Ht. Ang. Adj.	112941	112943
20" Stroller Handle	112938	112940
24" Stroller Handle	112939	112941
22" Recline	112938	112940
26" Recline	112939	112941

Please note cable kits 112938 - 112943 contain two cables and two e-rings.

Item Number	Part Number	Description	Quantity
1c, 2, 4-8	112988	Complete Dual Hand Tilt, Height Adjustable, 18-23" Height, Extra Short Depth Range, Arc	1
1e, 2, 4-8	112989	Complete Dual Hand Tilt, Height Adjustable, 18-23" Height, Short/Med/Long Depth Range, Arc	1
1d, 2, 4-8	112990	Dual Hand Tilt, Height Adjustable, 24-27" Height, Extra Short Depth Range, Arc	1
1f, 2, 4-8	112991	Dual Hand Tilt, Height Adjustable, 24-27" Height, Short/Med/Long Depth Range, Arc	1
1c, 3, 4, 6-8	112992	Dual Hand Tilt, Fixed Height, 22" Height, Extra Short Depth Range, Arc	1
1e, 3, 4, 6-8	112993	Dual Hand Tilt, Fixed Height, 22" Height, Short/Med/Long Depth Range, Arc	1
1d, 3, 4, 6-8	112994	Dual Hand Tilt, Fixed Height, 26" Height, Extra Short Depth Range, Arc	1
1f, 3, 4, 6-8	112995	Dual Hand Tilt, Fixed Height, 26" Height, Short/Med/Long Depth Range, Arc	1

Item Number	Part Number	Description	Quantity
1a, 3, 4, 6-8	112996	Dual Hand Tilt, Stroller Handle, 20" Height, Extra Short Depth Range, Arc	1
1c, 3, 4, 6-8	112997	Dual Hand Tilt, Stroller Handle, 20" Height, Short/Med/Long Depth Range, Arc	1
1b, 3, 4, 6-8	112998	Dual Hand Tilt, Stroller Handle, 24" Height, Extra Short Depth Range, Arc	1
1d, 3, 4, 6-8	112999	Dual Hand Tilt, Stroller Handle, 24" Height, Short/Med/Long Depth Range, Arc	1
1a, 3, 4, 6-8	113000	Dual Hand Tilt, Reclining, 22" Height, Extra Short Depth Range, Arc	1
1c, 3, 4, 6-8	113001	Dual Hand Tilt, Reclining, 22" Height, Short/Med/Long Depth Range, Arc	1
1b, 3, 4, 6-8	113002	Dual Hand Tilt, Reclining, 26" Height, Extra Short Depth Range, Arc	1
1d, 3, 4, 6-8	113003	Dual Hand Tilt, Reclining, 26" Height, Short/Med/Long Depth Range, Arc	1
1a, 7	112938	Cable, Tilt, Dual, Short, Arc	1
1b, 7	112939	Cable, Tilt, Dual, Medium, Arc	1
1c, 7	112940	Cable, Tilt, Dual, Long, Arc	1
1d, 7	112941	Cable, Tilt, Dual, XL, Arc	1
1e, 7	112942	Cable, Tilt, Dual, XXL, Arc	1
1f, 7	112943	Cable, Tilt, Dual, XXXL, Arc	1
2	002801	Lever Assembly Horizontal Tilt <i>Used with Height Adjustable Backrests</i>	2
3	002800	Lever Assembly Vertical Tilt <i>Used with Stroller Handle and Reclining Backrests</i>	2
4	001117	Cable Clip 1"	2
5	001118	Cable Clip 7/8" <i>Used with Height Adjustable Backrests</i>	2
6	100734	M5 x 16 PHMS ZC	2
7	102822	E-Ring, M6, YZ	2
8	001239	P-Clip 3/16"	2