
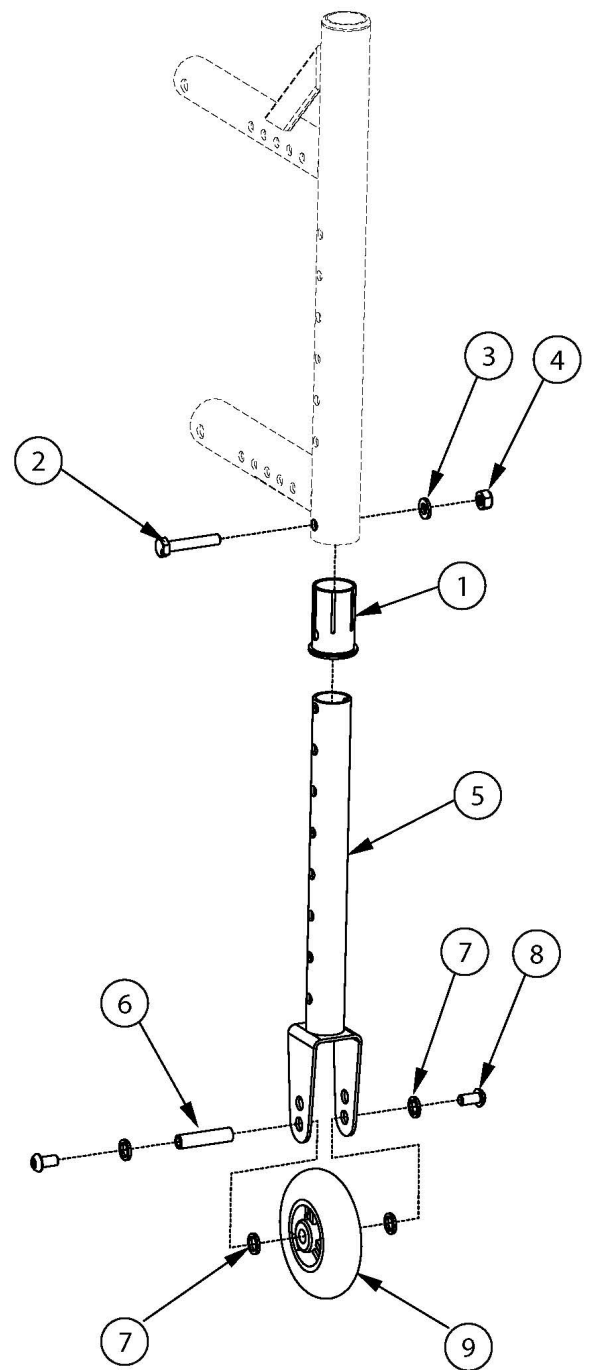


# Catalyst 5VX Transit Wheels

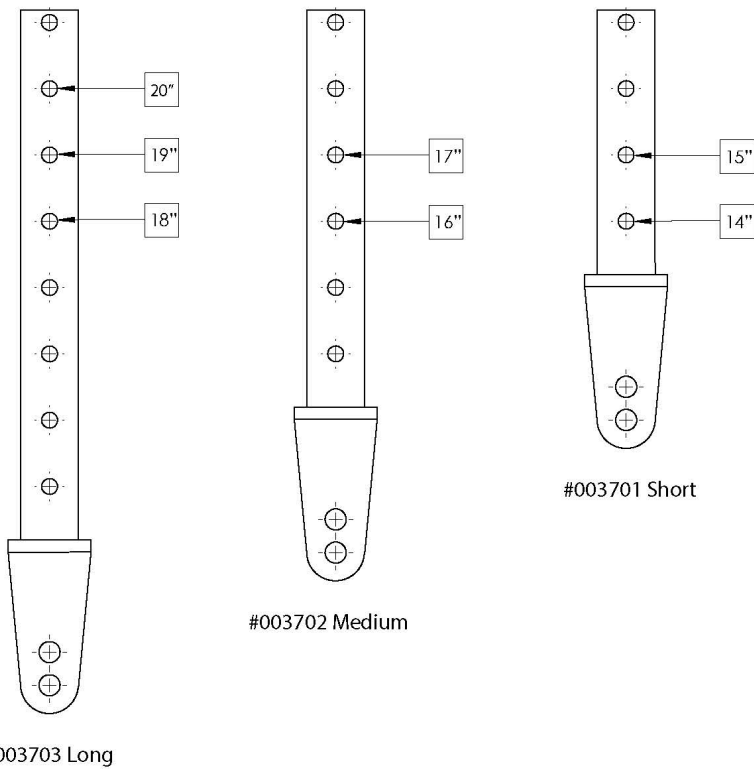

New rear frames required with serial numbers prior to C5053765, C5HD003546, or CVX0091137 <p>  
 Not available with Heavy Duty Option

Item Number	Part Number	Description	Quantity	Retail
1-4, 5a, 6-9	505421	Catalyst 5VX Transit Wheels Tower Assembly Kit, Short (PAIR) Used with 14" - 15" rear seat heights	1	\$145.00

Item Number	Part Number	Description	Quantity	Retail
1-4, 5b, 6-9	505422	Catalyst 5VX Transit Wheel Tower Assembly Kit, Medium (PAIR) <i>Used with 15.5" - 17" rear seat heights</i>	1	\$145.00
1-4, 5c, 6-9	505423	Catalyst 5VX Transit Wheel Tower Assembly Kit, Long (PAIR) <i>Used with 17.5" - 20" rear seat heights</i>	1	\$145.00
1-4, 5a, 6-9	506286	Catalyst 5VX Transit Wheels Tower Assembly Kit, Short (One Side) <i>Used with 14" - 15" rear seat heights</i>	1	\$72.50
1-4, 5b, 6-9	506287	Catalyst 5VX Transit Wheels Tower Assembly Kit, Medium (One Side) <i>Used with 15.5" - 17" rear seat heights</i>	1	\$72.50
1-4, 5c, 6-9	506288	Catalyst 5VX Transit Wheels Tower Assembly Kit, Long (One Side) <i>Used with 17.5" - 20" rear seat heights</i>	1	\$72.50
1	003501	Step Tube/Transit Wheel Sleeve, CAT5VX	1	\$3.15
2	100860	M6 x 35 HHCS P/THD ZC	1	\$1.58
3	100746	M6 Flat Washer Blk Zn	1	\$1.58
4	100658	M6 Nylock Nut Blk Zn	1	\$1.05
5a	003701	Transit Wheel Tower Short <i>Used with 14" - 15" rear seat heights</i>	1	\$94.55
5b	003702	Transit Wheel Tower Medium <i>Used with 15.5" - 17" rear seat heights</i>	1	\$94.55
5c	003703	Transit Wheel Tower Long <i>Used with 17.5" - 20" rear seat heights</i>	1	\$94.55
6	000347	M6 x 1.0 Threaded Barrel	1	\$9.06
7	101701	Washer 3.315/.350 x .425/.525 x .059/.067 18-8 F/W	4	\$2.63
8	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2	\$1.58
9	100790	Caster 3" Rollerblade	1	\$25.50