


Clik Camber Tube - Growth

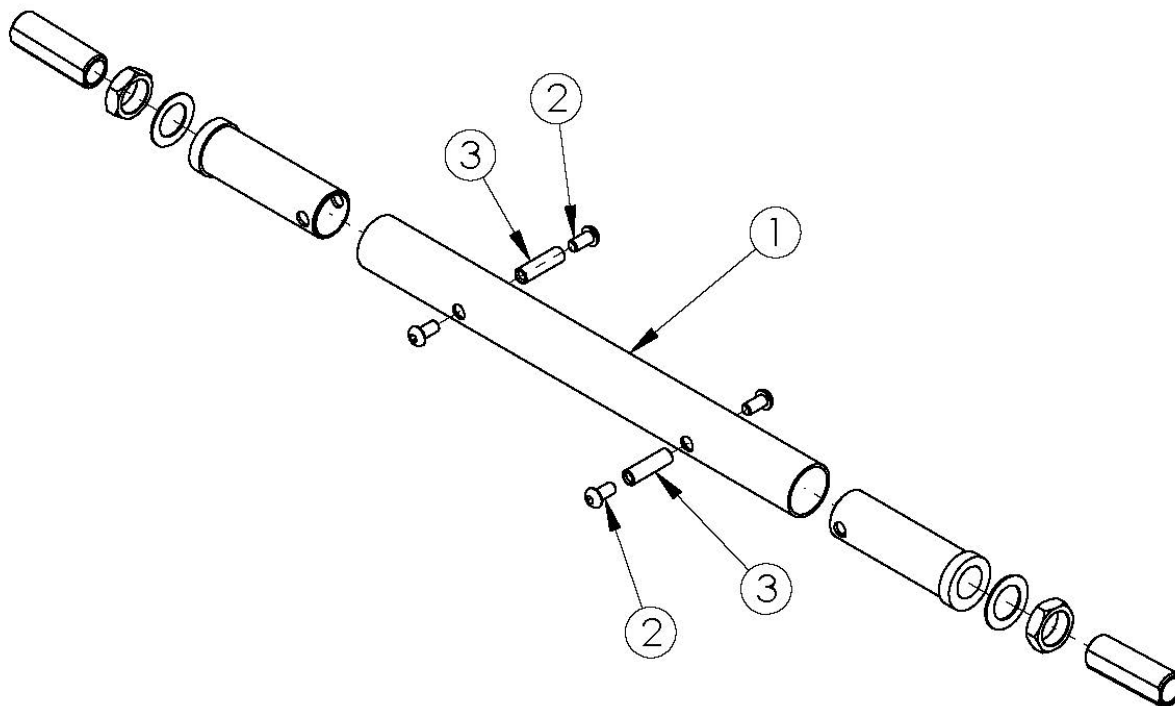

Black Camber Tube kits based on seat width. Color Anodized Camber Tube kits used on all chair widths. New camber tube required when changing the degree of camber by more than 2 degrees. If Wheelchair Drive Ready Option is present, replacement parts need to be processed through Alber USA. If OAD and 5th Wheel are present, please select the appropriate OAD Camber Tube and Standard Camber Tube when growing chair in width.

Item Number	Part Number	Description	Quantity
1a	503238	Camber Tube Rigid 8" Kit Black	1
1b	503239	Camber Tube Rigid 9" Kit Black	1
1c	500720	Camber Tube Rigid 10" Kit Black	1
1d	500721	Camber Tube Rigid 11" Kit Black	1
1e	003369	One Arm Drive Camber Tube 11" Kit, Clik / Rogue XP <i>Used if One Arm Drive is present. Does not include tube end plugs; add quantity 2 of 100532.</i>	1
1f	500722	Camber Tube Rigid 12" Kit Black	1
1g	003370	One Arm Drive Camber Tube 12" Kit, Clik / Rogue / Rogue XP	1

Item Number	Part Number	Description	Quantity
		<i>Used if One Arm Drive is present. Does not include tube end plugs; add quantity 2 of 100532.</i>	
1h	500723	Camber Tube Rigid 13" Kit Black	1
1i	003371	One Arm Drive Camber Tube 13" Kit, Klik / Rogue / Rogue XP <i>Used if One Arm Drive is present. Does not include tube end plugs; add quantity 2 of 100532.</i>	1
1j	500724	Camber Tube Rigid 14" Kit Black	1
1k	003372	One Arm Drive Camber Tube 14" Kit, Klik / Rogue / Rogue XP <i>Used if One Arm Drive is present. Does not include tube end plugs; add quantity 2 of 100532.</i>	1
1l	500725	Camber Tube Rigid 15" Kit Black	1
1m	003373	One Arm Drive Camber Tube 15" Kit, Klik / Rogue / Rogue XP <i>Used if One Arm Drive is present. Does not include tube end plugs; add quantity 2 of 100532.</i>	1
1n	500726	Camber Tube Rigid 16" Kit Black	1
1o	003374	One Arm Drive Camber Tube 16" Kit, Klik / Rogue / Rogue XP <i>Used if One Arm Drive is present. Does not include tube end plugs; add quantity 2 of 100532.</i>	1
1p	500727	Camber Tube Rigid 17" Kit Black	1
1q	003708	One Arm Drive Camber Tube 17" Kit, Klik / Rogue / Rogue XP <i>Used if One Arm Drive is present. Does not include tube end plugs; add quantity 2 of 100532.</i>	1
1r	500728	Camber Tube Rigid 18" Kit Black	1
1s	003709	One Arm Drive Camber Tube 18" Kit, Klik / Rogue / Rogue XP <i>Used if One Arm Drive is present. Does not include tube end plugs; add quantity 2 of 100532.</i>	1
1t	501749	Camber Tube Rigid Blue Anodized	1
1u	501750	Camber Tube Rigid Bright Green Anodized	1
1v	501751	Camber Tube Rigid Orange Anodized	1
1w	501752	Camber Tube Rigid Pink Anodized	1
1x	501753	Camber Tube Rigid Pewter Anodized	1
1y	501754	Camber Tube Rigid Purple Anodized	1
1z	501755	Camber Tube Rigid Red Anodized	1
1aa	501756	Camber Tube Rigid Gold Anodized	1
2	100667	M6 x 12 BTN SHCS BLKZC w/Patch	4

Item Number	Part Number	Description	Quantity
3	000066	M6 x 27 Threaded Barrel	2

Color Anodizing Samples

