


(Discontinued) Catalyst / Spark Seating Dynamics Rocker

Back

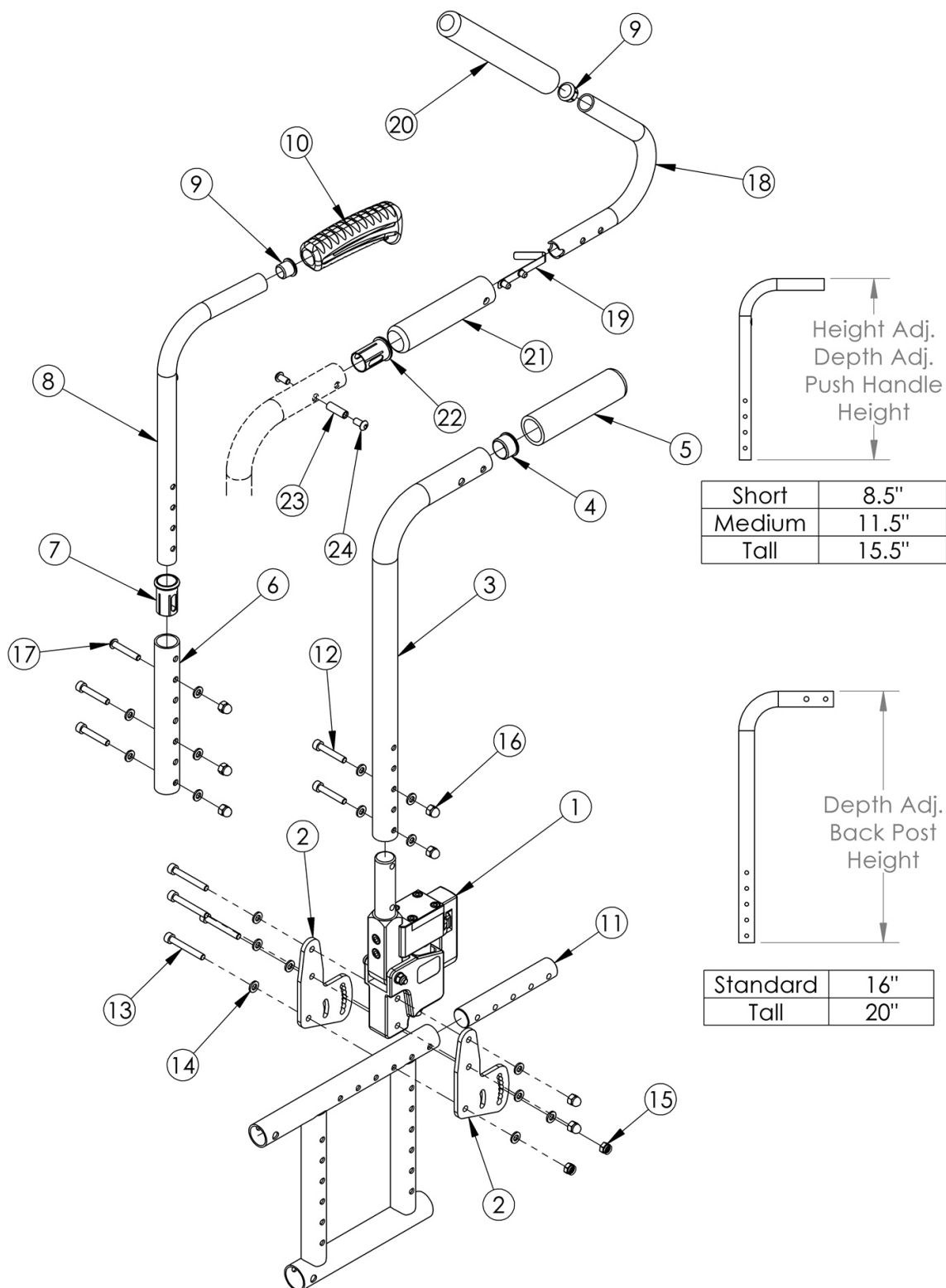

This backrest style discontinued February 8th, 2021 For complete assemblies, select Dynamic Rocker Back (PAIR), Frame Attaching Hardware Kit for Rocker Back (qty 2), choice of Depth Adj Rocker Back Tube Assembly (qty 2) OR Stroller Handle Back Post with Extension Complete Assembly (qty 2) Low Profile Push Handles are not available with Heavy Duty Option or Drum Brake If changing from another style backrest and the Drum Brake is present, a new Drum Brake Cable Kit may be required

Item Number	Part Number	Description	Quantity
13 - 16	502596	Frame Attaching Hardware Kit for Dynamic Back, (One Side) Catalyst	1
3a ,4, 5, 12, 14, 16	502595	Depth Adj Dynamic Back Tube Assm w/ Grip, Short, One Side CAT	1
3b ,4, 5, 12, 14, 16	502594	Depth Adj Dynamic Back Tube Assm w/ Grip, Tall, One Side CAT	1
6, 7, 8a, 9, 10a, 12, 14, 16, 17	502597	Depth Adjustable Height Adjustable Dynamic Back Tube Assembly, 16" - 18" Back Height, One Side	1
6, 7, 8b, 9, 10a, 12, 14, 16, 17	502598	Depth Adjustable Height Adjustable Dynamic Back Tube Assembly, 19" - 22" Back Height, One Side	1
6, 7, 8c, 9, 10a, 12, 14, 16, 17	502599	Depth Adjustable Height Adjustable Dynamic Back Tube Assembly, 23" - 24" Back Height, One Side	1
8a, 9, 10a	501654	Depth Adjustable Height Adjustable Upper Push Handle Short Assembly w/ Standard Grip	1
8b, 9, 10a	501655	Depth Adjustable Height Adjustable Upper Push Handle Medium Assembly w/ Standard Grip	1
8c, 9, 10a	501656	Depth Adjustable Height Adjustable Upper Push Handle Tall Assembly w/ Standard Grip	1
8a, 9, 10b	002762	Depth Adjustable Height Adjustable Upper Push Handle Short Assembly w/ Low Profile Grip	1
8b, 9, 10b	002763	Depth Adjustable Height Adjustable Upper Push Handle Medium Assembly w/ Low Profile Grip	1
8c, 9, 10b	002764	Depth Adjustable Height Adjustable Upper Push Handle Tall Assembly w/ Low Profile Grip	1
3a, 21 - 24	503080	Stroller Handle Depth Adj Dynamic Back Post Assembly, Short CAT	1
3b, 21 - 24	503081	Stroller Handle Depth Adj Dynamic Back Post Assembly, Tall CAT	1
4	100804	Tube End Plug 1"	1
6	002039	Depth Adjustable Height Adjustable Rocker Back Tube, Lower	1
7	000034	Sleeve 3/4" x 1 1/2"	1
9	100638	Tube End Plug 3/4"	1
10a	109432	Ergo Push Handle Grip 3/4" Standard with Instruction	1
10b	109430	Ergo Push Handle Grip 3/4" Low Profile with Instruction	1
11	000605	Depth Adjustable Frame Extension Tube, Catalyst/Spark	1
12	101810	M6 x 35 SHCS BLK ZC	2
13	101472	M6 x 45 SHCS BLK ZC	4

Item Number	Part Number	Description	Quantity
14	100746	M6 Flat Washer Blk Zn	13
15	100658	M6 Nylock Nut Blk Zn	2
16	100558	M6 Nyl Ins Domed Cap L/N Zc	4
17	101811	M6 x 35 BHCS Blk Zn	1
19	100533	Detent Button	1
22	000034	Sleeve 3/4" x 1 1/2"	1
23	000065	M6 x 23.68 Threaded Barrel	1
24	100667	M6 x 12 BTN SHCS BLKZC w/Patch	2